

# OPERATING INSTRUCTION MANUAL

Hereby explain:

This instrument is for life cosmetology,  
not for medical cosmetology.



**BEAUTY SERIES**  
**No. H500**

Before using the product, please read the Instructions carefully and keep it properly.

**First of all, I sincerely thank you for purchasing our instruments. This manual is written for better use of this instrument. Hope the company's products and services to bring you respect.**

### **Pay attention to**

The product must be used in strict accordance with the methods used in this manual. The Company shall be responsible for all the consequences caused, and the Company shall not assume any responsibility.

The Company does not make any form of security, including (but not limited to) an implied guarantee of the fact and fitness of it for a particular purpose.

The Company is not liable for errors contained in this information or for actual performance or use due to the provision of this manual.

This manual contains proprietary information protected by the patent law. Copyright, no photo, copy, copy or translation of any part of this manual without the written consent of the Company. The content contained in the manual can be changed without user notification.

### **The responsibility of the manufacturer**

The Company is only responsible for the safety and reliability of the instrument, that is, the assembly operation, readjustment, improvement and maintenance are conducted by the personnel approved by the Company, the relevant electrical equipment complies with the national standards, and the instrument is used in accordance with the operation instructions.

**This is a cosmetic instrument, can not be used in medical areas, otherwise the company will not assume any responsibility.**

If the beauty agency responsible for using this instrument cannot achieve a satisfactory operation plan, abnormal instrument failure will occur. Today, the company will provide a paid circuit diagram upon request, and will provide a list of calibration methods and other information to help the user by the appropriate, qualified technicians to repair the instrument parts that can be repaired by the user.

### **Pledge**

Manufacturing process and raw materials: The Company guarantees that the instruments (except accessories) will have no production process and raw material failure within one year from the date of shipment. Any inspection of the Company is not complete.

### **In this manual**

This user manual contains all the general information for this instrument. This is the starting point for new users to use the instrument, because it gives a general introduction to the system, explains how it works, and introduces the installation of the instrument and various operation methods. The required information can be found through the catalog.

This manual is read for people familiar with the cosmetic knowledge related to this instrument and experienced in using cosmetic manipulation equipment.

## 1. Notes

### 1.1 Notes for the use place

warn:

- 1) Please do not use and connect to the socket to avoid the fire occurrence;
- 2) Turn off the power switch during maintenance, so as not to cause fire and personnel electric shock;
- 3) The 100-220V  $\pm$  10%, 50HZ-60Hz  $\pm$  1 HZ power supply and effective grounding socket must be used;
- 4) The damaged or processed power cord is in danger of fire or electric shock;
- 5) Do not bend or twist the power cord to avoid fire or electric shock;
- 6) If the instrument is loose or damaged, please contact the company in time;
- 7) Do not use this instrument in unstable environments such as tilt, vibration and impact;
- 8) No water or debris enters inside the instrument. In case of foreign matter, please contact the company in time;
- 9) The power plug must be unplugged when the instrument moves;
- 10) When the instrument moves, it should avoid vibration as far as possible;
- 11) The instrument should be placed in a stable place, and at least 5cm of space should be left around, to ensure the air circulation, conducive to the heat dissipation of the instrument;
- 12) The instrument should not be covered with anything to prevent ventilation blockage;
- 13) Avoid the use of this instrument in the following environments: sunlight exposure; high humidity environment; near water environment; vibration, and tilt; strong magnetic field environment; electromagnetic wave and impact voltage occurrence; chemical storage, corrosive gas places;
- 14) This instrument shall not be placed near the radio, TV, photocopier, fax machine, etc.;
- 15) Do not use them with other instruments (such as microwave) to avoid mutual interference with the instruments and cause misoperation.

### 1.2 Notes before use

- 1) Must read the operation instructions carefully before starting up. The operator must be familiar with the operation manual and operation methods, and the instrument must be managed by special personnel;
  - 2) The operation parameters should be set under the guidance of professionals;
- pay attention to:
- 3) Verify whether the instrument is in a normal operation state before use;
  - 4) Ensure whether the grounding wire is correctly connected;
  - 5) To confirm that all the wires are correctly connected and safe;
  - 6) When using instruments, must use correctly, otherwise there may be danger, must be paid great attention.

### 1.3 Notes in Use

pay attention to:

- 1) When the instrument or the customer is abnormal during the operation, the operator shall immediately remove the operation handle from the customer and press the stop switch to stop the use of the instrument;

pay attention to:

- 2) During the operation of the instrument, to constantly observe the situation of beauty guests and instruments, and pay attention to whether there is any abnormal situation at any time;
- 3) The instrument shall be used correctly by the skilled operation technician, and the personnel other than the operation technician shall not use the instrument.

#### 1.4 Notes after use

pay attention to:

- 1) After the operation, the control handle is put back to the control handle hanger.
- 2) After using the operating handle, clean the probe and recycle the instrument (pack the solution bottle with water and clean it around) for the next use.

#### 1.5 Notes for failure, storage and inspection

warn:

- 1) Do not use the instrument when it is abnormal. In case of smoke and coke, can not operate, if continue to use, there is a danger of fire electric shock. Please press the stop button, turn off the power, unplug, and contact our company or dealer immediately;
- 2) Personnel other than technical personnel, absolutely can not repair and modify the equipment. This instrument is absolutely not allowed to open inside!

pay attention to:



- 3) Instruments and components must be checked regularly;
- 4) Do not use the cleaning tools that damage the protective coating of the instrument, such as brush, brush, etc.;
- 5) If the instrument is not used for more than 2 weeks, the instrument must be carefully checked before using it again.

#### 1.6 Warning and warning instructions

pay attention to:

When using this instrument near short wave or microwave equipment (1m), the instrument output may be unstable due to microwave interference.

## 2.Introduction to the instrument

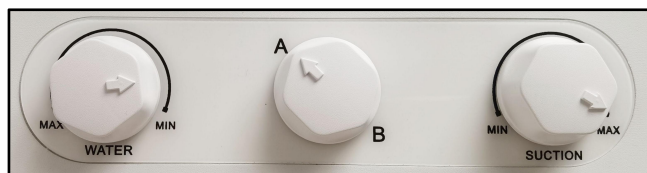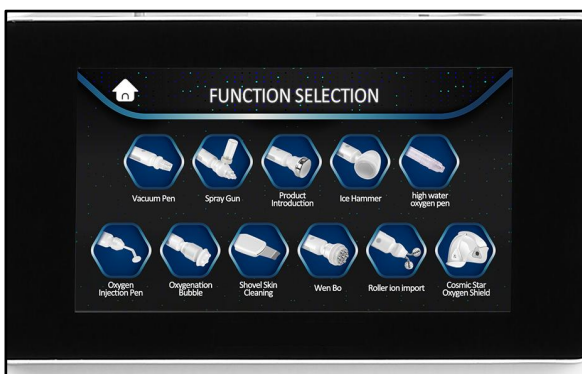

- 1.The amount of the water of the suction pen is adjusted less clockwise and more counterclockwise;
- 2.Straw pen, the solution bottle channel A / B bottle selection knob;
- 3.Sucking pen, suction size, clockwise large, counterclockwise small;
- 4.High pressure water and light pressure, clockwise small, counterclockwise large;

(Hd smart touch screen)



- 1.Solution bottle A / B / C bottle;
- 2.The oxygen wet hua pot of the cosmic star oxygen cover;
- 3.Oxygen flowmeter adjustment of the cosmic star oxygen cover.




(Hanging way of each handle)

1. Filter (the instrument must be installed during operation, otherwise causing suction, when the filter has water,Get rid of it in time! After the and then inserted.

2.Waste water bottle of the suction pen;  
(to absorb more than half of the sewage,remove and clean, and load on tight bottles.)

3.Power supply switch / power supply cord interface;

4.Install the handle according to the corresponding interface.

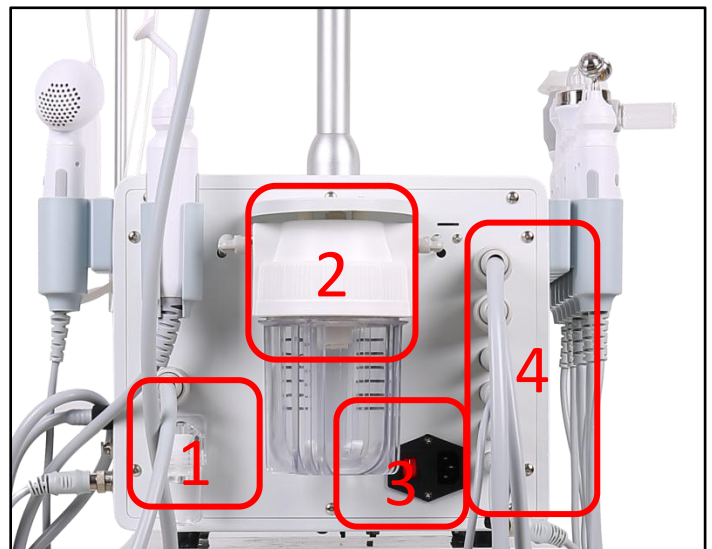

### 3. Installation instructions

#### 3.1 Unpack and check

3.1.1 Carefully remove the instruments and accessories from the packing box, and keep the packaging materials for later transportation or preservation. Count the attachment.

3.1.2 Check for any mechanical damage.

Note: immediately contact the sales department or agent if you have any problems.

3.2 The instrument shall be placed in a room with a temperature of 5°C ~40°C, a humidity not exceeding 80%, clean and ventilated, to avoid direct sunlight. And ensure that the vents are not blocked and that there is at least 5cm space around the instrument. Remember not to place items on the instrument, even if it is temporary, also do not allow it.

- 3.3 Installation of hangers, high pressure non-invasive water light, nano beauty plastic water light hangers to be larger, other hangers.
- 3.4 When installing the handle and linking the handle plug and plug parts, please install it one by one according to the hole position on the host machine.
- 3.5 Install the filter, plug in the power cord, and turn on the switch.

## 4. Operating procedures

- 4.1 Wear gloves to disinfect the host and handle (disinfect both before and after using the handle).
- 4.2 Before nursing, please remove all the metal accessories.

4.3 Start in, open the interface, click select traditional language or English.




4.4 Enter the handle selection interface.



4.5 Select the operating probe and click "Vacuum Pen" mode; remove makeup and clean the facial skin.

Enter the operation interface, it is recommended to install nutrient water or special small bubble water in A / B / C bottle;

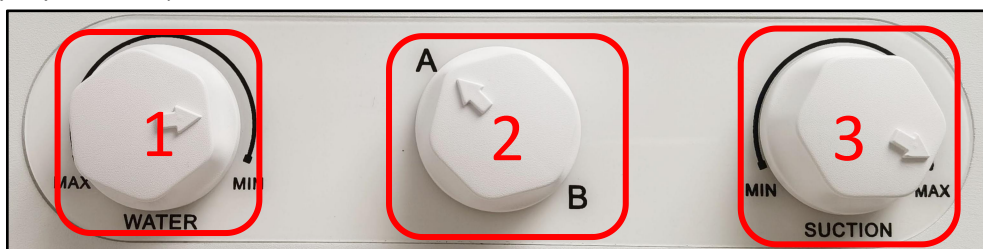


1. Rotate the "1" WATER knob to adjust the appropriate water flow size; (according to the icon, rotate less clockwise and more counterclockwise)
2. Turn the "2" knob and select any channel of A / B bottle;
3. Rotate the "3" SUCTION knob to adjust the appropriate suction size; (according to the icon, clockwise rotation and counterclockwise rotation to decrease)

Select the working channel, select A / B bottle, after any channel;

Select "TIME" and click " ◀ ▶ " to adjust the appropriate time;

After adjusting the water flow size / suction size, click start / pause;

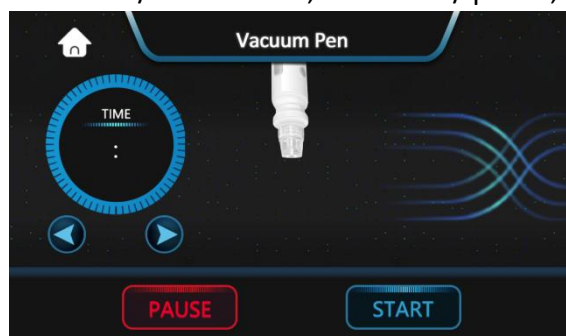

Select the operation head suitable for the customer's skin (hard head for normal skin, soft glue head for sensitive skin), click to start, do the test on the hand first, no problem, and then operate on the customer's face. Follow the face from the bottom to the top operation, can be fixed point operation or short distance pull (1-2cm) operation, face 1-2 times, T area can strengthen the

operation.

Note: When the wastewater bottle is full, the suction pen will then stop working automatically. Remove the wastewater bottle and install it to start the work again.

4.6 Select and click the "shovel skin Cleaning" mode to enter the operation interface, and apply uniform toner or pure water on your face.

Select mode: A is continuous mode, B is short intermittent mode, C is long intermittent mode;

Select "ENERGY" and click "◀ ▶" to adjust the appropriate energy;

Select "TIME" and click "◀ ▶" to adjust the appropriate time;

Click start / pause; the button on the handle can also control start / pause.



The handle is tilted 45° flat skin operation, along the facial skin from the bottom up operation. Energy starts from low, adjust according to the customer needs, clean 1-2 times, T area can strengthen the operation.

4.7 Select the operating probe, select and click "Oxygenation Bubble" mode to enter the operation interface;

Select "ENERGY" and click "◀ ▶" to adjust the appropriate energy;

Select "TIME" and click "◀ ▶" to adjust the appropriate time;

Click Start / pause;



With the oxygen bubble, essence gel and other products, the product coated the whole face, along the facial skin from the bottom to the top operation, using the use of oxygen bubble fixed point slow circle operation, the energy starts from low, according to the customer needs to adjust, operation 1-2 times.

4.8 Click "Spray Gun" mode to enter the operation interface. The product in the probe ampoule should be equipped with essence (it must be water agent. If there is viscosity, please use normal saline or pure water into water agent to avoid blocking).

Select "ENERGY" and click "◀ ▶" to adjust the appropriate energy;

Select "TIME" and click "◀ ▶" to adjust the appropriate time;

Click start / temporary; the button on the handle can also control start / pause.






Adjust the appropriate amount of water, follow the facial skin from the bottom up the operation, the mouth and nose slightly cover, according to the customer skin conditions for 2-3 times. It can operate in far range, but also in close range.

4.9 Select and click "Oxygenation Bubble " mode to enter the operation interface and apply uniform essence on the face;

Select mode: Table A continuous mode, B represents short intermittent mode, C represents long intermittent mode;

Select "TIME" and click " ◀ ▶ " to adjust the appropriate time;

Click Start / pause;

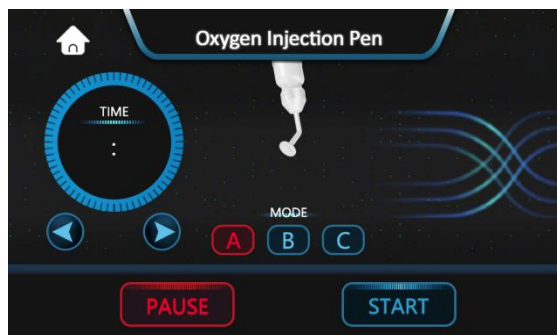

When giving oxygen injection, put the oxygen injection nozzle parallel to the skin, and operate from the bottom down the face, fixed point or slide, and operate 1-2 times according to the customer's skin condition.

4.10 Select and click "Product Introduction" mode to enter the operation interface and apply uniform nutrient solution / nutrient cream on your face.

Select mode: A is continuous mode, B is short intermittent mode, C is long intermittent mode;

Select "ENERGY" and click " ◀ ▶ " to adjust the appropriate energy;

Select " TIME " and click " ◀ ▶ " to adjust the appropriate time;

Click start / pause; the button on the handle can also control start / pause.



The energy is adjusted from 3. The energy is adjusted according to the customer's bearing capacity and circled along the five lines of the face.



4.11 Click "Wen Bo" mode to clean the facial skin and apply uniform nutrition solution / cream on the face.

Select mode: A is continuous mode, B is short intermittent mode, C is long intermittent mode;

Select "ENERGY" and click " ◀ ▶ " to adjust the appropriate energy;

Select " TIME " and click " ◀ ▶ " to adjust the appropriate time;

**Note that the handle must be completely flat to the skin, a single pole contact with the skin will cause customer discomfort!**

Click start / temporary; the button on the handle can also control start / pause.




Make the probe in full contact with the skin, the energy is adjusted according to the customer's endurance, and promote the five lines along the face.

4.12 Select and click on the "Ice Hammer" mode. Clean the facial skin and apply uniform nutrition solution / nutrition cream / facial mask.

Select "ENERGY" and click " ◀ ▶ " to adjust the appropriate energy;

Select " TIME " and click " ◀ ▶ " to adjust the appropriate time;

Click start / temporary; the button on the handle can also control start / pause.

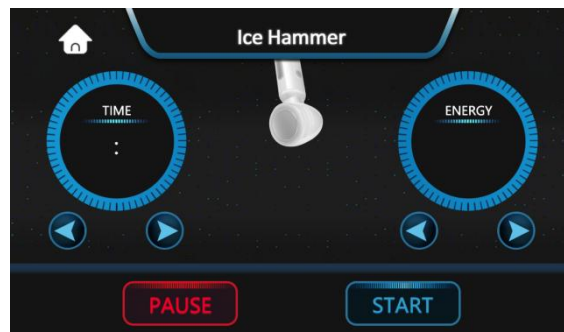

Cool, calm and soothing, follow the face from the bottom up, fixed point or slide operation (according to the product used by the skin).

4.15 Click "Roller Ion Import" mode to clean the facial skin and apply uniform nutrition solution / nutrition cream on the face.

Select mode: A is continuous mode, B is short intermittent mode, C is long intermittent mode;

Select "ENERGY" and click " ◀ ▶ " to adjust the appropriate energy (from low to high);

Select " TIME " and click " ◀ ▶ " to adjust the appropriate time;

**Note that the handle must be completely flat to the skin, a single pole contact with the skin will cause customer discomfort!**

Click start / temporary; the button on the handle can also control start / pause.




Make the probe and the skin complete contact, energy from low, adjust according to the customer's endurance, along the face to the five-line lifting operation.

4.6 Select "High Water Oxygen Pen" mode to enter the operation interface;

One end of the oxygen pen belt needle is connected to the saline bottle, the other end of the trachea is connected to the oxygen pen interface, the water and oxygen are injected into the human skin layer through the rapid pressure of oxygen, and the roller in the middle of the pipe controls the size of the water;

Select "TIME" and click "◀ ▶" to adjust the appropriate time;

Click Start / pause;

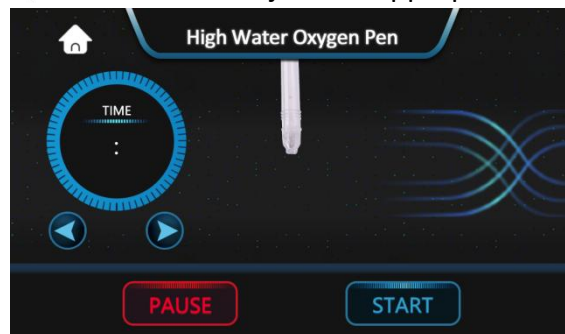

With normal saline or pure water, the probe is about 1cm from the skin along the face skin. When use, take the cotton pad to prevent water overflow to other places; slightly cover the mouth and nose, 2-3 times according to the customer's skin condition.

4.16 Click "Cosmic Star Oxygen Shield " mode; clean facial skin and apply facial mask to the face.

There are 7 manual modes of light: red light, green light, blue light, yellow light, purple light, green light, white light;

Please choose one of the modes of matching light according to your own situation or use needs.

Select "Music" and click "◀ ▶" to adjust the appropriate music;

Select "TIME" and click "◀ ▶" to adjust the appropriate time;

Select and click "Oxygen";

Click Start / pause;



Check the phototherapy mode to be used according to customer request or skin condition. If other light is needed, switch the light mode after 5-10 minutes.

Clean your skin, apply a mask and wear a shading eye mask. Select the startup function, and the mask is removed after 10-20 minutes.

## **5. End of operation**

### **5.1 After the operation**

#### **1) Overall cleaning**

Basic cleaning must be done to maintain prevention against bacterial infections. After the use of accessories, do not throw at will, should be cleaned with the clean cloth carefully clean, put on the hanging rack. It is recommended to use clean water or a little cleaner.

#### **2) Cleaning method of the suction pen**

After using the suction pen, you must use pure water to clean the channel, otherwise when the product is dry, it will affect the suction.

You can put pure water in A / B bottle circulation cleaning, adjust the water flow to the maximum, click "start" thumb hold the pen, long press for 10 seconds, cycle 3-5 times, so that the pure water can clean the whole pen. Then take out the tip of the pen and wipe the handle without water stains.

#### **Cleaning method of the suction pen tip**

In order to prevent infection, the suction tip must be cleaned. After use, use the brush with detergent (must be soft) to clean up the impurities of the suction head, and then soak it with 75% alcohol or put it into the ultraviolet sterilizer.

#### **3) Cleaning method of high-pressure water pointer**

If you use normal saline or other precipitated water, clean it up regularly to prevent the needle from blocking, and put in about 1 / 10 of the pure water. Spray in the bottle can finish.

#### **4) Cleaning method of the spray gun**

Wash after use every day. Put in about 1 / 5 of warm water and spray it in the bottle.

### **5.2 Turn off the main power supply switch and unplug the power cord if necessary.**

## **6. Precautions**

7.1: Note that when the waste water bottle reaches the upper limit, it stops working and alarms. Please throw it away in time to prevent damage to the instrument.

7.2: Customers need to remove their metal jewelry and do not play with their mobile phones.

7.3: The face has been microplastic parts to avoid prohibited operation.

7.4: Skin allergy, inflammation, sunburn, female three operation with caution.

7.5: No heart bypass patients.

7.6: skin cuticle thin or cheekbone red blood silk serious, use with caution.

7.7: facial surgery or do deep exfoliating, not to the recovery period of the people, use with caution.

7.8: After operation, you should apply hydrating mask for a week. Do not use efficacy products.

## **8. Maintenance and repair**

### **8.1 Maintenance**

#### **8.1.1 Daily inspection**

Managers must inspect the instruments and components.

A) Whether the power socket is reliably connected to the ground, please replace the power socket.

B) whether the power wire is deformed, broken, broken line: through the naked eye inspection, whether the wire is deteriorated, deformation, if some circumstances, fire may be caused by leakage. Change the new power cord immediately.

C) Control whether there is any abnormal or broken line of the handle: through the naked eye inspection, whether the output line is deteriorated or deformed, if so, please replace it immediately.

D) Please check whether the instrument is correct before use and stop use if incorrect.

#### 8.1.2 Cleaning

When cleaning the instrument, the dust must be wiped with a dry soft cloth and the stolen goods with a soft wet cloth. Before cleaning the instrument, the power switch must be turned off and the power cord disconnected.

8.1.3 For a long time (more than one year), it shall be carried out by professional maintenance personnel.

8.1.4 When the instrument is not used, it should be placed in a ventilated and dry place, and the instrument should be covered with a dust cover.

#### 8.2 Maintenance

8.2.1 The company's product quality problems host free warranty for one year (note: with the purchase bill and warranty card warranty; the warranty scope here is limited to the instrument host, excluding accessories and other consumables that need to be replaced regularly).

8.2.2 After the warranty period, please contact the manufacturer directly to provide the service guarantee.

8.2.3 To repair the machine, contact the customer service department or dealer of the company, do not repair it without authorization, and remove it by yourself. The company will not provide warranty.

### 9. Use permissions

9.1 The warranty period of this instrument is 1 year.(Note: except for man-made damage)

9.2 The warranty period of the accessories is 3 months.(Note: except for man-made damage)

9.3 The scrapped instruments and accessories beyond the service period shall be treated according to the relevant local regulations.

Maintenance log		
Product Name:_____ Date of purchase:_____		
Maintenance date	Cause of failure	Maintenance content

(The Company reserves the final interpretation of the information)