User Manual

AS161





Content

1.Safety Precautions 1
2.Contraindications ····· 2
3.Overview2
4.Control Panel Introduction
5.Technical Specification 5
6.Use Precautions 5
7.Maintenance & Cleaning 6
8.FAQ
9.Ask & Answer 8
10.Operation Guide
11.Warranty & Support 1

1. Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this device and related technology.
- 2.As a high-precision electronic device, this device should be stored properly. Avoid humid conditions, direct sunlight, and proximity to flammable or explosive materials to prevent damage.
- 3. Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the device in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the device on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should consult their doctor before using this device.
- 7. Ensure the device is applied on moist skin and avoid the operation on dry skin.
- 8. Power off the device when not in use to ensure its safety.
- 9.To maintain hygiene and extend the device's service life, sterilize it with saline.

2.Contraindications

Before using this device, it is essential to consult your doctor or seek advice from professionals. Details below:

- 1.Pregnancy or lactation.
- 2. Hypertension, heart disease, or with a pacemaker installed.
- 3. Unhealed wounds or ongoing recovery from recent surgery.
- 4. Epilepsy, severe diabetes, and hyperthyroidism.
- 5. Malignant tumors, hemophilia, and hemorrhage.
- 6. Skin disease or infectious disease.
- 7.Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
- 8.Recent plastic surgery (within three months), or presence of implants or metal objects in the treatment area.

3.Overview

Utilizing the most popular and safe technology, this 6-in-1 versatile beauty device provides comprehensive enhancements, including skin tightening and anti-aging benefits, ensuring a radiant transformation of your facial appearance. It's the ideal solution for those looking to improve their skin without the need for injections, medications, or side effects. Expect immediate results with its external application.

4.Control Panel Introduction



ABCD bottle adjustable swivel button

Ultrasound Switch RF Switch Scrubber Switch RF Energy Display/Set Abrasion Switch Ultrasound Energy Cold Switch Scrubber Energy 00 Display/Set 777 $\sqrt{}$ лл Spray Switch Mode selection: -Continuous : working mode ____: Intermittent Set working mode Time Display Waste Water Bottle: and Setting Turns red when full. Empty immediately.

5.Technical Specification

	Item Type: Beauty Machine	Material: ABS
	Color: As Pictures Show	Voltage: 110-240V
	Optional Plug: US Plug, UK Plug, AU Plug, EU Plug	
	Item Size: 25*40*38cm/9.84*15.75*14.96in	
	Package Weight: Approx. 700	

■ 6.Use Precautions

- 1. Strictly follow the instructions in the manual to use the device and train operator.
- 2.Before using the device, use a plug with a grounding pin and ensure the power socket is grounded.
- 3.Make sure the voltage of the device is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the device.
- 4.Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.

5

EN



- 5. During Hot & Cold treatment, avoid air vents, as poor heat dissipation may affect the device's service life.
- 6. During the Abrasion operation, adjust the suction intensity according to individual tolerance.
- 7.Clean the waste water bottle immediately after Abrasion treatment.

7. Maintenance & Cleaning

- 1.Disconnect the power cord when not in use and store the device in a dry, cool, and well-ventilated place.
- 2.Clean and sanitize the applicator with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.
- 3. After Abrasion treatment, brush the abrasion head with a disposable toothbrush, rinse with water, soak in alcohol for 3 minutes, then rinse again to remove alcohol residue.
- 4. After cleaning the device, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.
- 5.Disassembling the device and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8.FAQ

1. Will the skin become thinner and more sensitive after Abrasion treatment?

A: No, Abrasion is a non-invasive procedure that removes only the aged stratum corneum. As the skin naturally renews itself every 28 days, this treatment enhances smoothness, brightness, and refinement without increasing sensitivity.

2. Why is exfoliation necessary?

A: The stratum corneum, the outermost layer of the epidermis, consists of 15–20 layers of skin components that vary by body area. The skin's renewal cycle also changes with age—typically 28 days for teenagers and an individual's age plus 10 days for adults. However, various factors can disrupt this process, causing the stratum corneum to accumulate, leading to a dull complexion and sluggish skin metabolism. In such cases, exfoliation is essential to promote healthy skin renewal.

3.Can I apply makeup immediately after using this device?

A: Yes, the treatment enhances skin hydration and plumpness, ensuring smooth and well-blended makeup application. This non-invasive approach delivers fast, safe, and healthy results.

4. When can I expect to see results?

A: You will notice improved skin clarity and radiance immediately after the first session. Multiple treatments enhance hydration, brighten the complexion, reduce fine lines, and help slow the aging process.


6




5.Is the treatment addictive?

A: No, Abrasion treatment is a skincare procedure that enhances skin health and provides deep cleansing. It improves nutrient absorption while addressing concerns like dullness, dehydration, rough texture, and wrinkles. Discontinuing treatment simply allows the skin to return to its natural state without any dependency.

9. Ask & Answer

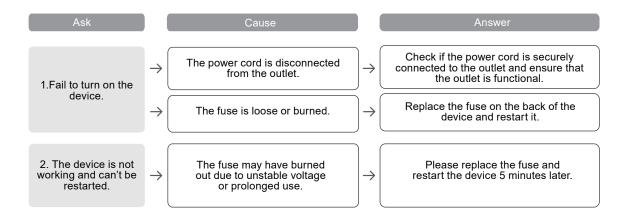


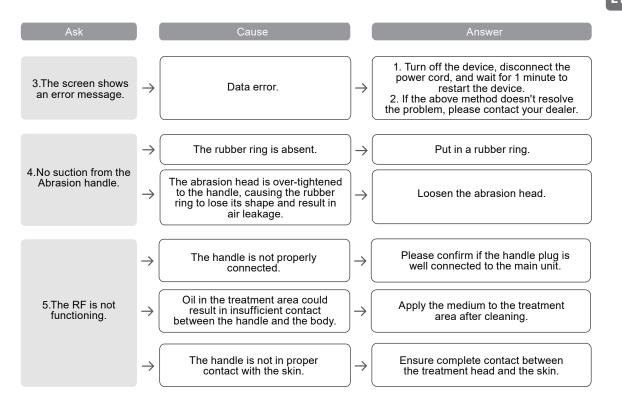




10.Operation Guide

Face:

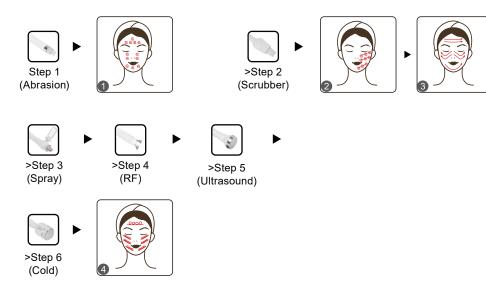

11.Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

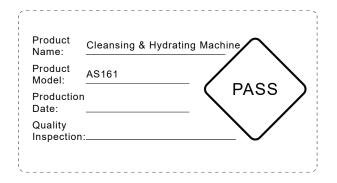

11