
User Manual

FC-23K5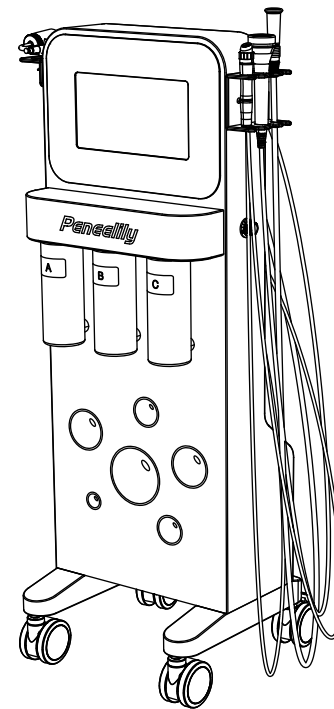

Thank you for choosing the Facial Skincare Device. Please read this user manual carefully before using the instrument!

This device uses microbubble vacuum adsorption to deeply cleanse hair follicles of impurities; a spray gun atomizes moisturizing essence to penetrate the skin's base and repair the skin barrier; a suction device promotes blood circulation and accelerates lymphatic drainage; ultrasonic waves refine the essence for better absorption; EMS microcurrents tighten the skin's contours; and EP electrical pulses enhance permeability. The complete system creates a closed-loop skincare routine, revitalizing the skin's health and vitality from the surface to the core.

Content

1. Benefits	1
2. Use Precautions	1
3. Contraindications	2
4. Technical Specification	2
5. Packing List	3
6. Installation	5
7. User Guide	9
8. Troubleshooting Guide	25
9. FAQ	27
10. Warranty and support	28

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1. Benefits

1. Improves hyperpigmentation
2. Removes dead skin cells, excess oil, blackheads, and whiteheads
3. Reduces fine lines and wrinkles
4. Improves enlarged pores
5. Improves facial puffiness and dullness
6. Improves sagging skin

2. Use Precautions


1. Please read this manual carefully before use to familiarize yourself with the product's operating instructions and related technical information.
2. Warranty service for this product is provided by the designated after-sales service provider.
3. When using the Hydro function, start at the lowest suction level and gradually increase it once your skin has adjusted.
4. This comprehensive skin care device is a high-precision electronic product and must be handled and stored with care. Do not place it in damp areas, in direct sunlight, or near flammable or explosive materials to prevent damage.
5. Turn off the power switch when the device is not in use, and ensure that the main power supply is turned off before leaving the area to ensure electrical safety.

3. Contraindications


Those with the following symptoms should use this instrument with caution. Please consult a doctor or professional before using it.

1. People with severely sensitive skin and those in the period of allergy
2. Those with implants on their face
3. Those with large areas of facial trauma
4. Patients with skin diseases and infectious diseases
5. Those who have recently undergone injection products, such as hyaluronic acid, water light, anti-wrinkle, or plastic surgery


4. Technical Specification

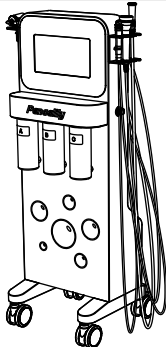	Power supply Input: AC110V-240v 50HZ-60Hz	Net Weight: 21KG
	Power: 32W	Gross Weight: 27KG
	Dimensions: 38*50*108CM	
	Input Power: 32W	

5. Packing list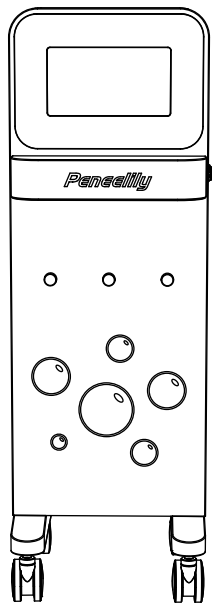

Main unit x1

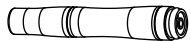
Hydro handle x 1

Hydro Hard Tip x 8

Hydro Silicone Tip x 6

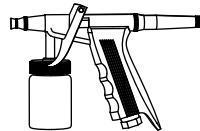Spray gun x 1

Hydro sanding head x 2

Suction Cup x 2


Hydro sealing head x 1

Suction connector x 1

Airbrush braided air hose x 1

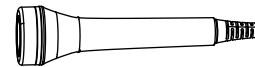
Import Hub handle x 1

Fuse x 4


Filters x 2

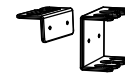
Accessory stand x 2


Waste water bottles x 1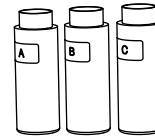

Water purification bottles x 3

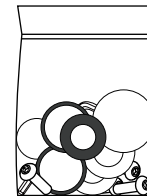Accessory package x 1

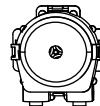peristaltic pump x 1

Peristaltic pump hose x 1

Power cord x 1


Instruction manual x 1


Before use, please check the instrument and accessories or attachments to ensure they are complete. Then remove all packaging materials.

6. Installation


1. Install all the accessory racks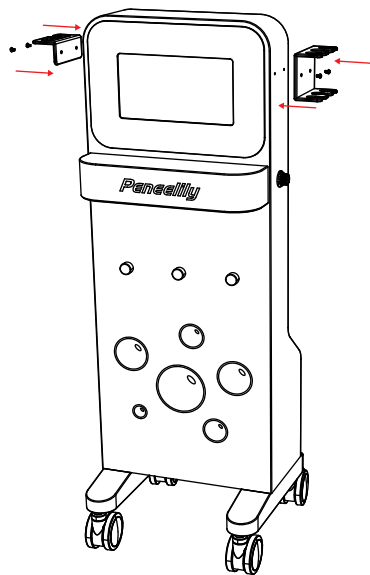


2. Connect the bottle to the Spray Gun handle. Install the Spray Gun's connecting cable, connecting one end of the braided tubing to the main unit and the other end to the Spray Gun.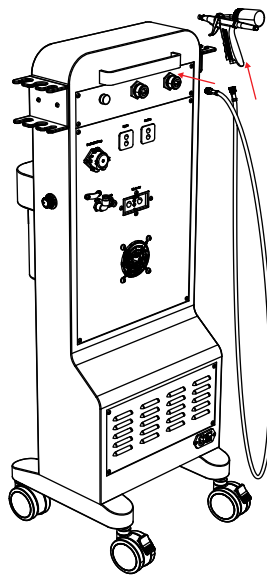


3. Install the connecting tube for the suction cup. First, connect one end of the white rubber tube to the main unit, and the other end to the suction cup.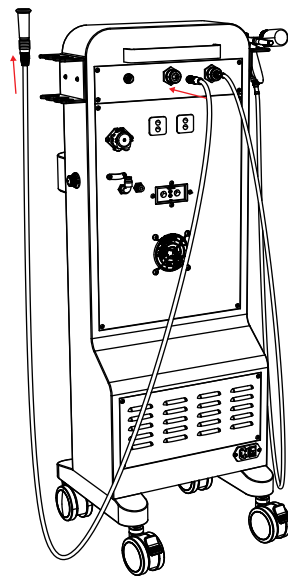


4. Install the Import Hub connection cable.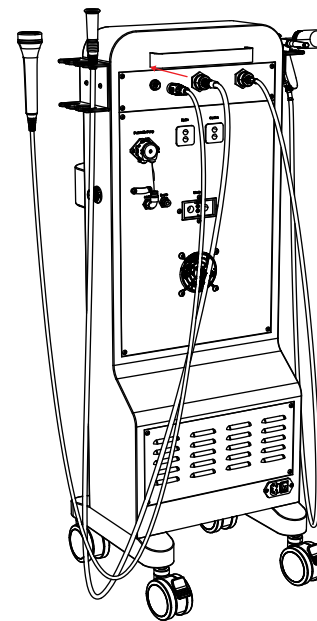



5. Install the Hydro connector: Firmly insert the blue tube into the drain outlet (Tip: When disassembling, press down the white retaining ring on the drain outlet and then pull out the tube); connect the white tube to the inlet on the left side of the peristaltic pump.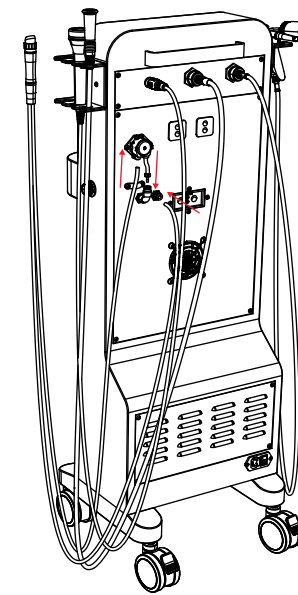

6. Install wastewater bottles and filters.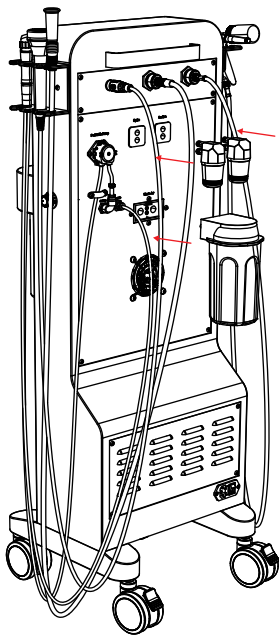

7. Install the solution bottle

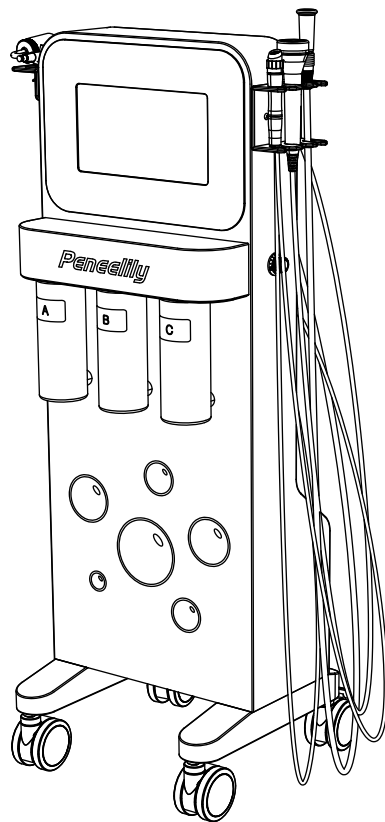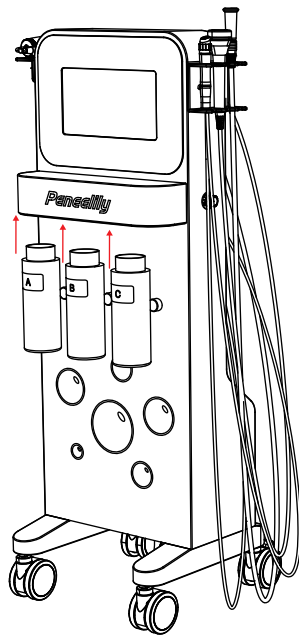
Replace the transparent rubber tubing on the peristaltic pump (only if the silicone tubing is damaged):

1. Press the left and right retaining clips of the peristaltic pump and remove the pump body from the main unit.


2. Press down on the upper and lower clasps of the pump body to open the pump housing.

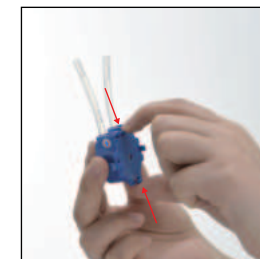
3. Remove the original transparent rubber tube.

4. Insert the new transparent hose into the slot of the pump body.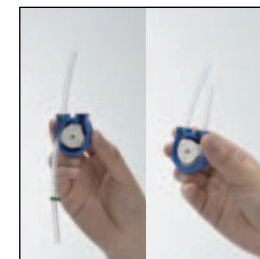



5. Secure the pump body housing and ensure that the snap fastener is properly engaged.

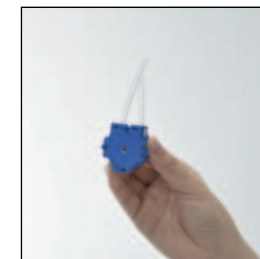6. Insert the assembled pump body back into the main unit and make sure the connection is secure.

7. User Guide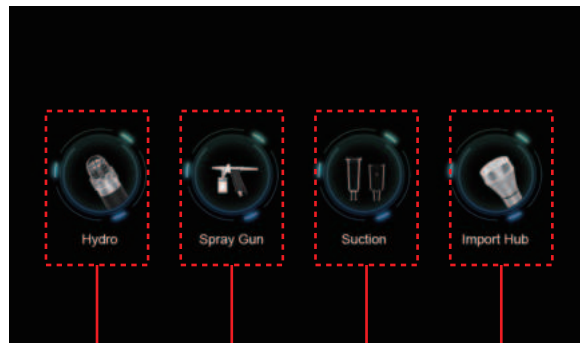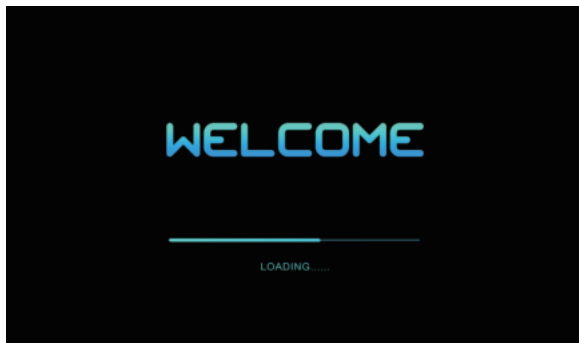

Hydro


Spray Gun

Suction

Import Hub

1. Hydro

1.1 From the main menu, select the "Hydro" icon to enter the function interface.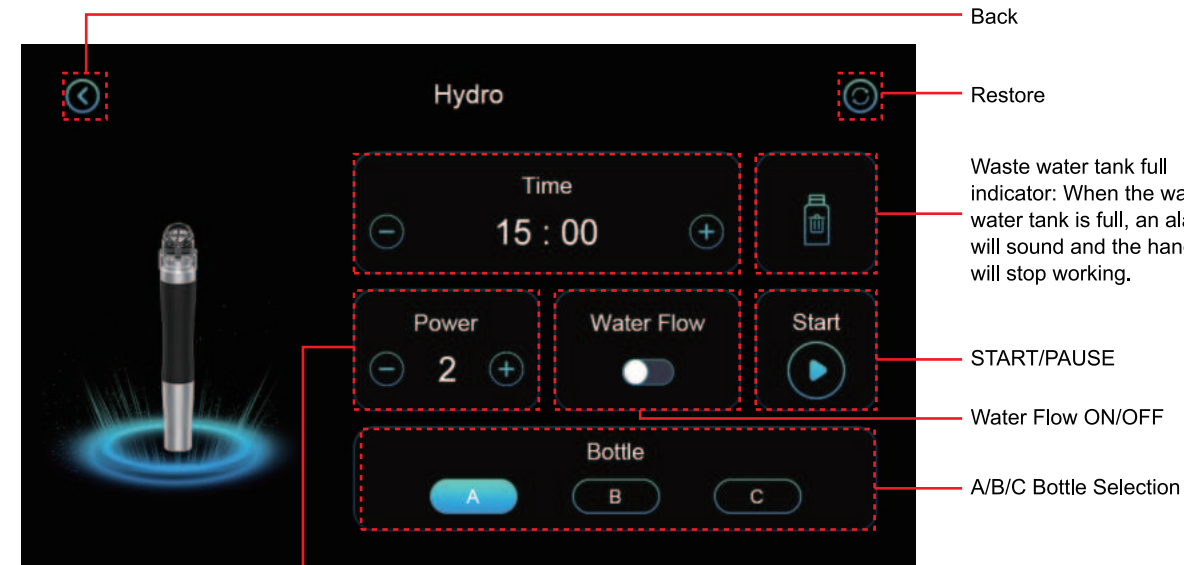

Back

Restore

Waste water tank full indicator: When the waste water tank is full, an alarm will sound and the handle will stop working.




START/PAUSE

Water Flow ON/OFF

A/B/C Bottle Selection

POWER
(water flow and suction force)
display and adjustment 0-5

1.2 How to Choose a Hydro Head

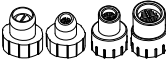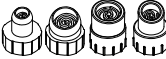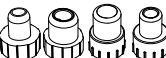
	Flat suction tip: Suitable for cleaning large areas of grease, dust and keratin.
	Straight suction tip: Suitable for cleaning dirt and makeup residue from linear areas of the face (such as the bridge of the nose, philtrum, and around the eyes).
	Spiral suction tip: Suitable for deep cleaning, effectively removing and extracting stubborn dirt, blackheads, and dead skin cells from the pore walls.
	Spot treatment tip: designed for targeted application on problem areas such as spots and blackheads

Note: If your skin is sensitive or you prefer a gentle cleansing method, it is recommended to choose the white soft silicone head. The transparent hard silicone head is suitable for normal skin.


	Note: Before installing a silicone tip, remove the rubber ring from the hydro dermabrasion handle to ensure the tip fits securely in place.
	Sealing head Note: When using the Hydro function for the first time, install the sealing head onto the hydro handle. Wait about 40 seconds after starting the device. Once water flows through the transparent tube, the unit is ready for use.
	Suitable for the cheeks and forehead
	Suitable for small areas around the eyes or nose

1.3 Hydro Installation


① Take a rubber ring and attach it to the handle (the machine you received already has a rubber ring installed).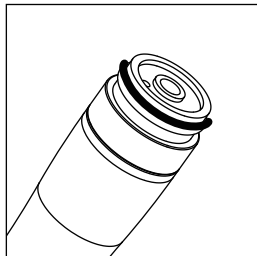

② Select the appropriate tip and attach it to the handle.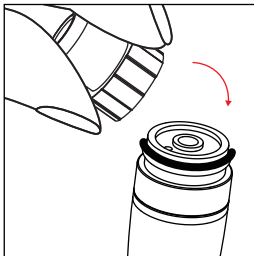


③ Remove the A, B, and C solution bottles.

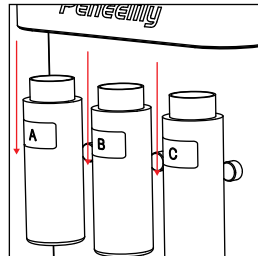④ Fill the A, B, and C solution bottles with small-bubble solution or purified water.

⑤ Reattach the solution bottles to the main unit.

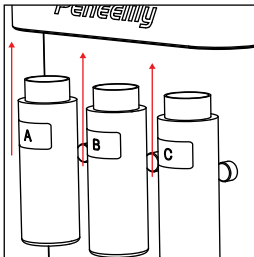1.4 Guidelines for Use

Recommended medium: Small bubble solution (AS1/AO3/SA2) + Blackhead Extraction Solution

Solution Guide:

AS1: Contains lactic acid, yeast extract, green tea extract, etc. Suitable for normal skin, recommended for full-face use to help deep cleanse and revitalize the skin.

AS2: Contains salicylic acid, lactic acid, sodium lactate, aloe vera and green tea extract, etc. Specially designed for sensitive skin and oily skin. Mainly used on the T-zone to control oil, clean pores and improve skin problems.

- ① Choose the appropriate hydro head and exfoliating product based on your individual needs and skin condition.
- ② Configure Setting: Please configure the device parameters based on your skin's specific needs.
- ③ Start from the chin, perform two to three operations, connect the three points into a line for one operation.

After completing the face treatment, use a circular motion to focus on cleaning the nose area.

Tip: during the process, gently wipe the skin with a damp cotton pad to remove residues as you go.

④ Recommended Frequency:

Oily Skin: Once pre week

Combination Skin: Once every 2 weeks

Normal/Dry Skin: Once every 3 to 4 weeks


1.5 Important Notes for Use ⚠

When using, it is recommended to start with the lowest suction force. Once the skin has adapted, gradually increase the suction force to avoid discomfort caused by excessive pulling of the skin.


1.6 Cleaning and Maintenance

Cleaning the Hydro Dermabrasion Tips:


① Use the included cleaning brush to remove residue from inside the tip under running water.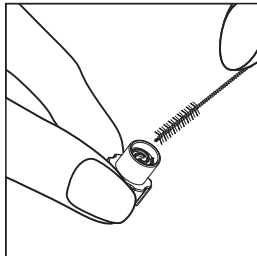

② Soak the tips in saline solution or rubbing alcohol for approximately 5 minutes.

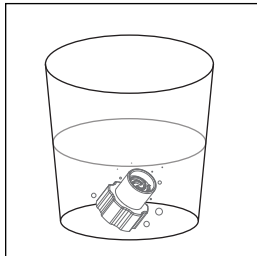
③ Rinse thoroughly with clean water and allow all parts to air dry completely before storing them in the storage case.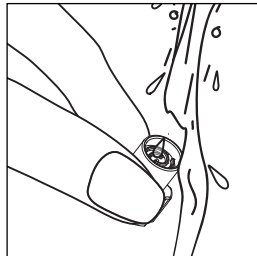


Flushing the Handle & Tube:

① Keep the silicone ring on the handle, then install the white sealing head onto the hydro handle.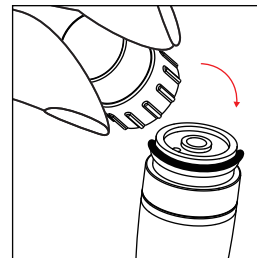


④ The cleaning is finished once all the water has circulated completely.

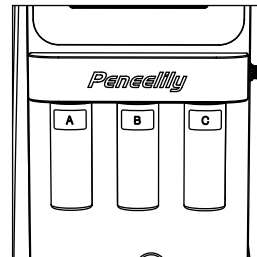② Pour clean water into any of Bottle A/B/C, and empty the waste water bottle.


③ Turn on the switch of the hydro handle and set the device power to maximum.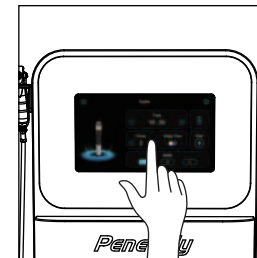

Note: Please clean the pipeline following the steps above after each use of the hydro handle. This can effectively extend the service life of the device.


2. Spray Gun

2.1 From the main menu, select the "Spray Gun" icon to enter the function interface.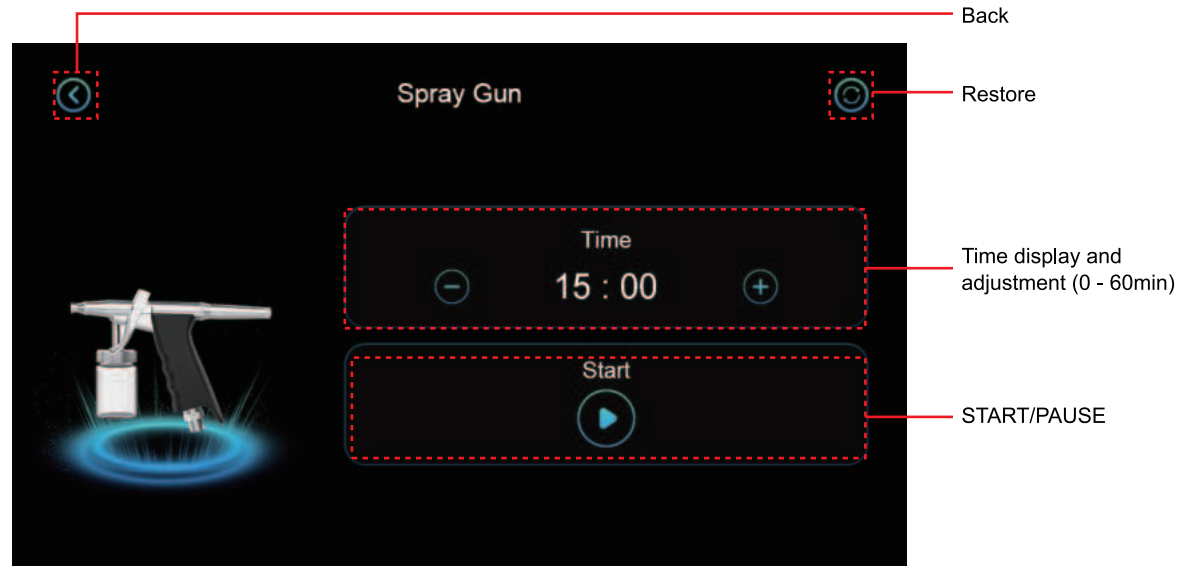


2.2 Spray Function Installation

① Remove the bottle and fill it with water (you can use purified water, serum, toner, saline solution, etc.; for serum, use a 1:10 dilution ratio with water, depending on your skin type).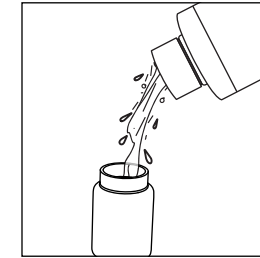



③ Pull the trigger gently to release the mist.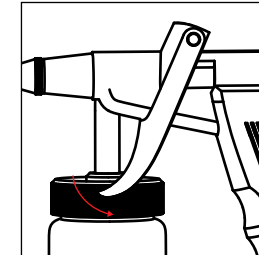

Note: If the trigger feels stuck, rotate the adjustment ring at the rear of the handle counterclockwise 2–3 turns.

② Reattach the bottle to the handle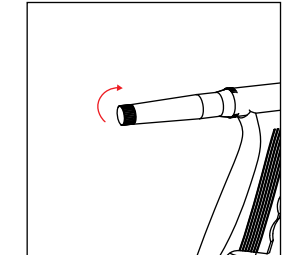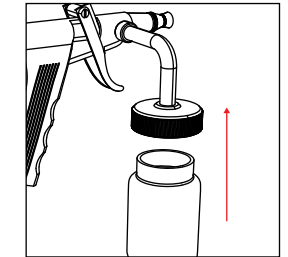

2.3 How to Use

Follow the steps below for an optimal spray application session.


- ① Fill the spray bottle with the desired solution.
- ② Hold the spray handle approximately 15–20 cm (6–8 inches) away from the face.
- ③ Apply the mist using slow, circular motions across the skin.

Recommended Usage Frequency:

Normal Skin: 2-3 times per week

Dry Skin: Once daily

Oily / Combination Skin: 2-3 times per week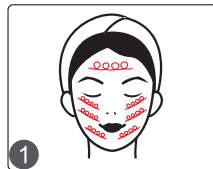

2.4 Cleaning the Spray Handle

- Fill approximately one-third of the bottle with clean water.
- Turn on the device and spray until the bottle is empty to flush the internal tubing.


2.5 Precautions

Avoid using highly concentrated liquids, as they may clog the spray nozzle.

If the spray nozzle becomes clogged, gently clean the opening with a fine needle.

3. Suction

3.1 From the main menu, select the "Suction" icon to enter the function interface.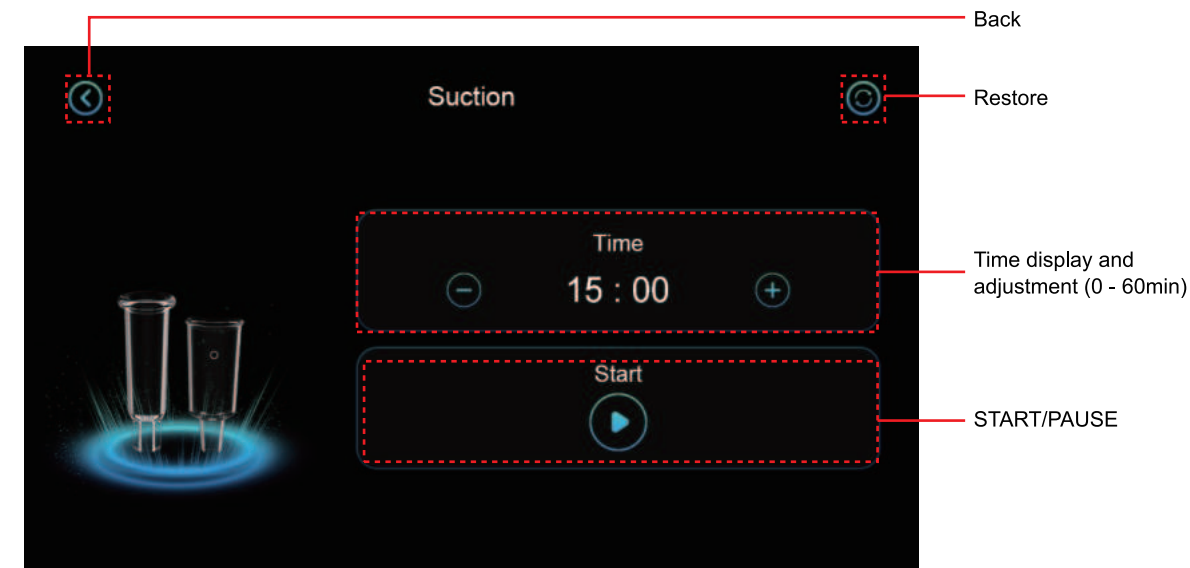


3.2 How to Choose a Cup


	Lymphatic drainage for normal skin.
	Lymphatic drainage suitable for people with sensitive skin (The cup features a small hole; pressing it with your thumb creates suction during use, and releasing it stops the suction.)

3.2 How to Install Suction


① Connect one end of the hose to the cup.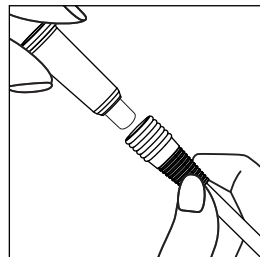

② Connect the other end of the hose to the machine.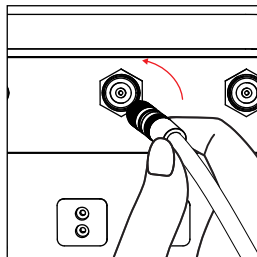

Note: After unscrewing the air intake port counterclockwise, you can see a filter cotton inside the instrument. This filter cotton should be replaced once a week.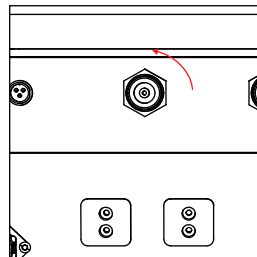

3.3 How to Use

Follow these steps to get the most out of your suction cup.

- ① Set the treatment course: Please adjust the device settings according to your skin's specific needs.
- ② Use the suction adjustment knob to adjust the suction level according to your comfort level.
- ③ Apply facial oil: Apply a generous amount of facial oil to the treatment area to ensure the device glides smoothly.
- ④ Glide from behind the ear down the side of the neck to the collarbone, then push horizontally toward the acromion and drain to the underarm.


Glide along the lower facial contour to behind the ear, then slide down the side of the neck to the collarbone, and push horizontally toward the acromion down to the underarm.

Start from the chin, glide across the mouth corners, nasal wings and lower eyelids up to the eyebrows and Yintang. Slide along the hairline to the sides of the ears, then move from behind the ears down the lateral neck to the collarbone, followed by horizontal pushing toward the acromion and draining down to the underarms.

⑤ Recommended frequency of use:


Oily skin: Twice a week

Dry, normal, or combination skin: 1–2 times a week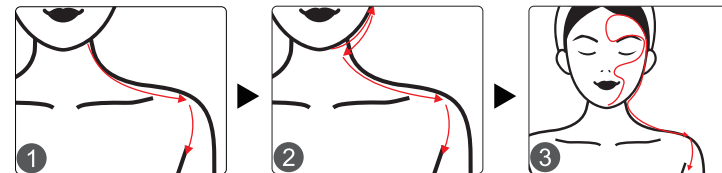


3.4 Precautions

When using this product, it is not recommended to hold it in one spot for too long, as this may cause skin irritation.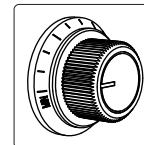

4. Import Hub

4.1 From the main menu, select the "Import Hub" icon to enter the function interface.

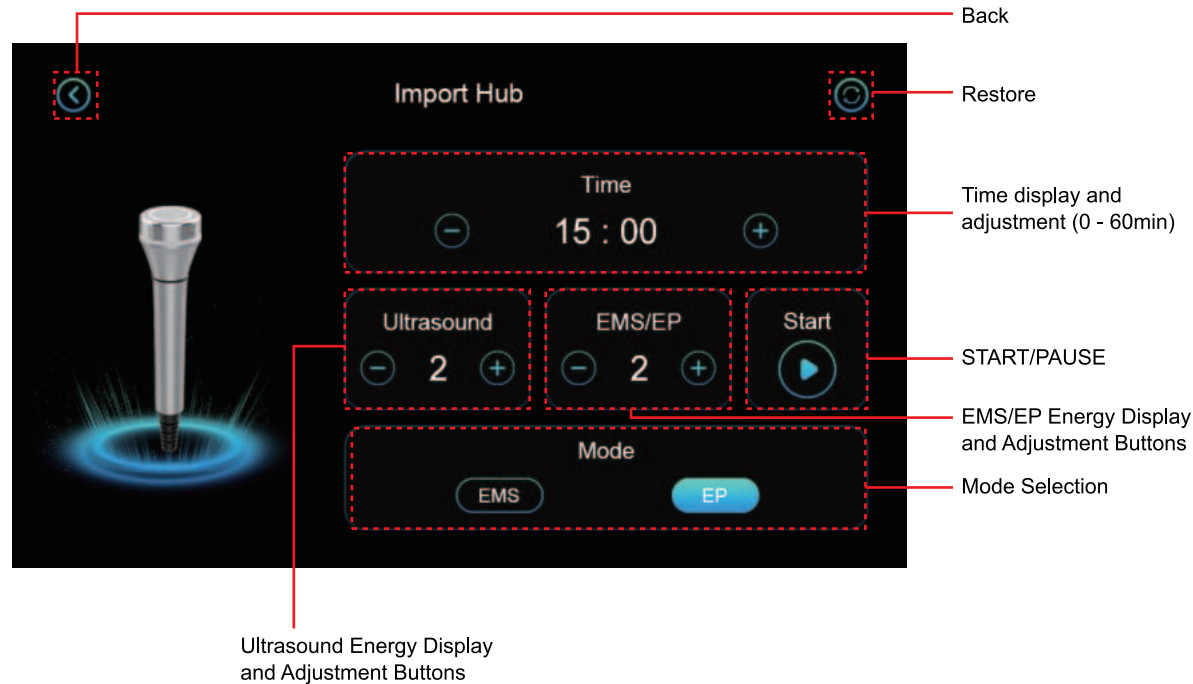4.2 How to Use

Follow these steps to get the most out of the Import Hub feature.

- ① **Configure Your Session:** Please configure the device parameters based on your skin's specific needs.
- ② **Apply Coupling Medium:** Use a sufficient amount of conductive gel or serum on the treatment area. This ensures smooth gliding of the device and promotes effective product infusion.
- ③ **Recommended Method:** Gently move the device handle across your face using continuous circular motions or upward lifting strokes, as demonstrated in the diagrams. Ensure consistent contact with the skin.

④ **Recommended Usage Frequency:**

Dry Skin: 2-3 times per week

Oily Skin: 2 times per week

Normal / Combination Skin: 1-2 times per week

4.3 Precautions

Keep the Device Moving: Do not hold the device stationary on one spot for an extended period, as this may cause skin discomfort.

Start Low, Go Slow: Always begin at the lowest intensity level. Gradually increase the level only after your skin has acclimated to the sensation.

8. Troubleshooting Guide

This section provides solutions to common operational issues. Please follow the steps below before contacting support.

1. No Suction From the Hydro Dermabrasion Handle

- ① Check that the filter and Waste Water bottle are installed correctly.
- ② Empty the Waste Water bottle if it is full.
- ③ Check if the Hydro handle connection cable is loose, not plugged in properly, or connected in the wrong position.

2. No water flow from the Hydro handle?

- ① The blue tube is not fully inserted. Push it firmly into place.
- ② There is insufficient water in Bottle A/B/C.
- ③ The transparent tube of the peristaltic pump is damaged. Please replace it with a new one.
- ④ Install the sealing head before first use, and wait about 40 seconds for normal water flow.
- ⑤ Check the operation interface and make sure the water flow switch is turned on.

3. Spray Function Not Dispensing Mist

- ① The liquid may be too concentrated and could be clogging the nozzle.

- ② The rear adjustment ring may be tightened too far.
- ③ Clean the nozzle gently using a fine needle if blocked.

4. Why is there no suction?

- ① Check whether the hose is connected to the suction port.
- ② Check whether the hose is securely connected to the handle.
- ③ Check whether the hose is damaged; if so, replace it.
- ④ Check whether the suction dial is set to medium or higher.

5. Need Additional Assistance?

If the issue continues after following the steps above, please contact Customer Support for further assistance.

9. FAQ

Q1: Why does my face turn red after use? Is this normal?

A: Yes, this is a common and temporary reaction. The device works on a vacuum principle. When the handle makes contact, it causes a brief increase in blood flow (capillary congestion) in the area, which may lead to temporary redness. This typically fades within a few hours. We recommend applying a moisturizer immediately after your session to soothe and hydrate the skin.

Q2: How soon can I expect to see results?

A: Many users notice smoother- and cleaner-looking skin after the first use. With regular use, the device may help improve the appearance of dullness, clogged pores, and uneven skin texture.

Q3: Will my skin become thinner or more sensitive with regular use?

A: No. The process is designed to cleanse pores by removing impurities like blackheads and whiteheads. It does not thin the skin's protective barrier. With consistent and correct use, the goal is to promote smoother, clearer-looking skin and should not induce sensitivit

Q4: Will my skin become thinner or more sensitive with regular use?


A: No. The process is designed to cleanse pores by removing impurities like blackheads and whiteheads. It does not thin the skin's protective barrier. With consistent and correct use, the goal is to promote smoother, clearer-looking skin and should not induce sensitivit.

10. Warranty & Support

A one-year warranty will be provided starting from the date of your purchase.

If you have any questions during use or any problems with the product, you can contact our customer service. We can also provide you with more professional skincare guidance.

You can scan the QR code below to watch the video on the installation and operation of the instrument.

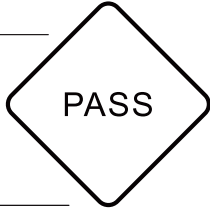Product Name:	Facial Skincare Device	
Product Model:	FC-23K5	
Production Date:		
Quality Inspection:		

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