


User Manual

MS-33J1

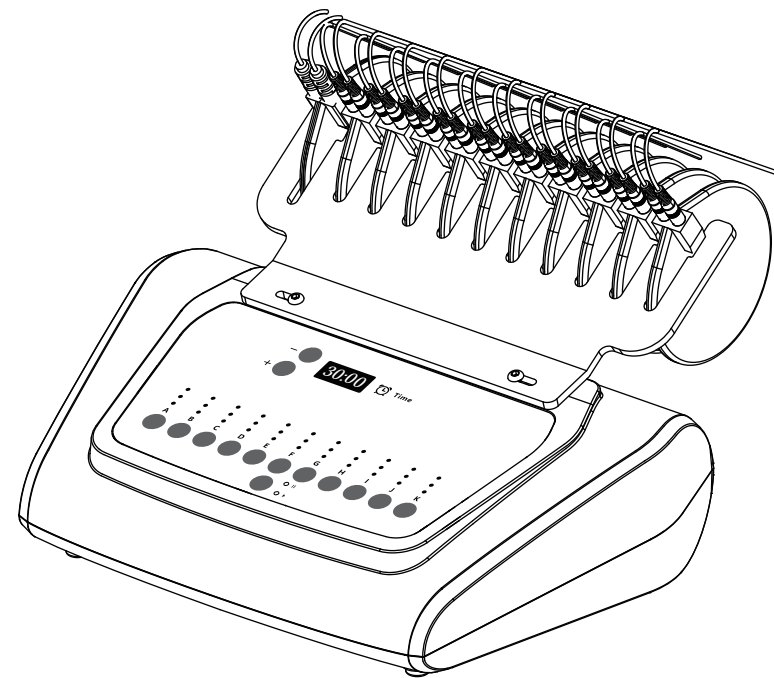

Content

| | |
|---------------------------------|---|
| 1.Safety Precautions | 1 |
| 2.Contraindications | 2 |
| 3.Overview | 2 |
| 4.Interface Function | 3 |
| 5.Technical Specification | 4 |
| 6.Use Precautions | 5 |
| 7.Maintenance & Cleaning | 5 |
| 8.FAQ | 6 |
| 9.Ask & Answer | 7 |
| 10.Operation Guide | 8 |
| 11.Warranty & Support | 9 |

1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 7.Ensure that the instrument is applied on moist skin and avoid the operation on dry skin.
- 8.Power off the instrument when not in use to ensure its safety.
- 9.If you are currently taking weight-loss drugs, please discontinue their use 1 - 2 months before using this instrument. Failure to do so may result in an extended course of treatment.
- 10.To maintain hygiene and extend the instrument's service life, sterilize it with saline.

2.Contraindications


Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details below:

- 1.Pregnant or lactating woman.
- 2.Individuals with hypertension, heart disease, or a pacemaker installed.
- 3.Individuals with unhealed wounds or those in the recovery period from surgery.
- 4.Individuals suffering from epilepsy, severe diabetes, and hyperthyroidism.
- 5.Individuals suffering from malignant tumors, hemophilia, and hemorrhage.
- 6.Individuals with skin disease or infectious disease.
- 7.Individuals have had a cesarean section within six months, a miscarriage within three months, or a natural birth within two months.
- 8.For your safety, refrain from using if you've had plastic surgery or have implants/metal objects near the treatment area within 3 months.
- 9.Individuals who are sensitive to the micro-current.



3.Overview

With the most popular and safest technology, this beauty instrument ensures you achieve your desired body figure. It provides the perfect solution for people seeking improvements in body figures without injections, drugs, or side effects. For external use with immediate effects.

4.Interface Function

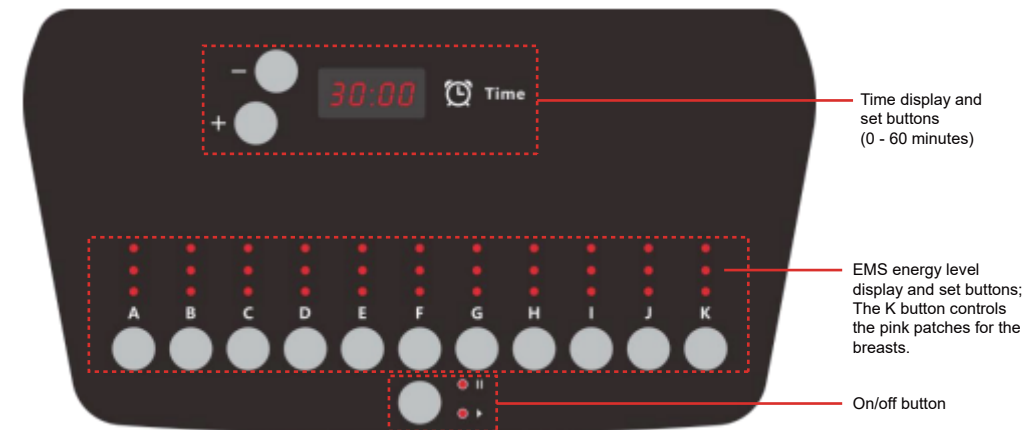

5. Technical Specification
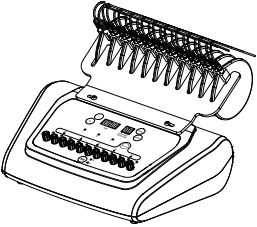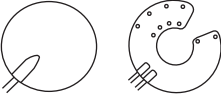
| | | |
|---|---|---------------------|
|  | Power supply Input: AC110V-220V 50Hz-60Hz | |
| | Power: 10W | Input Power: 10W |
| | Dimensions: 12.2 × 13 × 7.5 inches | |
| | Net Weight: 2.1KG | Gross Weight: 3.1KG |
|  | Frequency: 6Hz | |
| | Power: 10W | |

6. Use Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the instrument.
4. Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.
5. Stop the intake of weight drugs 1 - 2 months before the treatment, or extend the treatment course if you want to take the treatment immediately.
6. Do not use the instrument on an empty abdomen, and take the treatment 1 hour later after the meal.
7. Ensure full skin contact with the treatment head throughout the session to avoid uneven heat distribution.
8. Before beginning the microcurrent treatment, kindly inform the customer about the sensations associated with microcurrent to prevent any unnecessary concerns or discomfort.

7. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. Clean and sanitize the handle with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.

3. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.
4. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8. FAQ

1. Can EMS treatment aid in weight loss?

A: Yes. Unlike weight loss, it helps you achieve better fitness results. EMS, short for Electronic Muscle Stimulation, is also referred to as muscle electrical stimulation technology or muscle movement technology. Typically, human muscles initiate active movements in response to signals from the brain. However, EMS directly transmits electrical current stimulation to the muscles, prompting passive muscle contractions. This approach aims to achieve optimal fitness results.

2. Does the EMS treatment hurt?

A: No. EMS treatment employs micro-currents to stimulate the body, delivering effects similar to therapies such as massage and acupuncture. You may experience a mild current sensation during the treatment, but the entire process is comfortable and painless.

3. What are the effects of the instrument?

A: This treatment offers a range of positive effects, including body shaping, localized body contouring, and metabolism enhancement. It also enhances the function of internal organs and addresses sub-health concerns.


4. Will I regain weight if I stop the treatment?

A: Rarely. EMS treatment targets body adiposity rather than water retention, and since adiposity forms gradually, rebound is typically not a concern.

5. Do I need to diet during the EMS treatment?

A: During the EMS treatment, the micro-current stimulates the body's metabolism. However, excessive consumption of spicy, fried, and greasy foods can have a negative impact on metabolism. Therefore, maintaining a healthy diet and engaging in moderate exercise will enhance the treatment's effectiveness and promote overall well-being.

9. Ask & Answer

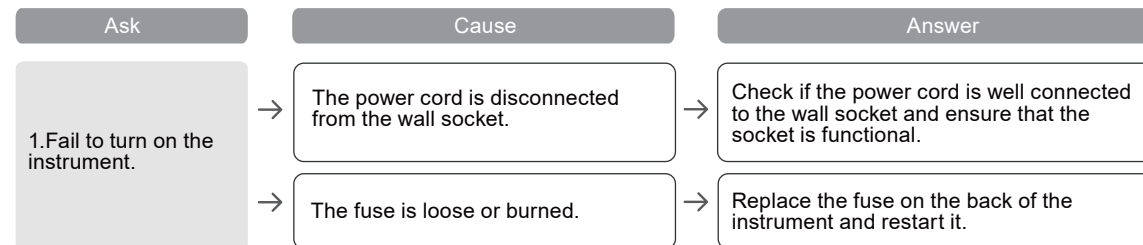


| Ask | Cause | Answer |
|---|--|--|
| 2.The instrument is not working and can't be restarted. | → The fuse may have burned out due to unstable voltage or prolonged use. | → Please replace the fuse and restart the instrument 5 minutes later. |
| 3.No current during operation. | → The cord of the EMS patch is not securely connected to the main unit. | → Please check if the cord is loose from the main unit. |
| | → You may have activated the wrong EMS patches. | → Please check the screen and ensure the required EMS patches are activated. |

10.Operation Guide


Arm

Step 1: (EMS patch 20 minutes)

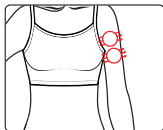


Abdomen

Step 1: (EMS patch 20 minutes)

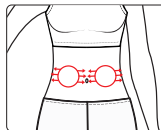


Leg

Step 1: (EMS patch 20 minutes)

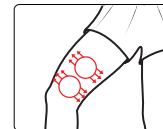

Breasts

Step 1: (EMS patch 20 minutes)

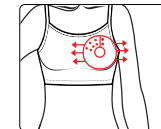

11.Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name: EMS Body Sculptor

Product Model: MS-33J1

Production Date: _____

Quality Inspection: _____

PASS