


User Manual

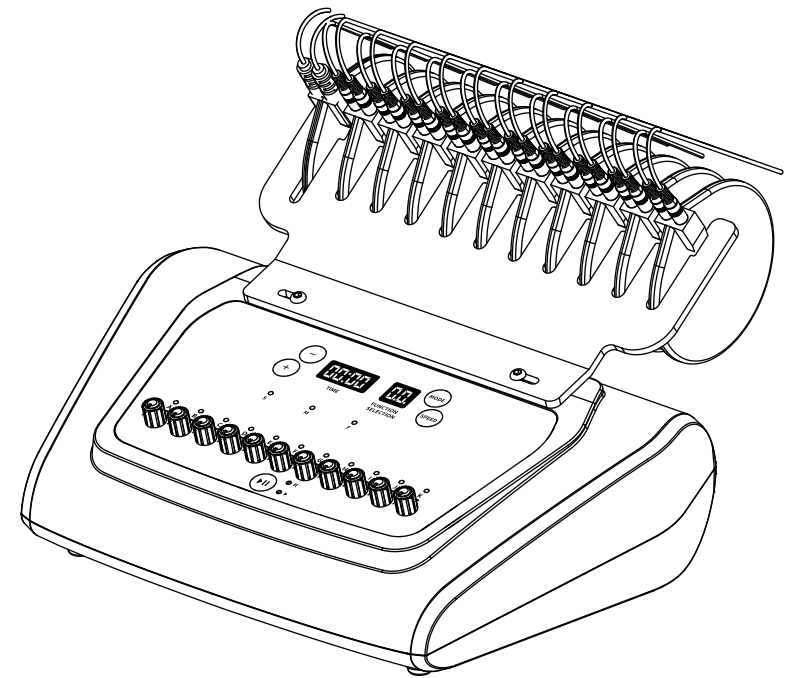

Content

1.Safety Precautions	1
2.Contraindications	1
3.Overview	2
4.Control Panel Introduction.....	2
5.Technical Specification	4
6.Use Precautions	4
7.Maintenance & Cleaning.....	5
8.FAQ	6
9.Ask & Answer	7
10.Operation Guide	8
11.Warranty & Support	9

1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 7.Power off the instrument when not in use to ensure its safety.
- 8.If you are currently taking anti-adiposity drugs, please discontinue their use for 1 - 2 months before using this instrument. Failure to do so may result in an extended course of treatment.
- 9.To maintain hygiene and extend the device's service life, sterilize it with saline.

2.Contraindications


Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details below:

1. Pregnancy or lactation.
2. Hypertension, heart disease, or with a pacemaker installed.
3. Unhealed wounds or during the recovery phase of surgery.
4. Individuals who are sensitive to the micro-current.
5. Epilepsy, severe diabetes, and hyperthyroidism.
6. Malignant tumors, hemophilia, and hemorrhage.
7. Skin disease or infectious disease.
8. Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
9. Had plastic surgery within 3 months or with implants or metal objects in the treatment area.


3. Overview

With the most popular and safest technology for skin tightening and body shaping, this EMS beauty instrument guarantees a radiant transformation for the user. It provides the perfect solution for people seeking improvements in body figures without injections, drugs, or side effects. Its external application yields immediate results.

4. Control Panel Introduction

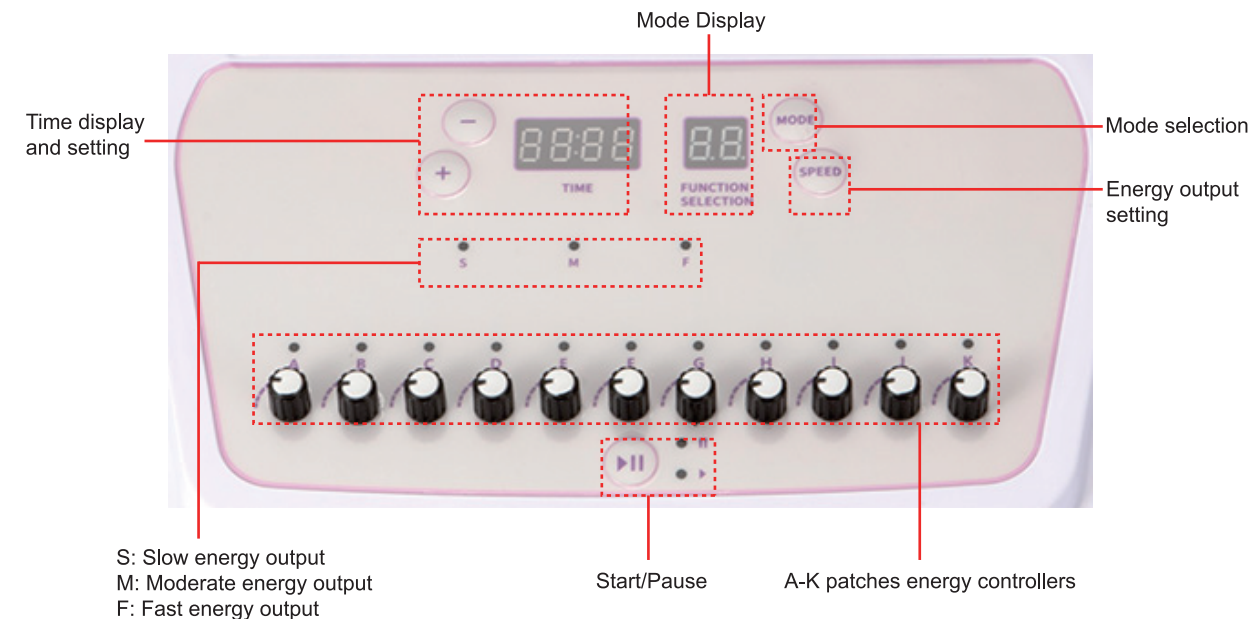

5. Technical Specification

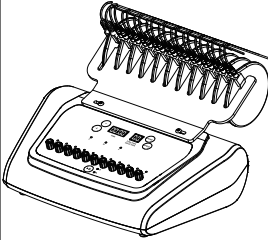	Power supply Input: AC110V-240V 50Hz-60Hz	
	Net Weight: 2.7KG	Gross Weight: 3.7KG
	Input Power: 16.5 W	
	Dimensions: 12.2 × 13 × 7.5 inches	
	Power: 16.5 W	Frequency: 250 Hz

6. Use Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the instrument.

4. Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.
5. Refrain from using the instrument on an empty stomach, and initiate the treatment one hour after a meal.
6. Before treatment, inform the recipient about microcurrent sensations to prevent concerns or discomfort.

7. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. Clean and sanitize the device with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.
3. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.
4. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8.FAQ

1.Any pain during the EMS treatment?

A: No, there isn't. The EMS treatment utilizes microcurrent to stimulate the body, providing a comfortable and enjoyable experience similar to traditional massage techniques.

2.What are the features of this instrument?

A: When applied to the body, this treatment effectively sculpts localized areas, boosts metabolism, promotes whole-body detoxification, and enhances internal organ function, all while addressing sub-health concerns.

3.Do I need to keep on a healthy diet during the treatment period?

A: It would be better if you could keep on a healthy diet, since after the Ultrasonic treatment, the metabolism will be accelerated, and eating spicy, greasy, and fried food will affect the metabolism. Moderate exercise and sweating can help the body excrete metabolic waste, and the weight loss effect will be more obvious.


9.Ask & Answer

Ask		Cause		Answer
1.Fail to turn on the instrument.	→	The power cord is disconnected from the outlet.	→	Check if the power cord is securely connected to the outlet and ensure that the outlet is functional.
2.The control panel shows an error message.	→	Data error.	→	1. Turn off the instrument, disconnect the power cord, and wait for 1 minute to restart the instrument. 2. If the above method doesn't resolve the problem, please contact your dealer.
3.Without the sensation of electric current during the treatment.	→	Check the connection port where the cable connects the host and the patch. The cable may be loose.	→	Securely connect the patch cable to the main unit.
		You may have applied the wrong patches.		Verify if the EMS patches you applied match those you selected on the control panel.
		The application of gel to the patch has been omitted.		Apply gel to the patch.

10.Operation Guide


Arm

Step 1: (EMS patch 20 minutes)

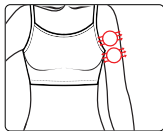


Belly

Step 1: (EMS patch 20 minutes)

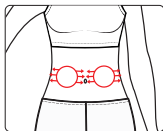


Leg

Step 1: (EMS patch 20 minutes)

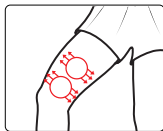

Chest

Step 1: (EMS patch 20 minutes)

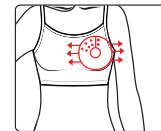

11.Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name: Versatile Beauty Enhancer

Product Model: MS-33J1U

Production Date: _____

Quality Inspection: _____

PASS