



MS-33P2

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us? Phone:(+86)

189 -48755929 Tel: 0755-86368500

Email:sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

Chapter 1: Safety Warnings

I. Electricity & Machinery Safety

- 1.The machine works with 100V~240V voltage input.
2. Please keep the voltage at stable condition.

II. Contraindication Safety

1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

III. Packing Including:

Dermabrasion main machine x1pc
PVC vacuum hose x1 pc
Red PVC vacuum hose x1pc
Diamond wands x3 pcs
Diamond tips x9 pcs
Cotton filter x1 pack
Glass tube x3 pcs
Spray Gun x1 pc
Skin Scrubber x1pc
Power Adapter x1 pc

Chapter 2: Before using

- I. Before use, clean face with lotion and finish massage face to accelerate blood circulation.

II. How to install machine.


- A. Dermabrasion accessories connection
 1. Install filter and rubber

Diamond dermabrasion connection



2. Connect hose with machine and glass tube.


Blackhead vacuum removal connection

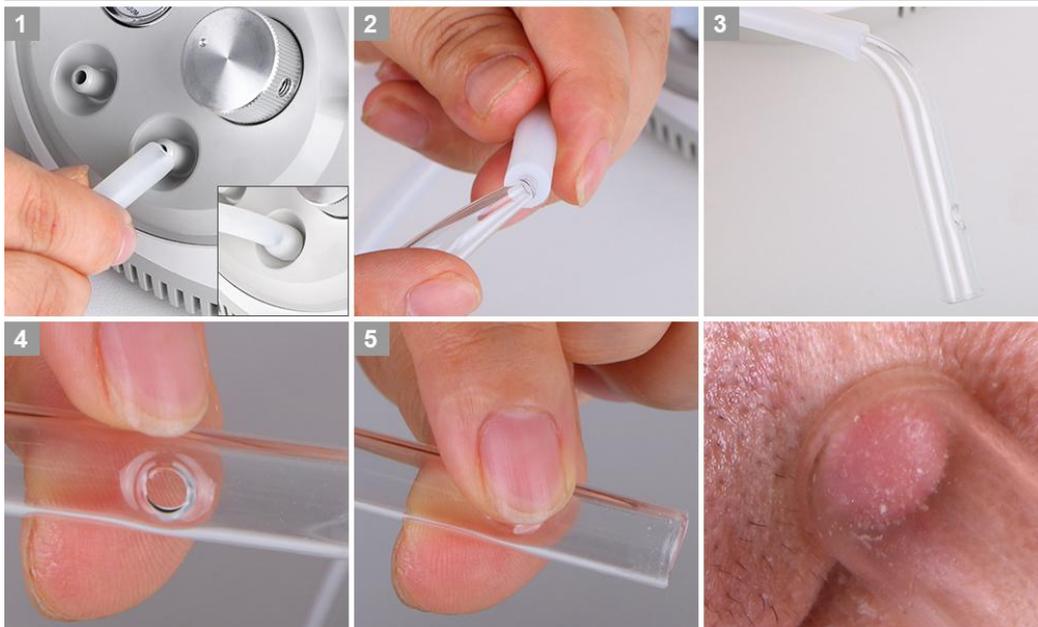

3. Connect Skin Scrubber



4. Spray gun connection

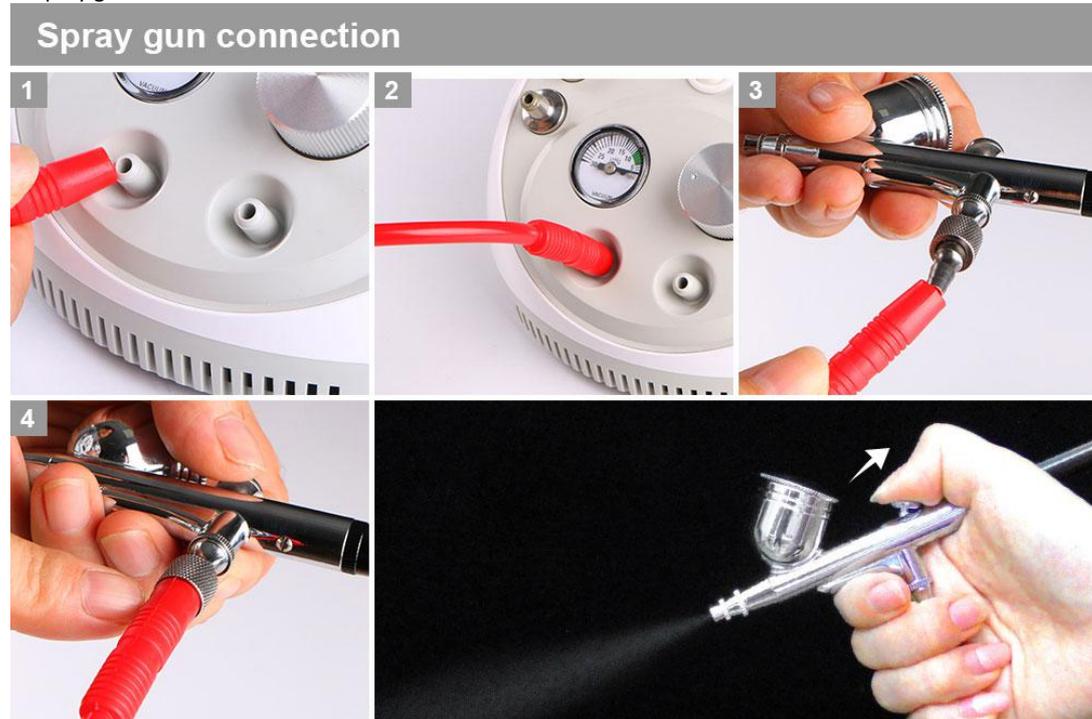

B. Connect power supply cord with the machine



C. Press the power bottom to start/finish treatment.



Chapter 3: Operation guide

I. Adjust the silver knob to set the suction of vacuum.
Left: Min Right:Max.



II. Choose the suitable microdermabrasion tips according to different treatment area and start the treatment, massage along the skin texture.

There are 9 different microdermabrasion tips for you choose:

1. Tips D240: Eyes and both sides of nose, for wrinkles, all blackhead and acne.
2. Tips D200: For Scars, fine lines and imperfections.

III. Operation guide

Operating Guide



STEP 1

Cleaning the facial skin and drying it.



STEP 2

Starting from the lower jaw, moving the rejuvenation from bottom to top.



STEP 3

Placing the rejuvenation instrument at the cheeks and moving from the inside to the outside.



STEP 4

Placing the rejuvenation instrument in the forehead area, moving from the middle to both sides.



STEP 5

Placing the rejuvenation instrument in the T zone, moving from top to bottom.



STEP 6

Using Toner to moisturize the skin, skin care products absorbing better.

Chapter 4: After Using

1. Do a hydrating mask after the treatment.
2. 10-15minutes/course, one course/3 days, 10 course/