



MS-5651

Instruction

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty(Except the consumable parts) . Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

1. Introduction to Manual

1.1 Purpose

This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user



Warning: Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.

For more information on training available please contact your local representative.

1.2 Conventions



Note: Notes designate information of special interest.



Caution: Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty



Warning: Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in Bold text

2. System Overview

2.1 Description

Unoisetion cavitation 2.0 technology

Many people has the experience of unbearable ring or noise sound in ears,it is the normal cavitation,our professional research teamhas overcome this problem and the buzzing noise has reduced yet after years of study,It is called cavitation 2.0 unoisetion technology

3D SMART RF --More comfortable ,more Intelligent

3D SMART RF, utilized a special technology of touch sensor , can commend inner RF procedure to start and stop working during treatment, normal RF when in working condition will transmit energy out whenever it touches skin or not, but 3D SMART RF won't do this because the touch sensor can detect the condition,the 3D SMART RF only work until the RF probe touch the skin and the light will be on, then the energy of 3D SMART RF will emit from low to high adjusted automatically upon skin's temperature

LED Photon

Red Light (650 - 730):

Stimulate the production of collagen. Collagen is the essential protein used to repair damaged tissue and to replace old tissue. Best for fine lines,reduce pore size, swollen after surgery.

Miracle BIO Wave

Micro BIO electrical beauty technology originates in Japan,with the purpose of substituting the traditional surgical face-lifting & wrinkle dispelling technology.This is popularly used in world top-class beauty salons and has shown great efficiency which help many customers solve diversified problems thoroughly.We have cooperated with Japanese companies and have developed this BIO devise, which regarded as the sixth generation of BIO series,is standing at a leading position in BIO tech.Therefore, it is the most valuable and effective BIO device in the world market.

Dual wavelength Lipo laser

One innovative invention, Wave 650nm accelerates blood circulation, and wave 980nm goes deep into the skin tissues, The two wavelength coordinate coherently to strengthen the fat burning effects, moreover, 980nm have very good effect on skin rejuvenation. if there are stubborn fats in your belly,arms, thighs, back or face, you can easily get rid of them with the help of This lipo laser slimming equipment. this is the new and natural fat reducing system that will help you to lose weight in short period without surgery. With professional technology specilized in beaty industry, This lipo laser can reduce weight up to 2-6 cm in one treatment. It is non-invasive and painless .You can get instant results!

Package Including

1 x Main Machine

8 x Big paddle with 17 laser(650NM 10 laser, 980nm 7 laser)//Each (Total 136 laser, 650NM 80 laser, 980nm 56 laser)

4 x Small paddle with 3 laser (650NM 2 laser, 980nm 1 laser)//each (Total 12 laser , 650NM 8

laser, 980nm 4 laser)

1 x holder for paddles

1 x Unioisetion RF Cavitation 2.0 Probe

1 x Octupole 3D SMART RF probe for body

1 x Quadropole 3D SMART RF Probe for arm/face/small area of body

1 x Bipolar 3D SMART RF for face and eye' area

1 x BIO Probe


1 x Holder for heads

1 x Power Supply Cord (Both 110v and 220v voltage (frequency: 50/60Hz) version are available.

Also we stock the USA, UK, AU, EU plug to suit for the country all over the world.)

2.2 Components and Features

2.2.1 Control Unit & Handpiece

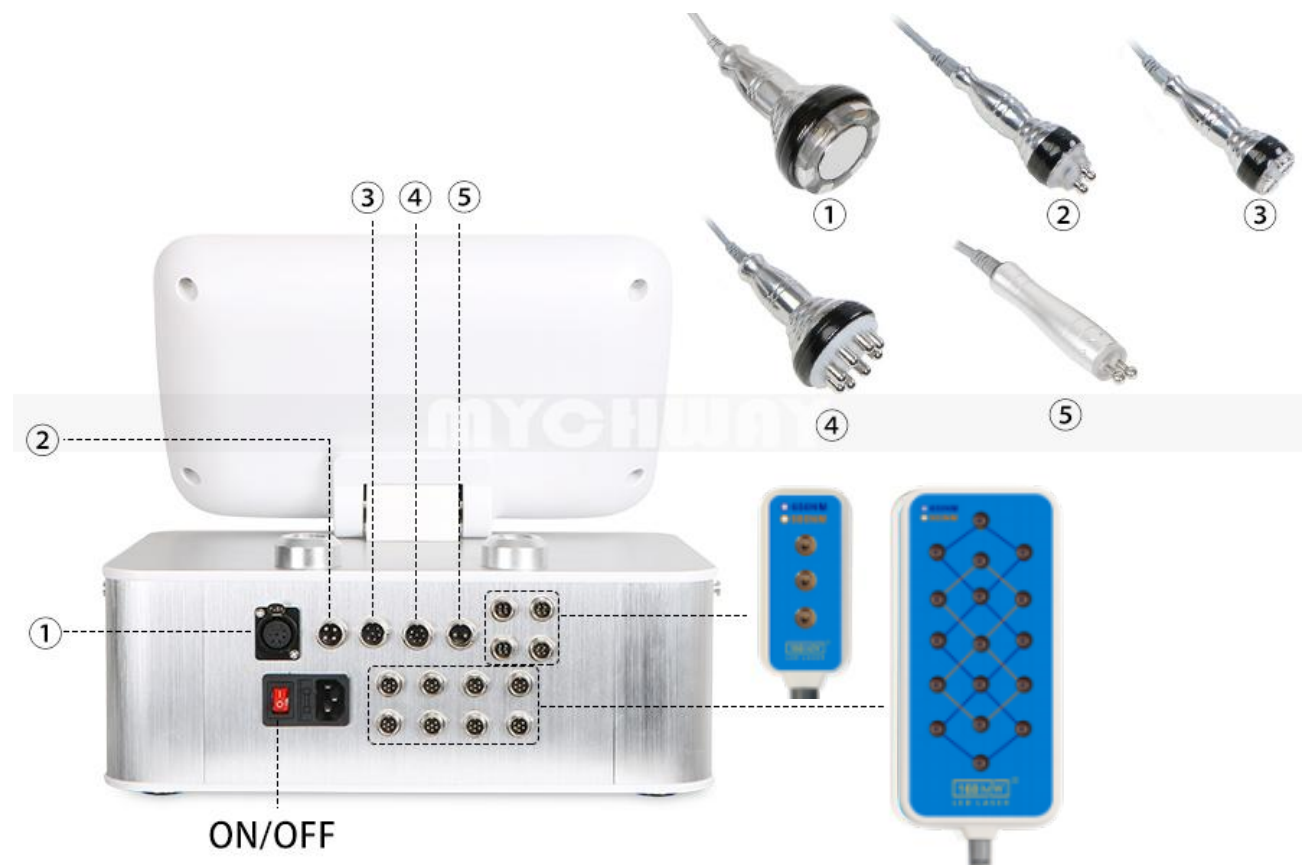



Unioisetion cavitation 2.0 rf head



BIO head



Face quadrupole rf head

Lipo laser pads



Bracket

Bandage

3. Treatment Guidelines.

3.1 Before operation

- * Please connect all parts well
- * Please connect power cord with power supply
- * Turn on the machine
- * Then will turn to this page





And Home page




3.2 Eye RF

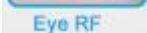

3.2.1 Press  button then login Eye rf operation display

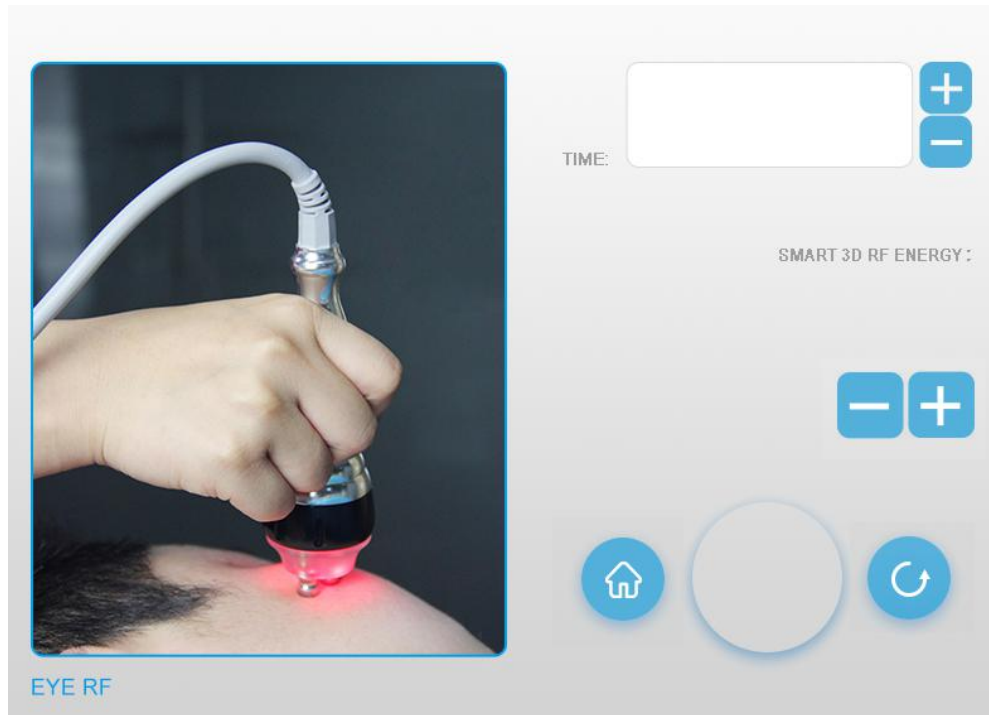

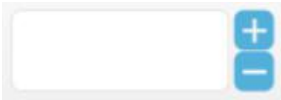
3.2.2 Please click  to set working time.


3. 2.3 Please click  to increase or decrease the ENERGY

3.2.4 Please click  to start or pause function.

3.2.5 Please click  to back to the home page

3.3 Face RF

3.3.1 Press  button then login face rf operation display

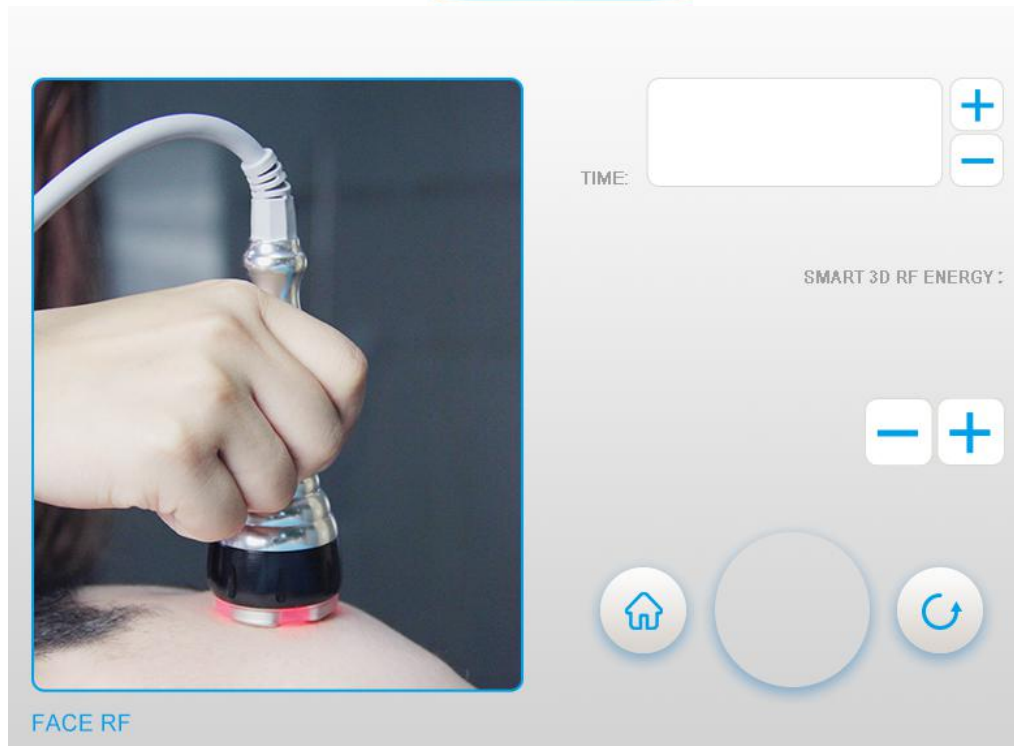


3.3.2 Please click  to set working time.


3.3.3 Please click  to increase or decrease the ENERGY

3.3.4 Please click  to start or pause function.

3.3.5 Please click  to back to the home page

3.4 Body RF

3.4.1 Press  button then login body rf operation display

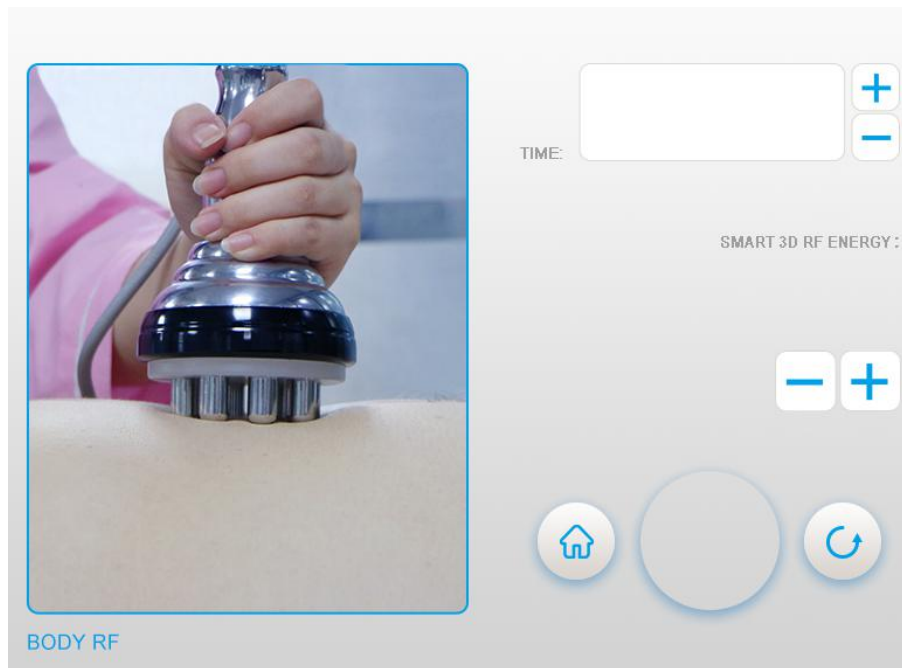


3.4.2 Please click  to set working time.


3. 4.3 Please click  to increase or decrease the ENERGY

3.4.4 Please click  to start or pause function.


3.4.5 Please click  to back to the home page


3.5 40K& RF

3.5.1 Press  button then login cavitation rf operation display

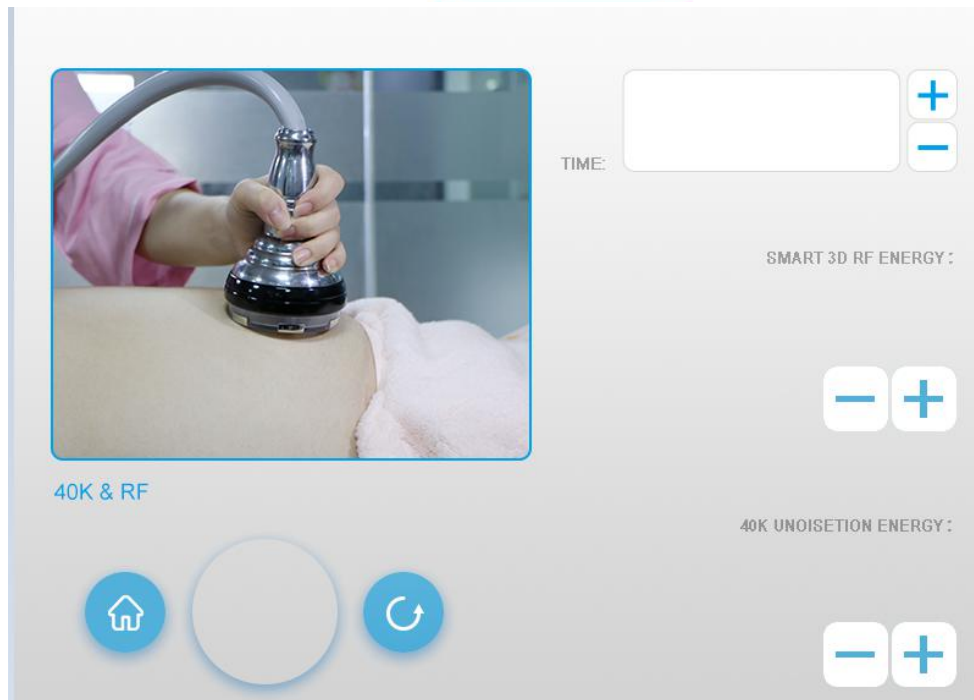

3.5.2 Please click  to set working time.

3. 5.3 Please click  to increase or decrease the rf ENERGY


3.5.4 Please click  to increase or decrease the cavitation ENERGY

3.5.5 Please click  to start or pause function.


3.5.6 Please click  to back to the home page

3.6 40K& RF

3.6.1 Press  button then login BIO operation display

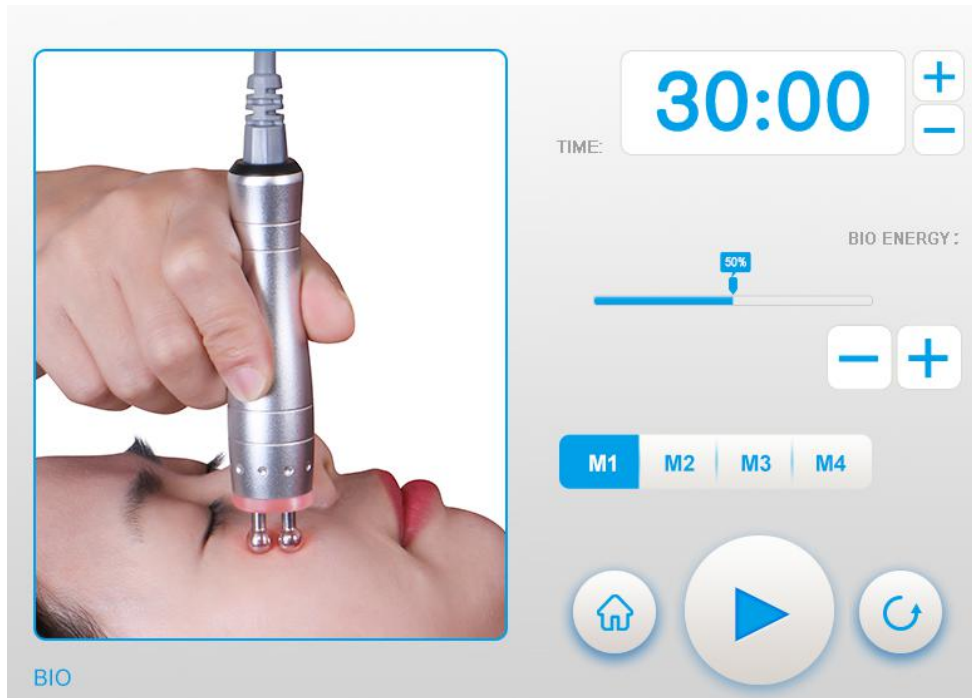

3.6.2 Please click  to set working time.


3. 6.3 Please click  to increase or decrease the BIO ENERGY


3.6.4 Please click  to choose working mode.

3.6.5 Please click  to start or pause function.

3.6.6 Please click  to back to the home page


3.7 Lipo laser

3.7.1 Press  button then login lipo laser operation display

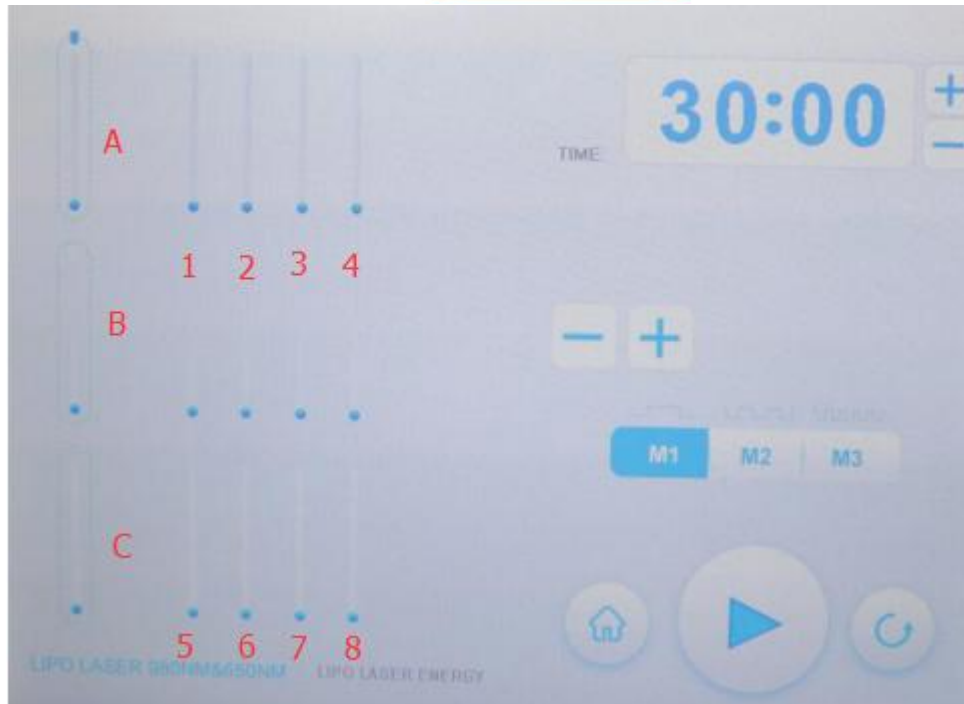

3.7.2 Please click  to set working time.

3.7.3 Click the left part to choose the energy of the lipo laser, A, B for the 8 big lipo laser lights, and C for 2 small lights, click 1.2.3.4.5.6.7.8 could choose each pad.

Click  to add or decrease energy.

3.5.4 Please click  to choose working mode.

3.5.5 Please click  to start or pause function.

3.5.6 Please click  to back to the home page

4. Treatment Guidelines.

1) Abdomen, thigh, buttock, arms slimming

Please clean the skin and then put the ultrasound gel on the treatment area, then turn on the

machine, set the time and energy. Please set the energy from low to high to get the best comfortable temperature, then follow this steps:

- 1, Please use the Sextupole rf head 5 mins
- 2, Please use the unioisetion 40k cavitation head 15-30min
- 3, Please use the Sextupole RF Vacuum head 10-15min
- 4, Use Lipo lasr pads around 25~30 min.

Work time: 2-3 times/ week

2) Face, eyes around,neck skin lifting:

Please clean the skin and then put the rf gel on the treatment area, then turn on the machine , set the time and energy, pls set the energy from low to high to get the best comfortable temperature, then follow this steps:

- 1,use face radio frequency head for face for 25-30 min
- 2,use eye radio frequency head for eye around for 25-30min
- 3, use bio head for face and eye for 10-15minutes


Work time: 2-3 times/ week

BUTTOCKS

ILLUSTRATIN of OPERATION

FACE

Treated time: 30 minutes

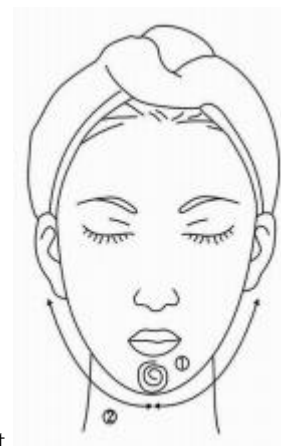

1. Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles.

4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes



1. From down to upper, pushing to the groin to dredge the lymph.

2. From down to upper, circling push by anticlockwise can help decomposing



fatness.



3. Also can push by come-and-go to decompose fatness.

4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.



3. Along the muscle of arm, pull up to lift and tight the muscle.

BACK

Treated time: 20 to 30 minutes



1. First by come-and-go, pushing the bladder nerve 2 to 3 times.




2. Circling by anticlockwise to stimulate the underarm lymph node.

3. On the back, by anticlockwise circling, can help to decompose the



fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help

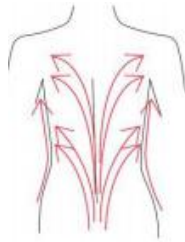

improving the back curve.

ARM

Treated time: 30 minutes

BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.
2. Circling to stimulate the lymph node.
3. By anticlockwise or come-and-go pushing, decomposing the fatness.
4. From the elbow to the outer, doing tightening gesture.



Facade

1. From the elbow to the outer, doing tightening gesture.


Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.




2. From small to large, by anticlockwise direction, circling push down.

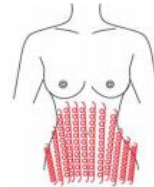

3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

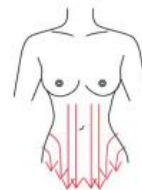

Attention:

1. Pregnant women or women during in menses.
2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.

4. Technical Specification

PowerSupplyInput: AC110V-220V 50HZ/60HZ

Power: 85W

Input Power: 85W

Dimensions: 12.2 * 20.8 * 14.5inches

NetWeight: 11.8kg

GrossWeight: 12.9kg

40K & RF

RF Frequency: 1MHz

Frequency: 40KHZ

Power: 85W

Body RF

Frequency: 1MHZ

Power: 75W

Face RF

Frequency: 1MHZ

Power: 65W

Eye RF

Frequency: 1MHZ

Power: 40W

Lipo Laser

Power: 55W

BIO

Frequency: 90KHZ

Power: 15W