


User Manual

MS-65F8

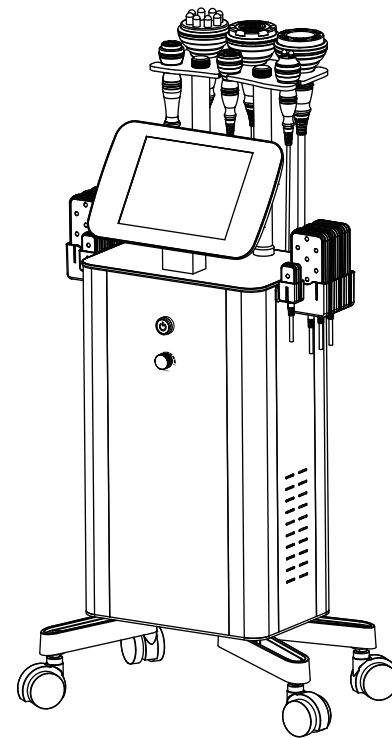

Content

1.Safety Precautions	1
2.Contraindications	2
3.Overview	2
4.Control Panel Introduction	3
5.Technical Specification	8
6.Use Precautions	10
7.Maintenance & Cleaning	11
8.FAQ	11
9.Ask & Answer	14
10.Operation Guide	16
11.Warranty & Support	23

1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.As a high-precision electronic device, this instrument should be stored properly. Avoid humid conditions, direct sunlight, and proximity to flammable or explosive materials to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the device in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the device on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pace-makers, etc.
- 6.Patients should consult their doctor before using this device.
- 7.Ensure that the instrument is applied on moist skin and avoid the operation on dry skin.
- 8.Power off the instrument when not in use to ensure its safety.
- 9.To maintain hygiene and extend the device's service life, sterilize it with saline.

2. Contraindications


Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details below:


1. Pregnancy or lactation.
2. Sensitivity to micro-currents.
3. Hypertension, heart disease, or with a pacemaker installed.
4. Unhealed wounds or ongoing recovery from recent surgery.
5. Epilepsy, severe diabetes, and hyperthyroidism.
6. Malignant tumors, hemophilia, and hemorrhage.
7. Skin disease or infectious disease.
8. Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
9. Recent plastic surgery (within three months), or presence of implants or metal objects in the treatment area.



3. Overview

With the most popular and safest technology for anti-aging, firming skin, and body shaping, this 7-in-1 beauty device ensures you achieve your desired body figure and radiant complexion. It provides an ideal solution for those seeking skin and body improvements without injections, drugs, or side effects. This non-invasive device delivers visible enhancements with each use.

4. Control Panel Introduction

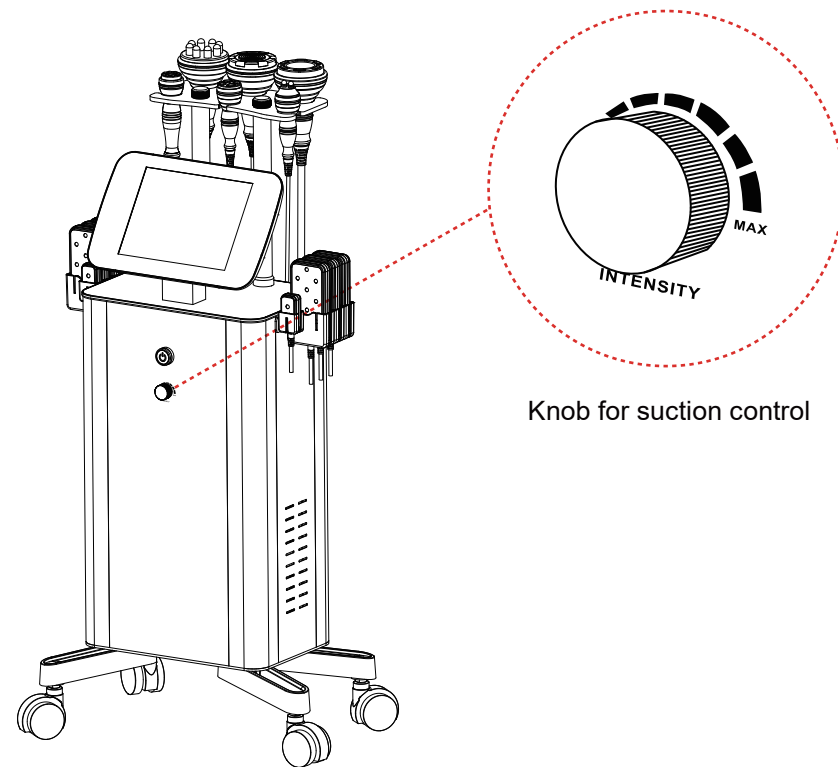

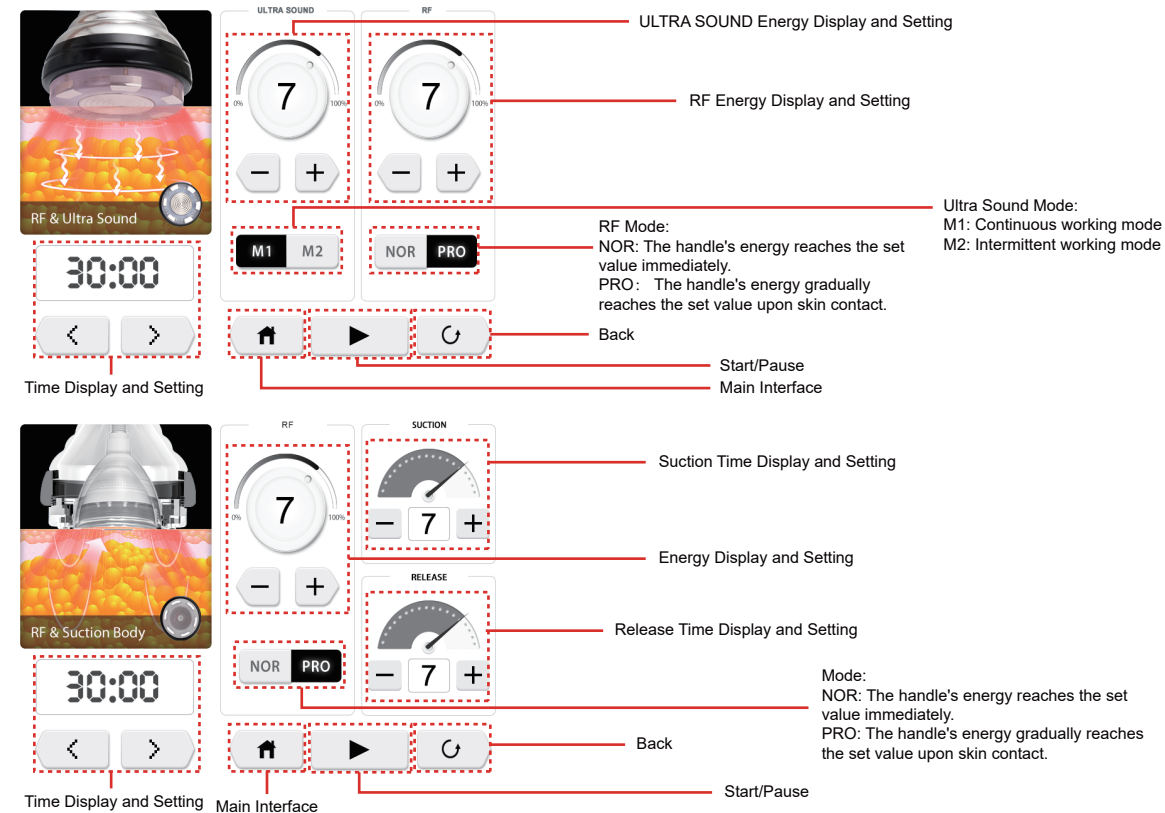

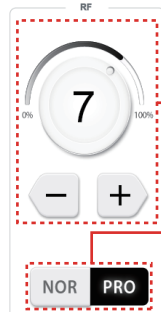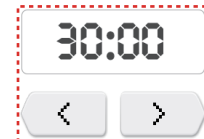

Energy Display and Setting



Mode:

NOR: The handle's energy reaches the set value immediately.
PRO: The handle's energy gradually reaches the set value upon skin contact.


Time Display and Setting

Main Interface



Back

Start/Pause

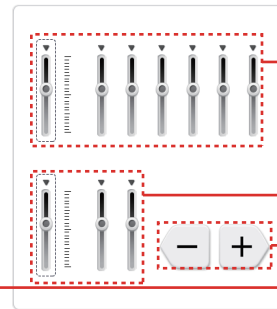

Energy Level of Big Laser Pad

Energy Level of Small Laser

Energy Level Set


Time Display and Setting

Main Interface



Back

Start/Pause

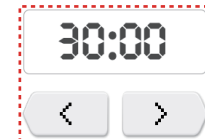

Energy Display and Setting



Mode:

NOR: The handle's energy reaches the set value immediately.
PRO: The handle's energy gradually reaches the set value upon skin contact.


Time Display and Setting

Main Interface



Back

Start/Pause

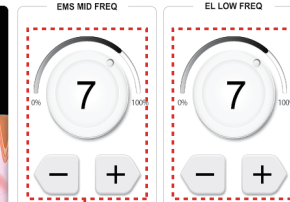

EL Energy Display and Setting

EMS Energy Display and Setting


Time Display and Setting

Main Interface

Start/Pause




Back

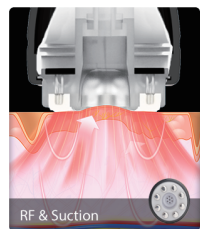

RF & Suction



Main Interface

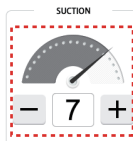

Suction Time Display and Setting

Energy Display and Setting



Release Time Display and Setting



Time Display and Setting




Back






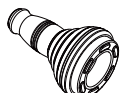
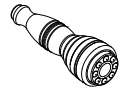


Start/Pause

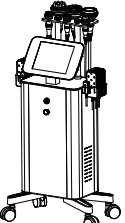Mode:
NOR: The handle's energy reaches the set value immediately.
PRO: The handle's energy gradually reaches the set value upon skin contact.

5. Technical Specification

	Power supply Input: AC110V-220V 50HZ-60HZ	
	Power: 85W	Input Power: 85W
	Dimensions: 14.5 × 21 × 44inches	
	Net Weight: 20KG	Gross Weight: 22.5KG

	Frequency: 5MHz		
	Power: 75W		
	Frequency: 5MHz		Wavelength: 650nm 160mw
	Power: 40W		Power: 26W
	RF Frequency: 5MHz		EL Frequency: 62.5Hz
	30K Frequency: 30KHz		EMS Frequency: 5Hz
	Power: 85W		Power: 15W
	RF Frequency: 5MHz	Airflow: 10LPM	
	Power: 75W	Noise level: < 65db (30cm away)	
	Vacuum: -75kpa		
	RF Frequency: 5MHz	Airflow: 10LPM	
	Power: 70W	Noise level: < 65db (30cm away)	
	Vacuum: -75kpa		

6. Use Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the instrument.
4. Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.
5. Refrain from using the instrument on an empty stomach, and initiate the treatment one hour after a meal.
6. Ensure full skin contact with the treatment head throughout the session to avoid discomfort.
7. Before EMS/EL treatment, inform the recipient about microcurrent sensations to prevent concerns or discomfort.
8. Refrain from using the RF&Ultrasound handle on areas such as the head, chest, breast, heart, or back.
9. Always start with the lowest suction intensity, and increase it gradually.
10. Ensure the filter cotton is installed inside the vacuum handle, as the foreign body may block the handle.

7. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. Clean and sanitize the handle with a damp cloth or saline, then dry it thoroughly with a towel to ensure it remains moisture-free.
3. Before cleaning the Vacuum & RF applicator, ensure to remove the filter cotton to prevent the growth of bacteria.
4. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.
5. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8. FAQ

1. How long does it take to see the results of RF treatment?

A: You might experience skin tightness right after the treatment, as the heat stimulation tightens the collagen tissue, and the RF stimulates the subcutaneous tissue bringing the continuous regeneration of collagen, leading to more noticeable results within one week.

2. Is RF safe for the skin?

A: RF firming and anti-wrinkle treatment is a non-surgical method that stimulates collagen regeneration and metabolism in the deeper layers of the skin without causing any harm. It's normal to experience temporary heat and redness in the treated area, which indicates increased circulation and should subside within a few hours. There's no need to be concerned about this reaction.

3. What is the duration of a body sculpting treatment?

A: Approximately 60 minutes, our skilled techniques and advanced device guarantee impressive results.

4. Liposuction or this device?

A: Liposuction surgery utilizes negative pressure to remove adipose tissue accumulations from specific areas, resulting in a body-shaping effect. It provides you with immediate results, but it also carries risks, requires anesthesia, and involves downtime. While our device offers a risk-free solution with no side effects. It sculpts your body, stimulates collagen regeneration for lifted skin, and creates the ideal figure for an overall improved appearance.

5. Is it safe to use ultrasound on the entire body?

A: NO. Ultrasound disrupts adipose tissue, which is then absorbed and metabolized by the human body. The power of ultrasound is quite strong, and the heart is particularly sensitive to sound waves due to being a hollow organ. Sound waves can cause vibrations in the heart, and there can be variations in how sound waves are conducted through muscle tissue and blood within the heart, resulting in back-and-forth reflections.

As a consequence, the valve may become dislodged from the heart muscle. Certainly, ultrasound treatment should be strictly avoided around the eye area to prevent the risk of retinal detachment, as well as areas such as the waist, back, chest, and the region around the heart.

6. Are there any side effects of ultrasound on the human body?

A: Ultrasound treatment is a non-surgical, non-invasive procedure that requires no incisions or anesthesia. It combines various technologies, including ultrasound (mechanical wave), cavitation, and ultrasound focusing. This process results in adipose tissue breakdown, dissolution, skin tightening, and body shaping. Importantly, it selectively targets low-density adipose tissue while safeguarding high-density tissues like blood vessels and nerves, ensuring no adverse effects on the body. You may notice a slight noise in your ears during the treatment, which is a normal occurrence and requires no cause for concern.

7. Why do I get tinnitus during the treatment?

A: When the ultrasound with concentrated energy vibrates strongly and has a frequency higher than 20KHz, it can penetrate the skin to reach the adipose layer that is 20mm deep. The energy causes high-speed friction and heat within the adipose tissue, which breaks them down and emulsifies them. However, this process can create noise that may be audible in your ears.

9. Ask & Answer

Ask	Cause	Answer
1.Fail to turn on the instrument.	→ The power cord is disconnected from the outlet.	→ Check if the power cord is securely connected to the outlet and ensure that the outlet is functional.
2.The screen shows an error message.	→ Data error.	→ 1. Turn off the instrument, disconnect the power cord, and wait for 1 minute to restart the instrument. 2. If the above method doesn't resolve the problem, please contact your dealer.
3.The RF is not functioning.	→ The handle is not properly connected.	→ Please confirm if the handle plug is well connected to the main unit.
	→ Oil in the treatment area could result in insufficient contact between the handle and the body.	→ Apply some gels to the treatment area after cleaning.
	→ The handle is not in proper contact with the skin.	→ Ensure complete contact between the treatment head and the skin.

Ask	Cause	Answer
4.RF&Ultra Sound handle is not functioning.	→ You might mistakenly assume that the handle is not functioning properly because of its low-noise design.	→ To verify handle functionality, refer to the RF & Ultrasound handle test.
	→ You may have forgotten to apply the gel or have used a product that is not designed for use with the instrument.	→ Please use professional gel for the RF&Ultra sound treatment.
	→ Insufficient pressure.	→ Apply more pressure during the operation.

10.Operation Guide

Face:

(1)Handle Selection: EMS&EL

Time: 10 minutes

Energy Level: 3

Consumables: Essence

Treatment Frequency: 1 - 2 sessions per week.

(2)Handle Selection: RF&Suction

Time: 10 minutes

RF Energy Level: 3

RF Mode: NOR

Suction Duration: 5

Release Duration: 3

Set the suction intensity according to the individual endurance.

Consumables: massage cream

Treatment Frequency: 1 - 2 sessions per week.


(3)Handle Selection: Lipo Laser

Time: 20 minutes.

Energy level: 3

Mode: M1

Treatment aids: strap

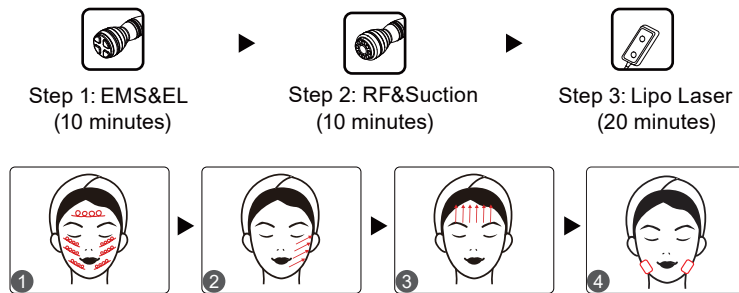

Eye:

(1)Handle Selection: Eye&RF

Time: 10 minutes

Energy Level: 3

Mode: NOR

Treatment Aids: Eye serum

Treatment Frequency: 1 - 2 sessions per week.



Arm:

(1)Handle Selection: Body&RF

Time: 15 minutes

Energy level: 3

Mode: NOR

Consumables: RF cream

Treatment Frequency: 1 - 2 sessions per week.

(2)Handle Selection: RF&Ultra Sound

Time: 15 minutes

Ultra Sound Energy Level: 3

Ultra Sound Mode: M1

RF Energy Level: 3

RF Mode: NOR

Consumables: gel

Treatment Frequency: 1 - 2 sessions per week.

(3)Handle Selection: RF&Suction Body

Time: 15 minutes

RF Energy Level: 3

RF Mode: NOR

Suction Duration: 5

Release Duration: 3

Set the suction intensity according to the individual endurance.

Consumables: essential oil

Treatment Frequency: 1 - 2 sessions per week.

(4)Handle Selection: Lipo Laser

Time: 20 minutes.

Energy Level: 3

Mode: M1

Treatment aids: strap

Treatment Frequency: 1 - 2 sessions per week.

Belly:

(1)Handle Selection: Body&RF

Time: 15 minutes

Energy Level: 3

Mode: NOR

Consumables: RF cream

Treatment Frequency: 1 - 2 sessions per week.

(2)Handle Selection: RF&Ultra Sound

Time: 15 minutes

Ultra Sound Energy Level: 3

Ultra Sound Mode: M1

RF Energy Level: 3

RF Mode: NOR

Consumables: gel

Treatment Frequency: 1 - 2 sessions per week.

(3)Handle Selection: RF&Suction Body

Time: 15 minutes

RF Energy level: 3

RF Mode: NOR

Suction Duration: 5

Release Duration: 3

Set the suction intensity according to the individual endurance.

Consumables: essential oil

Treatment Frequency: 1 - 2 sessions per week.

(4)Handle Selection: Lipo Laser

Time: 20 minutes.

Energy Level: 3

Mode: M1

Treatment aids: strap

Treatment Frequency: 1 - 2 sessions per week.



Step 1: Body&RF
(15 minutes)




Step 2: RF&Ultra Sound
(15 minutes)



Step 3: RF&Suction Body
(15 minutes)



Step 4: Lipo Laser
(20 minutes)

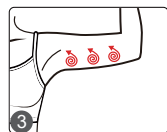



Step 1: Body&RF
(15 minutes)



Step 2: RF&Ultra Sound
(15 minutes)



Step 3: RF&Suction Body
(15 minutes)



Step 4: Lipo Laser
(20 minutes)



Back:

(1)Handle Selection: Body&RF

Time: 15 minutes

Energy Level: 3

Mode: NOR

Consumables: RF cream

Treatment Frequency: 1 - 2 sessions per week.

(2)Handle Selection: RF&Suction Body

Time: 15 minutes

RF Energy Level: 3

RF Mode: NOR

Suction Duration: 5

Release Duration: 3

Set the suction intensity according to the individual endurance.

Consumables: essential oil


Treatment Frequency: 1 - 2 sessions per week.



Step 1: Body RF
(15 minutes)



Step 2: RF&Suction Body
(15 minutes)

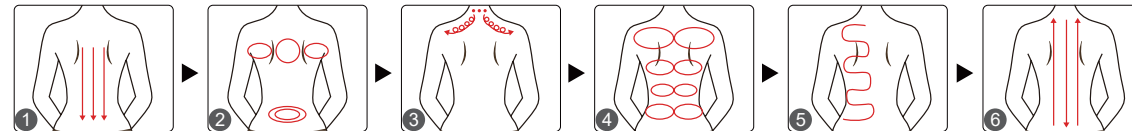

Leg:

(1)Handle Selection: Body&RF

Time: 15 minutes

Energy Level: 3

Mode: NOR

Consumables: RF cream

Treatment Frequency: 1 - 2 sessions per week.

(2)Handle Selection: RF&Ultra Sound

Time: 15 minutes

Ultra Sound Energy Level: 3

Ultra Sound Mode: M1

RF Energy Level: 3

RF Mode: NOR

Consumables: gel

Treatment Frequency: 1 - 2 sessions per week.

(3)Handle Selection: RF&Suction Body

Time: 15 minutes

RF Energy Level: 3

RF Mode: NOR

Suction Duration: 5

Release Duration: 3

Set the suction intensity according to the individual endurance.

Consumables: essential oil

Treatment Frequency: 1 - 2 sessions per week.

(4)Handle Selection: Lipo Laser

Time: 20 minutes.

Energy level: 3

Mode: M1

Treatment aids: strap

Treatment Frequency: 1 - 2 sessions per week.



Step 1: Body&RF
(15 minutes)




Step 2: RF&Ultra Sound
(15 minutes)



Step 3: RF&Suction Body
(15 minutes)



Step 4: Lipo Laser
(20 minutes)

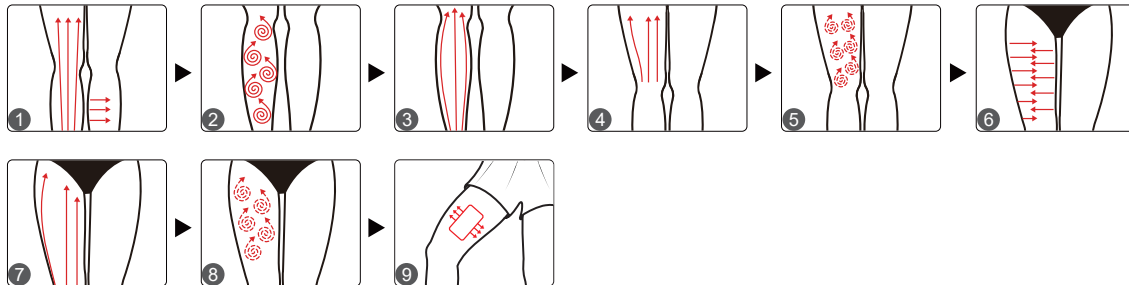

11.Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name: Versatile Beauty Enhancer

Product Model: MS-65F8

Production Date: _____

Quality Inspection: _____

