


## 3 in 1 micro-carving leather grinding instrument

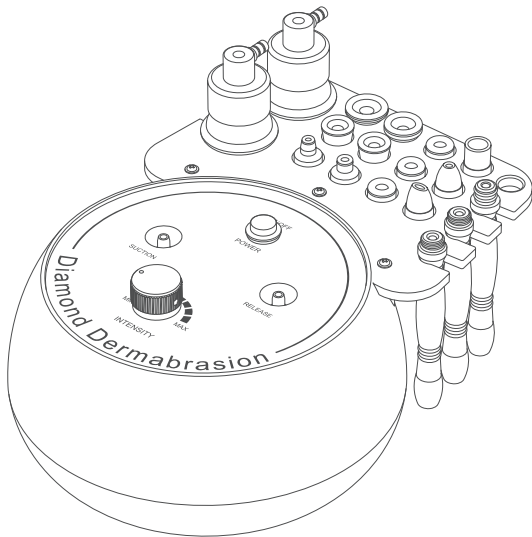

Please read this manual carefully before use

Dear users:

I am very pleased to introduce you to the 3 in 1 micro-carving leather grinding instrument, which is mainly used for dry skin, enlarged pores, clogged pores, and inelastic skin. This is a multifunctional beauty machine for face and body. Our goal is to obtain safe and effective beauty effects through the use of high end technology.

The 3 in 1 micro-carving leather grinding instrument machine is a special skin care machine that needs to be used professionally, and it needs to be operated by properly trained professionals. Any improper use will lead to undesirable onsequences. Therefore, we recommend that all personnel read this guide carefully and strictly follow the instructions before operation.

We believe that our high-quality products will bring you good returns.

In addition, you can rely on our worry-free after-sales service. thank you very much.

# INTRODUCTION

---

Microdermabrasion and high-pressure spray are currently the most popular beauty methods for deep cleansing and deep moisturizing. It is an effective substitute for all traditional facial cleansing and facial treatments in beauty salons. And easy to operate. It solves various skin problems for beauty lovers. The device does not require injections, drugs, and surgery, so there are no side effects.

During the whole process, it is used externally and has an immediate effect. Micro-sculpting microdermabrasion uses vacuum negative pressure suction technology to reduce internal air pressure, effectively remove dirt from the stratum corneum, suck out blackheads, excess oil and other pore garbage, realize free breathing and circulation of the skin, unclog pores, improve pores, and brighten skin tone. At the same time, the high-pressure spray can add moisture to the skin, cool and calm the skin.

## ADVANTAGE

---

1. Three-in-one beauty machine, which can improve facial care in an all-round way.
2. According to different requirements, multiple working probes can be replaced in different parts.
3. The micro-carved microdermabrasion probe can effectively improve various skin problems and help restore clean skin.
4. Painless and non-invasive during the process. Skip any recovery period without affecting any work and life plans.
5. No consumption, low cost and fast return.
6. Deep clean skin, replenish skin moisture, make skin more delicate.

## PRODUCT PRINCIPLE

---

### Micro-sculpted dermabrasion

Using special micro-carved tubes, inlaid with fine steel particles of different thicknesses, the skin will shed a superficial layer of dead skin cells by rubbing back and forth and controlled by the power of vacuum suction. It is non-invasive and physical. By removing dead skin cells on the surface of the skin, regeneration ability and skin quality can be improved. After use, the skin will shrink slightly, and the skin will become smoother and softer. At the same time, combined with manual massage techniques can also play a role in lifting the skin.

## Physical effect

Adopt vacuum negative pressure suction technology to reduce internal air pressure, effectively remove waste stratum corneum, and extract pores such as blackheads and excess grease. In addition, it helps to achieve free skin breathing and circulation, can unclog pores, improve pore size, and brighten skin tone.

## Glass tube

Adopt vacuum negative pressure suction technology to suck off the oil clogged in the pores and remove the dirt in the pores deeply. It can also remove blackheads and whiteheads.

## High pressure spray

Use high-pressure oxygen and water to spray water on the skin, with molecular particles smaller than 80 microns. The active water molecules and pure oxygen then quickly enter the deep layers of the skin, accelerating the absorption of nutrients in the skin. Enter the skin pores through high pressure, the molecules penetrate the dermis, promote cell regeneration, directly and quickly provide adequate nutrition to the skin, and at the same time remove the deep layer of the skin. Hyperbaric oxygen injection and nutrients can stimulate the regeneration of dermal fibrous tissue and promote metabolism, so as to achieve the effect of moisturizing, tenderness and radiance.

## Physical effect

The hydraulic jet nozzle is equipped with a pressurized oxygen pump. Using the advanced pressure swing adsorption principle, PSA quickly separates the high concentration of pure oxygen in the air, and mixes 80% of the pure oxygen molecules and 0.9% of the water molecules into the water through pulsed push, and transports them to the deep cells of the skin to replace them. The waste toxins can quickly produce energy, promote metabolism, and achieve a cleaning effect.

Through the vacuum tube as thin as three hairs, water and oxygen are rapidly atomized into the constituents, and injected into the skin at all levels through pulse injection to supplement the cell's demand for water and oxygen, thereby restoring cell vitality and making the skin moist and elastic.

## PRODUCT FUNCTION

---

1. Remove waste cuticle
2. Deep clean skin, improve rough skin.



3. Unclog the pores.
4. Shrink large pores.
5. Effectively remove strawberry nose, acne and blackheads.
6. Deep clean pores and dirt, thoroughly clean the skin.

## FOR PEOPLE

---

1. People with loose facial skin and lack of elasticity.
2. People with clogged pores and thick cuticles.
3. People with dull skin.
4. People with rough skin and large pores.
5. People with high turnover rate of oil secretion.

## NOT APPLICABLE TO THE CROWD

---

1. People with severely sensitive skin or in the allergic period.
2. People with a prosthesis in the nose.
3. Those with severe facial trauma.
4. Pregnant women and people with severe heart disease, severe three highs (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
5. People suffering from skin diseases and infectious diseases
6. People who have just received cosmetic injections, such as hyaluronic acid, intradermal injection, wrinkle removal or plastic surgery.


## NOTE AFTER USE

---

1. Wash your face with warm water within 3 days after use.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use irritating skin care products, including scrubs, exfoliating and AHAs within one week after surgery.
4. Avoid staying up late, smoking and drinking, eat more fruits and vegetables, and eat less greasy food.
5. Use the moisturizing mask once a day for three days.


# PRODUCT OPERATION INSTRUCTIONS

---

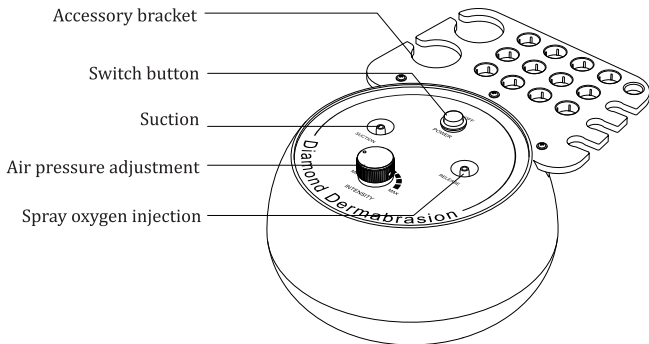


## PRODUCT ACCESSORIES

---

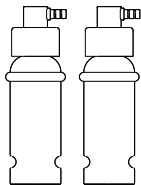


High pressure  
spray bottle\*2

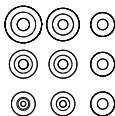

Micro carving  
head\*9









Blackhead  
suction\*3

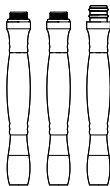

Micro-carved  
tube\*3


# MICRO-SCULPTED DERMABRASION

1. Fix the black rubber ring on the micro-carved tube.
2. Put the cotton filter into the micro-carving head to prevent dirt from entering the equipment, and check the cleaning effect at the same time.
3. Tighten the connection between the micro-carving head and the micro-carving tube.
4. Unscrew the screw on the other end of the micro-carved tube; after inserting the white hose, screw it back,
5. Adjust the suction strength, it is recommended to set it between 20%-80%. Avoid high suction, as it may cause congestion. Adjust clockwise to the maximum level; turn counterclockwise to adjust to the minimum level.
6. Insert the white hose connected with the micro-carved tube into the negative pressure suction socket of the host.

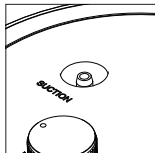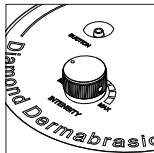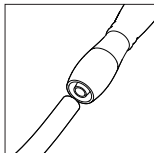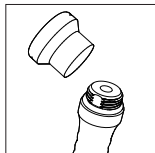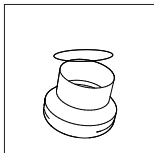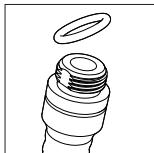

Choose the appropriate micro-carving head according to the area of the skin where it is used

Small skin: face, nose, lips, etc.




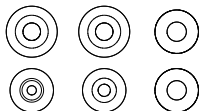


Large areas of skin: joints, face, etc.



# BLACKHEAD OPERATION INSTRUCTIONS

1. Put the cotton filter inside the blackhead parts to prevent dirt from entering the equipment, and check the cleaning effect at the same time.
2. Tighten the connection between the blackhead suction parts and the blackhead tube.
3. Unscrew the screw on the other end of the black tube; after inserting the white tube, screw it back,
4. Adjust the suction strength, it is recommended to set it between 20%-80%. Avoid high suction, as it may cause congestion. Adjust clockwise to the maximum level; turn counterclockwise to adjust to the minimum level.
5. Insert the white hose connected to the black head tube into the negative pressure suction socket of the host.

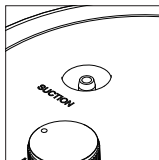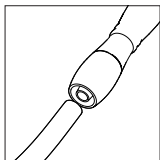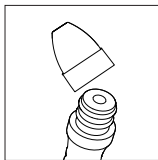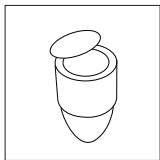

Choose the appropriate blackhead parts according to the area of the skin where it is used



## Small round hole tip

It is gentle and does not damage the skin, effectively removes aging keratin.



## Oval hole tip

It lifts the skin, tightens and removes blackheads, reduces sebum, and looks more youthful.





## Big round hole tip

With high suction power, it is easier to suck out blackheads in the T-zone of the face.

# HIGH-PRESSURE SPRAY OPERATION INSTRUCTIONS

---

1. Fill the bottle with essence/toner/purified water, etc.  
(Essence/toner: deep moisturizing and brightening the skin)  
(purified water: deep cleansing and moisturizing)
2. Insert one end of the red rubber hose into the air inlet
3. Insert the other end of the red rubber tube into the high-pressure spray socket of the host.
4. Press the top of the bottle to release the spray.

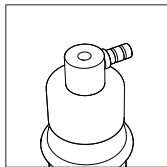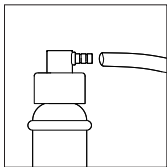

## INSTRUCTIONS

---

### Deep cleansing (≈60 minutes/once a week)

1. Remove makeup and clean the dirt on the surface of the face.
2. Apply blackhead gel and match with hot steam (apply toner on areas with large pores or clogged areas) for sensitive skin, it is recommended to use toner + cold steam (5-8 minutes)
3. Microdermabrasion: stay on the forehead for one second, then gently twist and move (forehead→nose→chin) (5 minutes)
4. Facial micro-sculpting: starting from the chin, point operation, and then connect 3 points to form a straight line and move in parallel. End with the temple. (3 minutes)
5. Do the same action on the other side. (3 minutes)
6. Glass tube operation: double clean the T zone (forehead, nose, chin). If you have more blackheads, you can clean the nose twice more. (3 minutes)

7. Forehead glass tube: slowly move the glass tube to the hairline. (2-3 times)
8. Nasal glass tube: slowly move the glass tube down the nose and nasolabialfold. (2-3 times)
9. Mouth glass tube operation: slowly move the glass tube down the parallel line, from one corner of the mouth to the other. (2-3 times)
10. Wash your face.
11. Apply moisturizing mask + cold steam. (10 minutes)
12. Clean your face with facial cleanser. (2 minutes)
13. Fill the water bottle with pure water.
14. High-pressure spray operation: from the chin to the earlobe, and then from the corner of the mouth to the door of the ear. Move in parallel while supporting the skin with free hands to keep the surface tight. (2 times)
15. Move the forehead hairline vertically while supporting the skin with free hands to keep the surface tight. (2 times)
16. Apply a facial mask. (15 minutes)
17. Wash your face.
18. Apply toner, face cream, sunscreen.

## Recommend

10 cleansing is a complete course of treatment.

After one use, you will see a clean, smooth and natural skin.

After a complete process, you will see that your skin is smooth and full of radiance.

After two complete treatments, your skin will be firm, smooth and elastic.

Three complete treatments can help your skin maintain elasticity, radiance and vitality.

## SAFETY PRECAUTIONS


---

If you have the following symptoms, please consult your doctor or professional before using this device:

1. People who are pregnant or breastfeeding.
2. Patients with heart disease or with a pacemaker.
3. Persons whose wounds have not healed and are recovering from surgery.
4. Patients with epilepsy, severe diabetes and hyperthyroidism.
5. Malignant tumors, hemophilia or severe bleeding.
6. People suffering from skin diseases and infectious diseases.

# 109L - HIGH PRESSURE OXYGEN INJECTION GUN\*1

---

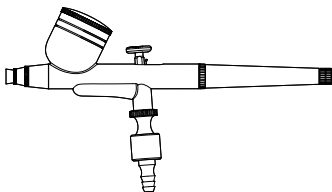

## PRINCIPLE




---

The high-pressure oxygen injection gun compresses the water and nutrients in the essence water into tiny molecules and injects them into the skin, so that the skin can absorb more nutrients. One injection of oxygen = five times of application.

## INSTRUCTIONS


---

1. Fill the top bottle with essence/toner/purified water, etc.  
(Essence/Toner: Deeply hydrates, brightens skin)  
(Pure Water: Deep cleans and moisturizes)
2. Insert one end of the red rubber hose into the air inlet
3. Insert the other end of the red rubber tube into the high pressure spray socket of the main unit.
4. Press the button on the top of the gun to release the spray.

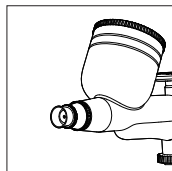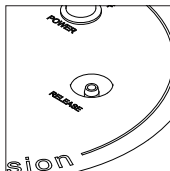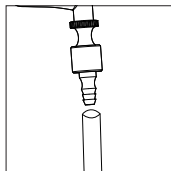


# 110L - GREASE HEAD\*3


---

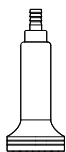

(Small)

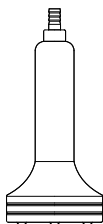

(Medium)

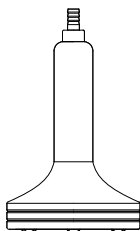

(Large)

## PRINCIPLE

---

Small scraping head: It can be used on the neck or other small areas.




Medium Gua Sha Head: It can be used on the meridians of the hand and has the widest range of use.

Large scraping head: It can be used to dredge the back or legs, and it can be better operated on a large skin surface.

## INSTRUCTIONS

---

1. Insert the white hose into the grease head connection port.
2. Insert the other end into the negative pressure suction port of the instrument.
3. Choose a suitable part of the body for massage.

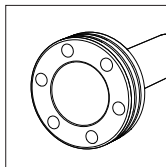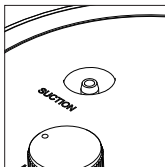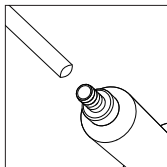



# PRECAUTIONS FOR PRODUCT USE

---

1. Before each operation, make sure that the power socket and grounding pin of the device are properly connected.
2. Make sure that the voltage of the equipment is appropriate. If the local voltage power supply is unstable, it is recommended that the user increase to the matching power supply voltage.
3. In order to ensure the use effect and normal service life of this device, please use the designated accessories provided or recommended by the original manufacturer.
4. The equipment should not be placed in a humid environment or near water, nor should it be directly exposed to the sun.
5. Do not place the device near a strong heat source, as this may affect the service life and normal use of the device.
6. Please do not use this device on the eyes, thyroid gland, parathyroid glands, testicles, pregnant women's abdomen and pacemakers.
7. Patients suffering from diseases should consult their doctors and obtain the doctor's permission before use.
8. Please turn off the power switch of this device, and make sure to turn off the main power of this device before the staff leave after daily use to ensure the safety of electrical products.
9. Strictly follow the instructions in the user manual to use the equipment or train the equipment operators.
10. Start with the lowest energy level and slowly accumulate.
11. When using this equipment, the operating part must be kept moist to avoid dry skin.
12. Avoid using it around the eyes and ears, skin injuries, or areas undergoing plastic surgery.
13. Adjust the vacuum suction pressure according to the customer's comfort level.
14. Remind customers to adjust their breathing when using hydraulic jets to operate the center column.
15. When operating the hydraulic ejector, ask the customer to close his eyes and mouth to prevent liquid from entering. Keep the distance between the bottle and the care area.
16. Clean the instrument with normal saline after operation to ensure that the instrument is clean and hygienic and prolong the service life of the instrument.
17. When operating the micro-carved dermabrasion setting, make sure to install the rubber ring to avoid no suction or weak suction.
18. When operating the micro-sculpted dermabrasion setting, make sure to install filter cotton to avoid dirt sucking into the pipe and damaging the machine.

19. Products used in high-pressure spray must be water-based. Toner containing impurities and deposits is not allowed.
20. Use a toothbrush or Q-tips to clean the micro-sculpted microdermabrasion patch, and sterilize it under ultra-ultraviolet rays or high temperature.
21. Be sure to remove the cotton filter after use.
22. Make sure to distinguish between the inhalation hole and the exhalation hole. Do not insert the hydraulic nozzle into the suction hole to avoid any possible damage to the machine. Do not operate negative pressure suction and high pressure spray at the same time.

## TROUBLESHOOTING

---

1. The instrument cannot be started, and the light on the back of the instrument does not light up?
  - A. Make sure that the power cord is connected to a working power outlet.
  - B. Make sure that the fuse tube on the back of the instrument is not loose or burned out.
2. Can't spray spray from spray bottle?
  - A. Please check if the nozzle is blocked.
  - B. Please check whether the spray bottle button is damaged.

## COMMON PROBLEM

---

- 1.Q: Will the skin become thinner and more sensitive after microcarving?
  - A: No. In this process, the microdermabrasion surgery is only used to remove the aging skin stratum corneum. Our skin regenerates every 28 days, so it only makes the skin smoother, more delicate and radiant. Allergy symptoms will not appear.
2. Q: Why do you need to exfoliate?
  - A: The stratum corneum is the layer where the cells above the epidermis are completely dead. Depending on your body, the stratum corneum is composed of 15 to 20 layers of dead cells. Human skin has a metabolic cycle, usually 28 days. This is the metabolic cycle of adolescents. For adults, the metabolic cycle should be actual age +10 years old.

However, due to various reasons, dead keratinocytes may not fall off in time, leading to thickening and aging of the stratum corneum. On the other hand, it also affects the color of the skin and the metabolism of the skin, so when the normal metabolic process itself cannot complete this work, other methods are needed.

3. Q: Can I put on makeup after using this instrument?

A: Of course, the skin after use is moist and plump, and the makeup time will actually be longer. This is a safe, healthy, fast and effective care method that will not pierce the skin or leave scars.

4. Q: How long will it take to see the results?

A: After one use, there will be obvious results, such as clean and transparent skin. The effect of continuous use is better, and it can improve the deep dehydration, darkening, fine lines and aging of the skin.

5. Q: Will the skin dependence?

























A: No, its function is to improve and deep clean the skin. It is the skin care and improvement of malnutrition caused by other factors. It is used to help the skin absorb nutritional products and solve the problems of dullness, yellowing, dehydration, roughness and wrinkles. If you do not repair, your skin will only restore its own secretion and aging, so no dependent factors are involved.

## PRODUCT PARAMETER

---

PRODUCT NAME	3 in 1 micro-carving leather grinding instrument	MATERIAL QUALITY	ABS material
RATED POWER	15W	PRODUCT TECHNOLOGY	micro skin grinding, high pressure spray, blackhead
NEGATIVE PRESSURE SUCTION	< - 60kpa	PRODUCT SIZE	200* 200* 105MM
RATED VOLTAGE	12V = 1A	EXTERNAL VOLTAGE	110/120--220/240V

# PRODUCT ACCESSORIES LIST

MODEL FUNCTION ACCESSORIES	108L	109L	110L
 GRINDING POLE			
 BLACKHEAD POLE			
 EXFOLIATING HEAD			
 CRYSTAL HEAD			
 WATERING CAN			
 SPRAY GUN			
 GREASE HEAD			

“” REPRESENTS THE ACCESSORIES OWNED BY THE MODEL

# PRODUCT WARRANTY

---

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name:	3in1micro-carving leather grinding instrument
Product Model:	SR-NF201
Production Date:	
Quality Inspection:	

PASS

- Warranty is not covered under the following circumstances.
- Failure in case of fire, water immersion, natural disaster, etc.
- Failure to use the product in accordance with the operating instructions.
- The failure of the product is caused by improper disassembly, disassembly and repair of the consumer after he / she falls on the ground.
- Consumables are not covered by warranty.

## WARRANTY INFORMATION

---

PRODUCT NUMBER	
PURCHASE DATE	
USER NAME	
ADDRESS	
WHERE TO BUY	
CONTACT NUMBER	
REASON FOR REPAIR	

I WISH YOU A HAPPY USE,  
YOUR SATISFACTION IS OUR POWER TO MOVE FORWARD