

Manual of fever master



What is【 448k temperature controller 】?

A Spanish scientist, Mr Josep Calbet, invented it for cancer patients.

Its instrument raises the temperature of 1-7 degrees for the cancer cells in the exploration layer of the body. When the cancer cells are about 35 degrees, they spread quickly, but when the I grade of cancer cells reaches above 39 degrees, Yu cells can't replicate themselves, and other cells will die.

Spanish scientists believe that the core technology of this instrument can also be applied to beauty. It is not only beautiful to improve the body temperature, but also can improve the temperature in the deep part of the body when the instrument is operated. The temperature in the body is closely related to your health. If the body temperature is raised by 1 degree, your body's immunity can be enhanced by 5-10 times.

Let me give you an example. When we have a cold, we will have a fever. When we have a fever, the temperature in our body will rise. At this time, our immune cells will be activated to attack and resist the disease. From the case, it can be analyzed that the temperature in our body has a great relationship with our immunity.

A few days ago, there were many similar beauty instruments on the market, but all of them were difficult to penetrate into the deep parts of the body. The 448k temperature controller can safely deliver Cong Nengjun to the deep parts of the body through the product frequency of 448KHz, which can reduce the internal temperature and immunity of the body.

Principle and efficacy of CAP

- Cap principle: cap is based on skin constant normal temperature and deep RF heating, which ensures the skin to be comfortable and not damaged, and then uses CAP to generate thermal conduction, which sticks to the deep skin, stimulates the detached and entrained colloidal particles in the body to move or vibrate rapidly, and generates heat energy. At the same time, under the action of polar molecular resonance technology, Heat the collagen tissue with cortex. When the heat energy of the deep-seated tissue of the skin reaches 45°C-60°C, it will naturally contract immediately, stimulating the secretion of more new collagen) to fill the gap of shrinking and lost collagen, rearrange it, and find a soft skin scaffold. Mosquitoes will finally achieve the effects of tightening the skin, flattening wrinkles, restoring skin elasticity and photometer.

efficacy

1. Lift and tighten, screw acne and improve wrinkles;
2. Burn fat and relieve pain.
3. Improve constipation and increase elasticity;
4. Tighten postpartum relaxation and relieve muscle pain: relax stiff joints.
5. Introduction of auxiliary essence

RES principle and efficacy

R. E. S Fei-frequency instrument is a slimming instrument that projects Qi-frequency energy fan into human body, generates deep heat from skin epidermis, decomposes body fat, and thus achieves the purpose of fattening. When the ring-frequency current is electrified to human tissue, because its detection amplitude is not small, it hardly produces ion movement, and it also does not produce electrochemical reaction or electro-gas decomposition. If the quotient frequency is transmitted to the human body, the human tissue will heat up. This is because when high-frequency current is applied to the human body, every time the current changes direction, the molecules that make up the tissue will vibrate, and the cows will graze each other for rotating, twisting and conflicting movements. This kind of molecular movement will make the human body tissue generate body heat, which is called probe heat.

RES efficacy

1 lymphatic ovulation

2. Blood ring to lift the nest

3 repair of cells, nerves, bones, muscles and ligaments

4 regulating endocrine

5 improve sleep

6 brighten skin tone

7. Improve the problem of chest drop and hyperplasia.

8 problems with body shape and orange peel on legs


9 lose weight

IO burns visceral fat

Technical schematic diagram of 448k temperature controller




448k 温控仪技术原理

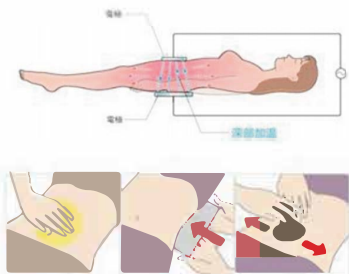

The 448k electronic control instrument uses two different electrodes to promote the positive and negative electrodes of electric high-voltage cells in the body to rub against each other sequentially to generate internal heat.

Operating interface

448K

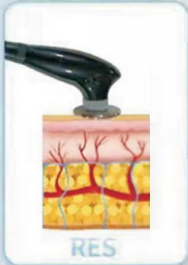

START

PAUSE




448K



START

PAUSE



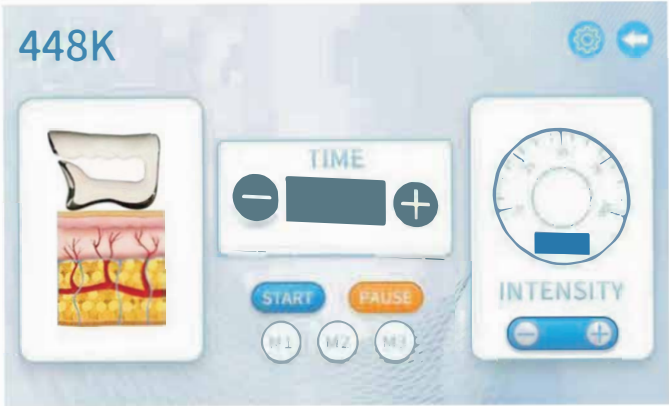

RES mode

"RES mode"

In order to achieve heat energy in the deep part of the body, you adopt the "RES-slap" method. At first, your body temperature is not very high, and the effect will not be reflected if your body temperature doesn't scare you, so you should "heat up" your deep part first. The effect of heating is that the blood in the body is trapped, and each cell can receive 1-minute "oxygen", and lymph is also rapidly encircled. Raising body temperature is the foundation of body top reform. Body temperature is also caught, of course, metabolism is also caught. Different parts of the operation time is different, about 30 minutes for one part.

1 efficacy

1. lymphatic detox
2. Increased blood circulation
3. Repair of cell nerve, iliac bone, muscle and incisor band
4. regulates endocrine.
5. improve sleep
6. improve skin color
7. Improve the problem of chest drop and hyperplasia.
8. body shape and leg wrinkles.
9. lose weight
10. burning visceral fat



CAP mode

[448k thermostat | facial care effect

[448k temperature controller L The main function of the face is muscle quality improvement and cell activation. You can enjoy the effect of a 448k temperature controller.

face

1. Anti-aging-improvement of wrinkles, spots and relaxation
2. Improve the effect
3. Facial cell activation
4. Promote the metabolism of granular column foundation
5. Promote the metabolism of the face
6. The wheelk fades away
7. Eye lift
8. Improve skin cell regeneration
9. facial fat burning-improve facial relaxation and double chin
10. Improve facial edema
12. Improve neck, neck stiffness, and relieve muscles.



The whole burying process

During nursing, in order to increase the efficacy of the 448k temperature controller, the body can be coated with firming cream, essential oil, slimming cream, etc.

448k renderings

Before & After



Matters needing attention of 448k temperature controller

- Do not use if you have the following symptoms

Matters needing attention of 448k temperature controller

Patients with acute diseases, patients with bleeding diseases above 38C.

Patients with hypertension, infectious diseases and tuberculous diseases.

Known tumor patients, heart patients, pregnant women

Allergic responder, hemophiliac.

Patients with steroid use and liver dysfunction caused by telangiectasia in the first stage, patients with primary explosion and patients with infectious diseases diagnosed by aldehyde.

Patients with facial melanosis, long-term hospital visits, and body fever.

Drug users, children under 120%.

— Safety precautions

1 This instrument is special for beauty salons and is also suitable for home users.

The operation of the instrument needs professional beauticians, and the operation of non-professionals may cause unnecessary harm.

When you receive the instrument, please check all the accessories according to the packing list in the manual and check whether the machine is damaged. If the machine is damaged during transportation, please prepare an instrument damage report, explain the missing parts or loss, and send a copy of the report to our company or distributor.

The power supply used should be consistent with the specified value of the power supply marked by the machine, otherwise the machine may not work or even burn out.

Please keep the original packaging of the instrument. When you need our assistance, you must wrap the machine in the original packaging before sending it to maintenance.

Please don't disassemble our instruments or try to do anything that is not explained in this manual. All maintenance work must be carried out by professional maintenance personnel recognized by our company.

Do not install or operate the instrument near the water source or when your hands are wet. Be careful not to spill liquid on the instrument.

Please don't put anything on the wire, and don't install the instrument in a place where the power cord is easy to be stepped on. seven

8 If the following situations occur, please unplug the instrument from the power socket and contact the professional maintenance station.

A) liquid splashes on the instrument, or the instrument gets wet or drenched:

B) The instrument emits peculiar smell, smoke or abnormal noise;

The power cord is scratched or damaged

D) the instrument falls or the shell is damaged.

Please unplug the power cord after use to ensure safety.

Do not insert any kind of articles into the opening of the instrument, otherwise there may be danger of Shi Huo or electric shock. If something falls into the instrument and you can't take it out smoothly, please unplug the power cord of the instrument and contact the dealer or our company. Don't put the instrument on a fixed cart, shelf or car. If you turn over the example, the instrument may be severely damaged.

In order to prolong the service life of this machine, the interval of continuous switching must be more than 1 minute.

13. The continuous working time of the instrument should not exceed 2 hours. After working for 1 hour every t, shut down the well and unplug the power supply, so that the instrument can rest for 10-30 minutes.

14. We have the right to modify the letter P provided in this manual without knowing it in advance. We reserve the right to interpret the said information.

Second, the use of precautions

Be sure to read the following carefully before using the secondary instrument.

Regular use of this instrument by regular servants will make your health better. It is recommended not to interrupt the use for a long time. If it is not strictly observed

No glottis and sex organs, and no idle work. The beautician must stop when he leaves.

When using this instrument, stop using it immediately if the body feels painful, or if the skin is red or leaves traces for several seconds. 4 slowly move the therapeutic head on the skin of the body, and don't stay in any grave of the skin for too long.

5 Don't use this instrument with embossed parts such as jewelry or earrings.

6 Do not use this instrument if there are metal objects in the body (such as gold wires or steel nails and pacemakers in the body). Children under the age of seven are forbidden to use this instrument, and children over the age of seven need a doctor's advice to use this instrument. Do not use this instrument.

9. It is not appropriate to use the instrument for physical therapy for heart offenders.

JO, if she has fever, infection, inflammation, skin ffj (gangrene, eczema, gangrene rash), excellent, not scarred or just scarred, severe

It is forbidden to use this instrument for chronic diseases (cancer, AIDS, hematological diseases) or blood coagulation.

11 If you have phlebitis, venous surgery (or venous sclerosis) within two months, varicose veins or numbness of limbs,

Please do not use this instrument. If in doubt, consult your doctor first.

Pregnant women are forbidden to use this instrument.

Unless instructed by a doctor, this instrument should not be used in the ear, inside the mouth, or on mucous membranes, nipples or paddles.

14. Idiotic patients are forbidden to use this instrument. People with mental illness are forbidden to use this instrument.

If you take drugs such as anti-inflammatory, anti-binding agents, antibiotics, antihistamines, etc. for a long time, or your skin will suffer from mild

If there is bruise due to collision, please consult your doctor before using this instrument.

After any plastic surgery, you can use this instrument, but you must follow the doctor's instructions.