User Manual

HR-AS64P

Content

1.Safety Precautions	1
2.Contraindications	1
3.Overview	2
4.Interface Function	3
5.Technical Specification	9
6.Operation Guide	10
7.Use Precautions	22
8.Maintenance & Cleaning	23
9.Common Faults and Their Solutions	24
10.FAQ	25
11.Warranty & Support	28

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1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure the performance throughout its service life.
- 4.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the device on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 7.Power off the instrument when not in use to ensure its safety.
- 8.To maintain hygiene and extend the instrument's service life, sterilize it with saline.

2.Contraindications

1. Pregnant women, breastfeeding mothers, patients with heart disease, malignant tumors, folliculitis, vascular diseases, serious major illnesses such as severe diseases, etc.
2. Patients with epilepsy, severe diabetes and hyperthyroidism.
3. Those with abnormal collagen metabolism, including a history of scars and poor wound healing.
4. Infection at the treatment site, herpes simplex, skin lesions, hematoma, etc.

5. Those whose wound from the recent plastic surgery has not been repaired.
6. Skin in the allergic period.
7. Patients with a pacemaker.
8. Photosensitive patients (sun-induced rash, erythema lupus), with symptoms such as redness, swelling and itching after sun exposure.
9. Those who have undergone photonic projects at the same operation site within 30 days.
10. Children and individuals under the age of 18 who are in the period of puberty development.

3.Overview

This multi-functional beauty machine employs advanced and safe hair removal and skin rejuvenation technologies, enabling you to effortlessly achieve a perfect transformation of hair removal and skin rejuvenation. It offers an injection-free, drug-free, and side-effect-free solution for those who love beauty, effectively addressing hair management and skin management issues. The entire process is safe and convenient, with the effects gradually becoming apparent after use, leaving the skin smooth and lasting, and bringing confidence and beauty.

4.Interface Function

Watering and drainage procedures:


Water filling operation

1. Install all accessories on the machine and insert the probe into the corresponding rack.
2. Remove the sealing caps of Affusion and Overflow, and connect the water filling funnel to Affusion.
3. Fill with pure water or distilled water until water flows out from Overflow, indicating that the water has been filled. Then remove the water filling funnel and reinstall the sealing caps on Affusion and Overflow.

Note: Before starting the instrument, the water filling process must be completed. Starting the device without water will cause equipment failure.

Drainage operation:


1. If you want to change the water or empty the water in the machine, first unscrew the sealing cap of Drainage and connect a funnel.
2. Unscrew the sealing cap of Overflow and wait for the water to drain.
3. Screw the sealing caps of Drainage and Overflow back on.

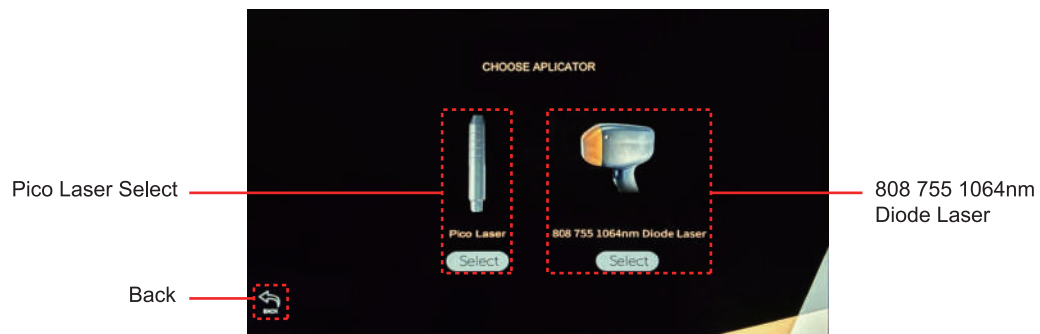
Pico Laser Interface:

808 755 1064nm Diode Laser Interface: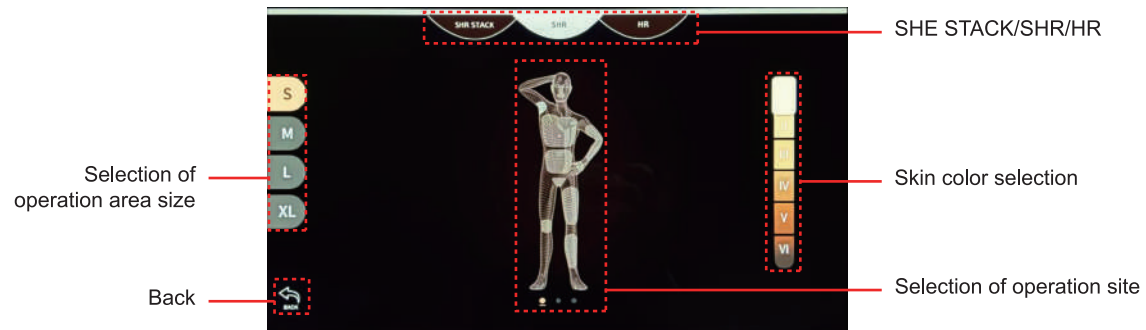

Time display and adjustment buttons

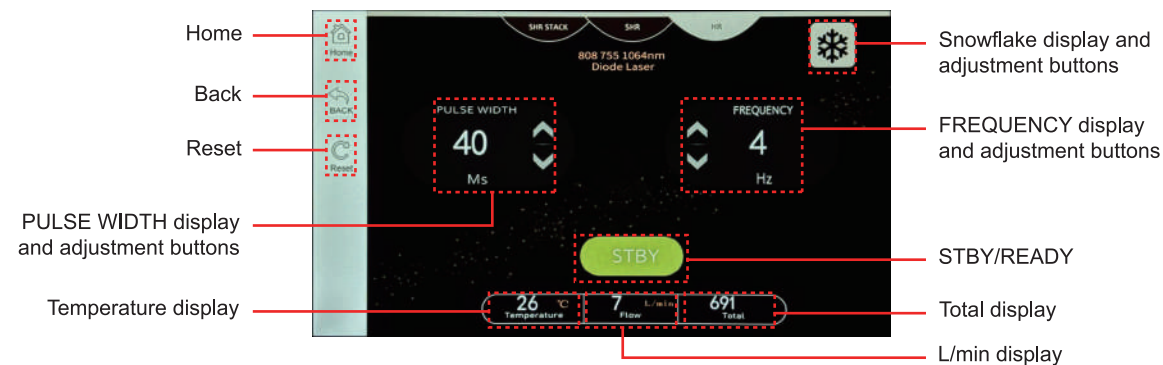5. Technical Specification

Screen size: 10.4 inches

Operating languages: English, Russian, German, French, Spanish, Portuguese

Light type: Semiconductor laser

Laser wavelength: 808nm

Working frequency: 1 - 10Hz adjustable

Pulse width adjustment: 10 - 400ms (adjustable)

Energy adjustment: 1 - 10J/cm²

Laser power: 6bar: 300W

808 power supply: 600W

Spot size: 15x15mm

YNG laser power: 300W

Output laser frequency: 1 - 10Hz continuously adjustable

Energy adjustment: ≥ 50 - 2000J/cm²

Laser wavelength: 1064nm, 532nm, 1320nm

Total power: 1200W

Output mode: Pulsed output

Cooling system: Semiconductor cooling + water cooling + air cooling for constant temperature

Shell material: ABS plastic

Power supply voltage: AC220V; AC110V (customizable)

Handle temperature: -6 to 1°C

6. Operation Guide

Arms/Back

(I) Function

Remove excessive hair from the targeted area

(II) Specific Operations

Preparation before operation: Shaving knife, disposable towel, tissue paper, gel, facial mask stick

1. Use the shaving knife to remove the hair from the surface of the skin in the area to be depilated

2. Clean up the removed hair

3. Apply a thick layer of gel to the area to be depilated

4. Put on goggles for the customer

5. The operator also puts on goggles

6. Adjust the parameters Select the treatment area, area size, and skin tone according to the recipient's condition, then tap "SHR".

Time: 5 minutes;

PULSE WIDTH: 50Hz;

FREQUENCY: 5Hz;

GRID SIZE: 3cm²;

Cooling Level: IV

7. Follow the stamping method, operate the entire area one line after another. First, operate horizontally, then vertically (5-8 times).

8. Use the facial mask stick to remove the gel and then wipe clean with a tissue.

(III) Post-operation precautions

1. After the operation, maintain a regular lifestyle and pay attention to a balance between work and rest.
2. Do not take a sauna, soak in a hot spring, swim, or expose yourself to direct sunlight within 7 days.

Lip hair / Finger hair

(I) Function

Remove excess hair in the operation area

(II) Specific operation

Preparation before operation: washbasin, disposable bag, disposable face towel, facial mask rod, shaving knife, makeup remover, facial cleanser, toner, essence, gel, mask, lotion/cream, sunscreen

1. Remove makeup and cleanse face
2. Use the shaving knife to scrape the hair in the area to be shaved clean
3. Clean up the scraped hair
4. Apply a gel with a thickness of approximately 0.5cm evenly
5. Put on goggles for the customer
6. The operator puts on goggles
7. Adjust parameters: Select the treatment area, area size, and skin tone according to the recipient's condition, then tap "SHR STACK".
FLUENCE: 45 J/cm²
STACK2: 5 Cooling Level: IV
8. Starting from the chin, apply the treatment in a line-by-line manner across the entire area. First, operate horizontally, then vertically (5-8 times).

9. Use the facial mask stick to remove the gel and then wipe clean.
10. Apply the facial mask.
11. Wash the face thoroughly, for 2 minutes.
12. Apply toner, essence, lotion/cream, and sunscreen.

(III) Precautions before and after the treatment

1. If you have used topical A acid ointment or skin-whitening products before, it is recommended to stop using them for 1 week before undergoing hair removal.
2. Do not undergo laser, exfoliation, or fruit acid rejuvenation treatments within one week before and after the treatment.
3. Maintain a regular lifestyle and balance work and rest.
4. Apply a hydrating mask at least once a day within one week after the treatment to avoid using irritating skincare products such as those containing fruit acid, salicylic acid, or alcohol.
5. Do not take a sauna, soak in a hot spring, swim, or expose yourself to direct sunlight within 7 days.
6. Strengthen sun protection. When going out, use physical sun protection measures such as holding an umbrella.

Legs / Armpits / Bikini

(I) Effects

Remove excess hair in the targeted area

(II) Specific Procedures

Preparation before operation: Shaving knife, disposable towel, tissue, gel, facial mask rod

1. Use the shaving knife to remove the surface hair in the area to be depilated
2. Clean up the removed hair
3. Apply thickly the gel on the area to be depilated
4. Put on goggles for the customer
5. The operator puts on goggles
6. Adjust parameters Select the treatment area, area size, and skin tone according to the recipient's condition, then tap "HR".
PULSE WIDTH: 40 Hz;
FREQUENCY: 4Hz;
Cooling Level: IV
7. Using the stamping method, operate the entire area one line after another, first horizontally and then vertically (5-8 times).
8. Use the facial mask stick to remove the gel and then wipe it clean with a tissue.

(III) Post-operation precautions

1. After the operation, maintain a regular lifestyle and pay attention to a balance between work and rest.
2. Do not take a sauna, soak in a hot spring, swim, or expose yourself to direct sunlight within 7 days.

Black-faced doll

(I) Effects

- (1) Deeply cleanse pores

- (2) Whitening and brightening the skin tone
- (3) Control oil secretion and shrink pores
- (4) Improve skin quality

(II) Specific operations

Preparation before operation: washbasin, disposable bag, disposable face towel, carbon paste, protective goggles, makeup remover, facial cleanser, toner, essence, cream, sunscreen

1. Use makeup remover to remove dirt, oil or makeup from the face thoroughly
2. Wash the face with facial cleanser and apply toner evenly
3. Apply carbon paste evenly on the entire face
4. Wait for the carbon paste to dry
5. Cover the customer's eyes with cotton pads and put on black protective goggles
6. The operator puts on the protective goggles
7. Select the black operation head and install it properly
8. Parameter adjustment: Settings:

ENERGY: 700Mj

FREQUENCY: 4Hz

Note:

- (1) Press the foot switch to release energy.
- (2) Press the red button on the back of the machine to control the red dot switch on the handle.
- (3) These parameters are for reference only. You can adjust them from low to high according to your personal tolerance.
9. Hold the handle operation port 3-5 cm away from your face. Start from your chin and apply the carbon powder on your face until it is all gone.

10. Remove the goggles and cotton pads.
11. Wipe your face clean with a face towel.
12. Apply a facial mask.
13. Clean your face after removing the mask.
14. Apply toner, essence, cream, and sunscreen.

(III) Precautions before and after the operation

1. Do not undergo laser, peeling, or fruit acid rejuvenation treatments for one week before the operation.
2. Do not expose yourself to excessive sunlight, sunbathing, or SPA within one month before the treatment.
3. Maintain a regular daily routine and balance work and rest after the treatment.
4. Apply a moisturizing mask at least once a day within one week after the operation.
5. Do not take a sauna, go on a hot spring bath, or swim within 7 days.
6. Strengthen sun protection. It is best to use sunscreen with an index of 30-50. When going out, take physical sun protection measures such as holding an umbrella.
7. Avoid using irritating skincare products and cosmetics within one week after the treatment, such as those containing fruit acid, salicylic acid, or alcohol.

Remove Yellowish Tone and Enhance Skin Texture

(I) Effects

- (1) Improve uneven skin tone and dullness
- (2) Fade fine lines and wrinkles
- (3) Improve skin texture and make the skin firm

(II) Specific Procedures

Preparation before operation: washbasin, disposable bag, disposable face towel, goggles, makeup remover, facial cleanser, toner, essence, lotion/cream, sunscreen

1. Use makeup remover to remove dirt, oil or makeup from the face
2. Wash the face with facial cleanser and apply toner evenly
3. Cover the customer's eyes with a cotton pad and put on black goggles
4. The operator puts on goggles
5. Select the 755nm operation head and install it
6. Parameter adjustment: ENERGY: 600Mj
FREQUENCY: 4Hz

Note:

- (1) Press the foot switch to release energy.
- (2) Press the red button on the back of the machine to control the red dot switch on the handle.
- (3) These parameters are for reference only. You can adjust them from low to high according to your personal tolerance.
7. Hold the handle operation port 3-5 cm away from your face, and operate the entire face evenly starting from the chin.
8. Remove the goggles and cotton pads.
9. Apply the facial mask.
10. After removing the mask, clean your face thoroughly.
11. Apply toner, essence, and cream.

(III) Precautions before and after the operation

1. Do not undergo laser treatment, peeling, or fruit acid rejuvenation beauty procedures for one week

before the operation.

2. Do not expose yourself to excessive sunlight, engage in sunbathing or SPA within one month before the treatment.
3. Maintain a regular daily routine and balance work and rest after the treatment.
4. Apply a moisturizing mask at least once a day within one week after the operation.
5. Do not take a sauna, go on a hot spring bath, or swim within 7 days.
6. Strengthen sun protection. It is best to use sunscreen with an index of 30-50. When going out, take physical sun protection measures such as holding an umbrella.
7. Avoid using irritating skin care products and cosmetics within one week after the treatment, such as those containing fruit acid, salicylic acid, or alcohol.

Facial Re-delineation of Eyebrows

(I) Effects

Remove the pigmentation on the eyebrows

(II) Specific Procedures

Preparation before operation: disposable towel, ointment, plastic wrap, repair solution, disposable sterile gloves

1. Wipe the operation area clean with a wet towel and apply the ointment
2. Cover the ointment with plastic wrap to enhance its effect and wait for 20 minutes
3. Remove the ointment and wipe it clean
4. Select the 1064nm/532nm operation head and install it

5. Adjust the parameters: ENERGY: 500Mj

FREQUENCY: 3Hz

Note:

- (1) Press the foot switch to release energy.
- (2) Press the red button on the back of the machine to control the red dot switch on the handle.
- (3) These parameters are for reference only. You can adjust them from low to high according to your personal tolerance.
6. Put on protective glasses for the customer and the operator
7. Put on disposable medical gloves
8. Move the instrument to the area of the eyebrows that needs to be washed, step on the foot pedal to start the operation, and blast in sequence along the area of the eyebrows that needs to be removed until one side of the eyebrows is completed, 1-2 times
9. Gently apply the repair liquid on the operation area with a cotton swab
10. The operation on the other side is the same as above

(III) Precautions before and after operation

1. For the next three days after the operation, the operation area must not be exposed to water and should remain dry
2. Do not expose to excessive sunlight, sunbathing or SPA within one month before the operation
3. After the operation, maintain a regular lifestyle and pay attention to a balance between work and rest

Removing Lip Tattoo (Removing Tattoo)

(I) Effects

Remove the pigments on the lips and remove the tattoo

(II) Specific Procedures

Preparation before operation: disposable towel, ointment, plastic wrap, repair solution, disposable sterile gloves

1. Wipe the operation area with a wet towel and apply the ointment immediately
2. Cover the ointment with plastic wrap to enhance its effect and wait for 20 minutes
3. Remove the ointment and wipe it clean
4. Select the operation head of 1064nm and install it
5. Adjust the parameters: ENERGY: 700Mj

FREQUENCY: 4Hz

Note:

- (1) Press the foot switch to release energy.
- (2) Press the red button on the back of the machine to control the red dot switch on the handle.
- (3) These parameters are for reference only. You can adjust them from low to high according to your personal tolerance.
6. Provide protective glasses for customers and operators
7. Put on disposable sterile gloves
8. Keep the operation head 3 cm away from the lips, step on the foot pedal to operate, and perform one burst at the lips, 1-2 times
9. Pause the instrument, replace the operation head (532nm), and prepare for operation
10. Keep the operation head 3 cm away from the lips, step on the foot pedal to operate, and perform 1-2 times

11. Apply the repair solution on the operation area

Note: Lip tattoos need to be treated within 6 months after the tattooing.

(III) Precautions before and after operation

1. Three days after the operation, the operation area should not be exposed to water and should be kept dry
2. Do not expose to excessive sun exposure, sunbathing or SPA within one month before the operation
3. Maintain a regular lifestyle and pay attention to balance between work and rest after the operation

Remove moles (remove superficial spots, remove birthmarks)**(I) Effects**

Remove moles, superficial pigmentation spots and birthmarks

(II) Specific procedures

Preparation before operation: disposable towel, ointment, plastic wrap, repair solution, disposable sterile gloves

1. Wipe the operation area clean with a wet towel and apply ointment, wait for 20 minutes
 2. Remove the ointment and clean it up
 3. Select the 532nm operation head and install it
 4. Adjust the parameters: ENERGY: 500Mj
- FREQUENCY: 3Hz

Note:

- (1) Press the foot switch to release energy.
 - (2) Press the red button on the back of the machine to control the red dot switch on the handle.
 - (3) These parameters are for reference only. You can adjust them from low to high according to your personal tolerance.
 5. Provide protective glasses for customers and operators
 6. Put on disposable sterile gloves
 7. Move the operating head to the area where the mole needs to be removed. Align the center red dot with the black mole, then step on the foot pedal to operate. Perform each mole operation 2-5 times
 8. After operating one mole, move on to the next one until all the moles to be removed are completed
 9. Apply the repair liquid to the operated area
- (Notes: The operation methods for removing spots, birthmarks, and moles are the same.)

(III) Precautions before and after the operation

1. The operated area cannot undergo laser, dermabrasion, or acid peeling beauty treatments
2. Do not over-expose to the sun or engage in sunbathing or SPA within one month before the operation
3. Maintain a regular lifestyle and pay attention to balance between work and rest
4. Do not get water on the operated area for three days after the operation, keep it dry
5. Do not take a sauna, go on a hot spring bath, or swim within 7 days
6. Strengthen sun protection, take necessary physical precautions when going out

7. Use Precautions

1. Before using the instrument, a plug with a grounding terminal must be used and ensure that the power socket of the instrument is truly grounded.
2. Ensure that the voltage of the instrument is compatible. If the local power supply voltage is unstable, we recommend that users add a power stabilizing power supply with matching power between the mains and the instrument.
3. Before treatment, please remove all jewelry from your body to avoid unexpected situations that may affect the therapeutic effect.
4. Patients who are currently ill should use this instrument with caution, unless they have received permission from a doctor.
5. Do not keep the probe stationary on the same skin during the operation to avoid skin burns. The probe must be tightly attached to the skin to prevent light leakage that may affect the effect or cause skin burns.
6. Delay the operation 15 days if you have recently sunburned or plan to go on a vacation in the sun.
7. People with dark skin tones such as Africans, Indians, and Native Americans should avoid this procedure as much as possible, even if their skin tone is lighter than that of people of the same race, they will absorb a large amount of laser and may damage the skin.
8. Do not use on eyebrows, eyelashes, or eyes.
9. Each time the energy adjustment is made, start from the lowest energy level to avoid skin burns caused by high energy. The standard for energy adjustment is that when the skin has a noticeable heat sensation, it is the standard for the energy adjustment value.
10. After continuous use for about 2 hours, turn off the machine and rest for 5 to 10 minutes to effectively extend the service life.
11. During operation, the user and the operator should wear protective glasses to protect the eyes from being burned by the laser.

8.Maintenance & Cleaning

1. When not in use, make sure to cut off the power supply and store it in a dry, cool and ventilated place.
2. Remove the medium from the handle surface, clean and disinfect the handle by wiping it with a wet towel or physiological saline, then dry the excess water with a dry towel to keep the handle dry.
3. After cleaning the machine, please cover it with a dust cloth or cling film. Before using it next time, disinfect the handle again.
4. The dust removal inside the machine should be operated by professionals. It is strictly prohibited to disassemble the machine for cleaning by yourself.

9.Common Faults and Their Solutions

Fault phenomenon	Cause Analysis	Solution
The machine is not powered on.	There is no power input, meaning no power is supplied to the instrument. This could be due to: 1) blown fuse 2) the emergency stop function not being released.	1) Replace the fuse 2) Release the emergency stop
808 755 1064nm Diode Laser Handheld Device Cannot Emit Light	1) The ready button on the screen was not pressed. 2) The button on the controller was not pressed.	1) Press the Ready button on the screen 2) Press the button on the controller
The instrument interface displays that the temperature is too high.	1)Insufficient water in the water tank; 2)prolonged continuous use.	1) Fill the water tank with water. 2) Stop using the instrument and rest for 15 minutes.

10.FAQ

1. Can hair removal be permanent with just one treatment?

Answer: No. Hair removal requires a full course of treatment to achieve satisfactory results. When undergoing laser hair removal, it needs to be treated every 3 to 6 weeks (depending on the area), as there are three cycles of hair growth: growth phase, resting phase, and regression phase. Hair removal is only at its best during the growth phase, and the hair growth cycles in different parts of the body vary. A targeted and effective treatment plan is formulated based on each person's hair growth condition, density, and different areas.

2. Does hair removal cause harm to the skin?

Answer: No harm. You can completely rest assured. Its working principle is the selective photothermal principle: that is, the selected wavelength only acts on the melanin in the hair follicle and does not affect the normal tissues of the skin. (That is to say, the light band only absorbs the melanin in the hair follicle, and there is no melanin around the skin, so there will be no harm.)

3. Will the hair become darker after hair removal?

Answer: After laser treatment, the remaining hair will become thinner and lighter in color. It will not become darker as a result of hair removal. While shaving, plucking, using hair removal cream, or waxing hair removal, people who use these methods for a long time will have thicker, harder, and darker hair.

4. Is laser hair removal suitable for everyone?

Answer: Although laser hair removal has a wide range of applications, it is not suitable for all people. Those with the following symptoms should pay attention:

- (1) History of keloid
- (2) Present infection or herpes infection in the treatment area
- (3) Have undergone hair removal with other methods within 4 weeks
- (4) Allergic to hydroquinone-based drugs or other bleaching agents
- (5) Allergic constitution

5. Is there a seasonal difference for hair removal?

Answer: Generally speaking, laser hair removal has no wounds and no recovery period. It can be treated at any time. Theoretically, it is non-seasonal. However, hair removal requires multiple treatments, so it is recommended to start the treatment as soon as possible so that you can freely display your beauty.

6. Can hair removal be done immediately after sunbathing?

Answer: It is not suitable to do sunbathing immediately after hair removal. Because after shaving, the hair follicles and the epidermis will fall off together, and the skin surface protection mechanism will decrease, so it is not advisable to sunbathe immediately.

7. Can the black face doll achieve results after just one treatment?

Answer: The black face doll not only uses the soft skin laser itself, but also integrates radiofrequency, medical skin care, etc. in the package. After one treatment, there will be significant changes in the skin, making it smooth, fair, with obvious improvement in fine lines, pigmentation, pores, etc. The effect can be seen within about a week.

8. How long does it take to see the effect after doing picosecond treatment?

Answer: Picosecond treatment works by directly breaking down the deposited pigment with a rapid and powerful energy, allowing it to be excreted through the skin lymph. It improves pigmented skin by enhancing skin repair and promoting collagen renewal and proliferation. The effect can usually be seen within about a week.

9. Can picosecond skin whitening remove the pigment completely in one treatment?

Answer: Generally, picosecond skin whitening cannot remove the pigment completely in one treatment. Because picosecond breaks down the deposited pigment, making it smaller. After one treatment, there will be a very obvious effect. However, to completely remove the pigment, a course of treatment is required. One course of treatment usually consists of 3 to 4 sessions. Generally, skin whitening requires 1 to 2 courses, and the specific number of courses depends on the individual's constitution.


10. What makes the black face doll different from ordinary skin rejuvenation?

Answer: The black face doll is a soft skin laser beauty method. It uses carbon powder as the medium, which enhances the soft skin laser's overall skin rejuvenation functions, such as shrinking pores, improving dullness, firming the skin, etc. Therefore, the beauty effect of the black face doll is more comprehensive and better.

11. Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name: Multifunctional comprehensive instrument
 Product Model: HR-AS64P
 Production Date: _____
 Quality Inspection: _____

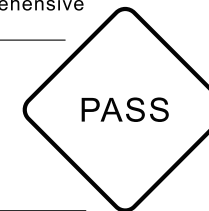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