



Inner Ball Roller

Please read the instruction manual carefully and follow the instructions before use.

Preface

Respected user:


Welcome to choose our company's latest product, the inner ball roller machine, which uses an innovative compression micro-vibration technology. This technology is realized by a 360° intelligent rotating cylindrical roller handle. A roller device composed of 50 silicone balls in the larger handle, used for body care; another roller device composed of 60 silicone balls in a smaller handle is used for face/neck care. Silicone balls are placed in a honeycomb shape with a specific density and diameter. The direction of rotation and pressure ensure that the micro-compression is transmitted to the tissue. The unique handle produces mechanical compression and micro-vibration on the skin, which cannot be replicated by hands or other techniques "Compression micro-vibration therapy" replaces the traditional method of "suction traction massage", using micro-compression instead of suction, and micro-vibration instead of traction. When applied to the body, it helps to improve lymphatic drainage and circulation, deep detoxification, relieve muscle stiffness and soreness, decompose subcutaneous fat deposits, eliminate cellulite, reduce cellulite and shape contours. After a few procedures, your body will immediately reduce its volume by 1-2 sizes! When applied to the face, it helps to improve the formation of blood vessels, thereby supporting the production of collagen and elastin, increasing oxygen delivery, helping to nourish and brighten tissues from the inside, in addition, it can relax and regulate muscles and help reduce expression wrinkles.

Appears, reduces swelling, fights tissue sagging, and rejuvenates the face significantly. Compression micro-vibration therapy is specially designed for non-invasive skin rejuvenation, body shaping and physical therapy. It is the latest generation of body care for the face, neck, shoulders, arms, chest, back, waist, abdomen, buttocks, legs and feet, etc. Compression micro-vibration therapy is a treatment that can reactivate blood circulation, a method that can care for complex diseases such as lymphatic stasis, lymphedema and cellulite, so it can improve skin nutrition and cell oxygenation, compression micro-vibration therapy It is an innovative combination between beauty and health.

Principle introduction


The inner ball roller apparatus is a non-invasive mechanical compression micro-vibration treatment. The principle is that the silicone ball rotates along the roller 360° to generate compression micro-vibration. As the ball rotates and exerts pressure on the skin, it produces a "pulsation compression" effect, realizing continuous push-pull kneading reciprocating motion, and the tissue will experience some pressure. And lifting action, will not squeeze or damage the skin, the tissue is exerted pressure to stretch the cells to naturally and deeply stimulate cell activity, blood flow and oxygenation, fat deposits are pressurized and thus loosened to finally decompose, reducing cellulite and Removes cellulite; also exerts pressure on deep muscle groups to fully soften and stretch, thereby reducing muscle stiffness and soreness, accelerating metabolism,

eliminating stagnation and fluid accumulation, conditioning tissues and re-firming skin tissues, reshaping your body. It can also stimulate fibroblasts, increase the production of collagen and elastin, increase blood flow and increase oxygen. As a result, wrinkles are smoothed, puffiness and eye bags are reduced, and the skin is rejuvenated and tightened. This technology is clinically proven to help tone muscles and sculpt the body and face to rejuvenate, as well as help redesign and tighten the chest.

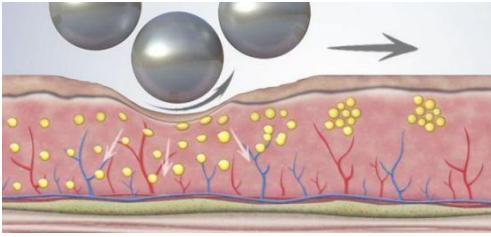


Reshaping action

Reshapes contours and smooths out wrinkles.

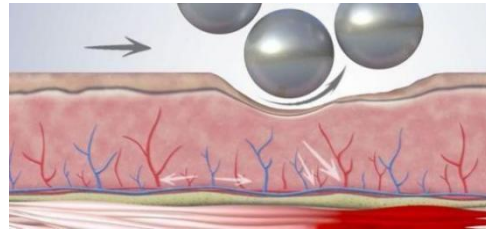


Muscle action

Tones muscles and tightens tissues.

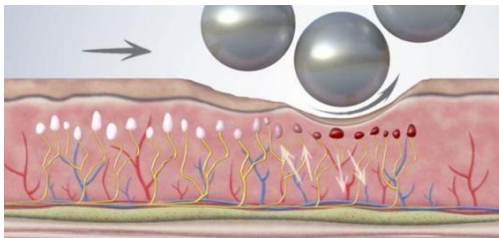


Sedation

Improve oxygen and eliminate pain.

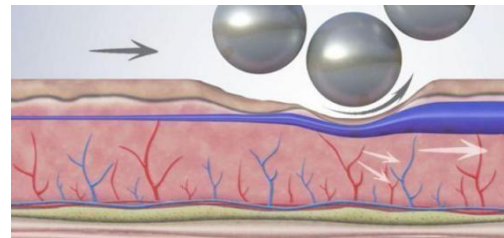

Drain action

Eliminate toxins and help lymphatic drainage.

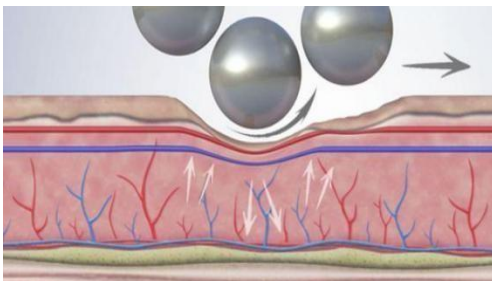

Vascular effect

Improve blood circulation and nourish tissues.

Features

1. Two handles for face & body, two handles can work at the same time.
2. Easier to replace the balls compare to other similar machine, one set of metal balls and two set of medical silicone balls. (Note: on market device it could not replaced).
3. The speed/rotation is 30% faster than the previous version, the speed is adjustable in 10 levels.
4. All handle wheels support rolling in two directions and can save five sets of operating parameters.
5. Each handle has 7 colors of photon lights.
6. The V-shaped handle with the unique two-way rotation principle enables the dual effects of squeezing and stretching the fat cells, better for promoting metabolism and fat reduction.
7. Injection molding case, high-end flight case and carton packaging.

Application part



Instrument instructions

A. Installation: Align the handle with the handle socket on the back of the device and place it on the rack; insert the power cord into the socket on the back of the device, turn on the power and turn on the power switch, and then press the power button to start the device.




B. Small handle: use for face, neck, shoulder(can control handle on the handle)



C. Large handle: use for the body(can control handle on the handle)



D.The replacement and installation of the roller built into the handle is shown in the figure below:

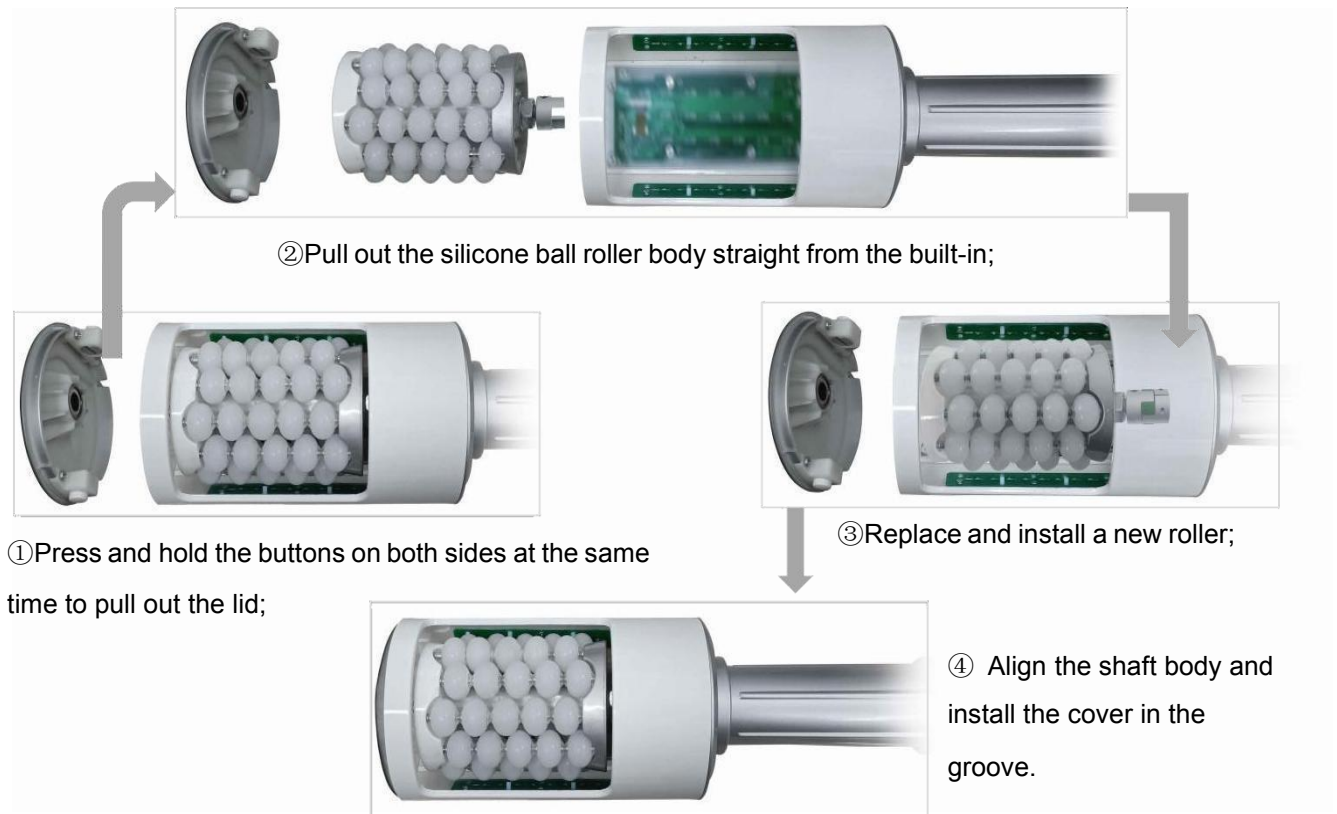

D . The 3rd latest with 360 Roller handles.



Function

1. Dissolve fat, tighten skin, body shaping
2. Tighten skin, increase skin elasticity, improve wrinkles
3. Stretch stiff muscle, relieve pain
4. Stimulate lymphatic blood vessels, remove unwanted fat and toxin from body
5. Activate collagen tissue, reduce cellulite

Advantage with 360 Roller handles


1. Unique patented roll design.
2. Shock-free, less labor consuming .
3. Automatic rolling balls, super comfortable.
4. Rolling massage and heat conduct.
5. Three output channels cover all body parts.

Before & After





Interface introduction

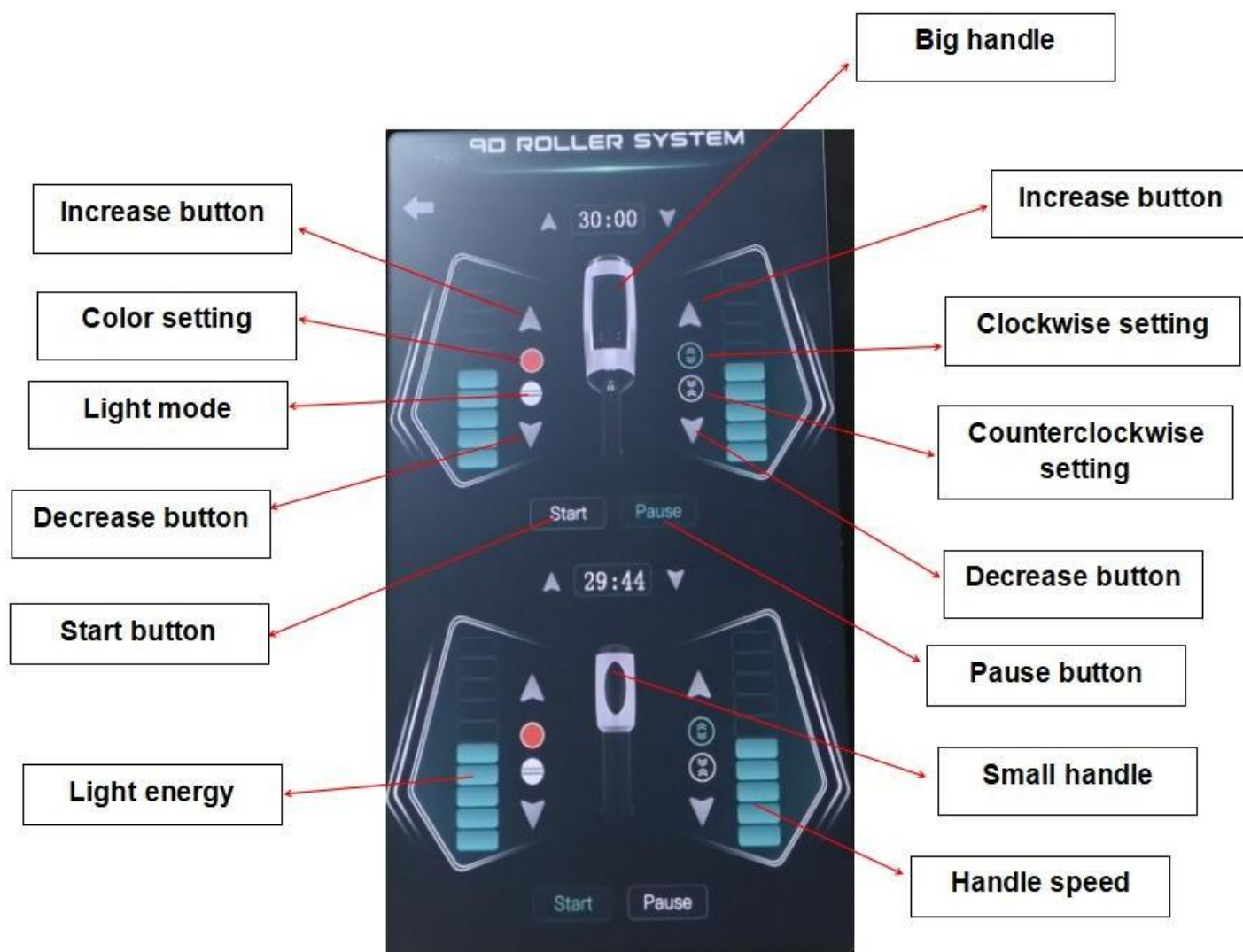A. Check whether the connection of the instrument is normal. Connect the instrument to the power supply and turn on the power switch. Press the power button and the machine will start up immediately, and the startup interface will appear.






introduction of specialized working interface



【】 Large handle; 【】 Small handle;

【】 Rotation Direction: Clockwise;

【】 Rotation Direction: Reverse;

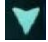【60:00】 Operating Time: 60 Minutes,

【】 Speed Gear:1-10 gears.Click  to increase the speed, Click  to decrease the speed.

Click 【】 Start; click 【】 Pause;

【】 light choose

【】 light mode

【】 Return button

Operation for the 3rd handle ,you can just unplug the small roller handle from machine and connect the The 3rd handle back to the machine ,and then operate it via small handle interface .

Common Problem

What is the Inner ball roller machine?

The inner ball roller instrument is a cutting-edge, globally recognized body shaping instrument. It uses innovative compression micro-vibration to improve lymphatic drainage, increase blood circulation, improve cellulite, reduce cellulite, reverse signs of aging, muscle conditioning and detoxification treatment. It can be used on face and body. The most popular areas for treatment are the thighs, buttocks and upper arms.

Is compression micro-vibration therapy safe?

Compression micro-vibration therapy is a non-invasive treatment. It is 100% safe and does not have any side effects.

How long is a single treatment?

It is suitable for any part of the body or face, but depending on the size of the area to be treated, the single time will vary from a minimum of about 45 minutes to a maximum of 1 hour and 30 minutes.

Will you feel pain during the treatment?

No, it is actually a very pleasant treatment. Most customers say that it feels similar to a deep tissue massage. The intensity/stress level gradually increases with each treatment and can be adjusted to your desired tolerance. There are no side effects and you can return to normal life immediately after treatment.

How often can the treatment be performed?

It is usually recommended to do it twice or three times a week. However, the minimum time required between treatments is 48 hours.

Product parameter

Product name	9D Inner ball roller slimming machine
Touch screen	10.1 inch large LCD
Output frequency	40-254Hz
Technology	Innovative compression micro-vibration technology
Large handle	50 silicone balls (for body care)
Speed of large handle	18000rpm
Small handle	60 silicone balls (for face/neck care)
Speed of small handle	18000rpm
Input voltage	AC110V~130V; 50HZ/60HZ
Output power	100-1000W
Fuse	5A
Ambient temperature	5°C~+40°C