

6. Accessories


Name	Qty
Power line	1 pc
Skin Scrubber	1 pc
RF handle	1 pc
Ultrasonic probe	1 pc
Cold hammer	1pc
Hydro facial probe	1 pc
Hydro facial probe head	7 pcs
Spray gun	1 pc
Mask	1 pc

H2 O2 hydra facial machine User manual



Dear customer, thanks for buying our product!
For the best capability and safety, please read this instruction manual before any operation.
Please reserve the instruction manual for future references.

1. Brief introduction:

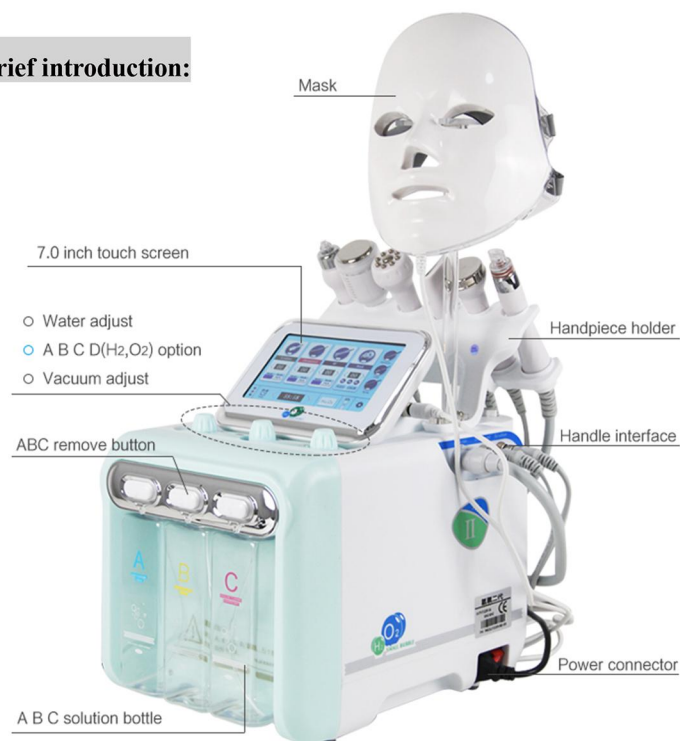

Aqua facial
handle



Ultrasound handle



Skin scrubber



RF handle



Spray gun



Cold hammer



Mask

4.7.2 Note


- 1) Don't watch the LED light source directly in long time.
- 2) If use disposable mask or cosmetics together with the LED mask, please choose high quality natural ingredients products.
- 3) The mask material is medical silicon, which will have some smell right after open the package, but smell will disappear very soon.
- 4) There is LED light sources and PCB board inside of the silicon mask. NO fold the mask, NO squeeze, No heavy press.
- 5) Use the LED mask on bed, after you finish treatment put the led mask back on the box, no left it on bed.
- 6) Don't use water to clean the LED mask, no dip into water. Please use wet cloth to clean LED mask. The soften mask can be cleaned by water.
- 7) Don't use any chemical liquid to clean the LED mask, and the soften mask.
- 8) Don't expose to sun light directly.

5. Maintenance service


- 5.1 The instrument is under warranty for one year except the accessories and the consumables. We would provide maintain service after the warranty period, but payable.
- 5.2 If you want to replenish or purchase the consumables and accessories, please contact the distributor for advice.
- 5.3 Breakdown cause by human as below is out of our warranty.
 - a) Breakdown cause by dismounting or transforming the instrument by customer himself/herself.
 - b) Breakdown cause by incorrect or unreasonable maintenance.
 - c) Breakdown cause by beating or dropping the instrument.
 - d) Breakdown cause by incorrect operation.
 - e) Damage cause by irresistible disaster like earthquake, fire disaster.
- 5.4 Welcome for any comment and suggestion.
- 5.5 When cleaning machines, please use dry soft towel to swipe the machine for the dust, and use wet soft towel to swipe the machine for the dirty.

to the texture of the muscles.

- 5) The whole working time is 15 minutes.

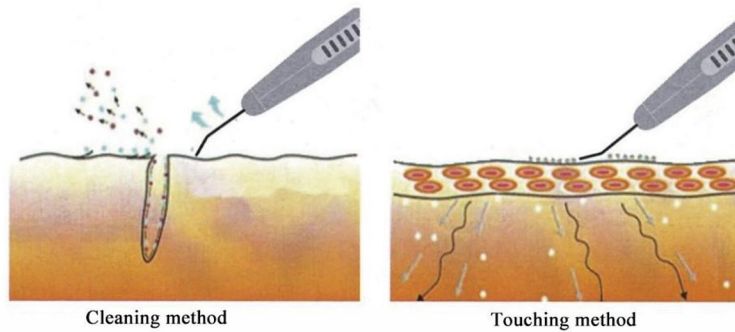

4.7 Using for Mask

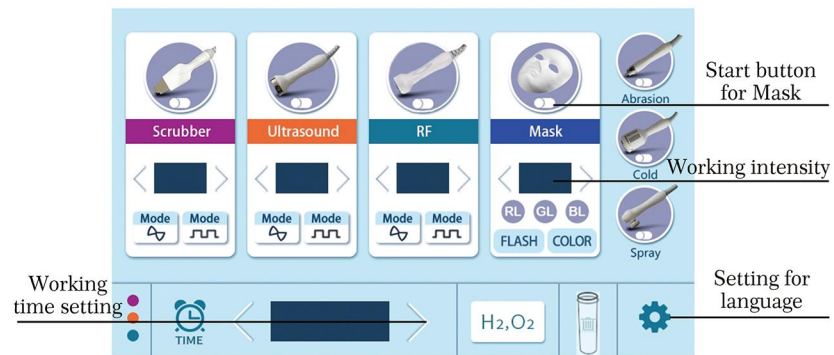

- 4.7.1 Solves the whitening, skin aging problem fundamentally, Let increased activity of cells; promote cell metabolism, make the skin secretion of collagen and fibrous tissue to fill itself. Accelerate blood reflux, increase skin elasticity and improve skin complexion, dull conditions, so as to achieve whitening, skin rejuvenation, wrinkle, repair, etc. Blemish effect. Instruments were based on the original technology upgrade, using seven different colors cold light, namely red, blue, green, yellow, purple, blue, laser light, each light has a different effect, you can handle different skin problems, the effectiveness of a variety of instruments, practical and affordable.

2. Caution

- 1) Please turn off the power if needs to change the fuse.
- 2) Please pull out the power cable if needs to move the machine.
- 3) The instrument should be placed on the temperature between 5-40 degree, and placed in clean indoor, also avoid direct sunlight. And make sure the vents are not blocked. Do not put anything on the instrument at any situation even temporary.
- 4) Install the instruments with probes with the picture below and install the handles on the accessory holder and put in the right places.
- 5) Do not use any irritating skin care products, cosmetic within 24 hours after treatment and pay attention to the sun light.
- 6) Using moisturizing mask for a week after treatment.
- 7) Those do not use the machine:
 - a) Pregnant women
 - b) Cardiovascular disease, kidney disease, cancer, diabetes
 - c) Bacterial infection site
 - d) Keloid patients
 - e) Whom with heart pacemaker
 - f) Whom with extreme sensitivity of skin diseases
 - g) Whom with Erythema
 - h) Whom with thin skin cuticle cannot use hydro facial handle.
 - i) People with thin skin cuticle cannot use RF for more than 20 minutes.
 - j) Cannot use the machine with severe allergy or severe acne.
- 10) If the patient feels uncomfortable, please turn off the machine in case of dangerous.

3.How to install the instrument


3.1 Description for the Back of the instrument



- 3.2 When moving the instrument, please remove all the liquid on the Bottle A,B,C,D, otherwise, liquid will enter the inside of the instrument during moving, causing damage to the instrument.
- 3.3 When using the instrument, if the instrument sounds beep beep, please observing the interface for the wasted bottle. If it is twinkling, please pour out the wasting water and install the wasted bottle to the instrument after done pouring out.


- 4.5.4 Using the ultrasonic handle stay close on the treatment area, and press the start button on the screen to start the ultrasonic function.
- 4.5.5 According to the contour line and do circling for the treatment area at about 5-8 minutes.

4.6 Using for Skin scrubber

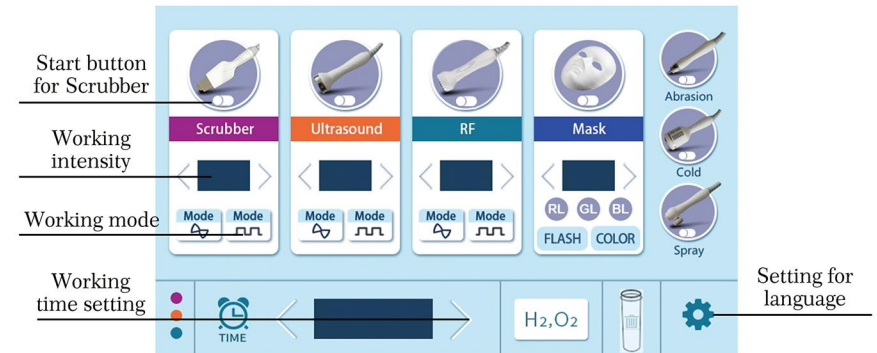


- 4.6.1 Cleaning the treatment area before using the treatment.
- 4.6.2 Adjusting the working time, intensity and mode before using.
- 4.6.3 Using the cleaning method to remove the dirt from the skin surface.
- 1) Using the facial cleanser to clean the skin.
 - 2) Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
 - 3) Applying enough water on the skin.
 - 4) The blade is tilted at 45 degrees, make sure it is close to the skin, moving up and down at moderate speed.
 - 5) The whole working time is 20 minutes.
- 4.6.4 Using the touching method to do lymphatic massage.
- 1) Using the facial cleanser to clean the skin.
 - 2) Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
 - 3) Applying enough skin care products on the skin.
 - 4) Make the skin scrubber head flating with the skin and moving it according

4.4 Using for RF

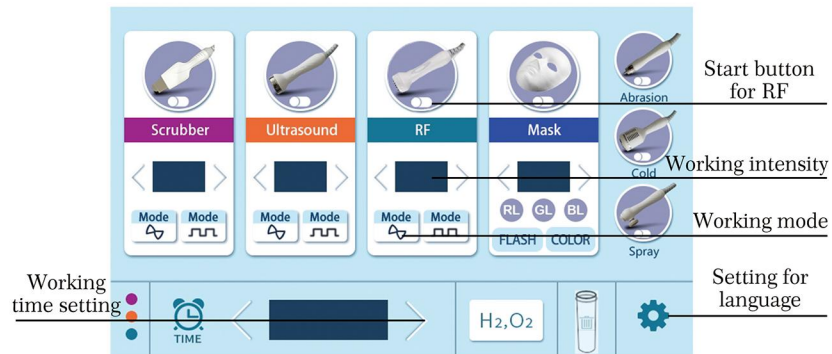

- 4.4.1 Cleaning the treatment area before using the instrument.
- 4.4.2 Apply suitable gel on the treatment area.
- 4.4.3 Adjusting the working time, RF intensity and RF mode before using.
- 4.4.4 Using the RF handle stay close on the treatment area, and press the start button on the screen to start the RF function.
- 4.4.5 According to the facial line and neck, do circling from bottom to top to realize detox. And operation time for each part should be around 10-15 minutes.



4.5 Using for Ultrasonic

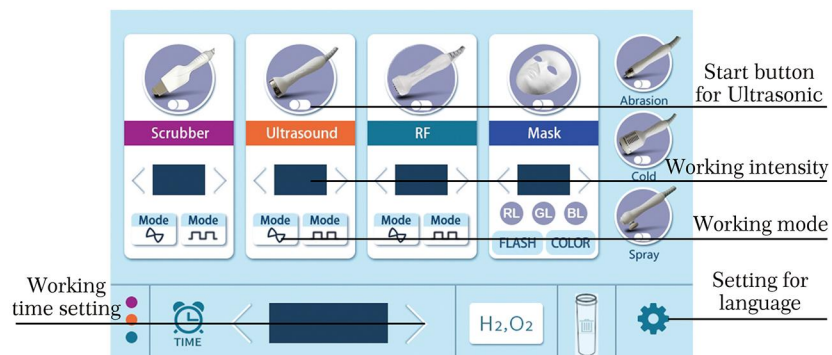

- 4.5.1 Cleaning the treatment area before using the treatment.
- 4.5.2 Apply suitable gel on the treatment area.
- 4.5.3 Adjusting the working time, ultrasonic intensity and ultrasonic mode before using.

4. Operation procedures and methods

4.1 Using for Aqua facial handle

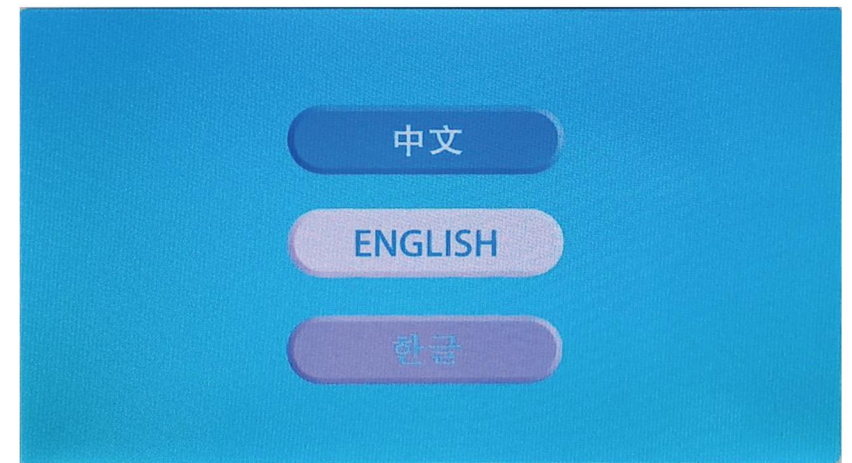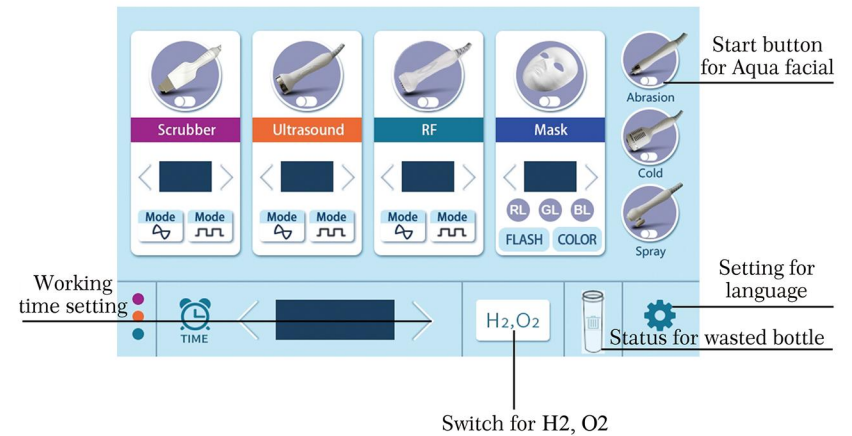


- 4.1.1 Cleaning the face by using facial cleanser.
- 4.1.2 Choosing the aqua facial cleaning head according to the situation for the skin. Choosing the liquid (A,B,C) by rotating the knob or the H2, O2.
- 4.1.3 Adjusting the flow level and pressure lever to start working, the whole cleaning procedure can be working at about 10 minutes.

Caution:

When using the H₂O₂ from the bottle D which on the back of the machine, first select the knob to the D position, and press the Button (H₂O₂), then please waiting for about 3 minutes, to make sure there are enough (H₂O₂) inside the bottle.


- 4.1.4 When done using the instrument, select the knob to position C to cleaning the aqua facial tube. First, selecting the flow level to the maximum intensity, and then selecting the pressure level to the maximum. Then turn on the machine, putting on the aqua facial head to the aqua facial probe. Then the tubes inside the machine will be cleaned automatically.

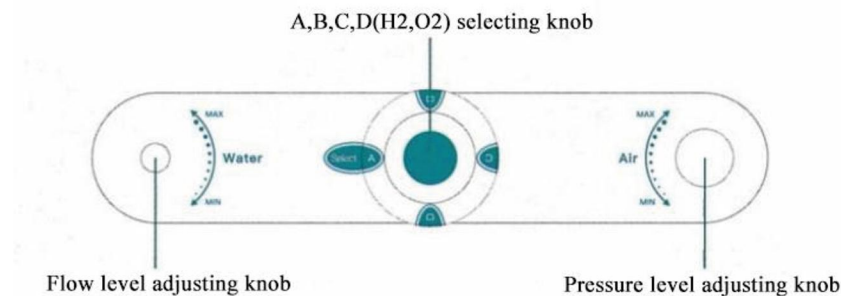

Caution:


When the liquids on the bottles are been using out, please input warm water to bottle A,B,C,D and cleaning the tubes by the methods for 4.1.4, to avoid blocking for the tubes.

4.2 Using for Spray handle

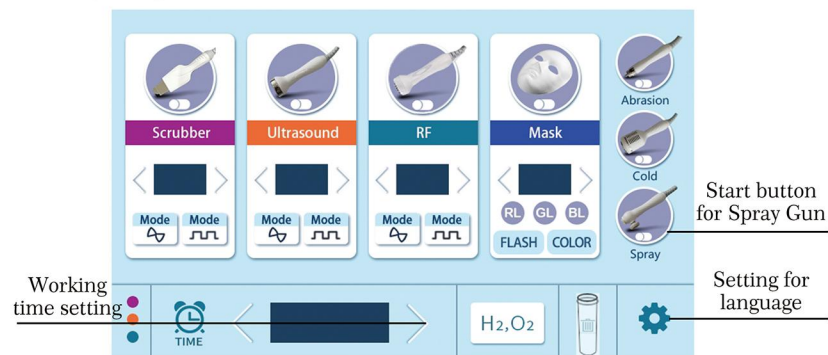

- 4.2.1 Cleaning the treatment area before using the handle.
 4.2.2 Input the dilution essence to the spray bottle.
 4.2.3 Press the start button for the spray gun on the screen.
 4.2.4 Press the button on the spray gun then the gun will start working and aim at the treatment area.

4.3 Using for Cold hammer

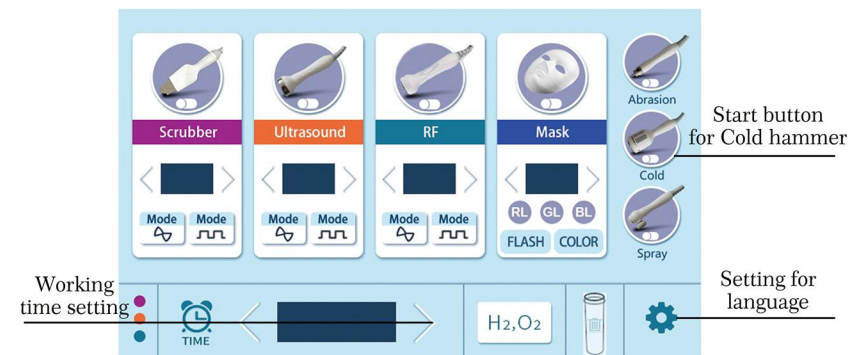

- 4.3.1 Cleaning the treatment area before using the cold hammer.
 4.3.2 Press the start button on the screen.
 4.3.3 Using the cold hammer on the treatment area.

Treatment on the face:

Start from ophryon and spread away, move upwards. As the photo shows.

1. From lower jaw to ear.
 2. From lower jaw to nose.
 3. From nose to Temple.
 4. From inner corner to the temple,
- do not get too close to the eyes and be tender.

