
LED User's Manual

CL-81K1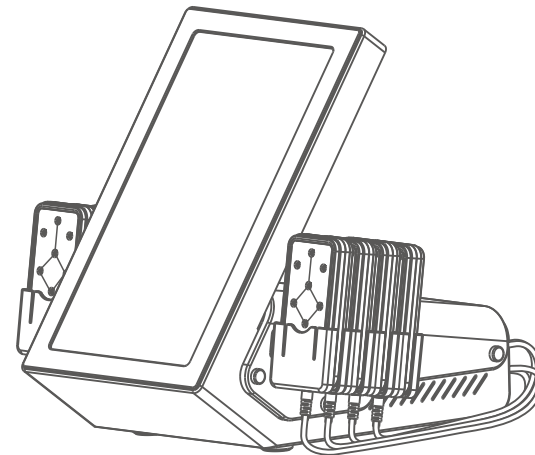

Foreword

Dear User:

Welcome to use our LED body shaping device, it is an efficacious LED beauty device for body shaping.

It should be operated by the professional, as the improper operation may cause adverse results to the human body.

Please read this manual carefully before operation, and follow the instructions in the manual during operation.

We believe our high-quality product will bring you excellent returns, and our perfect after-sales service will free your worries.

Thank you!

Content EN

1. Introduction	1
2. Precautions	1
3. Contraindications	2
4. Technical parameters	3
5. Packing list	3
6. User guide	4
7. Interface Introduction	7
8. Troubleshooting	9
9. FAQ	9
10. Treatments Introduction	10
Waist and Abdomen Sculpting	10
Arms Sculpting	12
Legs Sculpting	13
Back Sculpting	15
Buttocks Sculpting	16
11. Recommended frequency	18
12. After treatment	18
13. Warranty and support	18

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1. Introduction

LED body shaping device is the most popular beauty device for body shaping, with easy, convenient and fast operation, providing body shaping solution for you. No needles, no medicines, no surgeries, no side effects. With strong penetration, LED can penetrate epidermis and dermis to dissolve the subcutaneous fat by heat, the treatment is safe and non-invasive, which is popular among consumers.

2. Precautions

1. Strictly follow the instructions in the manual to use the instrument.
2. Before using the instrument, you must use a plug with a grounding pin and ensure that the power socket of the instrument is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we advise you to add a power-matched regulated power supply between the mains and the instrument.
4. Please use the designed accessories provided or advised by original manufacturer to sustain normal service life of the instrument and treatment effect.
5. Do not place the instrument in a wet place or near water, nor expose the instrument to direct sunlight.
6. Do not place the instrument near strong heat, as this may affect its service life and performance.
7. Remove off all the metal objects from your body to avoid unpredictable situation and affect the treatment.
8. Do not use the instrument on the eyes, thyroid, parathyroid, testicles, abdomen of pregnant women and pacemaker.
9. The instrument is not intended for use by patients, unless they have the permission of a doctor.

10. Please turn off the power of the instrument when not in use, and make sure to turn off the general power of the instrument before leaving after daily use to ensure the safety of instrument.

11. In case that you are taking other weight loss drugs, it is recommended to stop the drugs for 1 to 2 months before treatment. If you want to take the LED treatment immediately, the course of treatment should be extended.

12. Do not use the instrument on empty stomach, and take the LED treatment 1 hour later after the meal.

13. Fix the LED pads on clients first, then switch on the instrument, slowly adjust the intensity from low to high to ensure warmth and comfort of the client.

14. Make sure the light-emitting surface of LED pad is in full contact with the skin during procedure, and avoid partly contact.

15. Clean the instrument with saline after operation to ensure the hygiene.

3. Contraindications


1. People who had plastic surgery, with implants or metal objects at the treatment area in 3 months.
2. People who had injections in 3 months (hyaluronic acid, mesotherapy and botulinum).
3. People suffering from heart disease, epilepsy, severe diabetes, hyperthyroidism, malignant tumor, hemophilia, severe bleeding, or wearing pacemaker.
4. People who are experiencing skin irritation, with severe sensitive skin.
5. People with wound on skin.
6. Excessive aging group.
7. People in period of pregnancy, menstruation, lactation.
8. People with skin diseases and infectious diseases.
9. People who have unrealistic illusions about results.

4. Technical parameters


Product name	Magic Light Body Shaping
Product model	CL-81K1
Main unit material	Metal plate
Handle material	ABS
Input voltage	110V/220V 50~60Hz
Rated power	25W

Screen size	15.6 inches
Product dimensions	400*274*390mm
Packing dimensions	475*410*470mm
Product weight	8kg
Packing weight	10kg


5. Packing list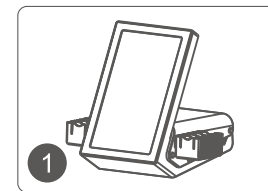


Main unit * 1

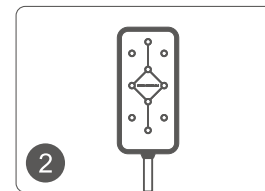LED pad * 8

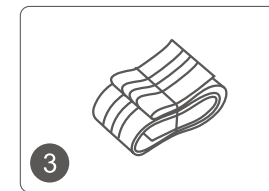Strap * 1 set


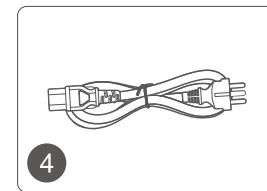
Power cord * 1


Note: Please check the instrument and its accessories or attachments before use, and remove all packaging materials after confirming that they are complete.

6. User guide

1. How to install.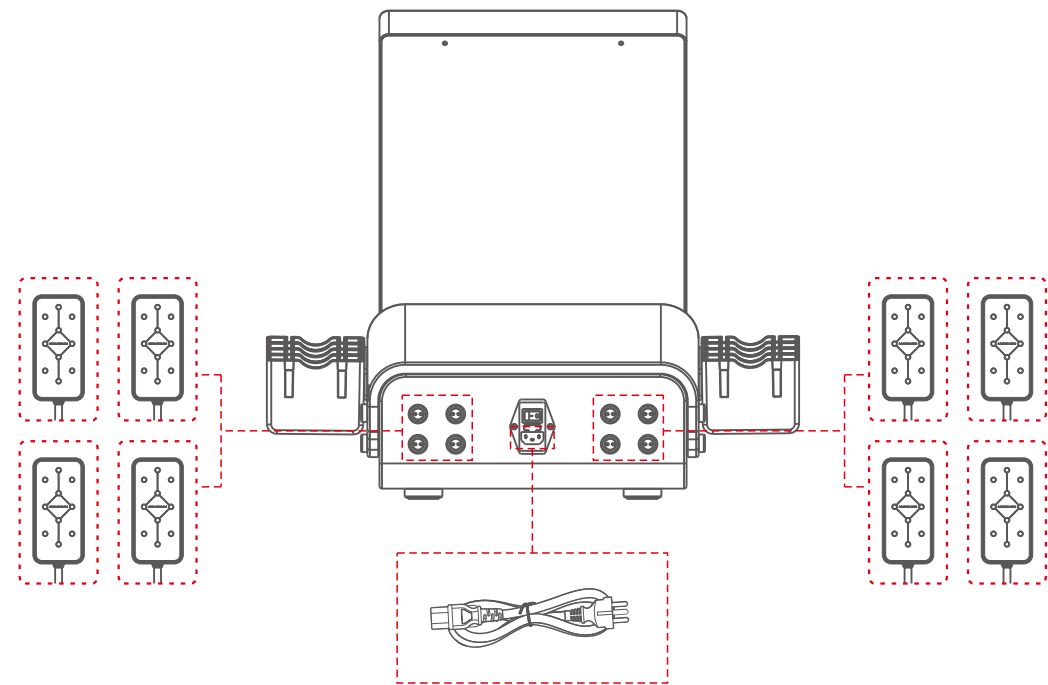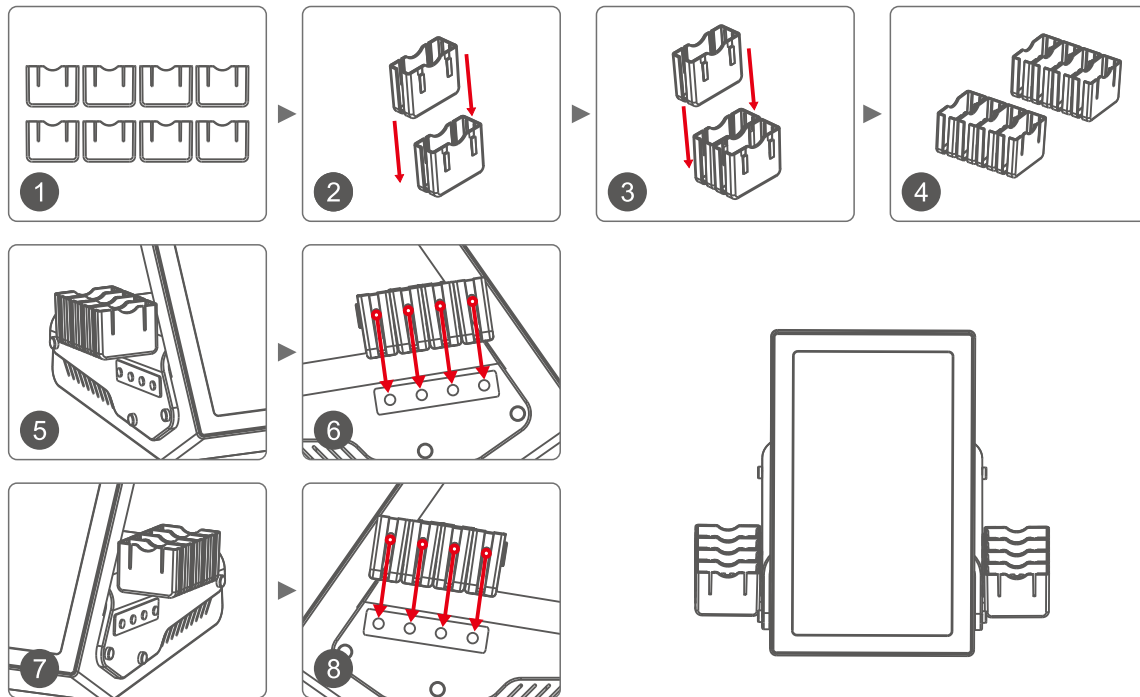

2. Please replace the fuse at the back of the machine when the machine is short-circuited or cannot be started.

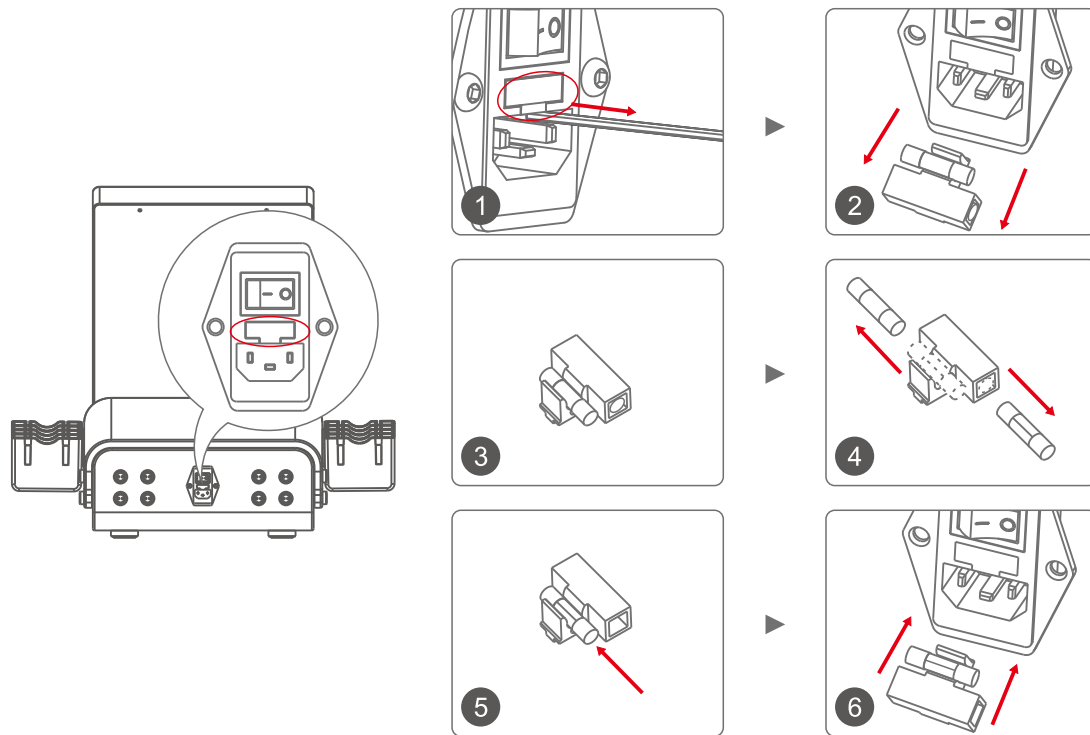
7. Interface Introduction


Welcome interface

Parameter setting interface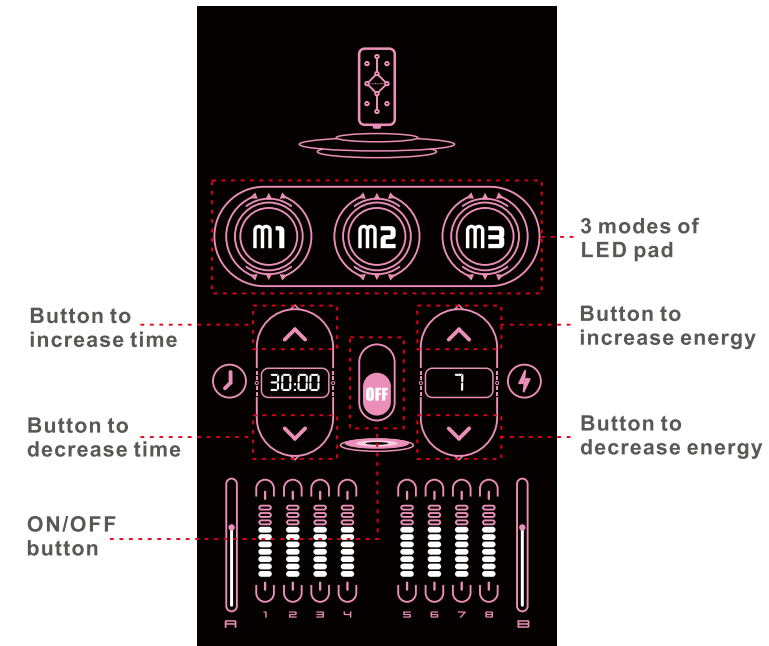

Parameter setting interface

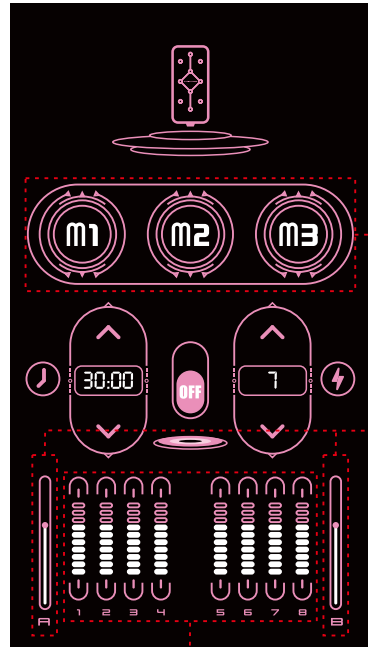**3 modes of LED pad**

M1: Continuous working mode


(lights on pad are always on, used for people who need strong weight loss)

M2: Intermittent working mode


(lights on pad flash slowly, used for result maintenance)

M3: Fast intermittent working mode

(lights on pad flash fast, used for first-time user or the beginning of treatment)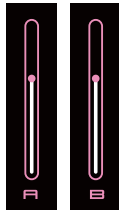

(1) Tap to set energy of all the LED pads uniformly.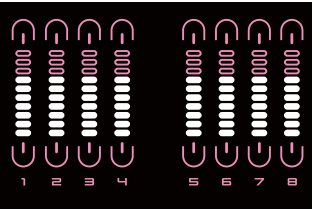

(2) Set energy of single LED pad, tap single LED pad, then tap energy set & display button to set.

8. Troubleshooting

Problem	Possible causes and solutions
1. Instrument fails to turn on and the button indicator is off?	→ A. Make sure the power cable is connected to a valid wall socket. → B. Check if the fuse is loose or burned.
2. No energy output from the LED pad?	→ A. Check if the pad cable is well connected with the main unit. → B. If the above method doesn't work, please contact with the dealer for assistance.
3. The instrument succeeded in starting up, but an error message appeared on the screen?	→ A. Please unplug the power cord, and wait for 1 minute before plugging in the power cord again and restart the instrument. → B. If the above method doesn't work, please contact with the dealer for assistance.

9. FAQ

1. LED body shaping treatment or liposuction surgery?

A: Compared to traditional liposuction, LED body shaping is the irradiation of fat by LED energy, without causing unevenness in the skin, and conversely, leaving the skin smooth and tight without sagging. In addition, the LED energy used in LED body shaping only works on the set fat layer, not hurting the surface skin while accelerating the self-repair of skin elastic fibers, leaving the skin of the treatment area tight and smooth.

2. Will I regain weight after LED body shaping treatment?

A: The number of fat cells in the body is fixed and remains constant as the age grows to adulthood. The increase in body weight is due to the increase in the volume of fat cells. LED body shaping treatment removes the local fat structure and reduces the volume of fat. The fact that there is little rebound case after LED body shaping treatment does not mean that there will be absolutely no rebound. Usually, it will not rebound if you keep a healthy diet.

3. Do I need to keep diet and take exercise during LED body shaping treatment?

A: The effect of LED body shaping treatment is remarkable, but we cannot only rely on it, if you don't keep a healthy diet after achieving idea weight after treatment, you are likely to regain the lost weight. It is important to know that there is no weight loss product in the world guarantees permanent effect. So, this treatment is an auxiliary weight loss method, and you need to stop the treatment after achieving the ideal weight. Please keep a healthy diet and take regular exercise.

10. Treatment & Application

Waist and Abdomen Sculpting

1. Efficacy


- (1) Improve cold limbs and irregular menstruation of females.
- (2) Improve the sagging skin and cellulites on waist and abdomen.
- (3) Tighten skin, improve stretches, increase skin elasticity.
- (4) Accelerate the metabolism and promote blood circulation.

2. Indications

- (1) People with cold limbs and irregular menstruation.
- (2) People with waist fat, belly fat, and sagging skin after delivery.
- (3) People with unattractive waist line due to a sedentary lifestyle.
- (4) People with stretch marks.

3. Recommended Application


Preparation: hot towel, essential oil, strap.

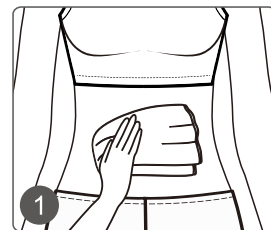
1. Clean the abdomen and waist (better with a hot towel).


2. Massage and relax abdomen and waist with the massage oil. (15 minutes)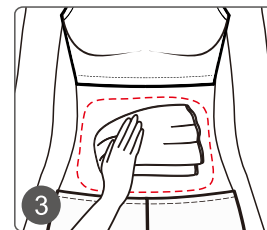


3. Clean the massage area.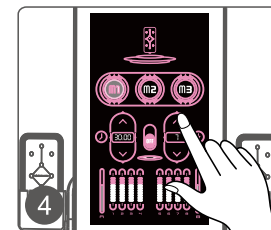


4. Set parameters on the main unit.
(1) Choose M1 mode.
(2) Set Time to 30 minutes.
(3) Set energy to 3-7.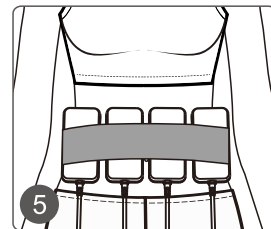

5. Fix the LED pads on waist and abdomen with the strap.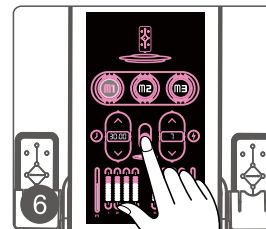

6. Tap ON/OFF button to start (30 minutes).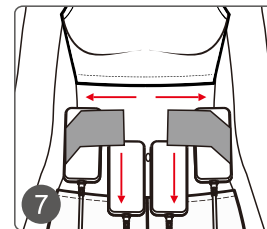

7. When the time is up, remove the LED pads, untie the strap.

Arms Sculpting

1. Efficacy

- (1) Stimulate the bottom layer of skin to regenerate collagen.
- (2) Improve the sagging skin on arms.
- (3) Slim the fat arms.
- (4) Accelerate blood circulation in arms, promote metabolism.

2. Indications


- (1) People with fat arms.
- (2) People with sagging and flabby skin on arms.
- (3) People with frequent soreness and numbness in the arms.

3. Recommended Application


Preparation: hot towel, essential oil, strap.


1. Clean the arms
(better with a hot towel).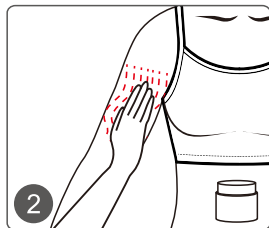


2. Massage and relax arms
with the massage oil.
(15 minutes)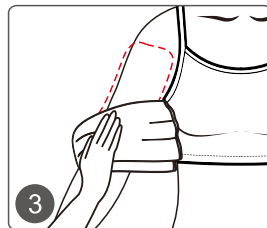


3. Clean the massage area.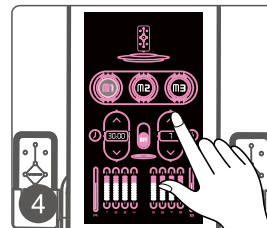


4. Set parameters on the main unit.
(1) Choose M1 mode.
(2) Set Time to 30 minutes.
(3) Set energy to 3-7.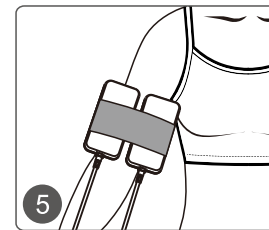

5. Fix the LED pads on arms
with the strap.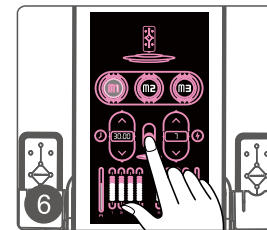

6. Tap ON/OFF button to start
(30 minutes).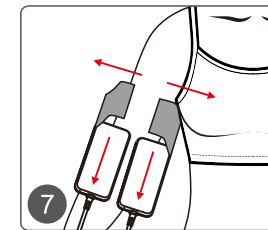

7. When the time is up, remove
the LED pads, untie the strap.

Legs Sculpting

1. Efficacy


- (1) Tighten skin, prevent skin sagging.
- (2) Stimulate collagen regeneration and smooth stretch marks.
- (3) Accelerate blood circulation and metabolism.
- (4) Prevents varicose veins.
- (5) Tighten excess leg fat and get rid of fat legs.

2. Indications

- (1) People with poor blood circulation in legs and metabolism.
- (2) People with rough and sagging skin on legs.
- (3) People with cellulite on legs or with bad leg proportions.

3. Recommended Application


Preparation: hot towel, essential oil, strap.

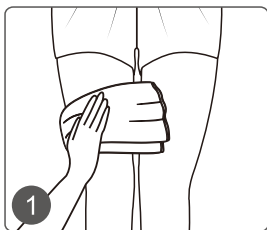
1. Clean the legs (better with a hot towel).


2. Massage and relax legs with the massage oil. (15 minutes)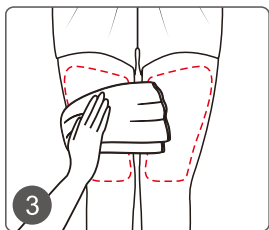


3. Clean the massage area.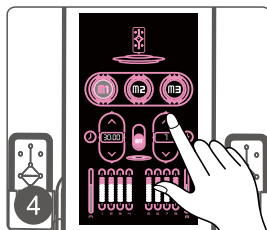


4. Set parameters on the main unit.
(1) Choose M1 mode.
(2) Set Time to 30 minutes.
(3) Set energy to 3-7.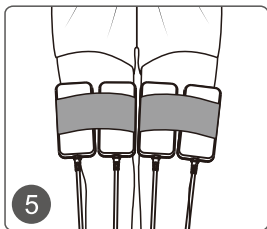

5. Fix the LED pads on legs with the strap.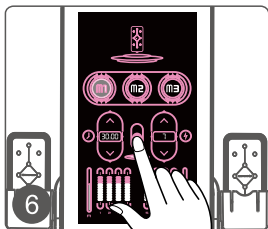

6. Tap ON/OFF button to start (30 minutes).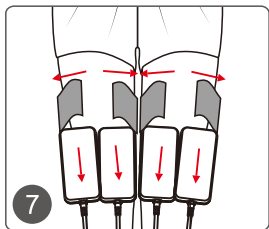

7. When the time is up, remove the LED pads, untie the strap.

Back Sculpting

1. Efficacy

- (1) Relieve back and shoulder tension.
- (2) Accelerate blood circulation in back, promote metabolism.
- (3) Improve blood supply to the head and improve sleep.
- (4) Improve fat deposits on back and clear back contour.

2. Indications


- (1) People suffering from shoulders and back soreness and neck pain.
- (2) People with insomnia and dreaminess, hypomnesia.
- (3) People with continue fatigue and sleepiness.
- (4) People with fat back.

3. Recommended Application


Preparation: hot towel, essential oil, strap.


1. Clean the back (better with a hot towel).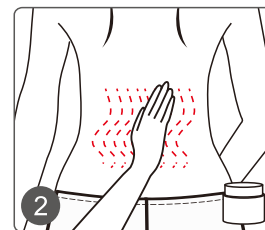


2. Massage and relax back with the massage oil. (15 minutes)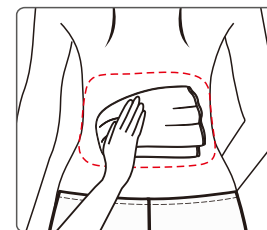


3. Clean the massage area.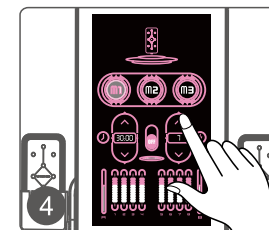


4. Set parameters on the main unit.
(1) Choose M1 mode.
(2) Set Time to 30 minutes.
(3) Set energy to 3-7.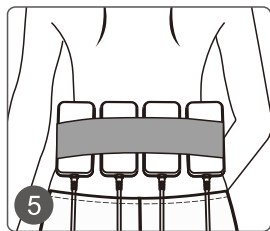

5. Fix the LED pads on back with the strap.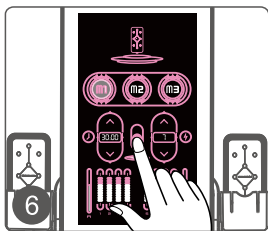

6. Tap ON/OFF button to start (30 minutes).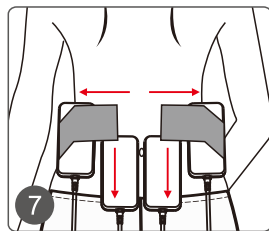

7. When the time is up, remove the LED pads, untie the strap.

Buttocks Sculpting

1. Efficacy


- (1) Improve blood circulation in buttocks, promote metabolism.
- (2) Improve painful menstruation, irregular menstruation, abnormal leucorrhea and other female disorders.
- (3) Improves sleep and female sexual function.
- (4) Activate ovarian function, stimulate glandular secretion, and increase conjugal affection.
- (5) Improve sagging butt, tighten skin and increase skin elasticity.

2. Indications

- (1) People with flabby buttocks or fat deposits on the buttocks.
- (2) People with unattractive buttocks shape or with flat buttocks.
- (3) People with cold buttocks.
- (4) People with dysmenorrhea, irregular menstruation, gynecological inflammation and other disorders.

3. Recommended Application


Preparation: hot towel, essential oil, strap.

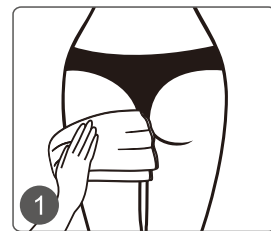
1. Clean the buttocks (better with a hot towel).


2. Massage and relax buttocks with the massage oil. (15 minutes)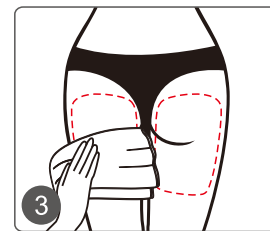


3. Clean the massage area.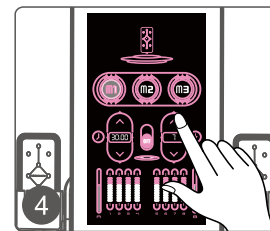


4. Set parameters on the main unit.
(1) Choose M1 mode.
(2) Set Time to 30 minutes.
(3) Set energy to 3-7.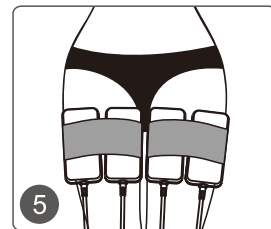

5. Fix the LED pads on buttocks with the strap.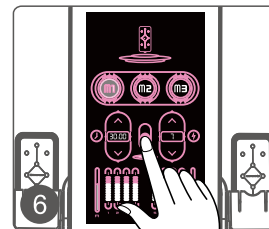

6. Tap ON/OFF button to start (30 minutes).

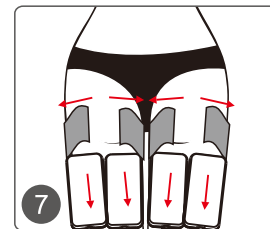7. When the time is up, remove the LED pads, untie the strap.

11. Recommended treatment frequency

A course of treatment is made up of 12 sessions, take 2 sessions a week, and 30 minutes for each treatment area. You can notice a visible effect after 1 - 2 courses, and 3 - 4 courses will achieve the best result. After 3 - 4 courses, it is recommended to take 1 course every 15 - 30 days.

12. After treatment

1. Keep warm, avoid cold food, or catch a cold after treatment.
2. Take a shower 4 - 6 hours later after treatment.
3. Drink more warm water to keep yourself hydrated.
4. Do not overeat or stay up late.
5. Avoid sauna, hot springs or strenuous exercise for 7 days after treatment.

13. Warranty and support

A one-year warranty is provided from the date of purchase.

Product Name: Magic light Body Shaping
 Product Model: CL-81K1
 Production Date: _____
 Quality Inspection: _____

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