

# User Manual

LS-78D3



# Preface

Dear Users,

We're pleased to present to you our latest 8-in-1 beauty machine, which incorporates scrubber, RF, ultrasound, hot&cold, photon&microcurrent, BIO and ion-based infusion for skin management. It is a professional beauty machine, which needs to be handled by professionals with proper training. Any improper use will result in adverse outcomes. Therefore, we advise any personnel to read this manual thoroughly and follow the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

# Table of Contents

Preface	
Part I	
Introduction	
Advantages	
Working Principles	
Facial Skin Firming&Anti-ageing	
Facial Thorough Cleaning	
Facial Soothing&Repairing	
Facial Skin Whitening	
Wrinkle Removal for Eye Area	
Neck care	
Part II	
1. Detailed Operations	
2. Technical Parameters	
3. Dos and Don'ts	
4. Troubleshooting & Solutions	
5. FAQs	
6. Packing List	
7. Simplified Treatment Steps	
8. Operational Diagrams	

# Part I

## Introduction

Scrubber, RF, ultrasound, hot&cold, photon&microcurrent, BIO and ion-based infusion are currently the most popular beauty approaches that focus on skin management. So, this machine which groups those as one can be an effective replacement of all facial treatments in beauty salons. Moreover, it is convenient and easy to operate, which can relieve skin problems for beauty-seekers. This machine requires no injection, no medication, and no operation, for which it has no side effects. It is used externally during the whole process and has an immediate effect. The scrubber uses high-frequency vibration and cavitation to cleanse skin thoroughly to soften and remove corneum, unblock pores, minimize pore size and smooth skin. Ultrasound handpiece uses high-frequency vibration to get rid of metabolites of skin and help with skin's absorption of nutrients. The ion-based nutrient introduction can promote the penetration of nutrients into skin to help improve skin's texture. RF heats deeper skin layers, stimulates collagen to regenerate and reorganize in dermis, and promotes blood circulation at skin bottom, which can achieve the results of skin firming and anti-aging. The BIO function, combined with photon therapy, can renew cells, promote metabolism, and relieve various skin conditions.

## Advantages

1. This 8-in-1 multifunctional beauty machine provides comprehensive facial treatments for people.
2. Ion-based introduction can not only remove unwanted substances to clean skin but also enable the introduction of nutrients into deep skin through the repulsion and attraction of anions and cations
3. RF can heat up dermis and then promote skin metabolism and collagen regeneration, thus delivering the results of skin lifting, firming and rejuvenating, wrinkle removal, and anti-ageing.
4. The BIO function can promote cell's vitality to rejuvenate skin. Based on photodynamics, the photon&microcurrent can renew cells, promote metabolism, and relieve various skin conditions.
5. The specially-designed scrubber can effectively remove the stubborn dirt and impurities on the area around nose (like nose wing and the tip of nose) and lips without damaging skin for its touching skin well, offering treatment receivers safe and comfortable experience.
6. It is painless and non-invasive during the process. There is no downtime, which means you can go back home as long as the treatment is finished without any inconvenience.
7. No consumption, low cost and quick returns.
8. It provides almost all facial treatments you can think of.
9. Unevenness, bleeding, and swelling will not appear after treatment.
10. It is suitable for all skin types, helping improve skin texture and smooth out fine lines.
11. The handpiece design is based on ergonomics. It is comfortable to hold it. And larger handpieces can be used to treat large-size areas.

## Working Principles


### Scrubber

Based on ultrasonic waves, this machine can produce up to 28,000HZ mechanical vibration, the energy of which has great penetrating ability on skin. The ultrasonic cavitation can make many bubbles form in the water. After those bubbles come into pores, harmful substances like

blackheads can be destroyed into smaller particles, which then will be moved to skin surface by the high-frequency vibration of ultrasonic waves. Sebum and other skin secretions which cause pore blockage will be emulsified. The purpose of killing bacteria can be also achieved at the same time. In this way, this machine can unblock pores, minimize pore size and smooth skin.

#### Physical effects

The high-frequency sound waves can work together with the use of softening products or exfoliating creams, in which corneum will be softened, sound waves can open up pores and then the high-frequency vibration will help get rid of harmful substances inside.

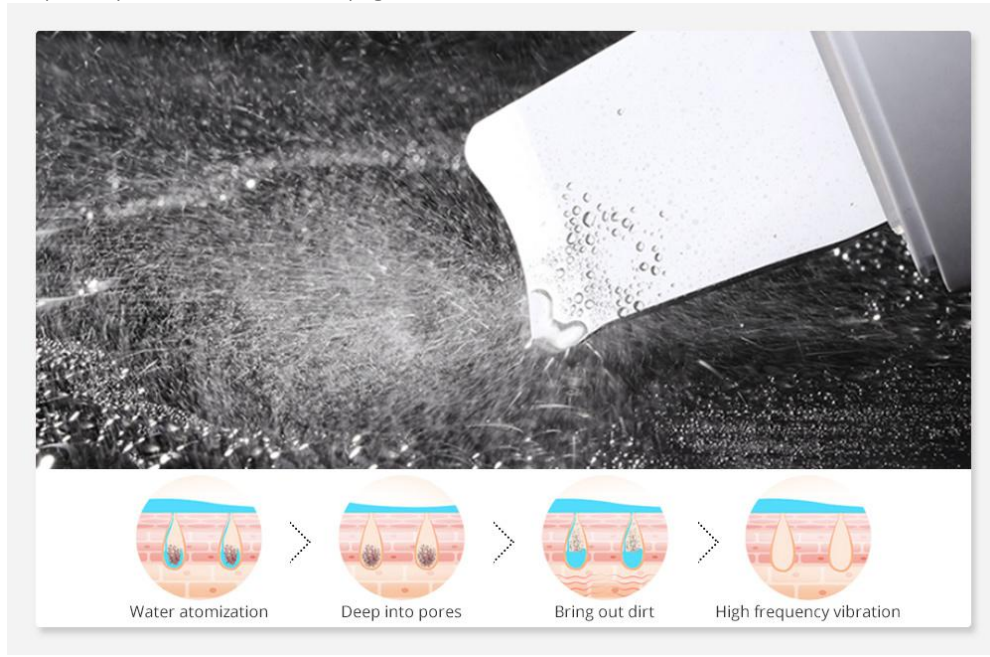

#### Face RF

RF wave can directly penetrate the skin and uses the resistance formed by the skin to produce energy, which raises the temperature of the bottom layer of the skin. Using the principle of dermis collagen protein tightening and stimulating collagen protein regeneration, it has two therapeutic effects: timely skin firming and lasting collagen protein regeneration.


After treatment, an obvious sense of contraction can be felt immediately at the treatment area, and the effect of skin firming and lifting can be seen about a month later, and the effect will become more and more obvious. The effect of RF treatment can last for about 18 months to 3 years depending on the area of treatment and individual maintenance methods, which, however, may be longer depending on individual maintenance methods. The latest research points out that better results can be achieved after repeated treatment.



#### BIO

It stimulates the energy contained in the ATPs of cells to restore the normal operation and

function of cells after its imitating human biological current enters into muscle cells through skin to provide energy for muscle and body. It can improve cell permeability to make it 30%-40% higher than usual. It can help the body better absorb nutrition, water, oxygen etc., and eliminate wastes, toxins and retained liquids as soon as possible. It can promote the synthesis of muscle fibers to make it 68% higher than usual, keep the skin moisturized, soft, firm and shiny, and increase collagen synthesis by 73% to reduce wrinkles and leave the skin softer, shinier and more elastic.

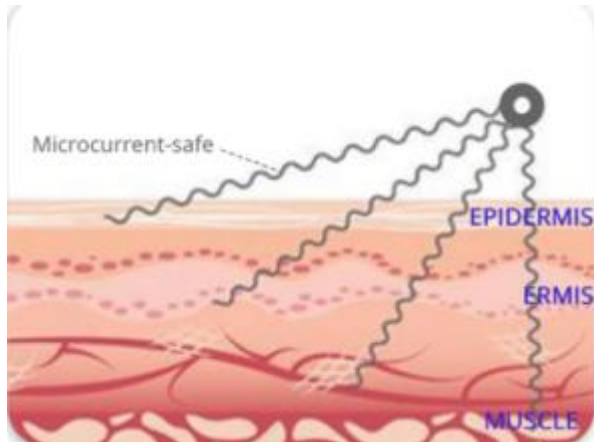

## Photon&Micrcurrent

LED photon therapy is based on photodynamics to activate deep cells and boost skin metabolism. After light is absorbed by the skin, the light energy is transformed into intracellular energy, which expands and strengthens microvessels and produces photochemical--enzymatic reaction with the skin, increasing the activity of Catalase and superoxide dismutase (SOD), which serve as the "guard" and "cleaning worker" of cells. The breakdown of adenosine triphosphate (ATP), a source of cellular energy, increases, and glycogen and protein content increase. Therefore, it promotes cell metabolism and synthesis. It stimulates basic fibroblast growth factor(BFGF) and the epidermal growth factor (EGF) to secrete, which can rearrange collagen fibers and elastic fiber, block and eliminate the formation of melanin, accelerate cell growth and blood circulation, and stimulate fiber cells to produce collagen to increase skin elasticity. It plays an important role in skin care such as repairing the aging skin and acne skin, fading spots, firming skin and alleviating sunburns.

### Biological effect

Based on photodynamics, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical--enzymatic reactions with the skin to increase the content of glycogen and protein. In this way, it promotes the metabolism and synthesis of cells.

**Red LED Light:** 635NM. Red LED light of specific wavelength heats up skin bottom, promote collagen regeneration, accelerates skin's micro circulation, and increase skin elasticity, leaving skin rosy and smooth and making skin look younger and have less wrinkles.  
Promote the regeneration of skin cells and activate active ingredients.

**Blue LED Light:** 462NM. Blue LED light of specific wavelength shrinks pores, removes skin grease, calm skin, fight acne, quickly lock in moisture and essence, leaving skin more elastic.  
Relieve skin inflammation, kill bacteria, get rid of acne.

**Green LED Light:** 527NM. Green LED light of specific wavelength helps to calm, repair skin, relieve pain, and promote blood circulation. It is suitable for those with thin corneum, and who

are prone to allergy and redness.

**Purple LED Light:** 600NM, dual frequency of red and blue LED lights. It combines the efficacy of two kinds of phototherapy and has a particularly good repairing effect especially for the treatment of acne and acne marks.


**Yellow LED Light:** 590NM. It can add energy to skin cells, promote the role of glands, assist digestion, treat skin diseases, enhance immunity, accelerate blood circulation, effectively repair red marks, red spots, and relieve swelling and pigmented spots, and is suitable for removing spots and brightening.

**White LED Light:** 670NM. It reaches deep into skin, accelerates metabolism of active cells, decomposes pigments, reduces fine lines and tightens skin. It can also relax skin, balance sebum production and control oil secretion.

**Turquoise LED Light:** 610NM, dual frequency of green and blue LED lights. It gradually enhances cell energy, and has positive impact on metabolism.

## Ultrasound

It makes high-frequency ultrasonic wave of 28,000 HZ/ second penetrate into the subcutaneous tissue at a depth of 4~6mm. Ultrasonic wave has high energy and strong permeability, which can drive facial cells to vibrate accordingly. It can not only drive pore grease out, but also change the volume of cells to improve blood circulation, which can stimulate cell regeneration and make the skin shiny and elastic. Secondly, because of the heat effect caused by its high-frequency vibration, it also has the function of quickly introducing nutrition into the skin, which can make the skin better absorb the nutrients of the skin care products.

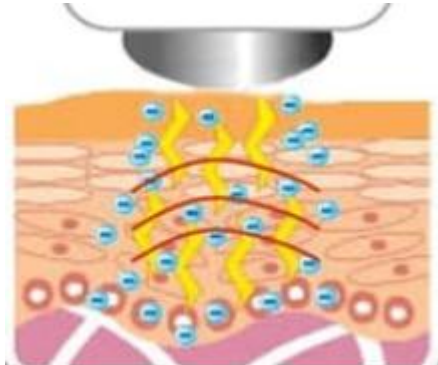

## Hot&Cold

It is based on thermal expansion and cold contraction. Our skin reduces the metabolic rate at 5 degrees Celsius, slows down the movement of cells and slows down the sensitivity of nerve endings, which inhibits pain and swelling of skin for allergic reactions. Due to thermal expansion and cold contraction, the temperature of 5 degrees Celsius causes the blood vessels to contract, resulting in pore contraction to restrain thick pores and firm the skin. Therefore, it can smooth facial wrinkles and crow's feet.

Physiological effects of 5°C :

1. It can constrict blood vessels, resulting in tightening of pores, which can greatly improve the size of pores.
2. It can strengthen myofibril, make the skin firm and elastic, and smooth the wrinkles and crow's feet.

3. It can constrict blood vessels, and reduce varicose veins and red veins effectively. It is easier to purify and detoxify venous blood in terms of harmful substances, and it has a significant effect on reduction and elimination of spots.

4. It can reduce the metabolic rate, inhibit and slow down the cell activity, reduce the sensitivity of the peripheral nerve to play a calming and soothing effect, and relieve skin swelling and pain, especially for allergic skin.

Physiological response of 40°C treatment :

1. 40°C makes people feel warm and comfortable, which will be able to relax muscles, keep the balance of vegetative nervous system and relieve pain and soreness. It can promote blood circulation, increase the oxygen content in the blood, and open pores and extend blood vessels to make skin absorb nutrients quickly, which is a milder and more comfortable absorbing way.

2. It can increase the metabolic rate of the skin, increase the heat energy content of the tissue, and make skin naturally eliminate the waste and toxins, which has a certain effect on the elimination of bags under the eyes and dark circles.

The alternating use of cold and heat hammer has multiplying effect on skincare, and can activate cells, and improve and purify the skin, which is a good instrument for health and beauty treatment.

## **Ion-based Introduction**

It uses direct electric current to form an electric field between skin surface and its deep layer, which will temporarily disable skin's function as a barrier to make skin absorb needed nutrients in an even way. Specifically, when the nutrients are applied to the skin in the form of dissolved substances, there will be ions. Affected by the electric field, those ions will move in specific directions. The cations will move to cathode and anions will move to anode. Ions of nutrients will permeate skin and there will be ion cluster locally so that the concentration of nutrient absorbed in the skin is several times higher than that of nutrients taken orally. In this way, its beauty effects can be delivered.

Biologic effects: it uses anions to make water-soluble nutrients permeate skin more efficiently to directly provide skin with nutrients and other needed substances through repulsion and attraction of ions

## **Facial Skin-firming&Anti-aging**

### **1. Needed handpiece**

Scrubber+hot&cold handpiece+Ultrasound+BIO+Face RF+Photon&Microcurrent

### **2. Effects**

1. Activate cells and increase skin's elasticity
2. Firm skin and smooth out fine lines
3. Relieve saggy condition of skin
4. Build face contour and make face have stereoscopic appearance
5. Shrink pores and plump up cheeks
6. Moisturize skin and promote skin's absorption
7. Brighten skin tone and fade pigmentation
8. Accelerate metabolism and blood circulation
9. Stimulate collagen regeneration and slow down skin ageing

### **3. Applicable range**

1. Those with saggy skin
2. Those with fine line on face, nasolabial folds and dynamic wrinkles
3. Those with unsatisfactory face contour and double chin
4. Those with rough skin and large pores
5. Those with saggy skin and edema after birth
6. Those with low skin metabolism and poor absorption

#### **4. Inapplicable Range**

1. Those who have just undergone plastic surgery with prostheses and metal materials, etc., implanted at the operation area.
2. Those who have experience in injection products recently, such as hyaluronic acid, or those who have mesotherapy, wrinkle removal, or plastic surgery.
3. Those who have hypertension, heart disease, diabetes, severe hyperthyroidism, malignant tumor, etc.
4. Those who are in allergic period and have severely sensitive skin and metal allergies.
5. Those who have trauma or wound on the skin.
6. Those who are of advanced years.
7. Those who are in menstruation, pregnancy, lactation, surgical recovery.
8. Those who have skin diseases and infectious diseases.
9. Those who are sensitive to electric currents.

#### **5. Do's and Don'ts after Operation**

1. Wash face with warm water within three days
2. Stay hydrated and use sunscreen
3. Do not use irritating products such as AHA, scrubbing and exfoliating products within one week
4. Do not stay up late, avoid drinking alcohol and smoking cigarettes and eat more fruits, vegetables and foods with less grease.
5. Apply face mask at least three times within a week

### **Facial Thorough Cleaning**

#### **1. Needed handpiece**

Scrubber+hot&cold handpiece+Ultrasound+BIO+Photon&Microcurrent

#### **2. Effects**

1. Cleanse deep skin layer and unblock pores
2. Slough off corneum
3. Supply skin with moisture and nutrients
4. Improve skin texture and increase skin's elasticity
5. Make skin smoother

#### **3. Applicable Range**

1. Those who have dry skin
2. Those who have large pores and rough skin that lacks glow
3. Those who have poor skin absorption
4. Those who have dark-toned skin or uneven-toned skin
5. Those who have oily, dry skin or have excess sebum production
6. Those who follow improper cleaning procedure

#### **4. Inapplicable Range**

1. Those who have severely sensitive skin or are having allergic reactions.
2. Those who have large lesions on face
3. Pregnant women or patients with severe heart diseases, severe hypertension, hyperglycemia, hyperlipemia, major diseases and so on
4. Those who have skin diseases or contagious diseases
5. Those who have just taken skin booster, and micro-needle treatment or who have incisions on skin.
6. Those who are sensitive to electric current.

## **5. Do's and Don'ts after Operation**

1. Wash face with warm water within three days
2. Stay hydrated and use suncream
3. Do not use irritating products such as AHA, scrubbing and exfoliating products within one week
4. Do not stay up late, avoid drinking alcohol and smoking cigarettes and eat more fruits, vegetables and foods with less grease.
5. Apply face mask at least three times within a week

## **Facial Soothing &Repairing**

### **1. Needed handpiece**

"Ion-" + hot & cold handpiece + Photon & Microcurrent

### **2. Effects**

1. Relieve redness, swelling, acne and allergic reaction on face.
2. Calm and soothe skin.
3. Minimize pore size.
4. Increase skin's elasticity.
5. Promote skin's resistance and lower skin's sensitivity.

### **3. Applicable Range**

1. Those who have sensitive, acne-prone skin or whose skin are vulnerable to redness, swelling and allergic reactions.
2. Those who have saggy skin without elasticity.
3. Those who have dark-toned, uneven-toned and dry skin and fine lines.
4. Those who have large pores, rough skin and excess grease secretion.

### **4. Inapplicable Range**

1. Those who have severely sensitive skin or are having allergic reactions.
2. Those who have prostheses in their noses.
3. Those who have large lesions on the face.
4. Pregnant women or patients with severe heart diseases, severe hypertension, hyperglycemia, hyperlipemia, major diseases and so on.
5. Those who have skin diseases or contagious diseases.
6. Those who have just taken skin booster, and micro-needle treatment or who have incisions on skin.
7. Those who have metal implants in body such as stent and pacemaker or who are sensitive to metals.

## **5. Do's and Don'ts after operation**

1. Wash face with warm water within 3 days.

2. Avoid sunburn and stay hydrated.
3. Do not use irritating products such as AHA, scrubbing and exfoliating products within one week.
4. Do not stay up late, avoid drinking alcohol and smoking cigarettes and eat more fruits, vegetables and foods with less oil.
5. Apply face mask three days in a row.

## **Facial Skin Whitening**

### **1. Needed handpiece**

Scrubber+hot&cold handpiece+"Ion-" +BIO+Face RF+Photon&Microcurrent

### **2. Effects**

1. Firm skin and smooth out fine lines
2. Relieve saggy condition of skin
3. Shrink pores and plump up cheeks
4. Moisturize skin and promote skin's absorption
5. Accelerate metabolism and blood circulation
6. Stimulate collagen regeneration and slow down skin ageing
7. Improve skin colour and make it have more clarity.

### **3. Applicable Range:**

1. Those with saggy skin
2. Those with fine line on face, nasolabial folds and dynamic wrinkles
3. Those with dull skin and uneven skin colour
4. Those with sallowness
5. Those with rough skin and large pores
6. Those with saggy skin and edema after birth
7. Those with low skin metabolism and poor absorption

### **4. Inapplicable Range**

1. Those who have just undergone plastic surgery with prostheses and metal materials, etc., implanted at the operation area.
2. Those who have experience in injection products recently, such as hyaluronic acid, or those who have mesotherapy, wrinkle removal, or plastic surgery.
3. Those who have hypertension, heart disease, diabetes, severe hyperthyroidism, malignant tumor, etc.
4. Those who are in allergic period and have severely sensitive skin and metal allergies.
5. Those who have trauma or wound on the skin.
6. Those who are of advanced years.
7. Those who are in menstruation, pregnancy, lactation, surgical recovery.
8. Those who have skin diseases and infectious diseases.
9. Those who are sensitive to electric currents.

### **5. Do's and Don'ts after Operation**

1. Wash face with warm water within three days.
2. Stay hydrated and use sunscreen.
3. Do not use irritating products such as AHA, scrubbing and exfoliating products within one week.
4. Do not stay up late, avoid drinking alcohol and smoking cigarettes and eat more fruits,

vegetables and foods with less grease.

5. Apply face mask at least three times within a week.

## **Wrinkle Removal for Eye Area**

### **1. Needed handpiece**

Face RF+BIO

### **2. Effects**

1. Relieve eye fatigue, dark circles, eye bags and edema.
2. Reduce lines around eyes and crow's feet.
3. Reduce dark spots and accelerate blood circulation.
4. Accelerate metabolism and prevent pigmentation.
5. Supplement eyes with nutrition and moisturize skin.
6. Firm and soften skin, lift the corners of the eyes.
7. Accelerate blood circulation around eyes and help skin absorb nutrients efficiently.

### **3. Applicable Range**

1. Those with wrinkles, fine lines, bags and dark circles around the eyes.
2. Those with dry skin, dry lines and dynamic wrinkles.
3. Those with eyes prone to fatigue and dryness.
4. Those who frequently face computer screens and mobile phones.
5. Those who often stay up late and have dark circles.
6. Those who often stay in a dry or hot environment.

### **4. Inapplicable Range**

1. Those who had just undergone plastic surgery.
2. Those who have hypertension, heart disease, diabetes, severe hyperthyroidism and malignant tumors.
3. Those who are having an allergic reaction and have severe sensitive skin or are allergic to metals.
4. Those who have skin lesions.
5. Those who are of advanced years.
6. Those who are pregnant or convalescents.
7. Those who have skin diseases and contagious diseases.
8. Those who are sensitive to electric currents

### **5. Do's and Don'ts after Operation**

1. Avoid excessive exposure to the sun. Suncream is encouraged to use.
2. After operation, the treated area is relatively dry and so being hydrated and protection against sunburn are encouraged.
3. Within 1-3 days after operation, it is better not to use alcohol, AHA or other exfoliating products.
4. Avoid facial washing with overheated water, taking a sauna, going for hot springs or other violent exercise within 7 days after operation.
5. Moisturizers and eye masks are encouraged to use. It will be preferable to use eye masks 3 times or more a week.

## **Neck Care**

### **1. Needed handpiece**

Face RF+Ultrasound+Photon&Microcurrent

### **2. Effects**

1. Reduce the flabbiness, roughness and dullness of the neck skin.
2. Firm skin and increase skin elasticity.
3. Remove double chin.
4. Accelerate lymph detox and improve facial skin.
5. Prevent neck and lymph diseases.

### **3. Applicable Range**

1. Those with rough and dull skin on the neck.
2. Those with lymphatic blocking
3. Those with sagging and inelastic neck skin.
4. Those who often bow their heads
5. Those who are not satisfied with their neck tone.

### **4. Inapplicable Range**

1. Those who have just undergone plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe hyperthyroidism and malignant tumors.
3. Those who are having an allergic reaction, or people with severe sensitive skin.
4. Those with skin trauma or wound.
5. Those who are of advanced years.
6. Those who are pregnant or convalescents.
7. Those with skin disease and other contagious diseases.


### **5. Do's and Don'ts after Operation**

1. Avoid sunburn. Keep the neck warm.
2. After operation, the treated area is relatively dry, moisturizing and sun protection is encouraged.
3. Within 1-3 days after operation, it is better not to use alcohol, AHA or other exfoliating products.
4. Avoid facial washing with overheated water, taking a sauna, go for hot springs or violent exercise within 7 days after operation.
5. Moisturizing and mask for the neck is encouraged, Use masks at least 3 times a week. Essence or neck cream is encouraged.
6. Avoid bowing head for a long time.

## Part II

### 1. Detailed Operation

After all are connected properly, the following interface will appear when the power switch is pressed.

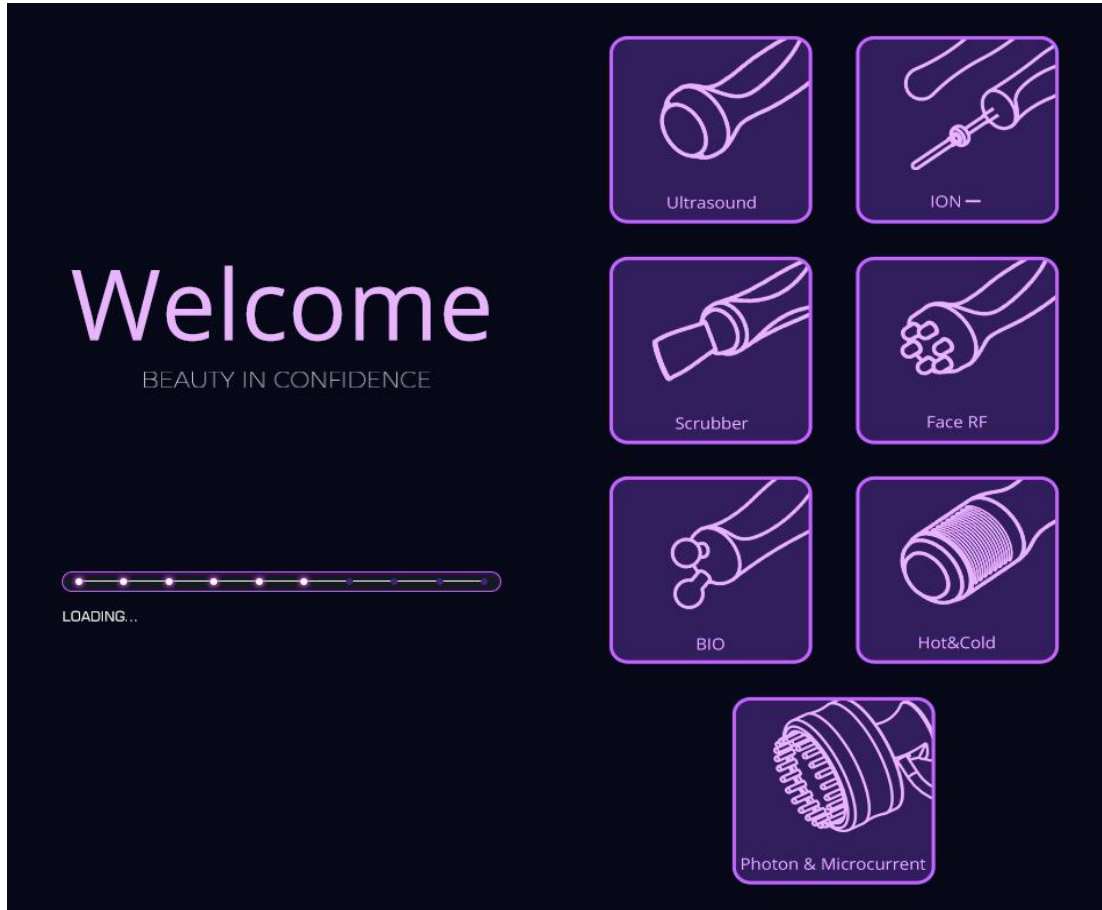

#### 1. Function Selection



time and adjusting buttons



button for decreasing time



button for increasing time



return button




start/stop button

#### 2. Detailed Operation of ULTRASOUND




choose to go to the following interface

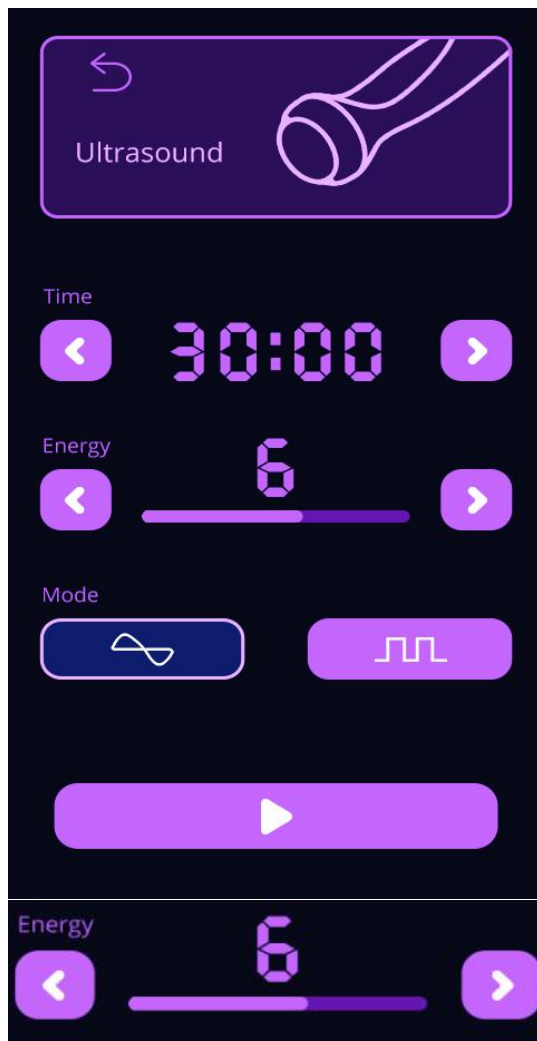

energy level and adjusting buttons



button for lowering energy level

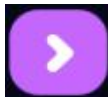

button for raising energy level



mode selection



Constant working mode, suitable for those with dry, rough and dark-toned skin.




Discontinuous working mode, suitable for those with sensitive skin or first-time users.

### 3. Detailed Operation of ION-




choose to go to the following interface

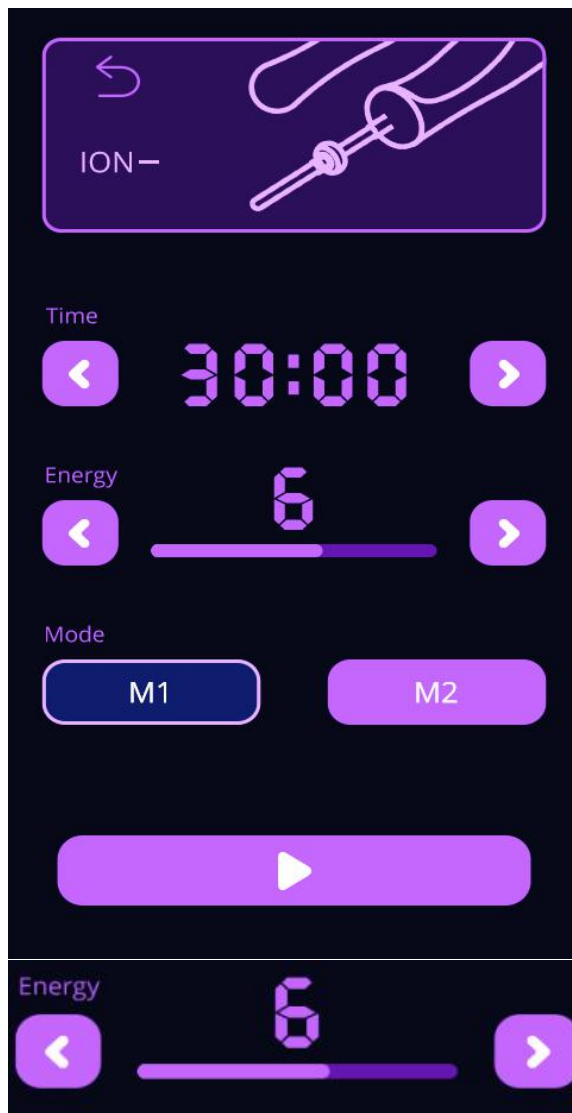


energy level and adjusting buttons



button for lowering energy level

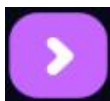

button for raising energy level

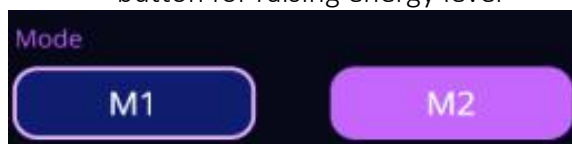

Working mode selection




Constant working mode, suitable for those with dry, rough and dark-toned skin.




Discontinuous working mode, suitable for those with sensitive skin or first-time users.

#### 4. Detailed Operation of Scrubber

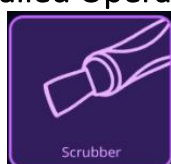

choose to go to the following interface

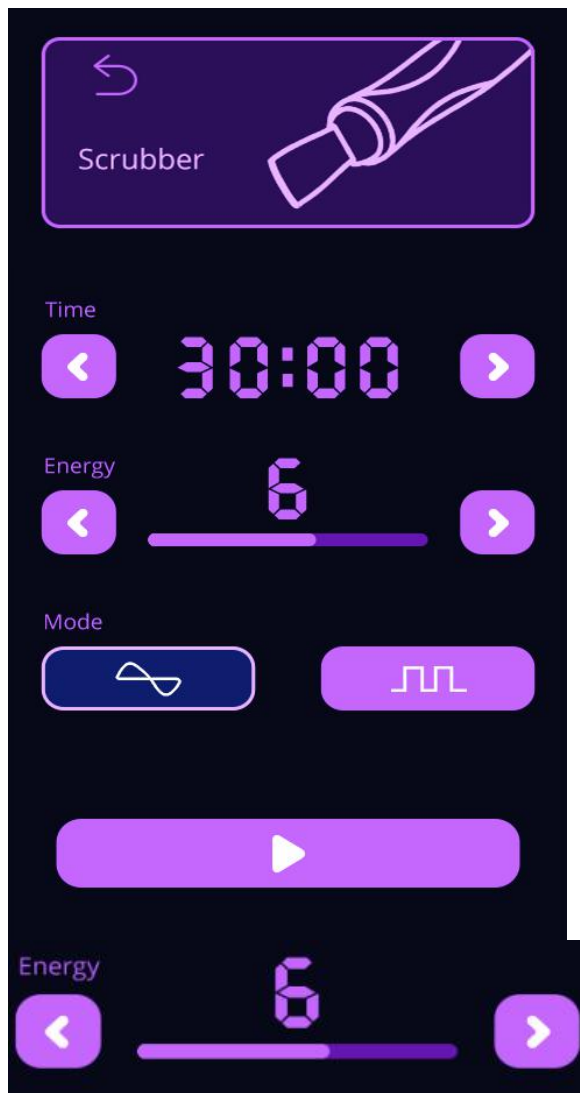

energy level and adjusting buttons



button for lowering energy level



button for raising energy level



working mode selection



Constant working mode, suitable for those who suffer from excessive sebum secretion and blockage in pores.




Discontinuous working mode, suitable for those with dry or sensitive skin.

## 5. Detailed Operation of Face RF



choose to go to the following interface

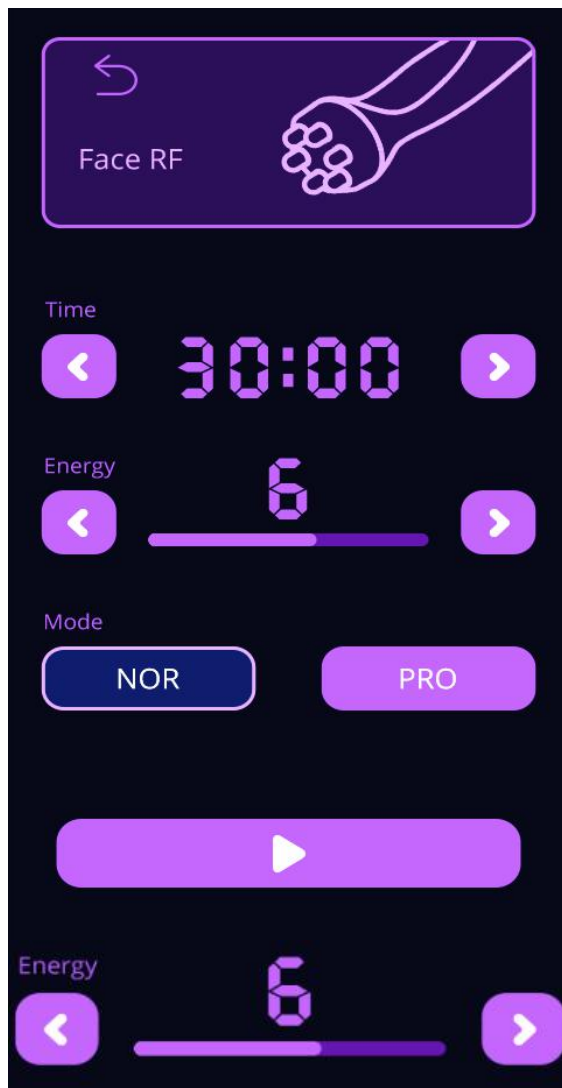


energy level and adjusting buttons



button for lowering energy level



button for raising energy level

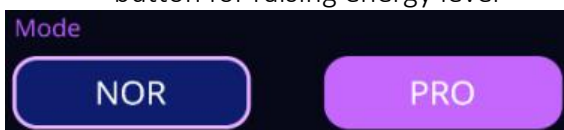

working mode selection



Fixed mode, the red LED light goes on and the energy of the handpiece directly reaches preset level after start button is pressed.




Smart mode, the red LED light goes on and the energy of the handpiece slowly reaches preset level after start button is pressed and the handpiece touches skin.

## 6. Detailed Operation of BIO



choose to go to the following interface

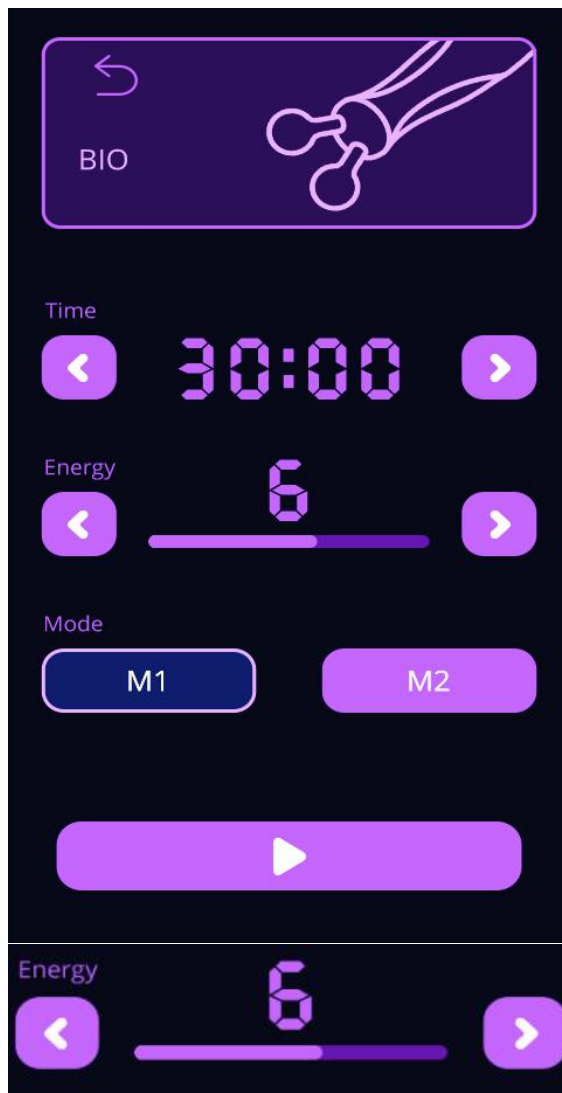


energy level and adjusting buttons



button for lowering energy level



button for raising energy level

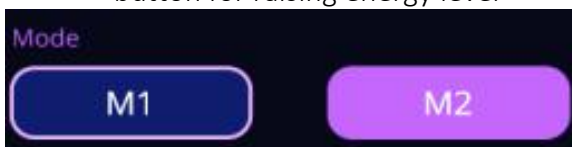

working mode selection



skin

Constant working mode, suitable for those with dry, rough and dark-toned




Discontinuous working mode, suitable for those with sensitive skin or first-time users.

## 7. Detailed Operation of Hot&Cold




choose to go to the following interface

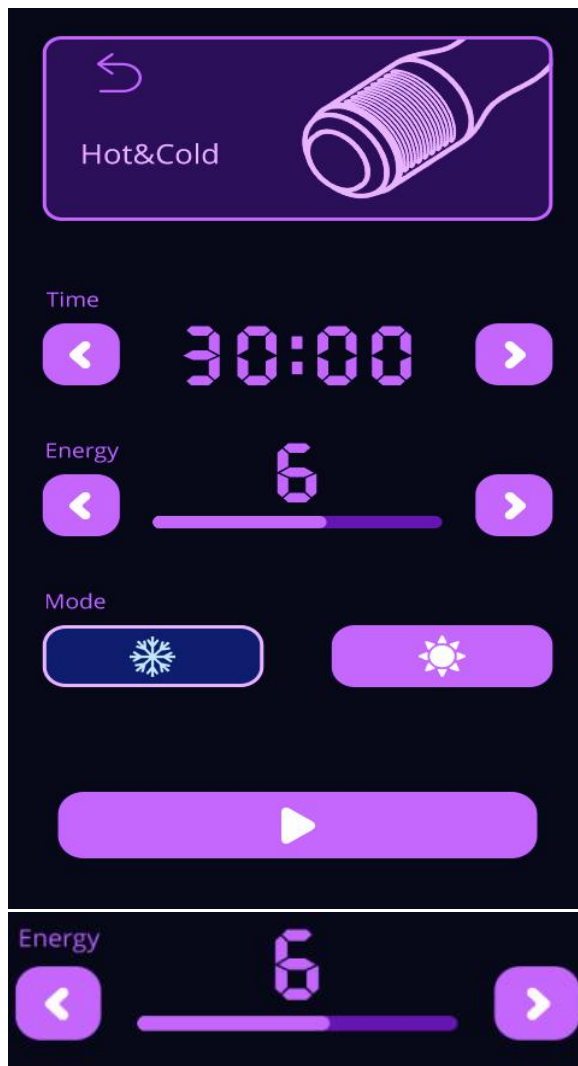


energy level and adjusting buttons



button for lowering energy level

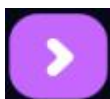

button for raising energy level

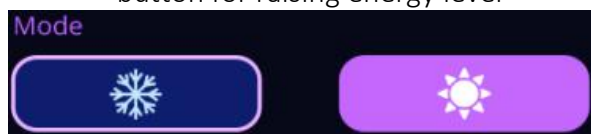

working mode selection



Cold mode. In this mode, it can soothe and repair skin, relieve acne, and contract pores.




Hot mode. In this mode, it can help expand pore size and help with nutrient introduction.

## 8. Detailed Operation of Photon&Microcurrent



choose to go to the following interface

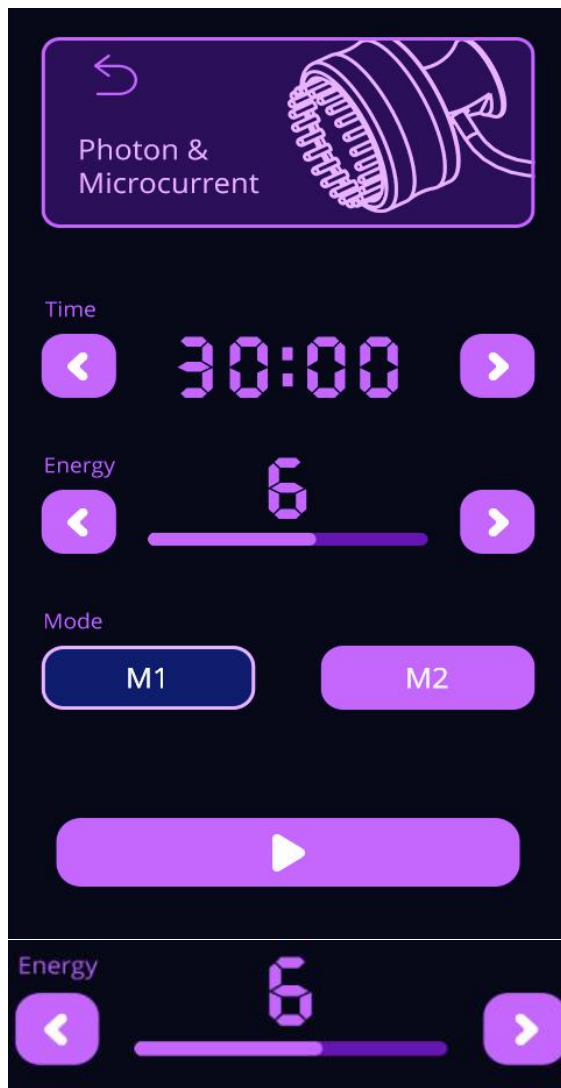


energy level and adjusting buttons



button for lowering energy level



button for raising energy level

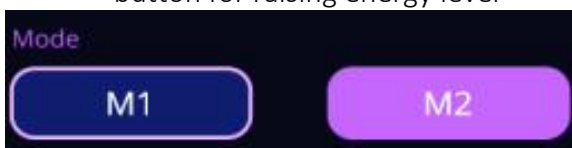

working mode selection



Constant working mode, suitable for those with dry, rough and dark-toned skin



Discontinuous working mode, suitable for those with sensitive skin or the first-time users.

**the functions of the buttons on the handpiece**



**OFF: Photon LED**



**AUTO: Automatic alteration of color (cycle through multiple twinkling colors)**



**MODE/ON: Mode selection (different colors can be chosen to stay on)**



**WAVE: Mono twinkling color (click on MODE to change to different color when needed.)**

### 3. Technical Specifications

Power supply Input:100V-240V

Power :190W

Cold Hot Hammer

Cold: 0-5 degree

Hot: 45 degree

Microcurrent &Photon

7 colors : Red, Blue, Green, Purple, Orange, Turquoise, White

Cold Hot Hammer

Cold: 0-5 degree

Hot: 45 degree

Microcurrent &Photon

7 colors : Red, Blue, Green, Purple, Orange, Turquoise, White

### 3. Dos and Don'ts

1. The plug with grounding pin must be used before the instrument is used and the power socket of the instrument is properly grounded.
2. Ensure that the voltage of the instrument is suitable. If the voltage of the local power supply is not stable, we suggest that the user add a voltage regulator with matching power between the mains and the instrument.
3. In order to ensure the treatment effect and normal service life of the instrument, please use

the specified parts provided or suggested by the original manufacturer.

4. The instrument should not be placed in damp places or near water sources, and should not be directly exposed to sunlight.
5. Do not place the instrument near a strong heat source, as this may affect the life and normal use of the instrument.
6. Before treatment, please remove all metal objects from the body first, so as to avoid unexpected conditions and influence the curative effect.
7. Please do not use the instrument on eyes, thyroid, parathyroid, testicles, abdomen of pregnant women, and pacemakers.
8. Patients who are suffering from illness should consult a doctor and get permission from the doctor before using.
9. Please turn off the power switch of the device, and ensure that the total power supply of the device is turned off before the staff leaves after daily use, so as to ensure the safety of the electrical products.
10. Set the energy at the lowest level at the beginning and then increase the level slowly after customer adapts to it.
11. Clean the instrument with normal saline after operation to ensure its cleanliness and hygiene and prolong its service life.
12. The handpiece can be used alone. Better results will be delivered if one handpiece works in conjunction with other handpieces.
13. When you use the cold&hot handle, keep your hands away from the steam outlet to avoid discomfort.
14. The treatment area should be kept moisturized during the operation and do not perform the treatment on dry skin.
15. The electrode must be held by the customer during the ion-based introduction treatment. Otherwise, there will be no electric current.
16. Normally, there will be slight stinging sensation of electric transmission during the ion-based introduction treatment.
17. Use machine or train machine operators in strict accordance with instructions in the user manual.

## **4. Troubleshooting & Solutions**

### **1. The instrument cannot be started, and the button light on the back of the instrument does not work?**

- A. Make sure the power cord is connected to the socket with normal power supply.
- B. Whether the fuse on the back of the instrument is loose or blown.

### **2. No RF output from the machine?**

- A. Please check whether the cord connects the handpiece to the machine properly
- B. Please check whether the treatment area has been cleaned. Grease or essential oil products may cause poor contact between the handpiece and human body to cause this problem.

### **3. Weakened energy output?**

- A. Please check whether the cord connects the handpiece to the machine properly
- B. Please check whether non-conductive grease and other substances are on the treatment handpiece, which can cause poor contact between the handpiece and skin to weaken the output.
- C. Please check whether the skin care product used is specified by this machine.

### **4. There is no physical sensation during the ion-based introduction treatment?**

- A. Please check the instrument is plugged in properly

B. Please check whether or not the customer holds the electrode in a wrong way. If there is still no physical sensation when the customer holds the electrode properly due to individual situation, wet cotton pad can be wrapped around the electrode stick to help with the electric transmission

**5. I can start the machine, but the screen shows erroneous message?**

A. Please unplug the power cord behind the instrument and wait for about 1 minute before restarting the instrument.

B. If the above method cannot solve the problem, please contact the instrument dealer for assistance.

## **5. FAQs**

**1. Q: Will skin become thinner and more sensitive after taking the scrubber treatment?**

A: No. It only removes the aging stratum corneum of the skin and unblock pores, minimize pore size and smooth skin in the process of treatment. Our skin will be updated every 28 days, so it will only make the skin smoother, delicate and transparent without making skin suffer from sensitive symptoms.

**2. Q: How long can I start seeing results of RF treatment?**

A: Normally, you can see the treatment results that day when you take the operation or within one week. Skin collagen tissue is heated and then contracts, and you can obviously feel the sensation of skin being tightened. Because RF works to stimulate subcutaneous tissue to promote collagen regeneration, so the more you take the treatment, the more noticeable the effects will be.

**3. Q: Is RF harmful to skin?**

A: RF treatment is non-surgical operation. It stimulates collagen regeneration at the skin bottom and boosts metabolism, and therefore has no damage to the skin. Local redness and hotness are the normal conditions of accelerated blood circulation, which will be relieved after a moment. So there's no need to worry about that.

**4. Q: Does the ultrasonic introduction help with acne scar removal?**

A: ultrasonic introduction can take advantage of the high-frequency vibration of the sound waves to introduce whitening nutrients into skin, inhibit the production of melanin and fade spots, thus effectively relieving the dark tone of skin and pigmentation spots caused by acne to restore skin's fairness. This treatment is safe and reliable without side effects. After-treatment care is simple and outdoor activities are allowed.

**5. Q: Does ultrasound treatment have side effects for human body?**

A: It is non-surgical and non-invasive treatment without anaesthesia. Generally speaking, it is not harmful to human body. Though it releases some energy, the energy is not high enough to do harm to body. Contrarily, it has accumulative effects that are good for the body.

**6. Does ion-based introduction treatment harm skin?**

A: no harm. With the influence of direct electric current, it uses the repulsion and attraction of ions to make skin absorb nutrients in a more efficient way to better skin. So, ions will not harm skin and contrarily help with skin care.

**7. Does BIO treatment harm skin?**

A : BIO treatment is non-surgical. It stimulates massage for muscles to restore elasticity.

Meanwhile, the stimulation of electric current for dermis can promote ATP production, which is the key substance for collagen regeneration. So, the micro-current can massage face, accelerate collagen regeneration and make face contour clearer to finally deliver the results of firming, lifting skin and reducing fat on face. It does no harm to the skin.

#### **8. How long will it take to see the effect?**

A: After one treatment session, you can see that the skin is clean and transparent. The effect of continuous treatment is more obvious, which can relieve aging conditions like the skin dryness, dullness and fine lines.

#### **9. Q: What is collagen?**

A: Collagen is a kind of biological macro molecule substance, a kind of white, opaque and non-branched fibrous protein. It can supplement the nutrients needed by all layers of the skin, enhance the collagen activity in the skin, retain moisture, nourish skin, slow down ageing, relieve facial relaxation, provide nutrients for the hair and so on. Collagen is a nutrient that human body must replenish to delay ageing. With the growth of age, collagen will gradually be lost. For women, collagen will begin to decrease since the age of 20. At the age of 25, collagen loss will reach its peak. At the age of 40, the collagen content is less than half of that one has at age of 18. It is the loss of collagen and water that causes the wrinkles on the face of the elderly. The breakage of collagen fibers and elastic fibers will cause skin tissue to be oxidized, atrophied and collapsed. The skin will be dry, wrinkled, flabby and inelastic. Therefore, in order to delay ageing, there must be plenty of collagen.

#### **10. Why do we need to exfoliate skin?**

A: The stratum corneum is a layer of tissue above the epidermis, on which the cells have completely died. Depending on different parts of the body, the stratum corneum is composed of 15 to 20 layers of dead cells. Human skin has a metabolic cycle, normally 28 days, which is the metabolic cycle of people in adolescence. If you are an adult, your metabolic cycle should be 10 days more than your age value. However, due to various reasons, it is possible that the aging and dead cells can not fall off in time, resulting in corneum hypertrophy, and the skin has no sense of transparency, which affects the skin color and the metabolic renewal of the skin. Therefore, in the case that there is no way to normally complete this normal metabolic process, it is necessary to use artificial methods to achieve the goal.

## **6. Packing List**

- 1 x Machine Body
- 1 x Ultrasound Handpiece
- 1 x Ion- Clip
- 1 x Stick-shaped Electrode
- 1 x Scrubber
- 1 x Face RF
- 1 x BIO Handpiece
- 1 x Hot&Cold Handpiece
- 1 x Photon&Microcurrent Handpiece
- 2 x Holder
- 1 x Power Cord

## **7. Simplified Treatment Steps**

### **1. Facial Skin-Firming&Anti-ageing(90 min)**

Remove makeup--clean face--apply hot steam and blackhead-removing liquid--scrubber--

“cold”handpiece--massage techniques--ultrasound--BIO--Face RF--“cold”handpiece--apply face mask--Photon&Microcurrent--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence, gel, RF cream, massage cream

## 2. Facial Thorough Cleaning(60 min)

Remove makeup--clean face--apply hot steam and blackhead-removing liquid--scrubber--“cold”handpiece--ultrasound--BIO--apply face mask--Photon&Microcurrent--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence, gel

## 3. Facial Soothing&Repairing(50 min)

Remove makeup--clean face--apply toner--“ion-”handpiece--“cold”handpiece--apply face mask--Photon&Microcurrent--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence

## 4. Facial Firming&Whitening(80 min)

Remove makeup--clean face--apply hot steam and blackhead-removing liquid--scrubber--“cold”handpiece--“ion-”handpiece--BIO--Face RF--“cold”handpiece--apply face mask--Photon&Microcurrent--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence, RF cream

## 5. Wrinkle Removal for Eye Area(45 min)

Remove makeup--clean the area around eyes--apply toner--massage techniques--BIO--Face RF--apply eye mask--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence, RF cream, massage cream



## 6. Neck Care(50 min)

Remove makeup--clean the area around eyes--apply toner--massage techniques--ultrasound--Face RF--apply neck mask--Photon&Microcurrent--apply eye mask--apply toner, essence, lotion/face cream, suncream.  
 Needed skincare products: essence, RF cream, massage cream

## 8. Operational Diagrams

Parameter Adjustment	Product	Techniques	Diagrams
<b>Facial Skin Firming&amp;Anti-ageing: 90 min once a week</b>			
<b>Face RF</b> Energy Level:3-7 Time: 10-15 min Mode:	Makeup remover+facial cleanser+blackhead-removing	1. Remove makeup and clean face, 5 minutes. 2. Apply toner, 1 minute 3. Apply blackhead removing liquid and hot steam( to the area where there are large	Technique5, 15

<p>NOR/PRO</p> <p><b>Scrubber</b> Energy Level:3-7 Time: 8-10 min Mode: discontinuous/constant</p> <p><b>Hot&amp;Cold</b> Energy level: 3-7 Time: 8-10 min Mode: cold</p> <p><b>Ultrasound</b> Energy Level:3-7 Time: 10-15 min Mode: discontinuous/constant</p> <p><b>BIO</b> Energy Level:3-7 Time: 10-15 min Mode: M1/M2</p> <p><b>Photon&amp;Microcurrent</b> Energy Level:3-7 Time: 10-15 min Mode: M1/M2 Choose the light with specific colour based on individual needs.</p>	<p>liquid+cold&amp;hot steam+essence+RF cream+face mask+LS-78D3</p>	<p>pores or severe blockage. Toner can be applied to cheeks. If you have sensitive skin, apply toner and cold steam on the whole face instead of blackhead removing liquid and hot steam), 5-8 min</p> <p>4. <b>Scrubber</b> treatment: use cotton pad to wet face, move the scrubber slowly on face at an inclination of 45°. Repeat the technique on nose area, 3-6 times.</p> <p>5. Lift upwards from jaw to earlobe, mouth corner to ear center, and nose wing to temple, 3 times.</p> <p>6. Do the same on the other side.</p> <p>7. Lift towards hairlines on forehead, 3 times.</p> <p>8. Wash face clean and apply essence evenly, 3 minutes.</p> <p>9. <b>“Cold” handpiece</b> treatment: move the handpiece upwards in circular motion from jaw to the back of ear, mouth corner to ear center and nose wing to temple line by line, 3-5 times.</p> <p>10. Repeat the technique on the other side.</p> <p>11. Move the handpiece in circular motion horizontally on forehead, 3 -5 times.</p> <p>12. Massage techniques: apply massage cream evenly to face and soothe face, 3 times.</p> <p>13. Apply digital pressing technique on the following acupoints (RN-24, DU-26, ST-4, ST-6, LI-20, BL-1, BL-2, EM3, TE-23, the temple, GB-1, ST-1, ST-2), 3 times.</p> <p>14. Soothe face, 3 times</p> <p>15. Alternately apply lifting technique from chin to ear lobe, mouth corner to Ermen(SJ21), and nose wing to temple, lift eye corner from lower eyelid to temple with</p>	<div data-bbox="1157 203 1294 421"></div> <p>Technique7, 17</p> <div data-bbox="1157 488 1294 705"></div> <p>Technique9, 24, 33</p> <div data-bbox="1149 772 1270 967"></div> <p>Technique11, 26, 35</p> <div data-bbox="1149 1034 1270 1229"></div> <p>Technique12, 14</p> <div data-bbox="1149 1296 1270 1491"></div> <p>Technique13</p> <div data-bbox="1149 1559 1270 1753"></div> <p>Technique22</p> <div data-bbox="1149 1821 1270 2016"></div> <p>Technique28, 29, 36, 37</p>
---	---	---	--






		<p>both hands, 3 times.</p> <p>16. Repeat the technique on the other side.</p> <p>17. Apply lifting technique towards hairline on forehead, 3 times</p> <p>18. Apply plucking technique with fingers alternately to lift face.</p> <p>19. Apply plucking technique with one hand towards hairline on forehead.</p> <p>20. One side, apply plucking technique with both hands alternately to lift face, 2-3 minutes</p> <p>21. Repeat the technique on the other side.</p> <p>22. Move in a zigzag motion on forehead with middle and ring fingers, slide to the front side of ears and lift back and forth for 3 times and then slide to the back of ears.</p> <p>23. Wash face clean and apply essence evenly.</p> <p>24. <b>Ultrasound</b> treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3-5 times.</p> <p>25. Repeat the technique on the other side.</p> <p>26. Move the handpiece in circular motion horizontally on forehead, 3 -5 times.</p> <p>27. Wash face clean and apply RF cream evenly, 3 minutes.</p> <p>28. <b>BIO</b> treatment: move the handpiece upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple, 3-5 times.</p> <p>29. Lift upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple with one hand and the handpiece, 3-5 times.</p> <p>30. Repeat the techniques above on the other side.</p>	 <hr/> <p>Technique31, 39</p>  <hr/>
--	--	--	--


		<p>31. Move the handpiece from forehead to hairline, 3-5 times.</p> <p>32. Clean face and apply toner evenly, 2 minutes.</p> <p>33. <b>Face RF</b> treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3-5 times.</p> <p>34. Repeat the technique on the other side.</p> <p>35. Move the handpiece in circular motion horizontally on forehead, 3 -5 times.</p> <p>36. Move the handpiece upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple, 3-5 times.</p> <p>37. Lift upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple with one hand and the handpiece, 3-5 times.</p> <p>38. Repeat the techniques above on the other side.</p> <p>39. Move the handpiece from forehead to hairline, 3-5 times.</p> <p><b>40.</b> Clean face and apply toner evenly, 2 minutes.</p> <p>41. <b>"Cold" handpiece:</b> the techniques are the same with the Face RF</p> <p>42. Apply face mask, 15 minutes.</p> <p>43. <b>Photon&amp;microcurrent</b> treatment: use the handpiece in stamping technique from jaw to earlobe, mouth corner to ear center, nose wing to temple and forehead to hairline, 5-8 minutes.</p> <p>44. Wash face clean, 2 minutes.</p> <p>45. Apply toner, essence, face cream and suncream.</p> <p>46. The end.</p>	
--	--	--	--

### Treatment Suggestions

One course of treatment consists of ten treatment sessions. After one treatment session, skin will be firmed and rejuvenated as the nutrients it needs are absorbed. After one course of treatment, skin will be lifted with glow. Two courses will make skin smooth and plump, leaving face contour clearer. Three courses will promote skin metabolism and detox processes, help fade pigmentation and skin darkness, slow down skin ageing and prevent skin from sagging, and stimulate collagen regeneration to leave skin smooth and firm with glow.

## Facial Thorough Cleaning: 60 min once a week

<p><b>Scrubber</b> Energy Level: 3-7 Time: 8-10 min Mode: discontinuous/constant</p> <p><b>Hot&amp;Cold</b> Energy level: 3-7 Time: 8-10 min Mode: cold</p> <p><b>Ultrasound</b> Energy Level: 3-7 Time: 10-15 min Mode: discontinuous/constant</p> <p><b>BIO</b> Energy Level: 3-7 Time: 10-15 min Mode: M1/M2</p> <p><b>Photon&amp;Microcurrent</b> Energy Level: 3-7 Time: 10-15 min</p>	<p>Makeup remover+facial cleanser+blackhead-removing liquid+cold&amp;hot steam+essence+RF cream+face mask+LS-78D3</p>	<ol style="list-style-type: none"> <li>1. Remove makeup and clean face, 5 minutes.</li> <li>2. Apply toner, 1 minute.</li> <li>3. Apply blackhead removing liquid and hot steam (to the area where there are large pores or severe blockage. Toner can be applied to cheeks. If you have sensitive skin, apply toner and cold steam on the whole face instead of blackhead-removing liquid and hot steam) 5-8 minutes.</li> <li>4. <b>Scrubber</b> treatment: use cotton pad to wet face, move the scrubber slowly on face at an inclination of 45°. Repeat the technique on nose area, 3-6 times.</li> <li>5. Lift upwards from jaw to earlobe, mouth corner to ear center, and nose wing to temple, 3 times.</li> <li>6. Do the same on the other side.</li> <li>7. Lift towards hairlines on forehead, 3 times.</li> <li>8. Wash face clean and apply essence evenly, 3 minutes.</li> <li>9. <b>"Cold" handpiece</b> treatment: move the handpiece upwards in circular motion from jaw to the back of ear, mouth corner to ear center and nose wing to temple line by line, 3-5 times.</li> <li>10. Repeat the technique on the other side.</li> <li>11. Move the handpiece in circular motion horizontally on</li> </ol>	<p>Technique 5</p>  <p>Technique 7</p>  <p>Technique 9, 13</p>  <p>Technique 11, 15</p>  <p>Technique 17, 18</p>  <p>Technique 20</p>
---	---	--	--

<p>Mode: M1/M2 Choose the light with specific colour based on individual needs.</p>		<p>forehead, 3 -5 times. 12. Apply essence evenly, 1 minute. 13. <b>Ultrasound</b> treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3-5 times. 14. Repeat the technique on the other side. 15. Move the handpiece in circular motion horizontally on forehead, 3 -5 times. 16. Clean face and apply gel evenly, 3 minutes 17. <b>BIO</b> treatment: move the handpiece upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple, 3-5 times. 18. Lift upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple with one hand and the handpiece, 3-5 times. 19. Repeat the techniques above on the other side. 20. Move the handpiece from forehead to hairline, 3-5 times. 21. Clean face, 2 minutes. 22. Apply face mask, 15 minutes. 23. <b>Photon&amp;microcurrent</b> treatment: use the handpiece in stamping technique from jaw to earlobe, mouth corner to ear center, nose wing to temple and forehead to hairline, 5-8 minutes. 24. Wash face clean, 2 minutes. 25. Apply toner, essence, face cream and suncream. 26. The end.</p>	
---	--	--	---

#### Treatment Suggestions

One course of treatment consists of ten treatment sessions. After one treatment session, pores will be visibly cleaned and skin will be hydrated and firmed. After one course of treatment, skin will become even-toned and condition of pores will be much improved. Three courses of treatment will leave skin moisturized and rejuvenated with glow and elasticity, which can make skin retain moisture for 24 hours.

## Facial Soothing&Repairing: 50 min once a week

**Hot&Cold**  
Energy level:  
3-7  
Time: 8-10  
min  
Mode: cold

**Ion-**  
Energy level:  
3-7  
Time: 10-15  
min  
Mode:  
M1/M2

**Photon&Microcurrent**  
Time: 10-15  
min  
Choose the  
light with  
specific  
colour based  
on individual  
needs.  
(no need for  
Microcurrent  
function)

Makeup  
remover+face  
cleanser+serum  
for skin  
soothing&repairing+facial  
mask for skin  
soothing&repairing+LS-  
78D3

1. Remove makeup and clean face, 5 minutes.
2. Apply toner, 1 minute.
3. Apply serum for skin soothing&repairing evenly, 1 minute.
4. Let the treatment receiver hold the stick-shaped electrode.
5. **Ion-** treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3 times.
6. Repeat the technique above on the other side.
7. Move the handpiece slowly and horizontally on forehead in a back-and forth manner, 3 times.
8. Move the handpiece in small spiral curve on the area around nose, 3-5 times.
9. Clean face, 2 minutes.
10. Apply some serum for skin soothing&repairing to face, 1 minute.
11. **"Cold" handpiece** treatment: move the handpiece upwards in circular motion from jaw to forehead line by line, 3 times.
12. Repeat the technique on the other side.
13. Move the handpiece in circular motion horizontally on forehead, 3 times.
14. Clean face, 2 minutes.
15. Apply skin-repairing facial mask, 15 minutes.
16. **Photon&microcurrent** treatment: use the handpiece in stamping technique from jaw to earlobe, mouth corner to ear center, nose wing to temple and forehead to

Technique6, 16



Technique8, 18







Technique13



Technique14



		hairline, 5-8 minutes. 17. Wash face clean, 2 minutes. 18. Apply toner, essence, face cream and sunscreen. 19. The end.	
<b>Treatment Suggestion</b> One course of treatment consists of ten treatment sessions. After one treatment, skin will be moisturized and smoothed. One course of treatment will help skin restore normal functions, which will not suffer from allergic reactions due to seasonal changes, exposure to cosmetic products and skin dryness. After two courses of treatments, skin blood circulation will be improved and you will see velvety skin.			
<b>Facial Firming&amp;Whitening: 80 min once a week</b>			
<b>Face RF</b> Energy Level:3-7 Time: 10-15 min Mode: NOR/PRO  <b>Scrubber</b> Energy Level:3-7 Time: 8-10 min Mode: discontinuous /constant  <b>Hot&amp;Cold</b> Energy level: 3-7 Time: 8-10 min Mode: cold  <b>Ion-</b> Energy level: 3-7 Time: 10-15 min Mode: M1/M2  <b>BIO</b> Energy Level:3-7	Makeup remover+facial cleanser+blackhead-removing liquid+cold&hot steam+essence+RF cream+face mask+LS-78D3	1. Remove makeup and clean face, 5 minutes. 2. Apply toner, 1 minute. 3. Apply blackhead removing liquid and hot steam( to the area where there are large pores or severe blockage. Toner can be applied to cheeks. If you have sensitive skin, apply toner and cold stem on the whole face instead of black head-removing liquid and hot steam)5-8 minutes. 4. <b>Scrubber</b> treatment: use cotton pad to wet face, move the scrubber slowly on face at an inclination of 45°. Repeat the technique on nose area, 3-6 times. 5. Lift upwards from jaw to earlobe, mouth corner to ear center, and nose wing to temple, 3 times. 6. Do the same on the other side. 7. Lift towards hairlines on forehead, 3 times. 8. Wash face clean and apply essence evenly, 3 minutes. 9. <b>"Cold" handpiece</b> treatment: move the handpiece upwards in circular motion from jaw to the back of ear, mouth corner to ear center and nose wing to temple line by line, 3-5 times.	Technique5  Technique7  Technique9, 14, 23  Technique11, 16  Technique18, 19, 26, 27






<p>Time: 10-15 min Mode: M1/M2</p> <p><b>Photon&amp;Microcurrent</b> Energy Level:3-7 Time: 10-15 min Mode: M1/M2 Choose the light with specific colour based on individual needs.</p>	<p>10. Repeat the technique on the other side. 11. Move the handpiece in circular motion horizontally on forehead, 3 -5 times. 12. clean face and apply essence evenly, 3 minutes. 13. Let the treatment receiver hold the stick-shaped electrode. 14. <b>Ion-</b> treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3-5 times. 15. Repeat the technique above on the other side. 16. Move the handpiece in circular motion horizontally on forehead, 3 -5 times. 17. Clean face and apply gel evenly, 3 minutes. 18. <b>BIO</b> treatment: move the handpiece upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple, 3-5 times. 19. Lift upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple with one hand and the handpiece, 3-5 times. 20. Repeat the techniques above on the other side. 21. Move the handpiece from forehead to hairline, 3-5 times. 22. Clean face and apply RF cream evenly, 2 minutes. 23. <b>Face RF</b> treatment: move the handpiece upwards in circular motion from jaw to earlobe, mouth corner to ear center and nose wing to temple, 3-5 times. 24. Repeat the technique on the other side. 25. Move the handpiece in circular motion horizontally on forehead, 3 -5 times. 26. Move the handpiece</p>	<div data-bbox="1157 203 1294 421"></div> <div data-bbox="1066 436 1284 470">Technique 21, 29</div> <div data-bbox="1157 488 1294 705"></div>
--	---	---

		<p>upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple, 3-5 times.</p> <p>27. Lift upwards from jaw to the back of ear, mouth corner to ear center, and nose wing to temple with one hand and the handpiece, 3-5 times.</p> <p>28. Repeat the techniques above on the other side.</p> <p>29. Move the handpiece from forehead to hairline, 3-5 times.</p> <p>30. Clean face and apply toner evenly, 2 minutes.</p> <p>31. <b>“Cold” handpiece:</b> the techniques are the same with the Face RF</p> <p>32. Apply face mask, 15 minutes.</p> <p>33. <b>Photon&amp;microcurrent</b> treatment: use the handpiece in stamping technique from jaw to earlobe, mouth corner to ear center, nose wing to temple and forehead to hairline, 5-8 minutes.</p> <p>34. Wash face clean, 2 minutes.</p> <p>35. Apply toner, essence, face cream and suncream.</p> <p>36. The end.</p>	
--	--	--	--

#### Treatment Suggestions

One course of treatment consists of ten treatment sessions. After one treatment session, skin will be firmed and rejuvenated as the nutrients it needs are absorbed. After one course of treatment, skin will be lifted with glow. Two courses will make skin smooth and plump, leaving face contour clearer. Three courses will promote skin metabolism and detox processes, help fade pigmentation and skin darkness, slow down skin ageing and prevent skin from sagging, and stimulate collagen regeneration to leave skin smooth and firm with glow.

**Wrinkle Removal for Eye area: 45 min 2-3 times a week**





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
<p><b>Face RF</b> Energy Level:3-7 Time: 10-15 min Mode: NOR/PRO</p> <p><b>BIO</b> Energy Level: 3-7 Time: 10-15 min Mode: M1/M2</p>	<p>Makeup remover+face cleanser+essence/eye cream/LS-78D3-eye mask</p>	<ol style="list-style-type: none"> <li>1. Remove makeup and clean face, 5 minutes</li> <li>2. Apply toner, 1 minute</li> <li>3. Apply massage cream evenly to eye area, soothe the area 3 times.</li> <li>4. Apply digital pressing technique on the following acupoints(BL-1, BL-2, EM3, TE-23, the temple, GB-1, ST-1, ST-2), 3 times.</li> <li>5. Use middle finger to lift from inner eye corner to temple back and forth, 3 times.</li> <li>6. Use middle finger and ring finger to lift from inner eye corner to temple along eyebrow, 3 times.</li> <li>7. Clean eye area clean.</li> <li>8. Apply essence(eye cream) evenly to eye area in circular motion, 3 minutes.</li> <li>9. <b>BIO</b> operation: adjust energy level, mode and time: 10 minutes.</li> <li>10. Move the handpiece to lift skin from lower eyelid to eye corner, 3-6 times.</li> <li>11. Move the handpiece to lift skin from lower eyelid to temple, 3-6 times.</li> <li>12. Move the handpiece to lift skin from lower eyelid to temple in small circles, 3-6 times.</li> <li>13. Move the handpiece to lift skin from lower eyelid to temple, 3-6 times.</li> <li>14. Lift from brow bone to hairline.</li> <li>15. Repeat the operation on the other side.</li> <li>16. Wash eye area clean, 1 minute</li> <li>17. <b>Face RF</b> operation: adjust energy level, mode and time: 10 minutes.</li> <li>18. Move the instrument to lift skin from lower eyelid to eye corner, 3-6 times.</li> <li>19. Move the instrument to lift</li> </ol>	<p>Technique3, 8</p>  <p>Technique5, 11, 19</p>  <p>Technique10, 18</p>  <p>Technique12</p>  <p>Technique14, 20</p> 
--	--	--	---

		<p>skin from lower eyelid to temple, 3-6 times.</p> <p>20. Lift from brow bone to hairline, 3-6 times.</p> <p>21. Repeat the operation on the other side.</p> <p>22. Wash eye area clean, 1 minute</p> <p>23. Apply eye mask, 15 minutes</p> <p>24. Remove mask and wash the area clean, 2 minutes</p> <p>25. Apply eye essence and eye cream.</p>	
--	--	--	--

### Treatment Suggestions

It is recommended to take this treatment 2-3 times per week. After one treatment, eye area is lifted and firmed and blood circulation is accelerated. After one month, fine wrinkles and black circles are reduced and skin colour is brightened. After three months, skin in eye area is totally firmed and rejuvenated with glossiness. If the customers keep using this machine, eye aging can be relieved and prevented.

## Neck Care: 50 min once a week
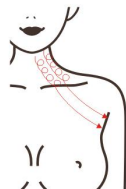
<p><b>Face RF</b> Energy Level:3-7 Time: 10-15 min Mode: NOR/PRO</p> <p><b>Ultrasound</b> Energy Level: 3-7 Time: 10-15 min Mode: discontinuous /constant</p> <p><b>Photon&amp;Microcurrent</b> Energy Level:3-7 Time: 10-15 min Mode: M1/M2 Choose the light with</p>	<p>Makeup remover+face cleanser+massage cream+essence+neck mask</p>	<ol style="list-style-type: none"> <li>1. Remove makeup and clean neck, 5 min.</li> <li>2. Apply toner, 1 min.</li> <li>3. Neck massage, rub oil into chest with both hands towards the back of the neck and press Gb-20 and Du-16 acupoint, 3 times.</li> <li>4. Alternately lift the lower jaw and double chin area to pass the back of ear to the armpit, 3 times.</li> <li>5. Push downwards with thumb and index finger splaying, tracing 3 meridians from the neck side to the armpit, 3 times.</li> <li>6. Push downwards with 4 fingers, tracing 3 meridians from neck side to the armpit, 3 times.</li> <li>7. Push downwards with bending fingers till the neck turns hot and slide to the armpit, 3 times.</li> <li>8. Push downwards with thumb and index finger</li> </ol>	<p>Technique3</p>  <p>Technique4</p>  <p>Technique5, 6, 7</p>  <p>Technique11, 18</p>  <p>Technique12, 19</p> 
--	---	---	---

<p>specific colour based on individual needs.</p>	<p>splaying from external collarbone to armpit, 3-5 times.</p> <p>9. Do the same on the other side.</p> <p>10. The end.</p> <p>11. <b>Ultrasound</b> treatment, use one hand and the handpiece to lift double chin area and then pass the back of ear to armpit, 3 times.</p> <p>12. Move the handpiece in circles along three lines on one side of neck to armpit, 3 times.</p> <p>13. Slide the handpiece from inner and outer sides of clavicle to the armpit, 3-5 times.</p> <p>14. Move the handpiece in circles on the whole neck area, 3 times.</p> <p>15. Do the same on the other side.</p> <p>16. Clean neck, 1 minute.</p> <p>17. Apply essence evenly, 1 minute.</p> <p>18. <b>Face RF</b> treatment, use one hand and the handpiece to lift double chin area and then pass the back of ear to armpit, 3 times.</p> <p>19. Move the handpiece in circles along three lines on one side of neck to armpit, 3 times.</p> <p>20. Slide the handpiece from inner and outer sides of clavicle to the armpit, 3-5 times.</p> <p>21. Move the handpiece in circles on the whole neck area, 3 times.</p> <p>22. Do the same on the other side.</p> <p>23. Clean neck, 1 minute.</p> <p>24. Apply neck mask, 15 minutes.</p> <p>25. <b>Photon&amp;microcurrent</b> treatment: use the handpiece in stamping technique from jaw to earlobe, mouth corner to ear center, nose wing to</p>	<div data-bbox="1134 197 1262 389"> </div> <div data-bbox="1066 398 1286 434"> <p>Technique14, 21</p> </div> <div data-bbox="1134 445 1262 638"> </div>
---	--	---

		temple and forehead to hairline, 5-8 minutes. 26. Wash face clean, 2 minutes. 27. Apply toner, essence, face cream and suncream. 28. The end.	
<b>Treatment Suggestions</b> A course of treatment consists of ten treatment sessions. After one treatment, skin becomes rejuvenated and smooth. After a full course, neck skin becomes smooth, and the neck lymphatic circulation is accelerated. After 2 courses of treatment, skin will be tightened, neck lines will be reduced, and the the condition of double chin will be relieved. After 3 courses of treatment, skin will be firmed with glow, lymphatic detox will be accelerated and some problems will be resolved such as acne and skin dullness. Moreover, skin aging will be prevented and skin will remain in a young state.			