

User Manual

MS-32J7



Preface

To our valued users,

Thanks for choosing our latest integrated beauty equipment. It's the latest skin cleansing and rejuvenating, and tightening and anti-wrinkles product with integration technology, which combines together diamond dermabrasion, RF, and LED Photorejuvenation technology. MS-32J7 is a professional beauty equipment which needs to be operated by people with professional training. Any improper use of this equipment may bring adverse consequence to human body. Therefore, we advise all the people to read this manual thoroughly and follow the instructions specified hereby strictly before operation.

We believe our quality product will offer you good returns, and our perfect after-sales service will make you worry-free.

Thank you and best regards.

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Facial Skin Cleansing & Rejuvenating
Anti-aging & Wrinkles Removing
Eyes Anti-aging
Neck Maintenance
Part II
Detailed Operations
Technical Parameters
Contraindications
Precautions for Use
Troubleshooting & Solutions
FAQs
Packing List
Operational Diagrams

Part I

Brief Introduction

Beauty equipment with diamond dermabrasion, RF, and LED Photorejuvenation is currently the most popular one that work on deep cleansing, skin rejuvenating and lightening, anti-aging and wrinkles removing. It can effectively replace all the facial programs in beauty salons, and can be operated easily, conveniently and swiftly, and solves skin problems for people pursuing beauty, and works safely and effectively, and requires no injection, no medication, no operation, and is without side-effects, and is operated externally during the whole process, and has an immediate effect. Diamond dermabrasion adopts vacuum suction technology. It effectively gets rid of old and useless stratum corneum and sucks out pore wastes like blackheads and excess grease by decreasing internal air pressure, thus achieving the effect of skin breathing freely and circulation, dredging pores, improving large pores, and brightening skin color. RF heats up deep skin, stimulates collagen hyperplasia and restructuring in skin dermis, and accelerates blood circulation in underlying skin, thus achieving the goal of skin tightening, plumpness, anti-aging, as well as senility preventing. Light energy emitted by LED face mask activates deep cells, accelerates cells growth, stimulates fibrocyte tissue producing collagen so as to achieve the effect of skin refining, fine lines fading, skin lightening and rejuvenating.

Advantages

1. 3-in-1 multifunctional beauty equipment that fixes facial care problems.
2. It has multiple operating heads, which can be changed according to different demands.
3. The special-designed diamond head can effectively improve facial skin problems, recover and cleanse skin.
4. It suits all kinds of skin, and can improve skin quality and flatten fine lines.
5. Painlessness, no invasiveness, no injury, without convalescence, you can leave immediately, and without affecting your normal work and life after treatment.
6. No consumption, low cost, but with quick returns.
7. It has a wider treatment range, including both body and face.
8. Unevenness, bleeding and swelling will not occur.
9. Doing physical therapy with 7 LED colors light wave, which can cleanse and lighten skin, and care skin deeply and evenly.

Skin Cleansing & Rejuvenating


Principle

Microdermabrasion (Diamond Dermabrasion)

Specialized dermabrasion tube embedded with tiny diamond particles of different sizes, by the means of rubbing back and forth and matched with vacuum suction, grinds off a shallow layer of epidermis, which is a non-invasive and physical dermabrasion. By removing dead skin cells on skin epidermis to make it regenerate and improve skin quality. After treatment, skin's fine lines lessen, and skin becomes smooth and flat and more tender. Meanwhile, matched with massage techniques, it can achieve the effect of lifting.

Physical Effect:

Diamond dermabrasion adopts vacuum suction technology. It effectively gets rid of old and useless stratum corneum and sucks out pore wastes like blackheads and excess grease by decreasing internal air pressure, thus achieving the effect of skin breathing freely and circulation, dredging pores, improving large pores, and brightening skin

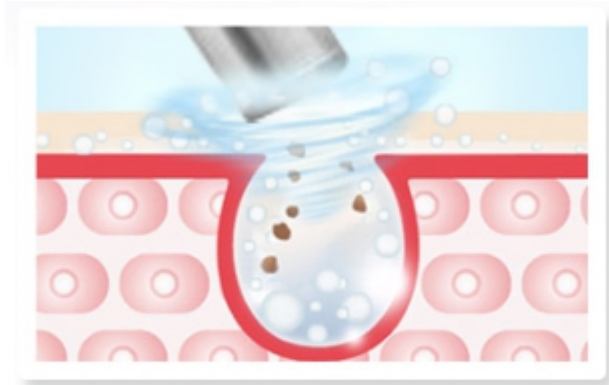

Dermabrasion

color. Balance skin barrier, remove dullness.

LED Photorejuvenation (LED Face mask)

It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, the light energy was transformed into intracellular energy which accelerates cell growth and blood circulation, stimulates fibrocyte tissue producing collagen, increases skin elasticity, and plays an important role in repairing aged cells, acne skin, fading spots, tightening and lifting, alleviating skin with sunburns, and effectively relaxes and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cell metabolism and synthesis.

Red Light: with a wavelength of 650NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity and makes it ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates skin cell regeneration and activates active ingredient through specific red light band.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, locks rapidly moisture and essence, makes skin more elastic, slows down skin inflammation, repairs dark acne, kills bacteria and removes acne through specific blue light band.

Green Light: with a wavelength of 527NM. It balances skin grease, soothes and repairs skin, eases pain, promotes blood circulation through specific green light band. It suits skin with thin stratum corneum and is prone to allergy and redness.

Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood


circulation, and effectively repairs red marks and red spots on skin, and relieves swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balance secretion, and control grease.

Turquoise Light: with a wavelength of 470NM. It calms and repairs skin, eases pain, alleviates allergy through specific ultramarine light band.



Phototherapy Skin Care

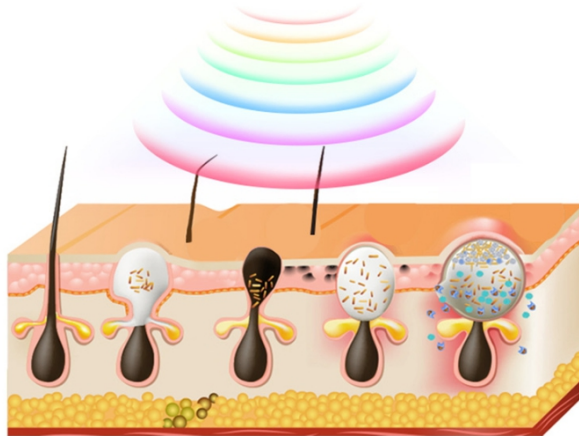

Effects

1. Get rid of old and useless stratum corneum.
2. Cleanse deep skin.
3. Dredge pores.
4. Calm skin and balance grease secretion.
5. Lighten skin color and skin.
6. Improve large pores and fade fine lines.
7. Strongly remove strawberry-like nose, acnes and blackheads .

Indications

1. People with slack skin.
2. People with clogged pores, or relatively thick stratum corneum.
3. People with dark or lackluster skin.
4. People with coarse skin or large pores.
5. People with a strong grease secretion.
6. People who are prone to have pimples, closed comedones, acnes, or with inflammation on underlying skin.
7. People with thin or impaired stratum corneum.
8. People with poor skin absorption, fine lines, wrinkles or color spots.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with prostheses in nose part.
3. People with extensive trauma on face.
4. Woman in pregnancy, or people with severe heart disease, serious Three Highs (hypertension, hyperglycemia, hyperlipidemia), serious diseases, etc.
5. People with skin disease or infectious disease.
6. People who just had injections, such as hyaluronic acid, mesotherapy, wrinkles removal, or plastic surgery.
7. People who are allergic or sensitive to light.

Notes (after treatment)

1. Wash your face with warm water within three days.
2. Do hydrating to maintain moisture, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliator, AHA (Alpha Hydroxyl Acid), etc.
4. Avoid staying up late and smoking and drinking alcohol. Eat more vegetables, fruits and light food.
5. Apply moisturizing mask one time each day within three days.


Anti-aging & Wrinkles Removing

Principle

RF

RF radiofrequency waves penetrate skin directly, and generate energy by utilizing electrical resistance effect formed by skin, which increases temperature on underlying skin. By using the principle of tightening collagen in dermis and stimulating collagen hyperplasia, it has two curative effects of lifting and tightening skin timely and regenerating lasting collagen.

The treated parts can experience obvious shrinkage instantly after treatment. The effect of tightening and lifting can be seen about one month later and will become more and more obvious. Depending on body parts that be treated and ways of maintenance of each person, the effect of RF can be maintained about 18 months to 3 years, or varies or extends even longer in line with ways of maintenance of each person. According to the latest research, a better effect can be achieved after multiple times of treatment.

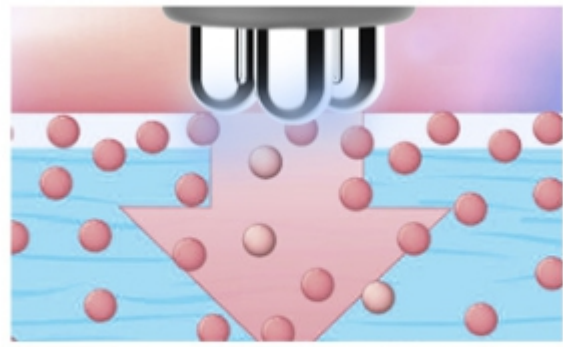

RF Wrinkles Removal

Rebuild plump skin with collagen regeneration

LED Photorejuvenation (LED Face mask)

It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, the light energy was transformed into intracellular energy which accelerates cell growth and blood circulation, stimulates fibrocyte tissue producing collagen, increases skin elasticity, and plays an important role in repairing aged cells, acne skin, fading spots, tightening and lifting, alleviating skin with sunburns, and effectively relaxes and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cell metabolism and synthesis.

Red Light: with a wavelength of 650NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity and makes it ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates skin cell regeneration and activates active ingredient through specific red light band.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, locks rapidly moisture and essence, makes skin more elastic, slows down skin inflammation, repairs dark acne, kills bacteria and removes acne through specific blue light band.

Green Light: with a wavelength of 527NM. It balances skin grease, soothes and repairs skin, eases pain, promotes blood circulation through specific green light band. It suits skin with thin stratum corneum and is prone to allergy and redness.

Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots on skin, and relieves swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balance secretion, and control grease.

Turquoise Light: with a wavelength of 470NM. It calms and repairs skin, eases pain, alleviates allergy through specific ultramarine light band.



Phototherapy Skin Care



Effects

1. Calm skin and balance grease secretion.
2. Fade fine lines, lighten and rejuvenate skin.
3. Improve wrinkles and color spots.
4. Tighten face and induct nutrient into deep skin.
5. Increase skin elasticity and refine skin.
6. Moisturize skin and increase skin absorptivity.
7. Accelerate blood circulation and metabolism.
8. Relieve double chin and firm skin.
9. Stimulate collagen hyperplasia and delay aging.

Indications

1. People with dark or lacklustre face.
2. People with slack or saggy skin.
3. People with fine lines, nasolabial folds or periorbital lines.
4. People with vague facial contour.
5. People with coarse skin or large pores.
6. People who are under long-term exposure of UV radiation at workplace.
7. People with loose skin, edema or puffiness after giving birth.

Contraindications

1. People who just undergone plastic surgery, or had prostheses and metal materials implanted in the treated parts.
2. People who recently had injections, such as hyaluronic acid, mesotherapy, wrinkles removal, or plastic surgery.
3. People with hypertension, heart disease, diabetes, severe thyroid, malignant tumors, etc.
4. People in allergic period, or have severely sensitive skin, or are allergic to metals.
5. People with skin trauma or wound.
6. People who are overly aging.
7. People in menstruation, pregnancy, lactation or surgical recovery.
8. People with skin diseases or infectious diseases.
9. People who have unrealistic illusion about the effects.
10. People who are sensitive or allergic to light.


Notes (after treatment)

1. Do not wash your face with overheated water within three days (but can wash with warm or cold water).
2. Do hydrating to maintain moisture, and protect yourself from the sun.
3. Do not enjoy hot springs or saunas, or do strenuous exercise, etc., within three days.
4. Apply facial masks at least three times in a week.
5. Avoid eating spicy and greasy food, and avoid staying up late and smoking and drinking alcohol. Eat more vegetables, fruits and light food.

Eyes Anti-aging

Principle

During eyes treatment, RF instrument heats up underlying skin of eyes rapidly through radiofrequency wave and stimulates growth of skin collagen, thus achieving the effect of tightening skin around eyes, removing eye bags and black eyes, improving red blood streak and fine lines.

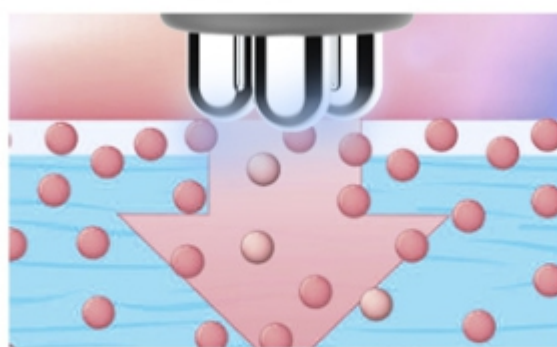

RF Wrinkles Removal

Rebuild plump skin with collagen regeneration

Effects

1. Relieve eye fatigue, black eyes, eye bags and edema.
2. Improve periorbital lines and crow's feet.

3. Fade pigment and accelerate blood circulation.
4. Accelerate metabolism and prevent pigment from sediment.
5. Replenish nutrition for eyes and moisten skin.
6. Tighten skin and make it tender, and lift the corner of eyes.
7. Accelerate blood circulation of eyes, and help with efficient absorption.

Indications

1. People with wrinkles, fine lines, eye bags or black eyes.
2. People with dry skin, dry lines or false wrinkles.
3. People whose eyes are prone to fatigues or dryness.
4. People who always face computer or cellphone.
5. People who always stay up late or with black eyes.
6. People who always exposure to dry or high temperature environment.

Contraindications

1. People who just undergone plastic surgery.
2. People with hypertension, heart disease, diabetes, severe thyroid, malignant tumors, etc.
3. People in allergic period, or have severely sensitive skin or are allergic to metals.
4. People with skin trauma or wound.
5. People who are overly aging.
6. People in pregnancy or surgical recovery.
7. People with skin diseases or infectious diseases.


Notes (after treatment)

1. Avoid being under the blazing sun and protect yourself from the sun.
2. It's better not to use alcohol, AHA(Alpha Hydroxyl Acid) or scrub cream within 1 to 3 days.
3. Avoid washing with overheated water, or enjoying hot springs, saunas or doing strenuous exercise within 7 days.
4. Drink more water and apply eye masks which should do at least 3 times in a week because the treated parts are relatively dry.

Neck Anti-aging

RF

RF radiofrequency waves penetrate skin directly, and generate energy by utilizing electrical resistance effect formed by skin, which increases temperature on underlying skin, heats up tissue rapidly and continuously, and stimulates growth of skin collagen so as to achieve the effect of lifting and tightening skin and making skin delicate.

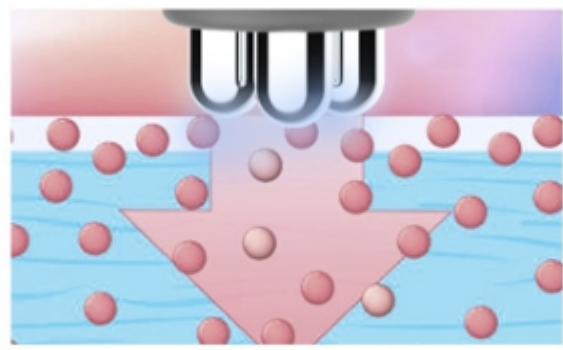

RF Wrinkles Removal

Rebuild plump skin with collagen regeneration

LED Photorejuvenation (LED Face mask)

It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, the light energy was transformed into intracellular energy which accelerates cell growth and blood circulation, stimulates fibrocyte tissue producing collagen, increases skin elasticity, and plays an important role in repairing aged cells, acne skin, fading spots, tightening and lifting, alleviating skin with sunburns, and effectively relaxes and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cell metabolism and synthesis.

Red Light: with a wavelength of 650NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity and makes it ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates skin cells regeneration and activates active ingredient through specific red light band.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, locks rapidly moisture and essence, makes skin more elastic, slows down skin inflammation, repairs dark acne, kills bacteria and removes acne through specific blue light band.

Green Light: with a wavelength of 527NM. It balances skin grease, soothes and repairs skin, eases pain, promotes blood circulation through specific green light band. It suits skin with thin stratum corneum and is prone to allergy and redness.

Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots on skin, and relieves swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balance secretion, and control grease.

Turquoise Light: with a wavelength of 470NM. It calms and repairs skin, eases pain, alleviates allergy through specific ultramarine light band.



Phototherapy Skin Care



Effects

1. Improve neck with slack, coarse and lacklustre skin.
2. Tighten skin and increase skin elasticity.
3. Relieve double chin.
4. Accelerate lymphatic detox and improve facial skin.
5. Prevent neck and lymphatic diseases.

Indications

1. People with coarse neck or dim skin color.
2. People with clogged lymph on neck.
3. People with slack and inelastic skin on neck.
4. People who always lower their heads.
5. People who are not satisfied with their neck skin color.

Contraindications

1. People who just undergone plastic surgery.
2. People with hypertension, heart disease, diabetes, severe thyroid, malignant tumors, etc.
3. People in allergic period, or have severely sensitive skin.

4. People with skin trauma or wound.
5. People who are overly aging.
6. People in pregnancy or surgical recovery.
7. People with skin diseases or infectious diseases.
8. People who are allergic or sensitive to light.

Notes (after treatment)

1. Protect yourself from the sun and keep neck warm.
2. It's better not to use alcohol, AHA (Alpha Hydroxyl Acid) or scrub cream within 1 to 3 days.
3. Avoid washing with overheated water, or enjoying hot springs, saunas, or doing strenuous exercise within 7 days.
4. Drink more water and apply neck mask which should do at least 3 times in a week, and apply essence or neck cream because the treated parts are relatively dry.
5. Avoid lowering head for a long time.

Part II

1. Detailed Operations

Detailed installation for the instrument is as follow:



Check the instrument and make sure it's properly connected. The following initial page will appear after having the power switch on.




1.1 Functions Selection



Time Display & Setting



Energy Level Display & Setting

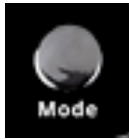

Modes Selection



Start/ Pause



Rotating Knob for Suction Setting (Suction can be adjusted in line with individual situation. It can't be set too high, otherwise redness may appear on treated parts.)

1.2 Detailed Operations for Eye RF


Tap "Mode", select "Eye RF", and enter the following page.



Time Display & Setting




Energy Level Display & Setting

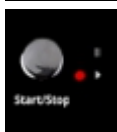

Start/ Pause

1.3 Detailed Operations for Diamond Dermabrasion

Tap "Mode", select "Dermabrasion", and enter the following page.

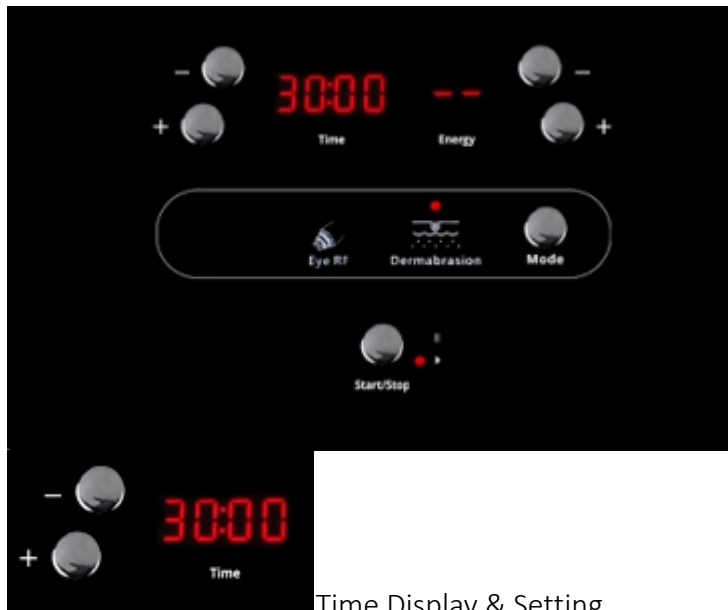

Time Display & Setting



Start/ Pause



Rotating Knob for Suction Setting (Suction can be adjusted in line with individual situation. It can't be set too high, otherwise redness may appear on treated parts.)



Black Rubber Ring (This rubber ring must be installed when operating, otherwise there is no suction or suction will be weakened.)



Filter Cotton (This filter cotton must be installed when operating, otherwise dirt may enter into the instrument causing damages to it. Meanwhile, the effect for cleansing can be seen.)



Select proper diamond head and install it on diamond dermabrasion wand.




Insert white rubber hose into wand.


There are 9 diamond operating heads in total, which can be chosen according to individual skin condition.



This is the biggest, namely No.1 diamond head (suitable for big area or body treatment)

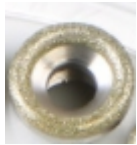


This is No.2 diamond head (suitable for big area or body parts treatment)

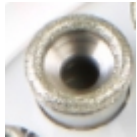

This is No.3 diamond head (suitable for big area treatment)




This is No.4 diamond head (suitable for big area treatment, face lifting, and exfoliating on big area)

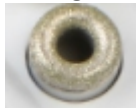

This is No.5 diamond head (suitable for cheek exfoliating, cleansing, and pores dirt sucking)



This is No.6 diamond head (suitable for sucking blackheads on face and nose, and clearing out obstruction in pores)

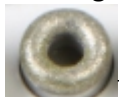

This is No.7 diamond head (suitable for face and nose cleansing and exfoliating.)




This is No.8 diamond head (suitable for blackheads sucking and dirt cleansing on root of nose)



This is No.9 diamond head (suitable for removing stubborn blackheads on root of nose or small area)

1.4 LED Photorejuvenation (LED Face mask)

Tap “Mode”, skip “Eye RF” and “Dermabrasion”. It’s LED Face Mask mode when red light is off.
Interface for remote control of LED face mask is as follow:

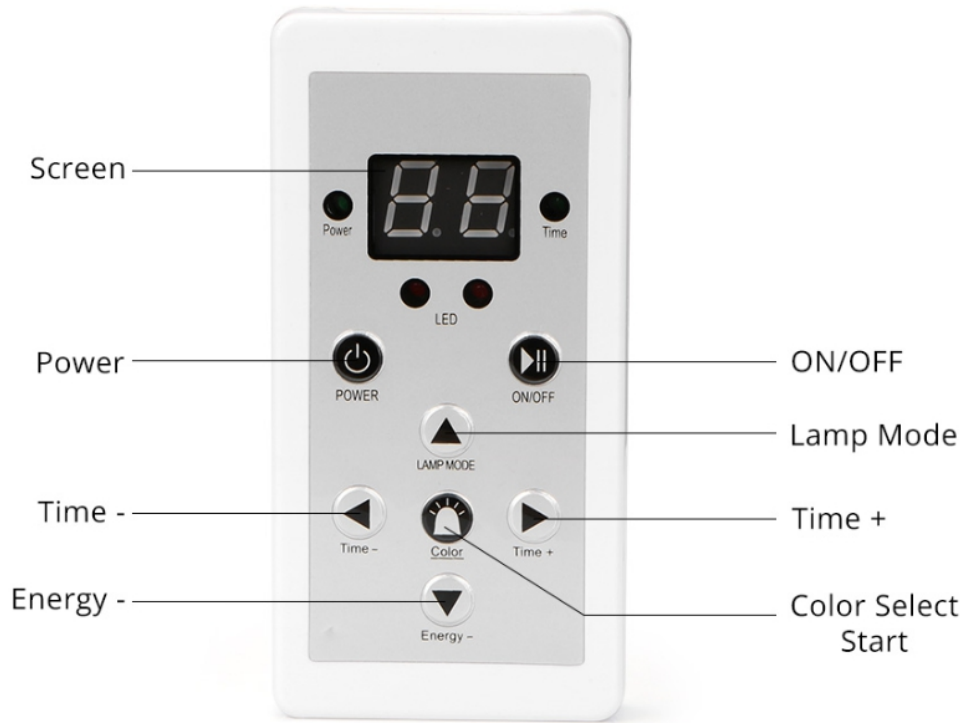

Choose LED face mask according to your demands.



Put mask on, set parameters, and then you can start LED Photorejuvenation treatment.

2. Technical Parameters

3 Pole RF Handle
Frequency: 1MHZ

Power: 30W

Max. Output Power: 8W

Photon

7 Colors: Red, Blue, Green, Purple, Orange, Turquoise, White

Vacuum Therapy Parameters

Power: 60W

Pressure: >250kPa

Air flow: >10L/minute

Noise Level: <70dB (30cm away)

3. Contraindications

People with the following situations should use the instrument with caution. Please consult a doctor or professional before using this instrument. The details are as follow:

- (1) Women in pregnancy, or lactation;
- (2) People with heart disease, or with heart pacemaker;
- (3) People with unhealed surgical wound, or in surgical recovery;
- (4) People with epilepsy, severe diabetes, or hyperthyroidism;
- (5) People with malignant tumors, hemophiliac or severe bleeding;
- (6) People with skin diseases or infectious diseases should use with caution;
- (7) People who have metal implants inside body, such as stent, pacemaker, or who are allergic to metals.
- (8) People who are sensitive or allergic to light.

4. Precautions for Use

- (1) Plug with ground pin must be used, and power socket that already factually grounded must be ensured before using the instrument.
- (2) To make sure instrument's voltage is adaptive. If voltage of local power supply is unstable, we suggest users add voltage regulator with matching power between the mains supply and the instrument.
- (3) To guarantee curative effect and normal service life of the instrument, please uniformly use specified parts provided or suggested by original manufacturer.
- (4) The instrument can't be placed in damp places or near water, and also can't be exposed to direct sunlight.
- (5) Don't place the instrument near a strong heat source since this may affect its service life and normal use.
- (6) Please remove all the metal objects from body before treatment so as to avoid unexpected situations which may affect curative effect.
- (7) Please restrain from using the instrument aiming at eyes, thyroid, parathyroid, testicles, pregnant woman's abdomen, pacemaker, etc.
- (8) People who are suffering from illness should use with caution unless he gets permission from a doctor.
- (9) Please turn off power switch of the instrument if no one uses it, and ensure main power is off after someone used it and before he left, thus safety of the instrument can be guaranteed.
- (10) Operating with the lowest energy level when just started, and increase the level gradually

after client got used to it.

- (11) It's prohibited to use the instrument on the rim of eyes, ears, the periphery of parts that have skin diseases, and parts that have plastic surgery.
- (12) Please set vacuum suction in line with client's comfort level when operating diamond dermabrasion.
- (13) Rubber ring must be mounted when operating diamond dermabrasion, otherwise there is no suction or suction will be weakened.
- (14) Filter cotton must be set when operating diamond dermabrasion, otherwise there is dirt after having skin filtered.
- (15) Clean the instrument with normal saline after operation so as to ensure its cleanliness and hygiene, as well as extending its service life.
- (16) LED face mask can be used separately, or be used with essence and facial mask device which can achieve a better effect.
- (17) LED face mask can't be soaked in water but can be cleaned with wet cloth.
- (18) Using instrument or training operators in strictly accordance with instructions specified in user manual.

5. Troubleshooting & Solutions

- (1) The instrument can't be started, and the lights on its back is not working?
 - A. Make sure the power cord is plugged into a good power socket.
 - B. Check whether the fuse tube on the back of the instrument is loose or burnt out.
- (2) The instrument is without RF output?
 - A. Please check whether the instrument handle and the plug connecting to equipment body are closely connected.
 - B. Please check whether the treated parts have been cleaned. Grease or oily essential products may cause poor contact between the handle and human body, which will result in no RF output.
- (3) The RF output is weakened?
 - A. Please check whether the handle and the plug connecting to equipment body are closely connected.
 - B. Please check whether there is non-conductive grease on the handle, which may cause poor contact and weaken the output.
 - C. Please check whether the product used is the adaptive product specified by the instrument.
- (4) Diamond dermabrasion is without suction?
 - A. Please check whether the rubber ring part of diamond head is twisted too tight that leads to air leakage.
 - B. Please check whether the rubber ring is installed.
 - C. Please check whether the rubber ring is properly connected to main machine, or whether there is damage to the rubber ring.
- (5) LED face mask is without output?
 - A. Please check whether the port connecting to connecting wire is loose.
 - B. If the above-mentioned solutions are not working, please contact the instrument dealer for assistance.

- (6) The instrument can be started, but there is error message on the monitor?
- A. Take out the plug in the back of the instrument, and wait for about 1 minute and re-plug and restart it.
 - B. If the above-mentioned solutions are not working, please contact the instrument dealer for assistance.

6. FAQs

(1) Q: How long does the effect of RF treatment can be seen?

A: Normally the effect can be seen on the same day or within a week. The skin collagen tissue, under heating, produces contraction, which results in an obvious skin tightening. RF can regenerate collagen continuously because it stimulates subcutaneous tissue. Therefore, the more you treat it, the more obvious the effect would be.

(2) Q: Is RF harmful to skin?

A: RF for tightening and anti-wrinkles is a non-surgical program. It stimulates collagen regeneration on underlying skin and accelerates metabolism. Therefore, it causes no harm to skin. Only local redness and heating will occur after treatment, which is a normal phenomenon caused by accelerated blood circulation, and it will disappear itself after a moment. Thus, there's no need to worry.

(3) Q: Does skin will become thin and sensitive after diamond dermabrasion treatment?

A: It will not happen. Diamond dermabrasion only gets rid of aged cuticle during treatment. Our skin refreshes every 28 days, thus diamond dermabrasion only makes skin become more and more smooth, tender, transparent and causes no sensitive symptoms.

(4) Q: Is there any dependency for this treatment?

A: There is no dependency. It's target at skin problems, help it improve, and cleanse deep skin, and maintains and prevents skin from having problems caused by other factors, and helps skin absorb nutrition, solve yellowish skin and lack of moisture, as well as rough skin. If quit in later stage, the skin will only back to previous conditions of secretion and senility, and it will not cause dependency.

(5) Q: How long does it take to see the effect?

A: Different skin can absorb different amount of LED light. It depends on individual body condition, age, skin condition, dietary, and drinking and smoking habits. Normally the effects can be seen within 4 to 8 weeks. For some people it can even be seen within 1 to 3 weeks, while for others it maybe seen within 9 to 11 weeks which is not happen very often.

(6) Q: Does it work using LED Photorejuvenation with face having acnes?



A: Of course. The specific blue wave band has the effect of shrinking pores, eliminating grease, relieving skin inflammation, repairing dark ances, doing sterilization and removing acnes. Different wave band has different effects, which can be chosen according to your individual demand.

7. Packing List

Main Equipment x1

Diamond Dermabrasion Wand x3
 Diamond Head x9
 Filter Cotton x1 pack
 Rubber Hose x1
 3-polar RF Handle x1
 LED Face Mask x1
 Power Cord x1
 Black Rubber Ring x1 pack
 Parts Rack x1
 Remote Control x1

8. Operational Diagrams




Parameters Setting	Product	Techniques	Diagrams
Skin Cleansing & Rejuvenating: 60 Minutes/ Once a Week			
Diamond Dermabrasion: Suction: 20%-80%. Suction and operating time can be adjusted according to different skin type (it's advised to have a trial on lower arms first after set the parameters before operating on face, and ask the client whether the suction is proper.) LED Face Mask:	Makeup Remover + Facial Cleanser + Educe Liquid + Base Mask + Cold & Hot Steam + Essence + Facial Mask + Instrument	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 minutes. 2. Apply educe liquid and use hot steam (apply the liquid on parts with large pores or with severe clog. Cheeks can be applied with toner, and face with sensitive skin can be applied with toner and used with cold steam. Hot steam and educe liquid is not advised.), 5 to 8 minutes. 3. Techniques for Diamond Dermabrasion: Touch the instrument closely to forehead and leave it there for 1 second, then revolve the instrument gently with fingers and bring it up (foreheadnosechin), 5 minutes. 4. Cheek Techniques: Start with chin, aligning three points and treating it again after every two to three points had been treated. Treat one line after another till Temple. 3 minutes. 5. Do the same on the other side, 3 minutes. 	Technique 3  <hr/> Technique 4  <hr/>


<p>Energy Level: 7 to 10 Time: 15 to 30 minutes</p> <p>Mode: It can be chosen according to client's demands.</p>	<ol style="list-style-type: none"> Clean face, 2 minutes. Apply moisturizing base mask, and wait for 2 minutes. Clean face, 2 minutes. Apply facial mask. LED Face Mask: 15 minutes (select the color and set the time and energy level, and then fasten it onto face.) Clean face, 2 minutes. Apply toner, essence and facial cream to prevent against the sun. Treatment done. 	
--	--	--



Course of treatment recommended:




One treatment a week, and ten treatments is a course. After one treatment, skin will become clean, flat and smooth. After a course, skin will become smooth and delicate and glossy. After two courses, skin will become tightened, transparent and elastic. After three courses, elasticity, lustre and ruddiness of skin will be maintained.

Anti-wrinkles & Skin Tightening: 60 Minutes/ Once a Week

<p>Eyes RF:</p> <p>Suggested time: 15 to 20 minutes</p> <p>Suggested energy level: 3 to 7</p> <p>LED Face Mask:</p> <p>Energy Level: 7 to 10</p> <p>Time: 15 to 30 minutes</p> <p>Modes: To choose color according to client's demand.</p>	<p>Makeup Remover + Facial Cleanser + Cold & Hot Steam + Massage Cream + Base Mask + Essence + Facial Mask</p>	<ol style="list-style-type: none"> Remove makeup and clean face, 5 minutes. Apply moisturizing mask and use hot steam, and wait for 10 minutes. Clean face, 2 minutes. Apply toner, 1 minute. Apply massage cream evenly on face and caress it. Repeat 3 times. Press Chengjiang (REN-24), Renzhong (DU-26), Dicang (ST-4), Jiache (ST-6), Yingxiang (LI-20), Jingming (BI-1), Cuanzhu (BI-2), Yuyao(EM3), Sizhukong (TE-23), Temple, Tongziliao (GB-1), Chengqi (ST-1), and Sibai (ST-2). Repeat 3 times. Caress whole face, and do it for 3 times. 	<p>Technique 5, 7</p>  <p>Technique 6</p>  <p>Technique 8, 17</p>  <p>Technique 10, 12, 19</p>
--	--	---	---

		<p>8. Lift in three lines respectively with two hands doing alternately, from chin to earlobe, from the corner of mouth to ear gate, from nose wing to Temple, from lower eyelid to Temple, and lift the corner of eye. Repeat 3 times.</p> <p>9. Repeat the same on the other side.</p> <p>10. Lift forehead towards hairline direction. Repeat 3 times.</p> <p>11. Flip and lift face with rotating finger. Repeat 3 to 5 times.</p> <p>12. Flip the forehead towards hairline direction with single hand. Repeat 3 to 5 times.</p> <p>13. One side: Lift and flip face with two hands doing it alternately. Do it for 2 to 3 minutes.</p> <p>14. Do the same on the other side.</p> <p>15. Move zigzag on whole forehead with middle and ring finger. Slide to the front of ear and lift to and fro 3 times, and pass over the back of ear.</p> <p>16. Treatment done.</p> <p>17. Eyes RF Techniques: Lift one line after another and upwards from chin to ear gate, from the corner of mouth to ear gate, from nose wing to Temple. Repeat 3 times.</p> <p>18. Do the same on the other side.</p> <p>19. Lift from forehead towards hairline direction.</p>	
--	--	--	---



		<p>Repeat 3 times.</p> <p>20. Clean face, 2 minutes.</p> <p>21. LED Face Mask: 15 minutes. (select the color and set the time and energy level, and then fasten it onto face.)</p> <p>22. Apply facial mask, and wait 15 minutes.</p> <p>23. Clean face, 2 minutes.</p> <p>24. Apply toner, essence and facial cream to prevent against the sun.</p> <p>25. Treatment done.</p>	
<p>Course of treatment recommended:</p> <p>Ten treatments is a course. After one treatment, the skin will become tightened and rejuvenated, and be replenished with the needed nutrition. After a course, the skin will be improved and turn glossy. After two courses, the skin will become smooth, plump and its outline will become clearer gradually. After 3 courses, metabolism and detox for skin will be enhanced. And it helps to fade pigment and dark skin, and to prevent and delay the aging, loosening and sagging of skin, and to prevent collagen hyperplasia on the bottom layer of skin, which will make skin tender and smooth and tightened and with gloss.</p>			
<h2>Eyes Wrinkle Removal: 25 Minutes/ 2 to 3 Times a Week</h2>			
<p>Eyes RF:</p> <p>Advised time: 10 to 15 minutes</p> <p>Advised energy level: between 3 and 7</p>	<p>Cleansing Oil + Facial Cleanser + Essence/Eye Cream + Instrument + Eye Mask</p>	<ol style="list-style-type: none"> 1. Remove make-up and clean face, 5 minutes. 2. Apply toner, 1 minute. 3. Apply essence (eye cream) evenly on eyes with hands moving circlewise. Repeat 3 times. 4. Techniques for Eyes RF: Set energy parameters, mode and time (about 10 minutes). 5. Lift instrument from lower eye lid to the corner of eye. Repeat 3 to 6 times. 6. Lift instrument from lower eye lid to Temple. Repeat 3 to 6 times. 7. Lift from lower eye lid moving circlewise to Temple. Repeat 3 to 6 times. 	<p>Technique 3</p>  <p>Technique 5</p>  <p>Technique 6</p>



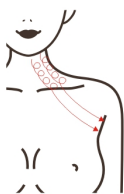

		<ol style="list-style-type: none"> Lift the instrument from the lower eye lid to Temple. Repeat 3 to 6 times. Lift instrument from brow ridge to hairline. Repeat 3 to 6 times. Treat the other side. Clean eyes, 1 minute. Apply eye mask, and wait for 15 minutes. Remove the mask and clean, 2 minutes. Apply eye essence and cream. Treatment done. 	 <p>Technique 7</p>  <p>Technique 8, 9</p> 
--	--	---	---

Course of treatment recommended:

It's advised to take this treatment 2 to 3 times in a week. After one treatment, eyes will be improved and tightened, and blood circulation for eyes will be accelerated. After one month, fine lines will be faded, black eyes relieved, skin color brightened. After three months, eye skin will be firmed, rejuvenated and with gloss. Stick to this treatment, eyes aging will be improved and prevented.

Neck Maintenance: 60 Minutes/ Once a Week

Eyes RF: Energy Level: between 3 and 7 Operating Time: 10 to 15 minutes LED Face Mask: Energy Level: 7 to 10	Makeup Remover + Facial Cleanser + Massage Cream + Essence + Neck Mask	<ol style="list-style-type: none"> Remove makeup and clean face, 5 minutes. Apply toner, 1 minute. Massage neck, and lift in the front of chest and apply oil with both hands, and move to the back of neck, and press Fengchi (GB20) and Fengfu (DU16). Repeat 3 times. Lift jaw and double chin with both hands doing it alternately, and lift to armpit via the back of ear, and pass under armpit. Repeat 3 times. 	<p>Technique 3</p>  <p>Technique 4</p>  <p>Technique 5, 6, 7</p>
--	--	--	--
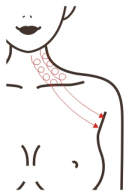
<p>Time: 15 to 30 minutes</p> <p>Mode: Red Light</p>	<ol style="list-style-type: none"> 5. Stroke three channels and collaterals on neck side with the V-shaped webbed area between thumb and index finger doing it circlewise, and till armpit and pass under there. Repeat 3 times. 6. Stroke three channels and collaterals on neck side with four fingers, and till armpit and pass under there. Repeat 3 times. 7. Rub neck side with kneeling fingers till it turns hot, and rub till armpit and pass under there. Repeat 3 times. 8. Stroke external collarbone with the V-shaped webbed area between thumb and index finger, and till armpit. Repeat 3 to 5 times. 9. Do the same on the other side. 10. Treatment done. 11. Eyes RF Techniques: Coupled with hands, lifting double chin to armpit via the back of ear, and pass under armpit. Repeat 3 times. 12. Move instrument circlewise on the side of neck till armpit in three lines respectively, and pass under there. Repeat 3 times. 13. Slide instrument from internal and external collarbones to armpit, and pass under there. Repeat 3 to 5 times. 14. Move instrument circlewise around neck. Repeat 3 times. 15. Do the same on the other side. 	<div data-bbox="1070 203 1342 414">  </div> <p>Technique 11</p> <div data-bbox="1070 481 1342 694">  </div> <p>Technique 12</p> <div data-bbox="1070 770 1342 983">  </div> <p>Technique 14</p> <div data-bbox="1070 1059 1342 1272">  </div>
--	--	---

		16. Wipe and wash the neck with hot towel, 2 minutes. 17. Apply neck mask, and wait for 15 minutes. 18. LED Face Mask: 15 minutes. (select a face mask and set the time and energy level, and then fasten the mask onto face.) 19. Wash with hot towel, and use hot compress for 5 minutes. 20. Apply toner, neck essence and neck cream to protect from the sun. 21. Treatment done.	
<p>Course of treatment recommended</p> <p>Ten treatments is a course. After one treatment, the skin will become rejuvenated and delicate. After a course, the skin on the neck will become smooth, and lymph circulation will be accelerated. After two courses, the skin will become tightened, and fine lines on neck faded, and double chin improved. After 3 courses, the skin will turn delicate and glossy, tightened and plump. And it helps to accelerate lymph detox, and improve darkness on the face, and remove acnes, as well as delaying the aging of skin and rejuvenating it.</p>			

