



# Vaginal Rejuvenation Device

## User Manual

Product Description .....01

Technolgy .....01

Function .....01

Product Structure .....02

Usage.....03

Steps .....03

Note.....03

Technolgy .....03

Normal after operation .....04

Parameter .....04

Storage .....04

Troubleshooting .....05

Maintain .....05

Warranty Regulations .....05

### Product Description

This instrument is a high-tech personal care device that gathers low and medium frequency wave free conversion technology into an all-in-one machine. It can be used to shrink, tighten, increase sensitivity and improve gynaecological inflammation (vaginitis, cervicitis, leucorrhoea, vaginal itching, etc.). This instrument can be used to reduce and tighten the vagina, increase sensitivity and improve gynaecological inflammation (vaginitis, cervicitis, leucorrhoea, vaginal itching etc.). The vagina itself is a natural barrier The vagina itself is a natural barrier to external bacteria. If it is lax, it is very susceptible to bacterial infections a passive movement of contraction, with the wave frequency running through the smooth muscle cell membranes. This causes a muscle contraction response and helps women to regularly eliminate toxins from their reproductive system.

### Technology

It actually use ems technolgo , When the EMS energy reaches the muscle tissue, an electric current runs through the smooth muscle cell membranes, causing a muscle contraction response. It helps women to eliminate toxins from the reproductive system on a regular basis. It keeps the reproductive system fresh, nourished, blood and Qi flowing and endocrine coordination. It also has the miraculous effect of contracting the vagina!


### Function

1. Tighten and shrink vagina, improve sensitivity
2. Improve gynaecological inflammation such as vaginitis, cervicitis and leucorrhoea
3. Training and repair of pelvic floor muscles

Made in china



### Product Structure

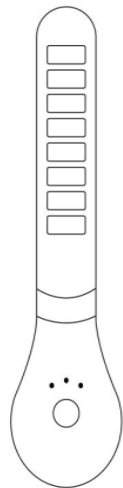

	One light flashing, energy level 1
	One light constantly on, energy level 2
	Two lights flashing, energy level 3
	Two lights constantly on, energy level 4
	Three lights flashing, energy level 5
	Three lights constantly on, energy 6

### Usage

1. Press and hold for 3 seconds to switch the power on/off
2. The green light flashes when the power is switched on, it is recommended to start from the 1st gear and adjust from low to high according to personal acceptance.

### Steps

1. Cleanse intimate areas
2. Place intimate gel
3. Press and hold the on/off button to turn on the instrument according to the strength of the individual (it is recommended to start from 1 position)
4. Slowly insert into the vagina, adjusting the parameters according to the strength of the individual (it is recommended to start from 1 position)
5. After 30 minutes of operation, slowly take out the instrument
6. After cleaning, put the intimate gel into the vagina for 20 minutes
7. During the recovery period of 24 hours, no intercourse, tub bath, hot springs, etc. are allowed.

### Technology

It actually use ems technolgo , When the EMS energy reaches the muscle tissue, an electric current runs through the smooth muscle cell membranes, causing a muscle contraction response. It helps women to eliminate toxins from the reproductive system on a regular basis. It keeps the reproductive system fresh, nourished, blood and Qi flowing and endocrine coordination. It also has the miraculous effect of contracting the vagina!

### Note

1. Do not place the instrument and related accessories in a humid environment, and often keep the instrument clean and should be handled with care, the wire used for the instrument should be placed properly and should not be twisted, folded or squeezed by foreign objects.
2. the probe must be thoroughly cleaned with alcohol wool

after each use, because the life of the probe depends on the cleanliness, also avoid scratching the probe.

3. 1-2 treatments per week.

### Normal after operation

1. Itching, redness and swelling of the vulva, due to mucus flowing out of the vagina with bacteria not cleaned Itching of the vulva.
2. Itching in the vagina
  - (1) Itchiness is caused by machine stimulation of the muscles of the vaginal wall contracting passively and regenerating the cells of the vaginal wall
  - (2) Inflammation in the vagina itself and the machine stimulates the internal environment of the vagina, resulting in a change in the vaginal environment and inflammation , so There will be itching

### Parameter

Product Name	Vaginal Rejuvenation Device
Voltage	7.4V
Dimension	203*54*52 (MM)
Weight	300g
Material	ABS+PC+Alloy
Operating time approx.	2h
Package size	25*9*6cm
Technology	EMS

### Storage

- Do not store this product in hot, humid or wet places or in areas exposed to sunlight.
- Keep out of the reach of children.

### Troubleshooting

If the following conditions apply, follow the corresponding instructions.

Phenomenon	Reason	Method
Work indicator not illuminated	Main unit is discharged	Please charge it
Does not work even if product is used correctly	Damaged wires or faulty components	Please contact the supplier

### Maintain

- Before storing in a bag or box, clean the surface of the machine with a wiping paper towel or flexible cloth (do not attach with skin care products or store it directly after wash with water).
- Do not use cleaning agents, banana water, alkaline detergents, alcohol, thinners, volatile oils, nail shampoos or other solvents that can damage the body of the machine. Do not use solvents such as hair oil, nail shampoo, etc.
- Do not leave this product in a condition where moisturising water or water is attached.

### Warranty Regulations

If the product fails or is damaged under normal use in accordance with the manual, it will be repaired free of charge within one year from the date of purchase.

Failure or damage caused by human error or not using the product according to the manual is not covered by the warranty.

Product failure or damage caused by user's own dismantling and repair, modification, etc. is not covered by the warranty.

Failure or damage caused by strong collision, excessive high or low voltage, or force majeure Not covered by the warranty.

In the event that the above is not covered by the warranty, we will determine whether the repair can be carried out and while repairing it We will charge for parts and repairs as appropriate.