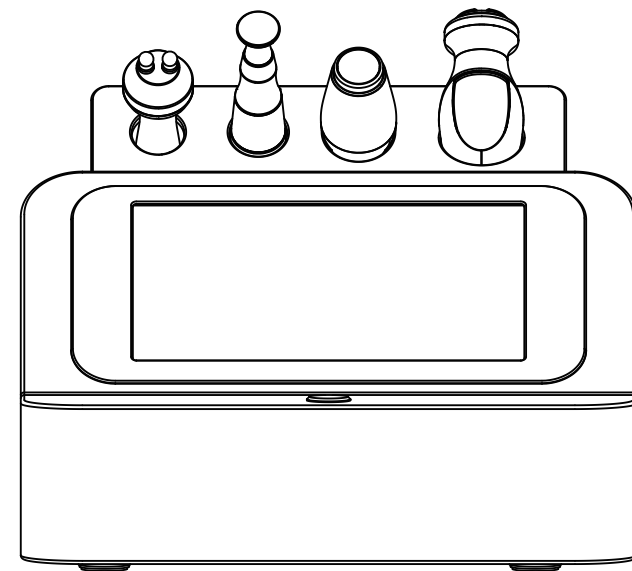
User Manual

PS-45K1

Content

1.Safety Precautions	1
2.Contraindications	1
3.Overview	2
4.Principle	3
5.Control Panel Introduction	16
6.Technical Specification	19
7.Use Precautions	20
8.Maintenance & Cleaning	20
9.FAQ	21
10.Ask & Answer	22
11.Operation Guide	23
12.Warranty & Support	30

Contenido

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1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 7.Ensure that the instrument is applied on moist skin and avoid the operation on dry skin.
- 8.Power off the instrument when not in use to ensure its safety.
- 9.To maintain hygiene and extend the device's service life, sterilize it with saline.

2.Contraindications

Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details below:

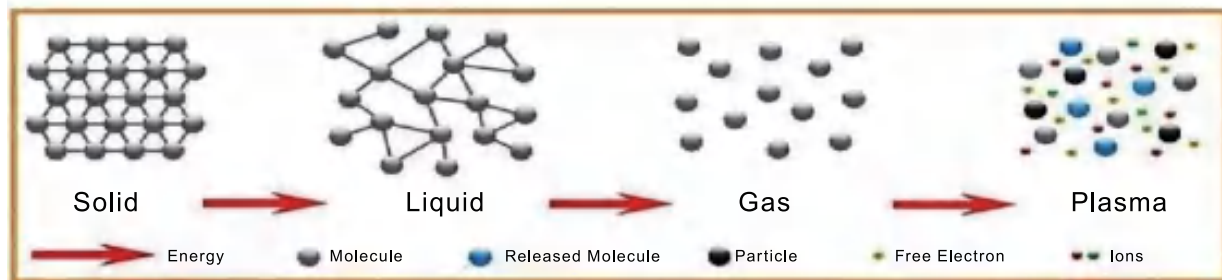
- 1.Pregnancy or lactation.
- 2.Individuals who are sensitive to the micro-current.
- 3.Hypertension, heart disease, or with a pacemaker installed.
- 4.Unhealed wounds or during the recovery phase of surgery.
- 5.Epilepsy, severe diabetes, and hyperthyroidism.
- 6.Malignant tumors, hemophilia, and hemorrhage.
- 7.Skin disease or infectious disease.
- 8.Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
- 9.Had plastic surgery within 3 months or with implants or metal objects in the treatment area.

3.Overview

Featuring the latest and safest technologies, including anti-aging, firming, acne removal, and plasma treatments, this 4-in-1 device guarantees a radiant facial transformation. It's the perfect solution for those seeking to improve their skin's appearance without injections, medications, or side effects. This non-invasive device delivers noticeable improvements with every use.

4.Principle

What is plasma?

When energy is applied to gaseous matter, it transforms into plasma, consisting of negatively charged free electrons and positively charged ions. The energy applied can include heat, AC, DC, or RF.

This product generates plasma using DC current as its energy source.

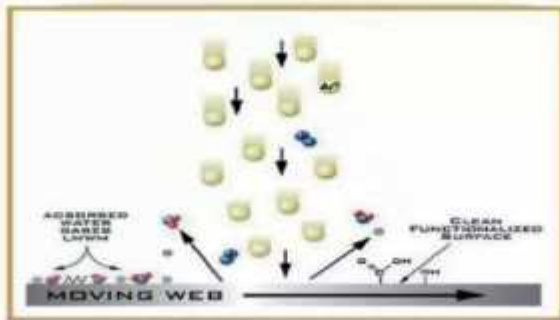
Fast healing, long-lasting results, with no scarring or pigmentation – powered by unique technology.

Mechanism

- Direct current generates spark discharge, and the heat from the spark makes the skin warm. Compared to AC discharge, DC discharge has a more concentrated effect on smaller areas of the skin. DC discharge does not harm the surrounding tissue, which is highly beneficial.
- The direct current discharge creates a conductive connection between the tip of the handle and the skin, with the tip positioned 4 mm away from the surface. The air between the treatment area and discharge point absorbs energy, ionizing the air, creating an electric current, and forming plasma.
- Plasma stimulates cell regeneration, promoting anti-aging, enhancing skin absorption, providing antibacterial cleansing, whitening and brightening the skin, reducing fine lines, boosting elasticity, lifting facial contours, and removing scars.


Plasma Lifting Treatment

Plasma treatment is a low-power, portable DC discharge system


that uses nitrogen and argon ions in the air to create microplasma.

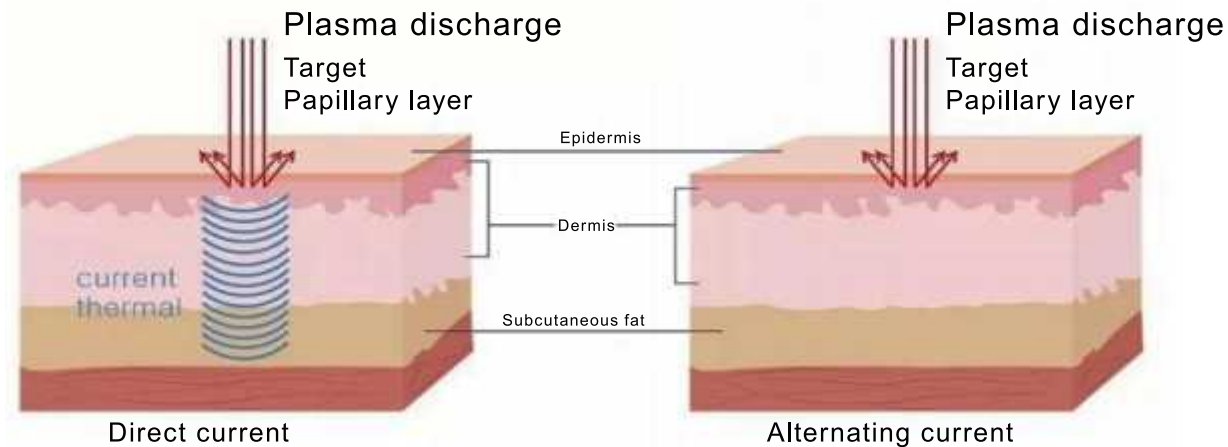What is plasma energy?


What is plasma energy?

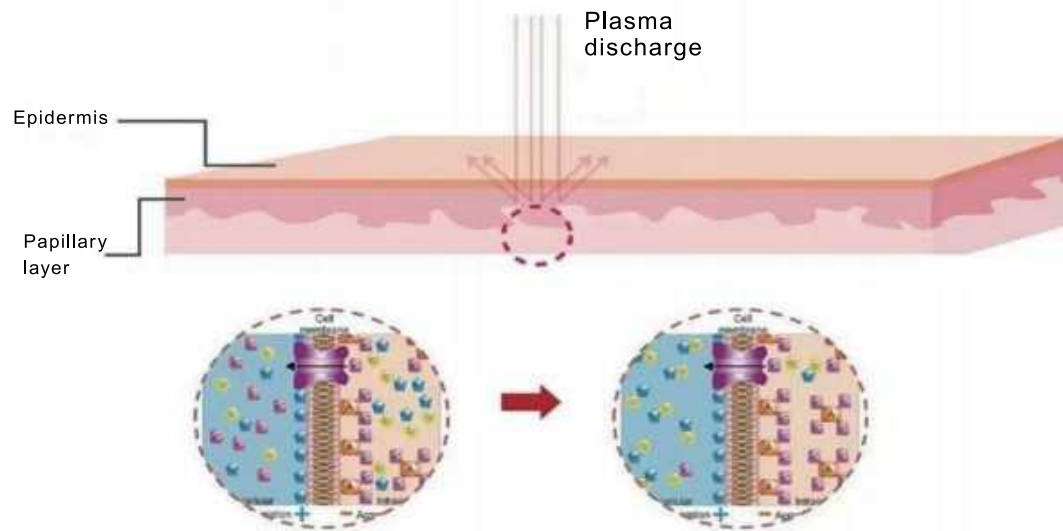Advantages and Benefits

- Unlike lasers, plasma technology works on all body areas as it doesn't rely on specific chromophores.
- Uses microcurrent to generate heat.
- Targets sagging areas, inducing a contraction effect.
- High-energy plasma creates micro-wounds in the papillary layer, stimulating collagen production and skin regeneration.

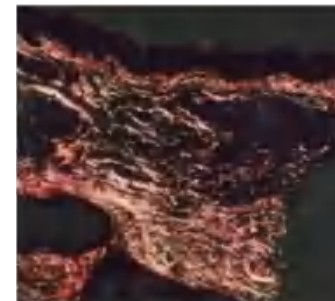
Advantages and Benefits

- New collagen is produced, improving skin elasticity.
- Plasma delivers effective results with low energy.
- Plasma energy stimulates skin regeneration.
- The glowing area represents newly produced collagen in the dermis.

Advantages and Benefits

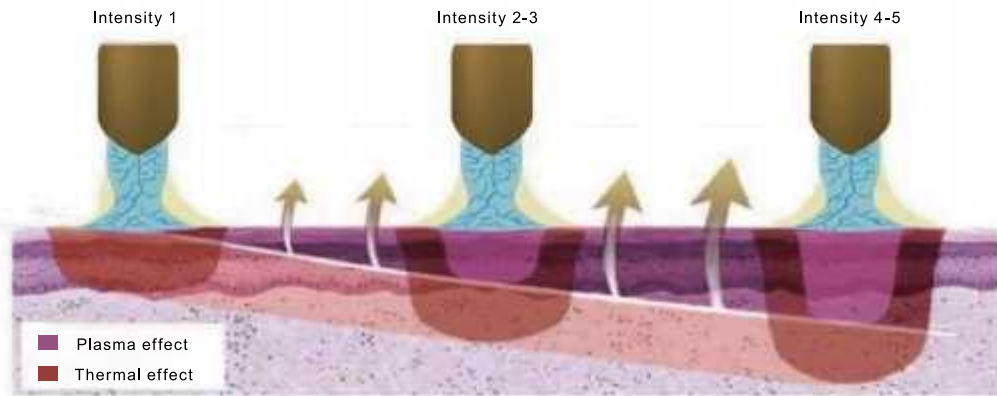Adjust the plasma energy to target the papillary layer directly, ensuring quick results.

Energy Penetration Depth by Tip Types:

Type	No. 1	No. 2	No. 3	No. 4	No. 5	No. 6
Tip Pics						
Contact skin with a medium applied.	2mm	2mm	1.5mm	/	/	/
Avoid skin contact; apply a repairing product after the procedure.	/	/	/	3mm	2mm	3mm
Benefits	Contours jawline, reduces double chin	Tightens skin, brightens tone	Smooths texture, reduces fine lines on the face and neck	Removes scars, moles, tattoos, pigmentation	Boosts nutrient absorption	Reduces crow's feet

Note: Avoid dropping the No. 4 and No. 6 tips during use, as this may affect their lifespan.

What is Ozone?

Ozone is produced by electrolyzing oxygen in the air using high-frequency voltage.

Effect of Ozone

Ozone targets the cell membrane of bacteria, damaging its components and destroying lipoproteins and lipopolysaccharides, leading to cell lysis and death. This process effectively kills bacteria, fungi, and mites, inhibits inflammation, and helps remove acne. After sterilization, ozone is reduced to oxygen, leaving no residue or secondary pollution, unlike chemical systems.

Advantages and Benefits of Ozone Plasma

- Applicable for all body areas.
- No wounds on the epidermis after treatment.
- Promotes skin regeneration.
- Stimulates new collagen production in the dermis.
- Enhances skin elasticity through collagen production.
- Non-invasive, painless, with no scabs, blood, or wounds. More convenient, safer, and effective.

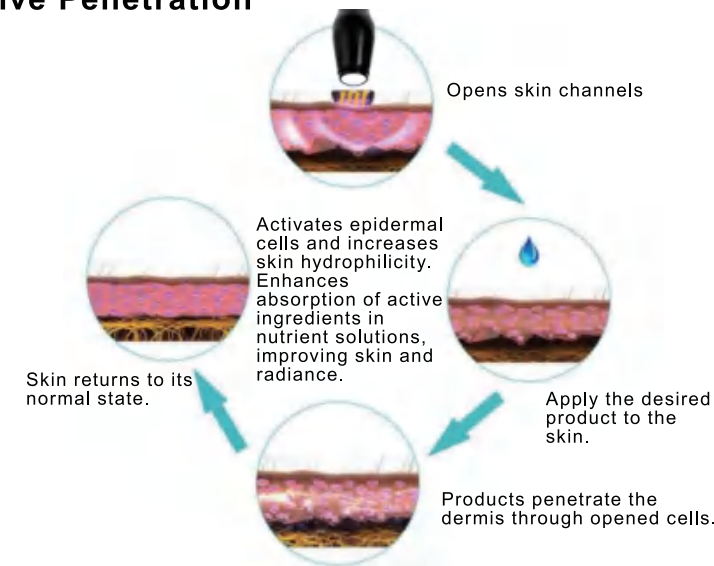
Main Effects of Ozone Plasma

- Enhances the absorption of active ingredients in skincare products, boosting their effectiveness.
- Kills bacteria and reduces inflammation, effectively removing acne, acne marks, blackheads, and preventing recurrence.
- Accelerates skin metabolism, promotes collagen regeneration, and helps combat skin aging.

- Alleviates skin itching caused by allergies and dryness.
- Inhibits melanin, brightens skin tone, refines texture, and shrinks pores.

Ozone Plasma Applications and Impact

1.Highly Effective Penetration

Ozone plasma activates the skin's epidermal cells, increasing its hydrophilicity. This enhances the absorption of active ingredients from nutrient solutions, helping the skin fully absorb their benefits for a glowing complexion.

2.Improves Skin Elasticity and Anti-Aging

As skin ages, the membrane potential of cells changes, affecting ion distribution. Ozone plasma helps regulate the passage of potassium and sodium ions, equalizing the membrane potential, tightening the skin, and improving elasticity.

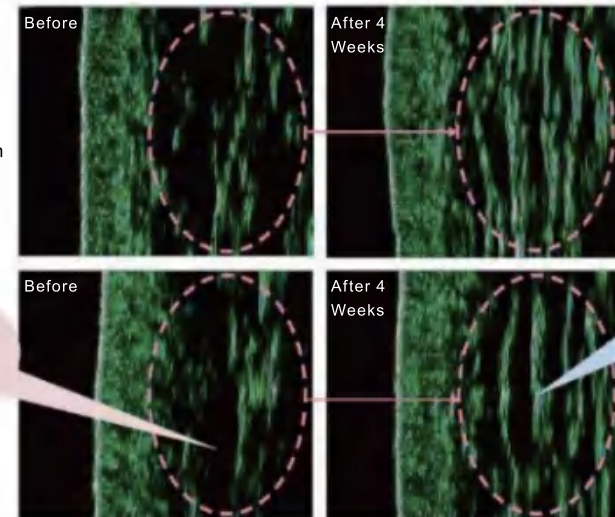
Skin
Regeneration

Stimulates collagen and fibroblast synthesis, prevents skin aging by optimizing cell regeneration.

Skin tissue is loose, appearing hollowed out.

Epidermal Thickness Difference
After Ozone Plasma Use

Tissue becomes firmer and tighter.

Ozone plasma effectively promotes skin regeneration by optimizing the cell regeneration cycle, stimulating the synthesis of fibroblasts, producing more collagen, and achieving anti-aging effects.

3.Acne and Pimple Removal

Ozone plasma exfoliates dead skin cells and kills 75% of acne-causing bacteria, particularly targeting inflammatory acne. It also inhibits sebum secretion, preventing the recurrence of acne.

4.Antibacterial and Anti-Inflammatory

The active oxygen atoms, molecules, and plasma radiation in ozone plasma destroy bacterial cell membranes, DNA, and proteins, providing effective sterilization and reducing inflammation.

5.Control Panel Introduction

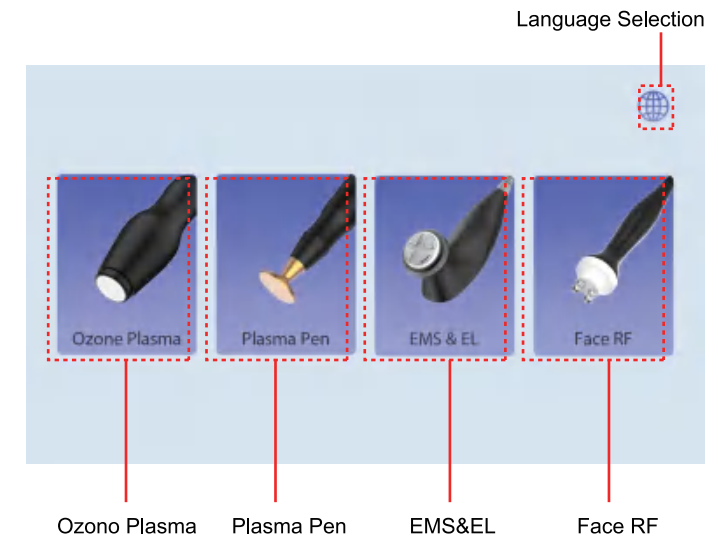
Welcome screen

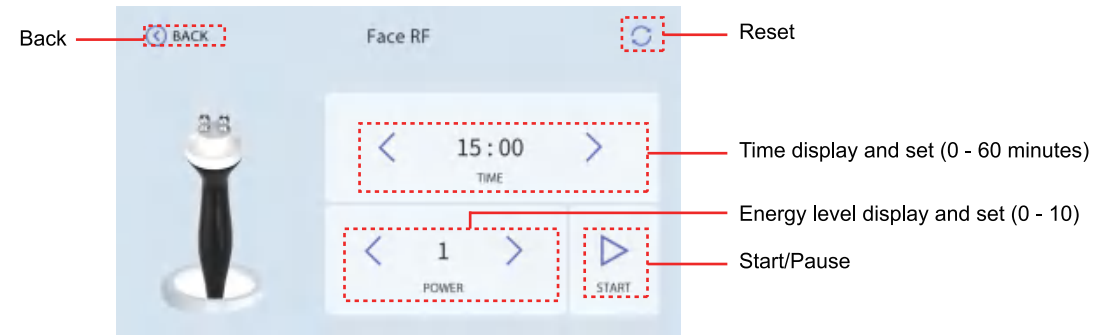
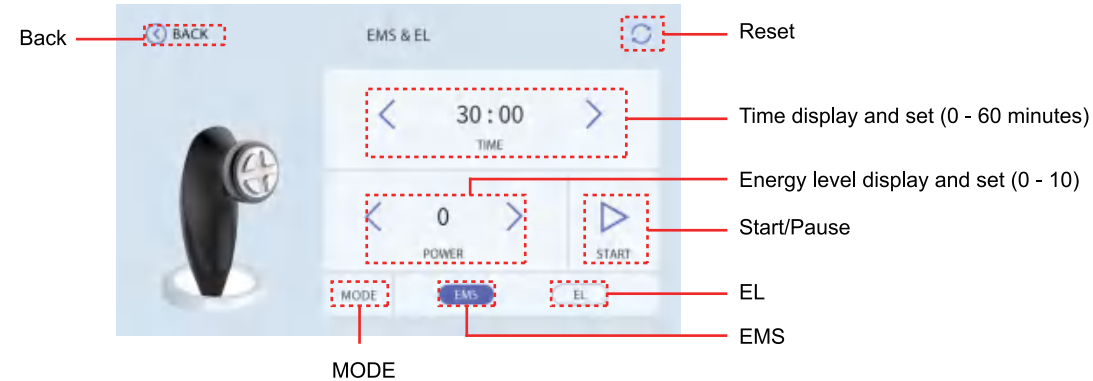
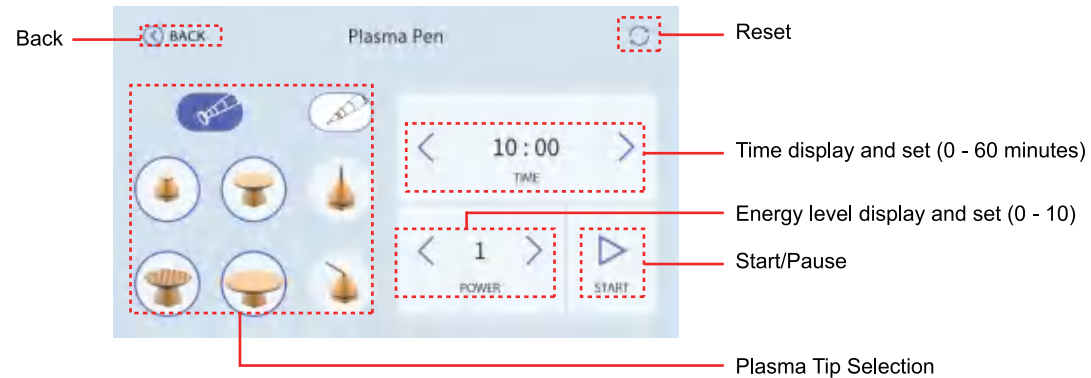
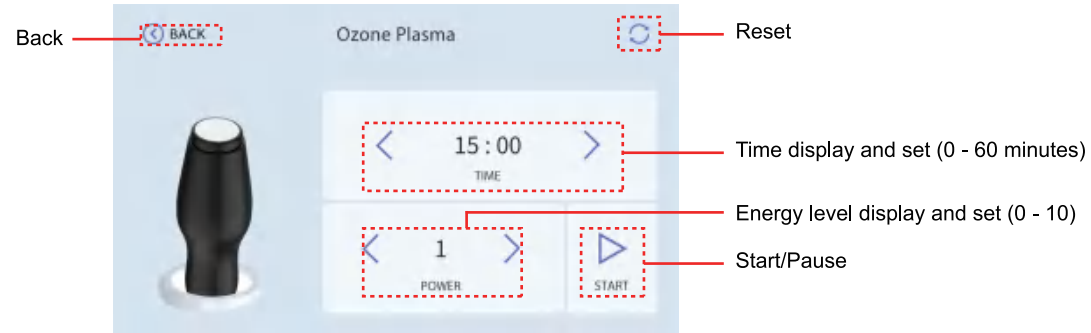
Language selection screen



The handle selection screen appears after selecting "English."

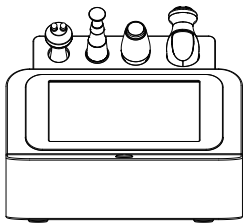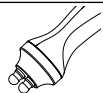6. Technical Specification

	Power supply Input: AC110V-220V 50Hz-60Hz		Net Weight: 7.6KG	
	Power: 50W		Gross Weight: 9.0KG	
	Dimensions: 36*33*31.5cm			
	Input Power: 50W			
	Power: 50W			Power: 20W
	Power: 24W			Pointed Tip Power: 19.8W
				Flat Tip Power: 19.9W

7. Use Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the instrument.
4. Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.
5. Ensure full skin contact with the treatment head throughout the session to avoid discomfort.
6. During the No. 4 tip operation, press the pedal to release energy and release the pedal to pause energy output.
7. During ozone plasma operation, a distinct ozone smell may be noticeable; this is normal.
8. Wipe the tip with saline before and after the No. 4 tip operation.
9. Kindly inform the recipient of a slight shock sensation from the microcurrent during EMS/EL treatment to avoid surprise.

8. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. Clean and sanitize the handle with a damp cloth or saline, then dry it thoroughly with a towel to ensure it remains moisture-free.
3. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.

4. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

9. FAQ

1. When can I expect the results of RF treatment?

A: You might experience skin tightness right after the treatment, as the heat stimulation tightens the collagen tissue, and the RF stimulates the subcutaneous tissue bringing the continuous regeneration of collagen, leading to more noticeable results within one week.

2. Is RF safe for the skin?

A: RF firming and anti-wrinkle treatment is a non-surgical method that stimulates collagen regeneration and metabolism in the deeper layers of the skin without causing any harm. It's normal to experience temporary heat and redness in the treated area, which indicates increased circulation and should subside within a few hours. There's no need to be concerned about this reaction.

3. What is the Mechanism of Mole Removal with Plasma?

A: The plasma pen removes moles by generating high-temperature plasma, which targets the mole, carbonizing it and allowing it to be naturally eliminated through the body's metabolism. This method minimizes damage to the surrounding skin, effectively removing the mole with minimal impact on adjacent healthy tissue.

10. Ask & Answer

Ask		Cause		Answer
1. Fail to turn on the instrument.	→	The power cord is disconnected from the outlet.	→	Check if the power cord is securely connected to the outlet and ensure that the outlet is functional.
	→	The fuse is loose or burned.	→	Replace the fuse on the back of the instrument and restart it.
2. The instrument is not working and can't be restarted.	→	The fuse may have burned out due to unstable voltage or prolonged use.	→	Please replace the fuse and restart the instrument 5 minutes later.
3. The screen shows an error message.	→	Data error.	→	1. Turn off the instrument, disconnect the power cord, and wait for 1 minute to restart the instrument. 2. If the above method doesn't resolve the problem, please contact your dealer.

11.Operation Guide

FACE

Ozone Plasma Handle Application

Handle Selection: Ozono Plasma

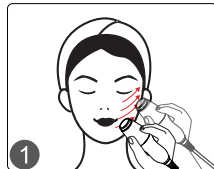
Time: 10 minutes.

Energy level: 3

Pairing product: Keep the skin surface dry without using any medium during application.

Note: Keep the skin surface dry during the procedure.

Treatment Frequency: 1 - 2 sessions per week.

Facial RF Handle Application

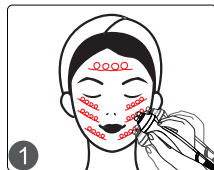
Handle Selection: Face RF

Time: 10 minutes

Energy level: 3

Pairing product: Serum

Treatment Frequency: 1 session per week.

EMS/EL Handle Application

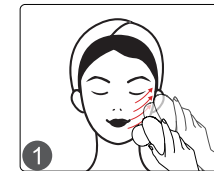
Handle Selection: EL/EL

Time: 10 minutes

Energy level: 3

Pairing product: Serum or massage cream.

Treatment Frequency: 1 session per week.

Plasma Handle Application (Avoid using the control panel while the pedal is pressed and the plasma pen is in contact with the skin.)

Infusion Treatment

Handle Selection: Plasma Pen

Plasma Tip Selection: No. 5

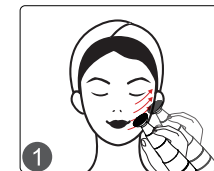
Time: 10 minutes

Energy level: 3

Pairing product: Gel

Note: Position the tip close to the skin without direct contact, then press the pedal. Maintain a 2mm distance between the tip and the skin throughout the operation.

Treatment Frequency: 1 session per week

Firming Treatment

Handle Selection: Plasma Pen

Plasma Tip Selection: No. 2

Time: 10 minutes

Energy level: 3

Pairing product: Gel

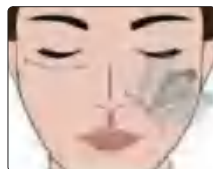
Plasma Tip Selection: No. 3

Time: 5 minutes

Energy level: 3

Pairing product: Gel

Gently press and rotate the treatment head around the nose, the corners of the mouth, and the area around the eyes.

Note: First, press the treatment head onto the skin and glide it smoothly. Then, depress the pedal and avoid lifting the handle during application.

Treatment Frequency: 1 session per week

Neck Lifting

Handle Selection: Plasma Pen

Plasma Tip Selection: No. 1

Time: 10 minutes

Energy level: 3

Pairing product: Gel

Plasma Tip Selection: No. 3

Time: 5 minutes

Energy level: 3

Pairing product: Gel

Press and rotate the handle over the wrinkle area.

Note: First, press the treatment head onto the skin and glide it smoothly. Then, depress the pedal and avoid lifting the handle during application.

Treatment Frequency: 1 session per week.

Mole Removal

Handle Selection: Plasma Pen

Plasma Tip Selection: No. 4

Energy Level & Time: Level 1 – 3, with each treatment lasting no more than 2 seconds.

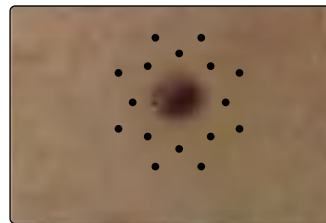
Pairing product: Keep the skin surface dry without using any medium during application.

Note: Position the tip close to the skin, then depress the pedal.

Technique:

1. Do not make direct contact with the skin using the tip; position it close to the skin and you should notice a spark on the skin after the pedal is depressed. Do not treat the target area for more than 2 seconds. This treatment should only be applied to the skin spot.

2. Treat the area around the spot twice, then treat the spot itself twice.

Post-Care for Mole Removal

▲ For extensive areas with deep pigmentation, two to three sessions may be needed; smaller areas with lighter pigmentation can often be resolved in one session.

▲ For deeper scars, repeat treatments at regular intervals are recommended to achieve optimal results.

▲ A slight burning sensation in the treated area is normal. Clean the area daily with a repair serum or erythromycin to promote healing. Avoid using water on the face for one week, and gently clean around the eyes with a cotton swab to prevent infection.

▲ If scabs become itchy during healing, refrain from picking them. They will naturally fall off in about 7-10 days, taking the pigment with them as they shed.

▲ Following scab detachment, temporary redness or hyperpigmentation may appear. Pigmentation will generally fade within one month and normalize within 2-3 months. Protect the skin from bright light for three months to prevent darkening.

▲ During the 1–2-month recovery period, avoid ginger (may affect skin texture), beef (may cause slight redness), soy sauce (may lead to slight darkening), and other photosensitive foods (e.g., celery, leeks, cilantro). Additionally, refrain from consuming alcohol or spicy foods.

▲ Avoid saunas and strenuous exercise to minimize infection risk and support a smoother recovery process.

Eye Lifting

Handle Selection: Plasma Pen

Plasma Tip Selection: No. 6

Energy Level: Level 1 – 2, with each treatment lasting no more than 2 seconds.

Pairing product: Keep the skin surface dry without using any medium during application.

Note: Do not allow the tip to make direct contact with the skin; position it close to the wrinkle area. You should notice a spark on the skin after pressing the pedal. Do not treat the target area for more than 2 seconds. This treatment should only be applied to the wrinkles at the corners of the eyes.

1. Schedule the treatment area.
2. Apply anesthesia to the scheduled area as indicated (superficial anesthesia).

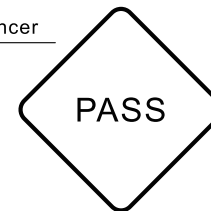
3. Spot
Perform the spot technique only on the corner of the eye for 2–3 seconds.

12. Warranty & Support

We offer a comprehensive one-year warranty and a one-month return and replacement policy starting from the date of purchase.

Product Name: Versatile Facial Beauty Enhancer
 Product Model: PS-45K1
 Production Date: _____
 Quality Inspection: _____

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