

User Manual

SR-040SE



Content

1.Safety precautions	1
2.Contraindications	1
3.Overview	2
4.Control Panel Introduction	2
5.Technical Specification	8
6.Use Precautions	9
7.Maintenance & Cleaning	9
8.FAQ	10
9.Ask & Answer	11
10.Operation Guide	12
11.Warranty & Support.....	14

1.Safety precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
- 2.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 6.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 7.Ensure that the instrument is applied on moist skin and avoid the operation on dry skin.
- 8.Power off the instrument when not in use to ensure its safety.
- 9.To maintain hygiene and extend the device's service life, sterilize it with saline.

2.Contraindications



Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details below:





1. Pregnancy or lactation.
2. Hypertension, heart disease, or with a pacemaker installed.
3. Unhealed wounds or during the recovery phase of surgery.
4. Sensitivity to micro-currents
5. Epilepsy, severe diabetes, and hyperthyroidism.
6. Malignant tumors, hemophilia, and hemorrhage.
7. Skin disease or infectious disease.
8. Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
9. Had plastic surgery within 3 months or with implants or metal objects in the treatment area.





3. Overview

As a versatile facial beauty device, it offers a radiant transformation using advanced technologies such as deep cleansing, hydration, and skin firming and lifting. It's the perfect solution for those seeking to improve their skin's appearance without injections, medications, or side effects. This non-invasive device delivers noticeable improvements with every use.

4. Control Panel Introduction

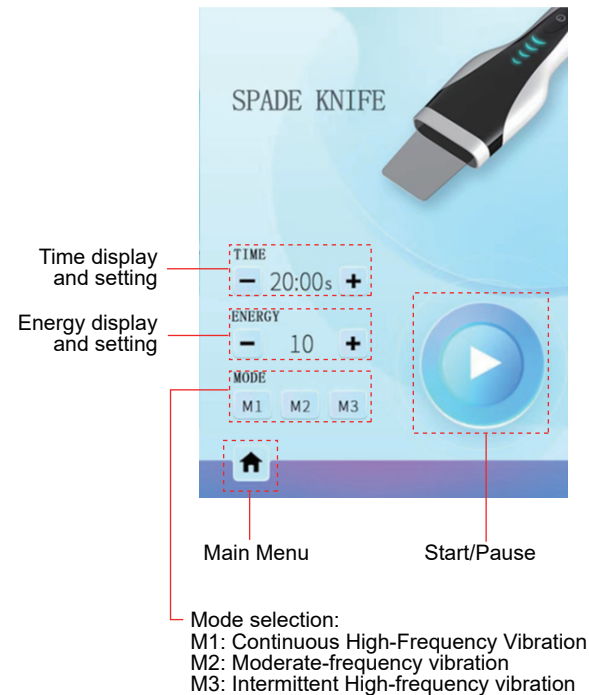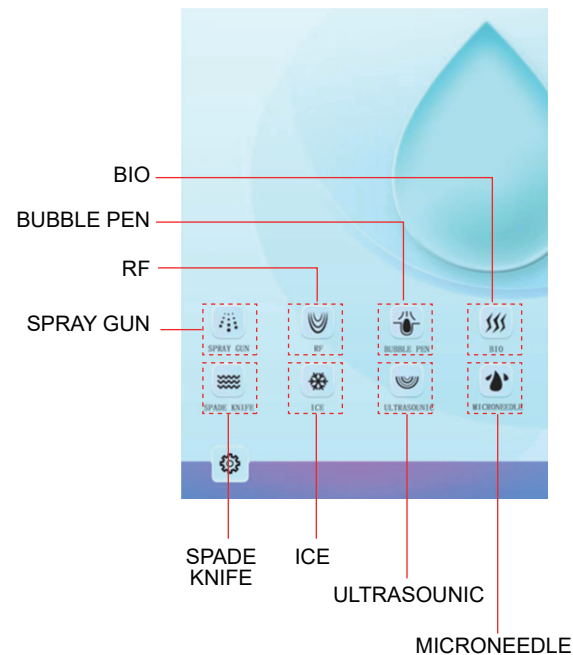

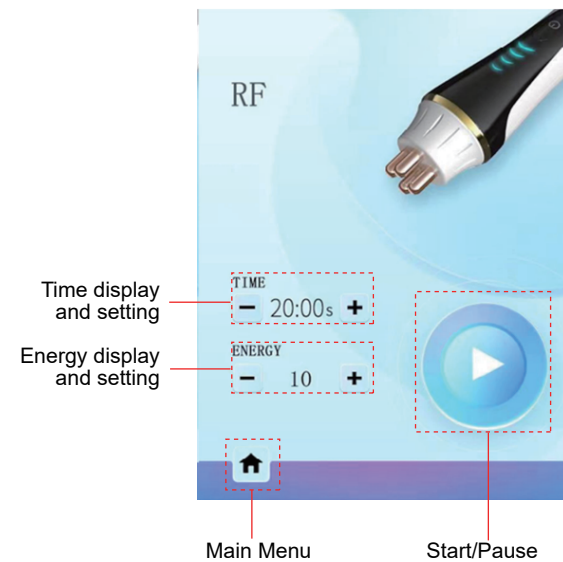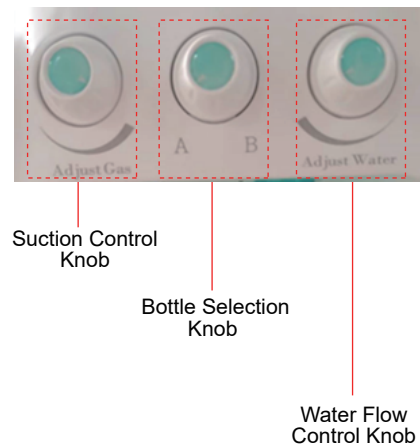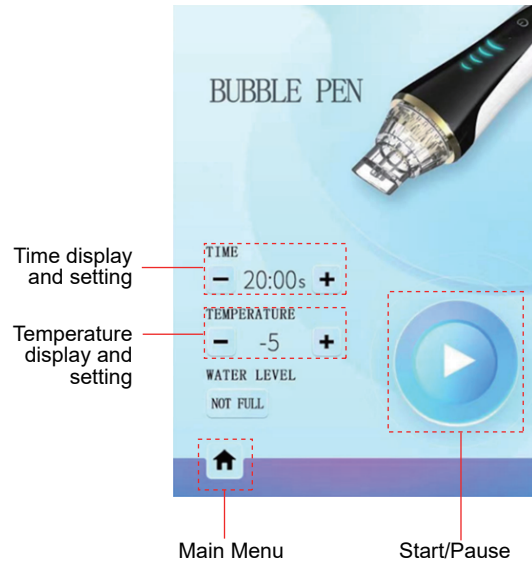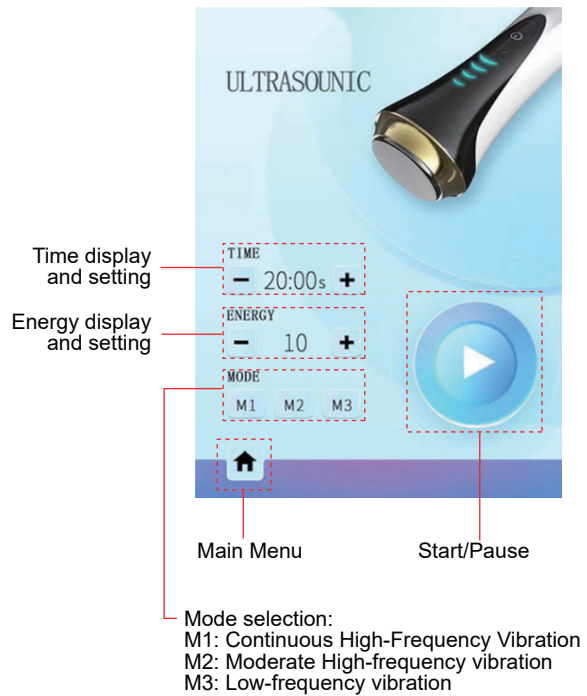

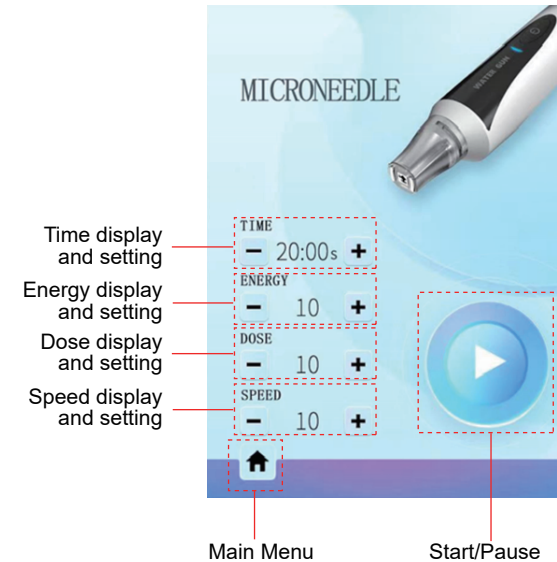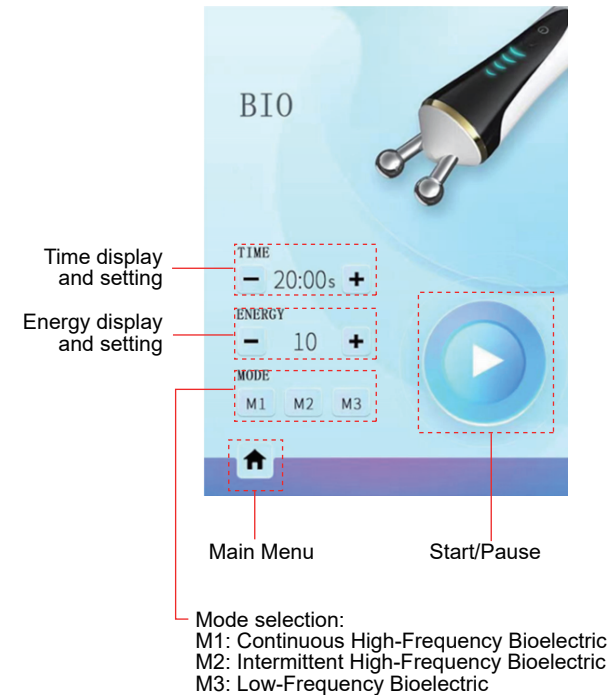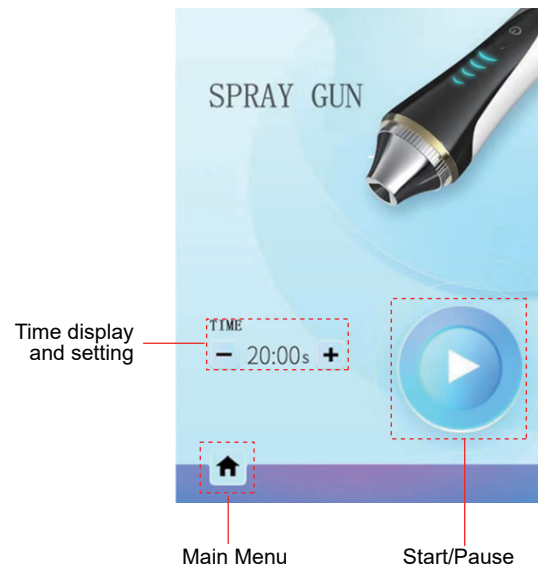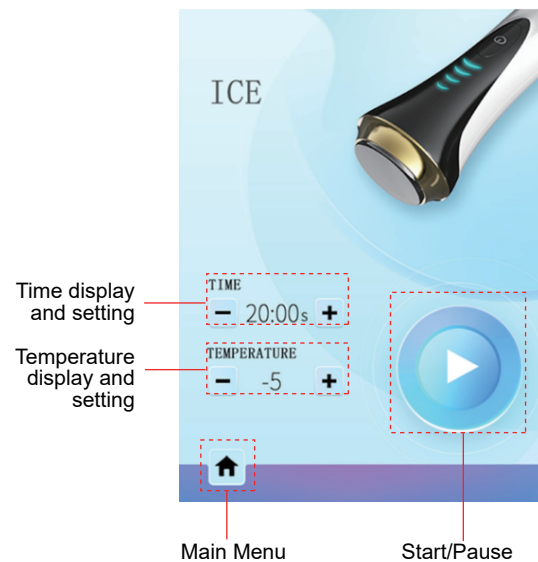

5. Technical Specification

Vacuum range	700mmHg
Output control	Touch Screen & Handle Switch
Gold RF	Fast response multipole (energy 10-level adjustment)
Electrophoresis	1MHZ (energy 10-level adjustment)
Ultrasound	1MHZ (energy 10-level adjustment)
Frozen refrigeration head	Water cooling & air cooling (energy 10-level adjustment)
High frequency scrubber skin knife	25KHZ (energy 10-level adjustment)
Ion airbrush pen	2000mmHg
Interface	10.1 inch touch screen
Input voltage	110-240V50-60HZ 150AV

6. Use Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the instrument.
4. Maintain the treatment head in full contact with the skin to avoid potential discomfort.
5. Make sure the treatment area skin is moist, do not operate on dry skin.
6. Choose a watery product instead of a thick one for the Spray Gun, as thick products may cause blockages and damage the device.

7. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. After use, rinse the Bubble Pen in saline, clean it with a brush, allow it to air dry, and store it in a dry condition.
3. Clean and sanitize the handle with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.

4. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.

5. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8. FAQ

1. Will the skin become thinner and more sensitive after using the Bubble Pen?

A: No, the Bubble Pen exclusively targets comedones, enhancing nutrient absorption for smoother, finer, and more translucent skin without causing sensitivity.

2. Why do I need exfoliation?

A: The stratum corneum is the outermost layer of the epidermis, consisting of 15–20 layers of skin components that vary depending on the body part. The skin's metabolic cycle also differs with age. For example, it's 28 days for teenagers, while for adults, it's their age plus 10 days. However, due to various factors, the stratum corneum may not shed as expected, leading to a thicker layer, a dull complexion, and slower skin metabolism. In such cases, additional exfoliation is essential to support healthy skin renewal.

3. Can I apply makeup immediately after using this device?

A: Yes, the treatment helps achieve moisturized and plump skin, ensuring seamless and well-blended makeup application. This non-invasive approach delivers fast, safe, and healthy results.


4. When can I expect to see results?


A: Immediately after the initial session, you will observe a noticeable improvement in your skin's clarity and transparency. Multi-sessions contribute to more visible effects including deeply hydrated skin, brightened complexion, reduced fine lines, and prevention of the aging process.

9. Ask & Answer

Ask	Cause	Answer
1. Fail to turn on the instrument.	→ The power cord is disconnected from the outlet.	→ Check if the power cord is securely connected to the outlet and ensure that the outlet is functional.
	→ The fuse is loose or burned.	→ Replace the fuse on the back of the instrument and restart it.
2. The instrument is not working and can't be restarted.	→ The fuse may have burned out due to unstable voltage or prolonged use.	→ Please replace the fuse and restart the instrument 5 minutes later.

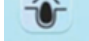

10 .Operation guide

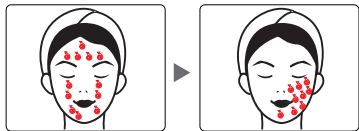
Cleansing and Hydrating (Step1:  mins)






(> Step2:  2mins)

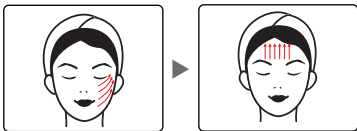

(> Step3:  5mins> Step4:  10mins)

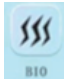

Facial Anti-aging (Step1:  10mins>Step2:  10mins>Step3:  10mins)



Microneedle ( 20mins)



11.Warranty & Support

We provide a one-year warranty and a three-month return and replacement policy from the date of purchase.

Product Name: Versatile Facial
Beauty Enhancer _____
Product Model: SR-040SE _____
Production Date: _____
Quality Inspection: _____

