
USER MANUAL

SR-MJ1809



Preface

The integrated management device will allow a VC solution to penetrate into the pore dermis layer, which can achieve deep whitening and stimulating collagen production. Of course, after skin care, there must be a high-quality mask to protect the skin, such as collagen, vitamins, green tea, and recycled series are good choices. In addition, the iontophoresis treatment of the skin comprehensive management device can also improve the absorption rate of hard skin, promote blood and lymph circulation, improve skin repair power, promote the discharge of skin and the human body, and at the same time, remove hair follicle endothelial fat and keratin. Prevents pigmentation and enhances the whitening effect.

The skin comprehensive management instrument is an indispensable skin beauty instrument, which can meet the basic needs of female friends to deeply clean the skin, whitening and hydrating, and burning fat and shaping one step at a time. The skin comprehensive management device can remove sebum and secretions, and can be used to clean the face with the blackhead derivation fluid, and can also play a sterilizing role throughout the cleaning process.

Catalog

Preface

Part I

Working principle

Applicable people and effect description
--

Part II

Detailed operation

Instrument technical parameters

User taboo

Precautions for use and maintenance

Operational precautions

Common fault handling methods

Frequently asked questions and answers
--

Package Included

Operation

Part I

I. Working Principle Of Double Cleaning

I. Cleaning shovel

Shovel knife can change technology, 28000 times per second, mechanical vibration energy emitted from the skin to the human body has obvious osmosis, through the medium of atomization, have stronger impact can instantly remove deep dirt from the pores, painless block pore reduction of acne, acne, make skin thoroughly clean.

The sebum and secretion induced by low-frequency emulsification can also be sterilized to spray light mist into the pores, remove sebum and secretion function, and remove dead skin.

Physical effects: high frequency vibration sound wave is combined with softening products or exfoliating cream to soften the damaged rough epidermal cuticle and open the pores with sound wave to remove the waste and wastes in the pores.

On the back of the instrument, high frequency vibration combined with the essence product can introduce the product into the skin, increase the elasticity of the skin and smooth away wrinkles.

II. Microdermabrasion

The second time is clean and tight, the skin is polished with diamond crystal specially processed for the patient's skin, and the dead cells of the skin are sucked out by vacuum to remove the deep dirt and wastes in the pores, so as to make the skin doubly clean and promote the growth of new skin. USES the negative pressure function to dispel the skin deep hair cysticercosis mite black head white head at the same time will block the pore dredge.

Biological effect: through negative pressure to produce suction, combined with frosting head operation, can affect the skin surface of the metabolites and pores deep black head, garbage, plug out, equivalent to the skin of the vacuum cleaner, to thoroughly clean the skin.

III. ION+ION-

It dissolves too much sebum in the skin and leads the impurities out of the body. It can form a weak electric field between the skin surface and the deep layer, temporarily breaking the skin barrier layer.

Biological effects: Use the principle of positive and negative ion anisotropic, we use

positive ions to excrete harmful substances such as metal ion components and residual aging substances deposited in the skin.

IV. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism and regenerative capacity soften tissue, stimulate the nervous system and cell function, and make the skin elastic.

Warming effect: By the warming action of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, the nutrients of the skin cells can be increased, the nerve excitability can be reduced, and the analgesic effect can be achieved, so that the muscle fibers of the tendon are relaxed. To the role of unlocking. The heat of the ultrasound is endogenous heat, 79%-82% of the heat is transported away from the active area by the blood, and 18%-21% is dispersed by heat conduction to the adjacent tissue. Therefore, the customer has no obvious heat feeling.

Chemical action: Ultrasound can enhance the catalytic ability, accelerate the metabolism of skin cells, change the PH value of the tissue to the alkaline direction, reduce the acidosis and pain associated with skin inflammation, and ultrasonic can improve the permeability of cell membrane and make nutrients and drug solutions. Poly, which is good for the skin to absorb nutrients, which is good for the drug to penetrate into the bacteria and improve the bactericidal ability.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can cause skin cells to vibrate, produce a fine massage effect, change the cell volume, and thus improve local blood and lymph. The circulation enhances cell permeability, enhances tissue metabolism and regeneration, softens tissue, stimulates the nervous system and cell function, makes skin shiny and elastic, and ultrasonic waves make diffusion faster by vibrating energy, and high-yield nutrient products Frequent vibration and thermal energy enter the skin more subtly. The warmth of ultrasonic waves can penetrate into the skin six centimeters, smoothing the circulation of blood and lymph, accelerating the blood circulation of cells, and initiating the biological effects of cell tissue, which can dilute various stains and make The skin is whitened, and cells in deep tissues are activated and regenerated.

V. Efficacys

1. Atomize the water, penetrate the pores, and bring out the dirt through high-frequency vibration.
2. Refine the nutrients into small molecular particles by high-frequency vibration, strengthen the skin absorption, and feed directly to the dermis layer.
3. Improve skin texture, large pores, wrinkles, thick keratin, purify the skin, remove excess

thick keratin, purify pores, break down fat and smooth pores, clear and smooth skin, etc.

4. Remove dark circles. Use eye care essence to repair and remove dark circles, eye bags and fine dry lines.

5. Increase the elasticity of the skin to prevent muscle relaxation, rough skin, wrinkles and skin aging.

6. Effectively discharge the metabolites on the surface of the skin and the blackheads, garbage and blockages deep in the pores, which is equivalent to the vacuum cleaner of the skin, and thoroughly clean the skin.

7. It is difficult to eliminate harmful substances by reducing the amount of metal ions deposited in the skin.

8. Prevent pigmentation and whitening.

9. Instantly remove dirt from the pores, and remove the acne, acne, and facial dark spots that block the pores without pain, and clean the skin.

VI. Applicable people

1. Dry skin and water shortage.

2. Large pores, rough skin, dull.

3. blackheads, whiteheads, pores, clogged, closed, acne, acne .

4. The skin is not well absorbed, and the bottom of the skin is dehydrated.

5. Dark skin and uneven skin tone.

6. Excessive secretion of oil from the skin, oily and dry skin.

7. Usually improper cleaning, the skin is not.

VII. Taboo people

1. Cardiovascular and cerebrovascular diseases, three high population.

2. People with severe skin allergies.

3. Patients with skin diseases and infectious diseases.

4. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth).

5. Skin damage, local symptoms such as inflammation.

6. Skin allergies.

7. During pregnancy and menstruation.

8. The skin has wounds or wounds.

VIII. Advantages of treatment

Microdermabrasion + cleaning shovel + ION+ + ultrasonic introduction four-in-one, ultrasonic uses high-speed small vibration to generate small foam, atomize water molecules, and in the joint action, discharge pores and skin tissue Sediments can also promote skin metabolism. The dermabrasion operation can deeply clean the blackheads and blockages in the pores, ION+ discharge the skin waste, and deepen the dirt in the pores to make the skin clean and clear. Give the skin a thorough cleansing.

IX. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.
5. Make sure to remove makeup before going to bed.
6. Avoid contact with high temperatures.

II Deep Moisturizing

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has an obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin are thoroughly cleaned. The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores with sound waves, and remove the garbage and waist deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. Spray bottle

Using strong pressure oxygen, the product and oxygen are simultaneously introduced into the air pressure equipment, and the granular water droplets are converted into a very small molecular weight to act on the skin, and the tiny water droplets are specially designed in the form of a spray at a speed of 200 M|sec. The nozzle is sprayed to inject water and oxygen into the bottom layer of the human skin at a very high speed.

Biological effect: The vacuum circuit is formed by vacuum negative pressure, and the high concentration of active oxygen and essence are fully combined, and the high-pressure release method directly acts on the skin, allowing nutrient molecules to promote the peeling of pores, which can deep cleans the surface while at the same time The skin provides long-lasting essential nutrients to make the skin healthy, moist, delicate and

shiny.

III. Ultrasonic

Warming effect: By the warming action of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, the nutrients of the skin cells can be increased, the nerve excitability can be reduced, and the analgesic effect can be achieved, so that the muscle fibers of the tendon are relaxed. To the role of unlocking. The heat of the ultrasound is endogenous heat, 79%-82% of the heat is transported away from the action area by the blood, and 18%-21% is dispersed by heat conduction to the adjacent tissue. Therefore, the customer has no obvious heat feeling.

Chemical action: Ultrasound can enhance the catalytic ability, accelerate the metabolism of skin cells, change the PH value of the tissue to the alkaline direction, reduce the acidosis and pain associated with skin inflammation, and ultrasonic can improve the permeability of cell membrane and make nutrients and drug solutions. Poly, which is good for the skin to absorb nutrients, which is good for the drug to penetrate into the bacteria and improve the bactericidal ability.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can make the skin cells vibrate, produce a micro-massage, make the skin shiny and elastic, and the ultrasonic waves make the vibration energy The diffusion speed is faster, and the nutrient product enters the skin more finely through high-frequency vibration and heat energy. The warmth of the ultrasonic wave penetrates into the skin six centimeters, smoothing the circulation of blood and lymph, accelerating the blood circulation of the cells, and starting the cell tissue organism. The effect is to deepen the moisture and enhance the absorption and penetration effect.

IV. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellulase) serving as the "guardian" and "cleaner" of the cells. And the activity of (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the

skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

V. Efficacy

1. Water atomization, deep into the pores, high-frequency vibration, bring out dirt.
2. High-frequency vibration, refine nutrients into small molecular particles, effectively strengthen skin absorption, nutrition directly to the dermis.
3. Improve skin texture, large pores, wrinkles, thick keratin, purify the skin, remove excess thick keratin, purify pores, break down fat and smooth pores, clear and smooth skin, etc.
4. Ultrasound on the skin can promote the normalization of fibrous tissue activity in the dermal tissue, thereby increasing the elasticity of the skin.
5. Accelerate cell growth and accelerate blood circulation.
6. Produces collagen, increases skin elasticity, repairs aging skin, acne skin, diminishes spots, enhances firmness, relieves sunburn, etc.
7. Passing nutrients into the skin with high frequency vibration and heat.
8. Deep hydration locks water to prevent water loss.

VI. Applicable people

1. Dry skin and lack of water.
2. blackheads, whiteheads, pores, clogged, closed, acne, acne.
3. Sensitive skin.
4. The skin appears aging, wrinkled skin.
5. Uneven skin tone, pigmented skin.
6. Requires skin firming, lifting, and bright people.
7. The skin is not well absorbed, and the bottom of the skin is dehydrated.
8. The skin is easy to peel off, remove makeup, not tender.
9. The skin appears dry lines, fine lines.

VII. Taboo people

1. Cardiovascular and cerebrovascular diseases, three high populations.
2. People with severe skin allergies.
3. Patients with skin diseases and infectious diseases.
4. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
5. Skin damage, local symptoms such as inflammation.
6. Skin allergies.
7. During pregnancy.
8. The skin has wounds or wounds.

VIII. Advantages of treatment

skin shovel + spray gun + ultrasonic + led micro-light color four-effect combination, deep into the skin after cleansing the skin, through the ultrasonic frequency along the direction of the pores into the skin dermis, to achieve long-term effective moisturizing, combined The effect of color light makes the skin accelerate the absorption of blood circulation, help the skin calm down and repair, and achieve deep hydration and lock water, so that your skin looks smoother, more tender and shiny.

IX. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.
5. Make sure to remove makeup before going to bed.
6. Avoid contact with high temperatures.
7. Avoid using products containing alcohol and hormones.

III. Microdermabrasion Firming Therapy

I. Microdermabrasion

Deep cleansing and firming, the dermabrasion uses diamond crystals specially processed for the patient's skin. It vacuums the skin's necrotic cells, removes deep pores and waste, and doubles the skin to promote new skin growth. Use the negative pressure function to remove the deep hair follicles of the skin. Aphids Blackheads Whiteheads At the same time, the blocked pores are dredged, and the skin is absorbed by negative pressure to achieve a firming effect.

Biological effect: The suction is generated by the negative pressure, combined with the operation of the sanding head, it can effectively discharge the metabolites on the surface of the skin and the blackheads, garbage and blockages deep in the pores, which is equivalent to the vacuum cleaner of the skin, and completely cleans the skin. At the same time, with the rotary lifting method, the effect of tightening can be achieved.

II. ION+ION-

It forms a weak electric field between the surface of the skin and the deep layer. It combines the deeper layers to replenish the essence of different functions in the skin (such as removing acne, moisturizing, wrinkle, and firming anti-aging). Supplementing the active ingredients into the skin can accumulate the surrounding skin cells as ion clusters,

promote the cells to continuously absorb the essence, and absorb the concentration several times higher than the extension or oral intake. The negative electrode can produce alkali action, stimulate nerves, promote blood circulation, soften fibrous tissue, and enhance firmness. The nutrient solution is ionized and introduced into the deep layer of the skin. The human body infiltrates the micro-current to diffuse the whole body tissue and activate the tissue, so that the essence can be introduced into the skin from the positive electrode to the negative electrode.

Biological effects: Using the principle of homopolar repulsive and heteropolar attraction, we use negative ions to make water-soluble nutrients (plant extracts) penetrate the skin more effectively and directly supply skin nutrients and needed substances.

III. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism, and ability to regenerate, soften tissue, stimulate nervous system and cell function, make skin firm and elastic

Warming effect: By the warming effect of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, and the nutrients of the skin cells can be increased. The heat of the ultrasonic waves is endogenous heat, and 79%-82% of the heat is transported by the blood from the action area. Walking, 18%-21% is dispersed by heat conduction to adjacent tissues. Therefore, the customer has no obvious thermal sensation, and the heat of the bottom layer of the skin effectively accelerates collagen regeneration, making the skin full and elastic.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can cause skin cells to vibrate, produce a fine massage effect, change the cell volume, and thus improve local blood and lymph. The circulation enhances cell permeability, enhances tissue metabolism and regeneration, softens tissue, stimulates nervous system and cell function, makes skin shiny and elastic, and the warmth of ultrasound penetrates into the skin six centimeters to make blood And the flow of lymph fluid is smooth, accelerates cell blood circulation, initiates biological effects of cell tissue, stimulates cell regeneration, and activates and regenerates cells of deep tissues. Achieve a firming effect.

IV. EMS

Mainly through micro-current to stimulate the acupuncture points on our face, let the cells at the bottom of our face become active, improve cell activity, accelerate facial metabolism and promote collagen formation in the dermis. It has whitening, rejuvenation, wrinkle, and skin. Elasticity, improve skin condition, repair damaged skin, reduce pores, firm skin, anti-aging, and anti-oxidation effect. The EMS model is also called micro-electric

stimulation, which can relax the muscle spasm, increase the movement, and make the facial lines clearer, thus achieving the effect of tightening and lifting the face.

Reduces and prevents muscle atrophy, increases local blood circulation and muscle pre-workout.

Biological effects: use weak currents to stimulate muscles and lymph, contract muscles, and achieve edema and lifting.

V. Magic gloves

The energy mode is effectively transmitted to the human body, and the important acupoint stimulation of the human body achieves the purpose of massage, and has the magical effect of skin shaping. Activates cells and restores skin elasticity, thus achieving beauty equipment that solves some skin problems and delays skin aging.

VI. Efficacy

1. Enhance cell activity, promote muscle movement and restore skin elasticity.
2. Accelerate blood circulation of microvessels, enhance cell permeability, and effectively supply nutrients to muscle tissue and skin.
3. The electric ions generated by the micro-current can penetrate into the skin deeply, helping to repair the elastic fibers and collagen layer of the skin, thereby stretching and reducing the fine lines.
4. Firmly enhance facial contours and create beautiful lines.
5. Anti-oxidant, whitening, blemishes, firming skin.
6. Accelerate skin metabolism and release toxins.
7. Supply the skin with the nutrients needed to rejuvenate the skin and delay aging.

VII. Applicable people

1. A person with a lack of luster on the face, a rough skin and a blackhead.
2. Drinking regularly and often insomnia.
3. The face is prone to long spots and long acne.
4. The contour of the face is not clear, and the meridians are blocked..
5. The skin appears slack, sagging, not tight and full of people
6. Dark yellow skin, dull.
7. Poor skin metabolism, rough, uneven people.
8. Dry skin, lack of water and wrinkles.

VIII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. During pregnancy, recovery period.

-
4. There are pustules on the face.
 5. Just doing a complete surgery.
 6. Cardiovascular and cerebrovascular diseases, three high population.
 7. Patients with skin diseases and infectious diseases.
 8. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
 9. Skin damage, local symptoms such as inflammation.
 10. Skin allergies.
 11. During pregnancy and menstruation.

IX. Advantages of treatment

Microdermabrasion + ultrasonic + EMS operation + magic gloves four effects in one, deep cleansing the skin, ultrasonically introduce nutrients deep into the skin, promote the activity of the dermal tissue, thereby increasing the skin's elasticity, through EMS micro-current operation It stimulates facial acupuncture points and accelerates cell activity, promotes collagen regeneration, and combines with magic gloves to make skin firm and enhance elasticity.

X. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.
5. Make sure to remove makeup before going to bed.
6. Avoid contact with high temperatures.
7. Avoid using products containing alcohol and hormones.

IV. Iontophoresis

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has an obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin are thoroughly cleaned. The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores

with sound waves, and remove the garbage and waist deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. ION+ION-

It forms a weak electric field between the surface of the skin and the deep layer. It combines the deeper layers to replenish the essence of different functions in the skin (such as removing acne, moisturizing, wrinkle, and firming anti-aging). Supplementing the active ingredients into the skin can accumulate the surrounding skin cells as ion clusters, promote the cells to continuously absorb the essence, and absorb the concentration several times higher than the extension or oral intake. The negative electrode can produce alkali action, stimulate nerves, promote blood circulation, soften fibrous tissue, and enhance firmness. The nutrient solution is ionized and introduced into the deep layer of the skin. The human body infiltrates the micro-current to diffuse the whole body tissue and activate the tissue, so that the essence can be introduced into the skin from the positive electrode to the negative electrode.

When the beauty product is applied to the skin in a dissolved state, ions can be decomposed, and under the action of a direct current electric field, the ions move in a direction, the cation moves from the anode end to the cathode, and the anion moves from the cathode end to the anode direction, the drug or essence The permeation ions penetrate into the skin and form an ion pile locally. The local drug absorption concentration of the skin is several times higher than the oral drug concentration, so that the corresponding effect is achieved.

Biological effects: Using the principle of homopolar repulsive and heteropolar attraction, we use negative ions to make water-soluble nutrients (plant extracts) penetrate the skin more effectively and directly supply skin nutrients and needed substances.

III. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism, and ability to regenerate, soften tissue, stimulate nervous system and cell function, make skin firm and elastic.

Warming effect: By the warming effect of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, and the nutrients of the skin cells can be increased. The heat of the ultrasonic waves is endogenous heat, and 79%-82% of the heat is transported by the blood from the action area. Walking, 18%-21% is dispersed by heat conduction to adjacent tissues. Therefore, the customer has no obvious thermal sensation, and the heat of the bottom layer of the skin effectively accelerates collagen regeneration, making the skin full and elastic.

Biological effects: high power and high energy can cause skin cells to vibrate, produce micro-massage, change cell volume, thereby improve local blood and lymph circulation, enhance cell permeability, and improve tissue metabolism and Regeneration ability, soften tissue, stimulate nervous system and cell function, make skin shiny and elastic. The ultrasonic wave makes the diffusion speed faster by the vibration energy, and the nutrient product enters the skin more finely by the high-frequency vibration and thermal energy. The warm energy of the ultrasonic wave penetrates into the skin six centimeters, smoothing the circulation of blood and lymph, and accelerating the blood circulation of the cells. It activates the biological effects of cell tissue, can dilute various pigmentation, whiten skin, and activate and regenerate cells in deep tissues.

IV. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellalase) serving as the "guardian" and "cleaner" of the cells. And the activity of superoxide dismutase (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

V. Efficacy

1. Highly concentrated active agent penetrates the skin.
2. Improve hard skin absorption rate.
3. Promote blood and lymph circulation.
4. Promotes skin repair.
5. Quickly replenish oxygen to the skin and restore vitality.
6. Activate cells and restore elasticity.
7. Moisturizing whitening, smoothing the skin, delaying aging.
8. Prevent pigmentation and whitening.
9. Increase skin nutrient absorption rate and prevent water and nutrient loss.

-
10. Deep hydration, whitening, brighten skin tone.
 11. Strengthen the penetration of cells, and introduce non-penetrating nutrients into the skin.

VI. Applicable people

1. Dry skin and water shortage.
2. Large pores, rough skin, dull.
3. Skin aging wrinkles.
4. Uneven skin tone, pigmented skin.
5. Requires skin firmness.
6. Poor skin absorption, lack of water at the bottom of the skin.
7. Skin lacks elasticity, not full of smooth people.
8. Skin lacks nutrition, dark yellow, long spots, wrinkled skin.
9. The pores are clogged and the stratum corneum is too thick.

VII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. During pregnancy, recovery period.
4. There are pustules on the face.
5. Just doing a complete surgery.
6. Cardiovascular and cerebrovascular diseases, three high population.
7. Patients with skin diseases and infectious diseases.
8. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
9. Skin damage, local symptoms such as inflammation.
10. Skin allergies.
11. During pregnancy and menstruation.

VIII. Advantages of treatment

Skin shovel + electric ion + ultrasonic + LED light therapy, four-effect combination, can be imported into VC essence, vitamin essence, collagen essence, etc., with cleaning and ultrasonic introduction to double the effect, can deepen the skin yellowing, dull Matte, uneven skin tone, long spots, wrinkles and other issues.

IX. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.

-
5. Make sure to remove makeup before going to bed.
 6. Avoid contact with high temperatures.
 7. Avoid using products containing alcohol and hormones.

V. Cell Regeneration

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has an obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin are thoroughly cleaned. The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores with sound waves, and remove the garbage and waist deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism, and ability to regenerate, soften tissue, stimulate nervous system and cell function, make skin firm and elastic.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can cause skin cells to vibrate, produce a fine massage effect, change the cell volume, and thus improve local blood and lymph. The circulation enhances cell permeability, enhances tissue metabolism and regeneration, softens tissue, stimulates nervous system and cell function, and makes skin shiny and elastic. The ultrasonic wave makes the diffusion speed faster by the vibration energy, and the nutrient product enters the skin more finely by the high-frequency vibration and thermal energy. The warm energy of the ultrasonic wave penetrates into the skin six centimeters, smoothing the circulation of blood and lymph, and accelerating the blood circulation of the cells. It activates the biological effects of cell tissue, can dilute various pigmentation, whiten skin, and activate and regenerate cells in deep tissues.

III. EMS

Mainly through micro-current to stimulate the acupuncture points on our face, let the cells at the bottom of our face become active, improve cell activity, accelerate facial metabolism and promote collagen formation in the dermis. It has whitening, rejuvenation, wrinkle, and skin. Elasticity, improve skin condition, repair damaged skin, reduce pores, firm skin, anti-aging, and anti-oxidation effect. The EMS model is also called micro-electric stimulation, which can relax the muscle spasm, increase the movement, and make the facial lines clearer, thus achieving the effect of tightening and lifting the face.

Reduces and prevents muscle atrophy, increases local blood circulation and muscle pre-workout.

Biological effects: use weak currents to stimulate muscles and lymph, contract muscles, and achieve edema and lifting.

IV. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellulase) serving as the "guardian" and "cleaner" of the cells. And the activity of superoxide dismutase (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

V. Efficacy

1. Improve hard skin absorption rate.
2. Highly concentrated active agent can penetrate skin.
3. Promote blood and lymph circulation.
4. Promote skin repair.

-
5. Promote the discharge of skin and body aging.
 6. Prevent pigmentation and whitening effect.
 7. Quickly replenish oxygen to the skin and restore vitality.
 8. Anti-aging antioxidant effect.
 9. Accelerate cell growth, promote collagen regeneration, and accelerate blood circulation.
 10. Deep cleansing of skin dirt, effect sterilization.
 11. Increase skin shine and elasticity.
 12. Firming and eliminating edema.

VI. Applicable People

1. Dry, dehydrated skin.
2. Oily, mixed skin, extremely dry and oily skin.
3. Large pores, rough skin, no light.
4. Thin cuticle, need skin regeneration, repair.
5. Requires skin firming, lifting, and bright people.
6. Aging, wrinkled skin.
7. Collagen loss, slack, not full.

VII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. During pregnancy, recovery period.
4. There are pustules on the face.
5. Just doing a complete surgery.
6. Cardiovascular and cerebrovascular diseases, three high population.
7. Patients with skin diseases and infectious diseases.
8. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
9. Skin damage, local symptoms such as inflammation.
10. Skin allergies.
11. During pregnancy and menstruation.

VIII. Advantages of treatment

Skin shovel + ultrasonic + EMS + LED light therapy, four-effect combination, It can promote blood circulation, promote skin oxygen supply and nutrient supply, stimulate cell activity by current stimulation, accelerate facial metabolism and blood circulation, promote collagen regeneration, and achieve faster cell regeneration and nutritional supplementation. Leaves skin healthy & firm.

IX. IX. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.

-
2. Avoid touching, grasping, buckle, etc.
 3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
 4. Do not change skin care products frequently.
 5. Make sure to remove makeup before going to bed.
 6. Avoid contact with high temperatures.
 7. Avoid using products containing alcohol and hormones.

VI. Calming Soothing

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin is thoroughly cleaned.

The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores with sound waves, and remove the garbage and waste deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. Spray bottle

Using strong pressure oxygen, the product and oxygen are simultaneously introduced into the air pressure equipment, and the granular water droplets are converted into a very small molecular weight to act on the skin, and the tiny water droplets are specially designed in the form of a spray at a speed of 200 M|sec. The nozzle is sprayed to inject water and oxygen into the bottom layer of the human skin at a very high speed.

Biological effect: The vacuum circuit is formed by vacuum negative pressure, and the high concentration of active oxygen and essence are fully combined, and the high-pressure release method directly acts on the skin, allowing nutrient molecules to promote the peeling of pores, which can deep cleans the surface while at the same time The skin provides long-lasting essential nutrients to make the skin healthy, moist, delicate and shiny.

III. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellulase) serving as the "guardian" and "cleaner" of the cells. And the activity of superoxide dismutase (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

IV. Hot and cold hammer

Use the principle of air conditioner. The use of special wafer ceramics, energy absorption, and the ability to achieve -5 degrees of refrigeration in an instant, the heat energy generated by the absorption of energy by the wafer is heated by a physical heat to achieve a heating effect of 40 degrees, and the fan discharges excess heat. And the effect of constant temperature and freezing alternately. Promotes metabolism, relieves allergic reactions, shrinks pores, calms skin, whitens skin.

Biological effects: the use of refrigeration and heating to the skin, the cooling of the tissue temperature, blood vessels and veins contraction, reduce metabolic rate, inhibition of slowing cell activity, reduce peripheral nerve sensitivity, calming and soothing, offset skin Red, swollen, bloated, painful, especially allergic skin. Achieve calming the skin, redness and swelling, shrink pores and other effects. When heating, the tissue can be heated, the temperature is raised, the blood circulation is adjusted, the metabolism is accelerated, and the product is absorbed.

V. Efficacy

1. Promote product absorption and penetration
2. Soothes and calms skin
3. Eliminate redness, swelling, heat, pain, itching, discomfort

-
4. Replenishes water, prevents moisture loss, and locks in moisture
 5. Relieves acne muscles, fragile skin
 6. Shrinks pores, delicate skin, smoothes fine lines
 7. Repairs skin barriers, enhances skin resistance and repairs after sunburn
 8. Cleans skin with excess dirt and keratin

VI. Applicable People

1. Dry skin and lack of water.
2. Large pores, blockage, rough skin, dull.
3. Sensitive skin.
4. Thin cuticle, red blood.
5. Acne muscle, sensitive muscle.
6. The pores are thick and rough.
7. Dark complexion, dull.
8. Oily and dry, mixed skin.
9. After the exposure, the skin appears red and swollen.

VII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. During pregnancy, recovery period.
4. There are pustules on the face.
5. Just doing a complete surgery.
6. Cardiovascular and cerebrovascular diseases, three high population.
7. Patients with skin diseases and infectious diseases.
8. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
9. Skin damage, local symptoms such as inflammation.
10. Skin allergies.
11. During pregnancy and menstruation.

VIII. Advantages of treatment

Skin shovel + spray gun + cold hammer + LED light therapy, four effects in one, calming the skin, alleviating sensitivity, redness and other discomfort, shrinking pores, LED light can be anti-inflammatory and sterilizing, and promote skin absorption and soothing effect. You can use moisture essence, aloe vera extract, etc. to quickly restore skin moisture to enhance the calming effect.

IX. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.

-
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
 4. Do not change skin care products frequently.
 5. Make sure to remove makeup before going to bed.
 6. Avoid contact with high temperatures.
 7. Avoid using products containing alcohol and hormones.

VII. Acne Care

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin is thoroughly cleaned.

The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores with sound waves, and remove the garbage and waste deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. Hot and cold hammer

Use the principle of air conditioner. The use of special wafer ceramics, energy absorption, and the ability to achieve -5 degrees of refrigeration in an instant, the heat energy generated by the absorption of energy by the wafer is heated by a physical heat to achieve a heating effect of 40 degrees, and the fan discharges excess heat. And the effect of constant temperature and freezing alternately. Promotes metabolism, relieves allergic reactions, shrinks pores, calms skin, whitens skin.

Biological effects: the use of refrigeration and heating to the skin, the cooling of the tissue temperature, blood vessels and veins contraction, reduce metabolic rate, inhibition of slowing cell activity, reduce peripheral nerve sensitivity, calming and soothing, offset skin Red, swollen, bloated, painful, especially allergic skin. Achieve calming the skin, redness and swelling, shrink pores and other effects. When heating, the tissue can be heated, the temperature is raised, the blood circulation is adjusted, the metabolism is accelerated, and the product is absorbed.

III. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism, and regenerative capacity, soften tissue, stimulate the nervous system and cell function, and make the skin elastic.

Warming effect: By the warming action of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, the nutrients of the skin cells can be increased, the nerve excitability can be reduced, and the analgesic effect can be achieved, so that the muscle fibers of the tendon are relaxed. To the role of unlocking. The heat of the ultrasound is endogenous heat, 79%-82% of the heat is transported away from the active area by the blood, and 18%-21% is dispersed by heat conduction to the adjacent tissue. Therefore, the customer has no obvious heat feeling.

Chemical action: Ultrasound can enhance the catalytic ability, accelerate the metabolism of skin cells, change the PH value of the tissue to the alkaline direction, reduce the acidosis and pain associated with skin inflammation, and ultrasonic can improve the permeability of cell membrane and make nutrients and drug solutions. Poly, which is good for the skin to absorb nutrients, which is good for the drug to penetrate into the bacteria and improve the bactericidal ability.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can cause skin cells to vibrate, produce a fine massage effect, change the cell volume, and thus improve local blood and lymph. The circulation enhances cell permeability, enhances tissue metabolism and regeneration, softens tissue, stimulates nervous system and cell function, makes skin shiny and elastic, and ultrasonic waves make diffusion faster by vibrating energy, and high-yield nutrient products Frequent vibration and thermal energy enter the skin more subtly. The warmth of ultrasonic waves can penetrate into the skin six centimeters, smoothing the circulation of blood and lymph, accelerating the blood circulation of cells, and initiating the biological effects of cell tissue, which can dilute various stains and make The skin is whitened, and cells in deep tissues are activated and regenerated.

IV. Spray bottle

Using strong pressure oxygen, the product and oxygen are simultaneously introduced into the air pressure equipment, and the granular water droplets are converted into a very small molecular weight to act on the skin, and the tiny water droplets are specially designed in the form of a spray at a speed of 200 M|sec. The nozzle is sprayed to inject water and oxygen into the bottom layer of the human skin at a very high speed.

Biological effect: The vacuum circuit is formed by vacuum negative pressure, and the high concentration of active oxygen and essence are fully combined, and the high-pressure

release method directly acts on the skin, allowing nutrient molecules to promote the peeling of pores, which can deep cleans the surface while at the same time The skin provides long-lasting essential nutrients to make the skin healthy, moist, delicate and shiny.

V. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellalase) serving as the "guardian" and "cleaner" of the cells. And the activity of superoxide dismutase (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

VI. Efficacy

1. Cleans skin and removes blockages of keratin and pores.
2. Calms skin, relieves redness, swelling, heat, pain, itching, etc.
3. Shrink pores and lock in moisture.
4. Promote circulation, and metabolism, speed up product absorption.
5. Lightens pigments and tightens skin.
6. Strong hydration to prevent water loss.
7. Increase elasticity and smooth fine lines.
8. Reduce acne outbreak rate and inhibit acne growth.
9. Anti-inflammatory sterilization, reducing the chance of acne breeding.
10. Thoroughly cleanse the skin and prevent clogging.
11. Relieve acne marks, pigmentation.

VII. Applicable people

1. Acne muscle, oily skin.

-
2. Large, rough, unsmooth pores.
 3. Pigmentation, uneven skin tone.
 4. Blackhead, thick pores in the stratum corneum.
 5. Fine lines, dull skin.
 6. Acne repeatedly.
 7. Sensitive skin.

VIII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. During pregnancy, recovery period.
4. There are pustules on the face.
5. Just doing a complete surgery.
6. Cardiovascular and cerebrovascular diseases, three high population.
7. Patients with skin diseases and infectious diseases.
8. There are metal patients in the body (cardiac pacemaker, steel plate, metal teeth, etc.).
9. Skin damage, local symptoms such as inflammation.
10. Skin allergies.
11. During pregnancy and menstruation.

IX. Advantages of treatment

Skin shovel + Hot and cold hammer + Ultrasound + spray gun + LED light therapy + High frequency, six effects, biological acne, calm the skin, deep solution to acne problems, combined with color light and anti-inflammatory sterilization of the skin, promote pox Eliminate pox and repair skin.

X. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.
5. Make sure to remove makeup before going to bed.
6. Avoid contact with high temperatures.
7. Avoid using products containing alcohol and hormones.

VIII. Overall Skin Conditioning

I. Cleaning shovel

The unique energy-changing technology, the energy emitted by 28,000 mechanical vibrations per second has obvious osmotic effect on human skin. Through the atomization transmission of the medium, a strong impact force can instantly remove the deep dirt in the pores, and it is painless. The discharge of acne, acne that blocks the pores, and the skin is thoroughly cleaned.

The sebum and secretions that are clogged with low-frequency emulsified pores can also be sterilized, spray light mist into the pores, remove sebum and secretions, and remove dead skin.

Physical effects: high-frequency vibration sound waves combined with softening products or exfoliating cream to soften the damaged rough skin cuticle layer, and open the pores with sound waves, and remove the garbage and waste deep in the pores by high-frequency vibration.

The back side of the instrument can be molecularly introduced into the skin by high-frequency vibration and essence products to increase skin elasticity and smooth wrinkles.

II. Microdermabrasion

The second time is clean and tight, the skin is polished with diamond crystal specially processed for the patient's skin, and the dead cells of the skin are sucked out by vacuum to remove the deep dirt and wastes in the pores, so as to make the skin doubly clean and promote the growth of new skin. USES the negative pressure function to dispel the skin deep hair cysticercosis mite black head white head at the same time will block the pore dredge.

Biological effect: through negative pressure to produce suction, combined with frosting head operation, can affect the skin surface of the metabolites and pores deep black head, garbage, plug out, equivalent to the skin of the vacuum cleaner, to thoroughly clean the skin.

III. ION+ION-

It forms a weak electric field between the surface of the skin and the deep layer. It combines the deeper layers to replenish the essence of different functions in the skin (such as removing acne, moisturizing, wrinkle, and firming anti-aging). Supplementing the active ingredients into the skin can accumulate the surrounding skin cells as ion clusters, promote the cells to continuously absorb the essence, and absorb the concentration several times higher than the extension or oral intake. The negative electrode can produce

alkali action, stimulate nerves, promote blood circulation, soften fibrous tissue, and enhance firmness. The nutrient solution is ionized and introduced into the deep layer of the skin. The human body infiltrates the micro-current to diffuse the whole body tissue and activate the tissue, so that the essence can be introduced into the skin from the positive electrode to the negative electrode, and the skin can be discharged to the skin to make the skin more pure and thorough. , beautiful and beautiful.

When the beauty product is applied to the skin in a dissolved state, ions can be decomposed, and under the action of a direct current electric field, the ions move in a direction, the cation moves from the anode end to the cathode, and the anion moves from the cathode end to the anode direction, the drug or essence The permeation ions penetrate into the skin and form an ion pile locally. The local drug absorption concentration of the skin is several times higher than the oral drug concentration, so that the corresponding effect is achieved.

Biological effects: Using the principle of homopolar repulsive and heteropolar attraction, we use negative ions to make water-soluble nutrients (plant extracts) penetrate the skin more effectively and directly supply skin nutrients and needed substances.

IV. Ultrasound

Mechanical action: The ultrasonic power is strong and the ability is large. The action on the face can cause the skin cells to vibrate, produce a fine massage effect, change the cell volume, thereby improving the local blood and lymph circulation, enhancing the cell permeability and improving the tissue. Metabolism, and regenerative capacity, soften tissue, stimulate the nervous system and cell function, and make the skin elastic.

Warming effect: By the warming action of ultrasonic waves, the temperature of the skin surface can be increased, the blood circulation can be accelerated, the nutrients of the skin cells can be increased, the nerve excitability can be reduced, and the analgesic effect can be achieved, so that the muscle fibers of the tendon are relaxed. To the role of unlocking. The heat of the ultrasound is endogenous heat, 79%-82% of the heat is transported away from the active area by the blood, and 18%-21% is dispersed by heat conduction to the adjacent tissue. Therefore, the customer has no obvious heat feeling.

Chemical action: Ultrasound can enhance the catalytic ability, accelerate the metabolism of skin cells, change the PH value of the tissue to the alkaline direction, reduce the acidosis and pain associated with skin inflammation, and ultrasonic can improve the permeability of cell membrane and make nutrients and drug solutions. Poly, which is good for the skin to absorb nutrients, which is good for the drug to penetrate into the bacteria and improve the bactericidal ability.

Biological effects: high power, high energy, and a micro massage of 1 million to 3 million times per second, which can cause skin cells to vibrate, produce a fine massage effect, change the cell volume, and thus improve local blood and lymph. The circulation enhances cell permeability, enhances tissue metabolism and regeneration, softens tissue, stimulates nervous system and cell function, makes skin shiny and elastic, and ultrasonic waves make

diffusion faster by vibrating energy, and high-yield nutrient products. Frequent vibration and thermal energy enter the skin more subtly. The warmth of ultrasonic waves can penetrate into the skin six centimeters, smoothing the circulation of blood and lymph, accelerating the blood circulation of cells, and initiating the biological effects of cell tissue, which can dilute various stains and make the skin is whitened, and cells in deep tissues are activated and regenerated.

V. EMS

Mainly through micro-current to stimulate the acupuncture points on our face, let the cells at the bottom of our face become active, improve cell activity, accelerate facial metabolism and promote collagen formation in the dermis. It has whitening, rejuvenation, wrinkle, and skin. Elasticity, improve skin condition, repair damaged skin, reduce pores, firm skin, anti-aging, and anti-oxidation effect. The EMS model is also called micro-electric stimulation, which can relax the muscle spasm, increase the movement, and make the facial lines clearer, thus achieving the effect of tightening and lifting the face.

Reduces and prevents muscle atrophy, increases local blood circulation and muscle pre-workout.

Biological effects: use weak currents to stimulate muscles and lymph, contract muscles, and achieve edema and lifting.

VI. Magic gloves

The energy mode is effectively transmitted to the human body, and the important acupoint stimulation of the human body achieves the purpose of massage, and has the magical effect of skin shaping. Activates cells and restores skin elasticity, thus achieving beauty equipment that solves some skin problems and delays skin aging.

VII. LED light therapy

LED high-tech optical rejuvenation beauty instrument uses the principle of photodynamics to activate deep cells to make the skin better metabolism. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin - an enzymatic reaction, and increases the catalase (cellulase) serving as the "guardian" and "cleaner" of the cells. And the activity of superoxide dismutase (SOD). Increases the breakdown of adenosine triphosphate (ATP), a source of cellular energy. Increase glycogen and protein content to promote cell metabolism and synthesis. Stimulates the secretion of basic synthetic fiber growth factor (BFGF) epidermal growth factor (EGF), rearranges collagen fibers and elasticity, and blocks and blocks the formation of melanin, accelerates cell growth, accelerates blood circulation, and stimulates fibrous tissue. Produces collagen, increases skin elasticity, plays an important role in repairing aging skin, acne skin, diminishing spots, lifting firmness, relieving sunburn and other care.

Biological effects: Using the principle of photodynamics, activate deep cells to allow the skin to better metabolize. After the light is absorbed by the skin, the light energy is converted into intracellular energy, which relaxes and strengthens the microvasculature and produces a photochemical reaction with the skin-enzymatic reaction, increasing the glycogen and protein content, thereby promoting cell metabolism and synthesis.

VIII. Hot and cold hammer

Use the principle of air conditioner. The use of special wafer ceramics, energy absorption, and the ability to achieve -5 degrees of refrigeration in an instant, the heat energy generated by the absorption of energy by the wafer is heated by a physical heat to achieve a heating effect of 40 degrees, and the fan discharges excess heat. And the effect of constant temperature and freezing alternately. Promotes metabolism, relieves allergic reactions, shrinks pores, calms skin, whitens skin.

Biological effects: the use of refrigeration and heating to the skin, the cooling of the tissue temperature, blood vessels and veins contraction, reduce metabolic rate, inhibition of slowing cell activity, reduce peripheral nerve sensitivity, calming and soothing, offset skin Red, swollen, bloated, painful, especially allergic skin. Achieve calming the skin, redness and swelling, shrink pores and other effects. When heating, the tissue can be heated, the temperature is raised, the blood circulation is adjusted, the metabolism is accelerated, and the product is absorbed.

IX. Efficacy

1. Double deep cleansing, thoroughly cleans the skin, leaving the skin clean and transparent.
2. Removes deep dirt and waste from pores (aphids, blackheads, whiteheads, sebum, garbage, blockages, etc.).
3. Accelerate absorption, accelerate blood circulation and metabolism.
4. Promotes blood and lymph circulation, and releases toxins.
5. Prevents pigmentation and whitening to brighten skin tone.
6. Replenish nutrients to the skin and restore vitality.
7. Activate cells, stimulate collagen regeneration, increase elasticity.
8. Firming, lifting skin, smoothing wrinkles, preventing sagging, sagging.
9. Calms skin, stabilizes skin condition, smoothes skin's bad mood.
10. Improves roughness and does not smooth skin.
11. Anti-inflammatory, soothing acne muscles and sensitive muscles.

X. Applicable people

1. Dry and dehydrated, rough and dark skin, dull.
2. Excessive skin keratin, blackheads, whiteheads, closed mouth, acne, pore metabolites,

-
- excess sebum, pores clogged garbage and dirt, etc.
 3. Dry lines, fine lines, aging skin begins to appear.
 4. Thin cuticle, damaged skin barrier, sensitive or allergic.
 5. Often sunbathing, as well as long-term office workers.
 6. Long spotted pox.
 7. The face is prone to long spots and long acne.
 8. Slightly red bloodshot skin, sensitive and fragile skin.
 9. Drinking and drinking people often staying up late.
 10. The oil secretion is too strong, the pores are thick, the skin is not transparent, and the product cannot be absorbed.
 11. The skin loses its elasticity and begins to sagging.
 12. Uneven skin tone, pigmentation.

XI. Taboo people

1. During pregnancy and menstruation.
2. Inevitably strong (day) light exposure crowds in daily work.
3. Those who are allergic to photosynthesis and drugs and those who are allergic to strong light sources.
4. Skin sensitive and susceptible to allergies.
5. Epilepsy patients and scars and physical fitness.
6. Your upper eyelids and areas with open wounds.
7. Malignant tumor, diabetic patient.
8. The use of functional skin care products should be stopped after 1-3 months, the operation parts should be stopped for 1-2 days after using essential oils, blackheads need to be cleared for 3 days before treatment.
9. Expected value is too high.
10. Metal allergic.
11. Skin disease population and patients.
12. Serious heart disease or people with pacemakers.
13. Vascular stent, metal, plate fixation after fracture surgery.
14. Gold wire implanted, implanted with dentures or metal teeth.
15. People who are allergic.
16. Recovery period.
17. There are pustules on the face.
18. a person who has just finished a plastic surgery.
19. Cardiovascular and cerebrovascular diseases, three high population.

XII. Advantages of treatment

Deep double clean, double hydrating whitening, solve the fundamental skin problem. Create a healthy and perfect skin.

XIII. Precautions after operation

1. Pay attention to sun protection, avoid sunbathing just after operation.
2. Avoid touching, grasping, buckle, etc.
3. Pay attention to the mask to replenish moisture to prevent dryness and accelerate metabolism.
4. Do not change skin care products frequently.
5. Make sure to remove makeup before going to bed.
6. Avoid contact with high temperatures.
7. Avoid using products containing alcohol and hormones.
8. Drink plenty of water and keep your mood.
9. Sufficient sleep and rest time.

Part II

I. Detailed Operation

Hot and cold hammer



Hot and cold hammer Running State

1. Press  button to open the hot and cold hammer function.
2. Press  setting treatment time, increase treatment time.
3. Press  setting treatment time, reduce treatment time.
4. Press  button to Regulatory energy attenuation(1~9 Level).
5. Press  button to regulatory energy enhancement(1~9 Level).
6. Press  can choose(A, B)operation mode.
7. Press  button and the machine starts to run.

BIO and Photon



BIO and Photon Running State

8. Press  button to open the ion function.
9. Press  setting treatment time, increase treatment time.
10. Press  setting treatment time, reduce treatment time.
11. Press  button to Regulatory energy attenuation(1~9 Level).
12. Press  button to regulatory energy enhancement(1~9 Level).
13. Press  can choose(A, B)operation mode.
14. Press  button and the machine stars to run.

EMS



EMS Running State

1. Press  button to open the ion function.
2. Press  setting treatment time, increase treatment time.
3. Press  setting treatment time, reduce treatment time.
4. Press  button to Regulatory energy attenuation(1~9 Level).
5. Press  button to regulatory energy enhancement(1~9 Level).
6. Press  can choose(A, B)operation mode.
7. Press  button and the machine stars to run.

Ultrasound



Ultrasound Running State

1. Press  button to open the ion function.
2. Press  setting treatment time, increase treatment time.
3. Press  setting treatment time, reduce treatment time.
4. Press  button to Regulatory energy attenuation(1~9 Level).
5. Press  button to regulatory energy enhancement(1~9 Level).
6. Press  can choose(A, B)operation mode.
7. Press  button and the machine stars to run.

Dermabrasion/ Spray



Dermabrasion/ Spray Running State

1. Press  button to open the ion function.
2. Press  setting treatment time, increase treatment time.
3. Press  setting treatment time, reduce treatment time.
4. Press  button and the machine starts to run.

ION+ION-



ION+ION- Running State

1. Press  button to open the ion function.
2. Press  setting treatment time, increase treatment time.
3. Press  setting treatment time, reduce treatment time.
4. Press  button to Regulatory energy attenuation(1~9 Level).
5. Press  button to regulatory energy enhancement(1~9 Level).
6. Press  can choose(A, B)operation mode.
7. Press  button and the machine starts to run.

II. Instrument Technical Parameters

Power supply Input: 100V-240V

Power: 190W

Ultrasonic handles for face

Frequency: 3MHZ

Power: 50W

Hot & cold hammer

Temperature: 0~5 degree centigrade

Power: 10W

Ultrasonic shape head

Frequency: 3MHZ

Power: 50W

3MHZ ultrasound scrubber

Frequency: 3MHZ

Output Power: 5W

Number of vibrations per minute: 1620000 times

Double deep cleanse-dermabrasion

Total power: 14W

Voltage: 12V

The diameter of the micro-carving head: 22mm, 20mm, 16mm, 16mm, 15mm, 15mm, 15mm, 08mm, 06mm

Length of micro-carved rod: 110mm

Suction range: 0 to -80Kpa

III. User Taboos

1. Pregnant or lactating women.
2. Heart disease or with a pacemaker.
3. The surgical wound has not healed yet.
4. Patients with epilepsy, severe diabetes and hyperthyroidism.
5. Patients with malignant tumors, patients with hemophilia or severe bleeding.
6. Patients with skin diseases and infectious diseases should be used with caution.

IV. Precautions For Use And Maintenance

1. Remove the treatment head every time you use the instrument. Clean it with water and keep it in a safe place.
2. The instrument must have a plug with a grounding pin before use and ensure that the

instrument's power outlet is properly grounded.

3. Ensure that the voltage of the instrument is adapted. If the local power supply voltage is unstable, we recommend that the user add a power-matched regulated power supply between the mains and the instrument.
4. For the purpose of the treatment of the instrument and to ensure the normal service life, please use the specified accessories provided or recommended by the original manufacturer.
5. Do not place the instrument in a humid place or near a water source, and do not expose the instrument to direct sunlight.
6. Do not place the instrument close to a strong heat source, as this may affect the life of the instrument and normal use.
7. Please remove all metal objects from your body before treatment to avoid unpredictable conditions and affect the efficacy.
8. Please do not use it in the eyes, thyroid gland, parathyroid gland, testis, pregnant abdomen and atrial pulse generator.
9. Patients who are ill should also use this instrument with caution unless approved by a doctor.
10. When the instrument is not in use, please turn off the power switch of the instrument and ensure that the total power of the instrument is turned off before the personnel leave after daily use to ensure the safety of the electrical products.
11. Use the instrument or train the instrument operator in strict accordance with the instructions in the instruction manual.

V. Operational Precautions

1. If you feel unwell during use, or if you find any abnormality in the product, please turn off the power immediately and stop using it.
2. If the strength is too high, there may be cramps. If this happens, please stop using it immediately.
3. Please do not use it for young children.
4. Please place it in a place that is not easy for infants and young children to touch.
5. If the product is abnormal, please stop using it immediately. Please unplug the power cord after use.
6. When starting to use, please do not adjust the intensity too high, please use from low intensity.
7. If the skin is red and swollen and itching, please reduce the output intensity and choose the strength that suits your skin.
8. When using the freezing head, avoid touching the air outlet to avoid being burnt.
9. When using microdermabrasion, check if the gasket is installed, otherwise it will leak or not enough suction.
10. The face must be wet when using the shovel and avoid dry skin operation.
11. Avoid wearing metal objects when using this product. Please remove the metal accessories, the body should not do it in metallic substances.
12. The product used in the lance must be a watery product.
13. Carefully avoid injecting into the mouth and in the nostrils when spraying oxygen.
14. Clean the cuvette and the inner tube hole before and after each lance. Avoid tube

plugging caused by product drying. Cleaning method: Pour pure water into the small cup, then press the control button down and pull back, and quickly spray the water out of the cup.

15. When sucking blackheads, it is best to put a proper amount of cotton paper in the glass tube to prevent dirt from being sucked into the air duct and damage the main unit. Don't stay in the face for too long, so as not to cause bruises

16. If there is no suction when starting up, it is normal, and it will return to normal after a while. The suction size can be adjusted with the metal knob, which is large clockwise.

17. When using the spray bottle, do not insert the catheter into the black hole to prevent the lotion from being sucked into the air pump and damaging the main unit.

18. It is normal for the squeaky sensation of current to flow through the operation of the ionization operation.

19. When the cleaning shovel is cleaned, the groove is outward. The operation will see the water droplets splashing in the water, and the skin part of the instrument is not covered with water drops. Put the cleaning shovel upside down and apply the flat to the skin for import and lifting.

VI. Common Fault Handling Methods

1. Does the instrument operate without feeling?

A. Check if the instrument socket is plugged in.

B. Check if the handle link is inserted.

C. Check if the instrument has contact products.

2. The instrument does not start, and the button light on the back of the instrument does not light up?

A. Make sure the power cord is connected to a valid power outlet.

B. Is the fuse tube on the back of the instrument loose or burnt?

3. Does the instrument have no current output?

A. Please check if the instrument handle and the connector connected to the body are tightly connected.

B. Please check whether the treatment part has been cleaned. The grease or oily essential oil product may cause the instrument's treatment handle to be in poor contact with the human body, resulting in no current output.

4. Is the current output weakened?

A. Please check if the instrument handle and the connector connected to the body are tightly connected.

B. Please check if there is any non-conductive grease and other substances on the handle of the treatment. This will cause the contact to be weak and the output to be weakened.

C. Please check if the product used is the adapter specified by the instrument.

5. Does the instrument have no pressure or little pressure?

A. Please turn off the instrument and check if the hose of the instrument is tight.

B. Please check if the handle cover is tight and the product is not watery.

C. Please check if the hose is damaged or leaking, resulting in poor air pressure.
D. If the above method cannot be processed, please contact the dealer of the instrument for assistance.

6. The instrument can start, but the screen has the wrong information?

Unplug the power plug from the back of the instrument and wait for about 1 minute before plugging in and restarting the instrument.

B. If the above method cannot be processed, please contact the dealer of the instrument for assistance.

VII. Frequently Asked Questions And Answers

1. Can you make up after doing this instrument?

You can make up your makeup. After you finish, your skin will be moist and full. At this time, make up your makeup. This is a way of care that does not break the skin and has no wounds. It is safe, healthy and fast.

2. How long does it take to see the effect?

Once you have done it, you can see that the skin is clear and transparent. The continuous effect is more obvious, which can improve the deep water shortage, dullness, fine lines and aging of the skin.

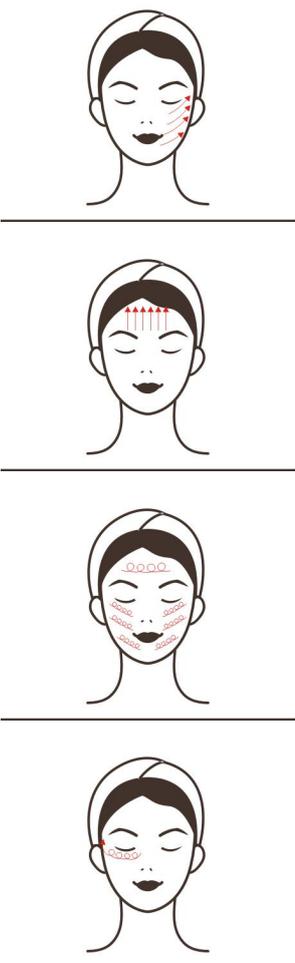
3. Is there any dependency on this machine?

No dependence, this is to improve and deep clean the skin for the skin problem. It belongs to the maintenance and resistance to the skin due to other factors, helping the skin to absorb nutrients and solve the skin's dark yellow, water-deficient, rough, wrinkled skin. If you don't do it later, you just return to the skin's own secretion and aging, and will not cause tolerance.

VIII. Package Included

2 x Ultrasonic handles for face
1 x Hot & cold hammer
2 x INO+/INO-
2 x 3MHZ ultrasound scrubber
3 x Diamond Dermabrasion tip
3 x Diamond Dermabrasion plastic wand
2 x Microcurrent Set
1 x Microcurrent & Photon
1 x Magic gloves
1 x Main machine
1 x Plastic holder
1 x Power supply(We offer 100-240V, AU/EU/UK/AU plug)

IX. Operation

Parameter Adjustment	product	Operation Technique	Diagram
Double clean: 70 minutes, once a week			
Adjust energy: 1-5 levels Mode: A, B	Cleansing Oil + Cleanser + Microdermabrasion + Cleaning Shovel + Electric Ion + Ultrasound + Essence + Mask	<p>Cleansing Cleanup 5 minutes</p> <p>Apply blackhead derivation solution + thermal spray, 5-10 minutes (sensitive muscle can be about 2 minutes, blackhead can be as long as 10 minutes)</p> <p>Wipe the face with a damp cotton pad and clean the face with a skin shovel for 5-10 minutes (the nose part with blackheads can be cleaned)</p> <p>Clean the face</p> <p>Microdermabrasion operation 5-10 minutes (select the appropriate operating head and suction, the face and forehead are pulled up in a line, the nose can be pulled and pulled)</p> <p>Clean the face</p> <p>Apply facial cleansing cream or cleansing lotion or skin lotion, and transfer with electric ions for 5-10 minutes.</p> <p>Ultrasonic circular motion into the facial essence 5-10 minutes (can be introduced into the moisturizing essence)</p> <p>Ultrasonic ring is introduced into the eye essence for 3-5 minutes (a small circle around the eye, the whole circle of the eye is introduced)</p> <p>Apply a cleansing hydrating mask for 15 minutes</p> <p>End</p>	
<p>Treatment recommends</p> <p>Ten times for a course of treatment, the skin is clean and transparent after one finish, and the pores are refreshing and smooth. After a course of treatment, the pores of the skin are improved, the roughness and the strawberry nose are alleviated, and the skin is stabilized after three treatments, which helps the subsequent products to absorb better and makes the skin smooth and smooth.</p>			

Deep Moisturizing: 60 minutes, once a week

<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser + Cleaning Shovel + Ultrasound + BIO and Photon + Essence + Mask</p>	<p>Cleansing makeup, 5 minutes Wipe the face with a damp cotton pad and clean the face with a skin shovel for 5-10 minutes (the nose part with blackheads can be cleaned) Clean the face Spray gun with toner for secondary cleaning and hydration 3-5 minutes (one line, one line, upward circular injection) Ultrasonic circular introduction of facial essence 5-10 minutes (can be introduced into moisturizing essence) Ultrasonic ringing introduces eye serum, 3-5 minutes Apply moisturizing mask for 15 minutes + micro-light color seal operation face (optional red light) Clean the face Rubbing toner, essence, face cream, sunscreen End</p>	 <hr/> 
---	--	--	---

Treatment recommends

Ten times for a course of treatment, after one time, the skin is tender and delicate, and a course of treatment relieves dryness, lack of water and peeling. After three treatments, it stabilizes the skin, keeps the skin moisturized and prevents water loss.

Microdermabrasion Firming Therapy: 90 minutes, once a week

<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil +Cleanser +Microdermabrasion + Electric Ion + Ultrasound + EMS + Magic gloves + Essence + Mask</p>	<p>Cleansing makeup, 5 minutes Microdermabrasion operation 5-10 minutes (select the appropriate operating head and suction, the face and forehead are pulled up in a line, the nose can be pulled and pulled) Apply facial cleansing cream or cleansing lotion or skin lotion, and transfer with electric ions for 5-10 minutes. Clean the face Ultrasonic introduction of facial essence 5-10 minutes (From the chin, a line of squats in a line up and up, can be</p>	 <hr/> 
---	---	---	---

		<p>introduced into the firming anti-wrinkle essence)</p> <p>Ultrasonic introduction of eye essence for 3-5 minutes (a small circle around the eye, the whole eye is introduced)</p> <p>EMS operation (both hands start from the chin at the same time, lining up the line and lifting up the circle to select the appropriate operation of the head) 5-10 minutes</p> <p>Magic gloves are inserted into the plug and then put on the magic gloves, choose energy (slowly adjusted from small to large, according to the customer's ability to choose) 5-10 minutes (both hands from the neck began to line up the line up, the forehead to the hairline Direction improvement)</p> <p>Apply mask, 15 minutes</p> <p>Clean the face</p> <p>Toner, essence, face cream, sunscreen</p> <p>End</p>	    
--	--	--	--

Treatment recommends

Ten times for a course of treatment, the skin is clean and transparent after one finish, and the pores are refreshing and smooth. It has a firming effect, a treatment to relieve rough skin and improve skin contour. Three courses of treatment stabilize the skin condition, leaving the skin moisturizing and preventing moisture loss. The skin is firmed and lifted to achieve anti-aging effects.

Iontophoresis: 70 minutes, once a week

<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser + Black export liquid + Cleaning Shovel +</p>	<p>Cleansing makeup, 5 minutes</p> <p>Apply black export liquid + thermal spray for 5-10 minutes (sensitive muscle can be about 2 minutes, many blackheads can last up to 10 minutes)</p> <p>Wipe the face with a cotton pad and use</p>	
---	---	--	---

	<p>Electric Ion + Ultrasound + BIO and Photon + Essence + Mask</p>	<p>the skin shovel for 5-10 minutes (the nose part with blackheads can be focused) Clean the face Apply facial cleansing cream or cleansing lotion or skin lotion, and transfer with electric ions for 5-10 minutes. Ion the first line of the electric ion to lead the circle into the vc essence or collagen essence 5-10 minutes (the cotton piece is wrapped around the ion-implanted rod and fixed after the vc essence is introduced) Ultrasonic introduction of facial essence 5-10 minutes (From the chin, a line of squats in a line up, can be introduced into the whitening essence) Ultrasonic introduction of eye essence for 3-5 minutes (a small circle around the eye, the whole eye is introduced) Apply mask + color seal stamp operation 15 minutes (color light can choose red light operation) Clean the face Toner, essence, face cream, sunscreen End</p>	
<p>Treatment recommends</p> <p>Ten times for a course of treatment, the skin is clean and transparent after one time, clear excess keratin, so that the essence is well absorbed and infiltrated, a treatment to brighten the skin, repair the skin, promote the metabolism of the skin, and tighten the skin after three treatments. Shiny, natural white. Improves skin's own absorption and metabolism.</p>			
<p>Cell Regeneration: 70 minutes, once a week</p>			
<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser +Cleaning Shovel + EMS + BIO and Photon + Essence + Mask</p>	<p>Wipe the face with a cotton pad and use the skin shovel for 5-10 minutes (the nose part with blackheads can be focused) Clean the face Ultrasonic introduction of facial essence5-10 minutes (From the chin, a line of squats in a line up, can be introduced into the whitening essence) Ultrasonic introduction of eye essence for 3-5 minutes (a small circle around the eye, the whole eye is introduced)</p>	

		<p>EMS operation (both hands start from the chin at the same time, lining up the line and lifting up the circle to select the appropriate operation of the head) 5-10 minutes</p> <p>Apply mask 15 minutes + led color seal stamp operation (led light can choose red light)</p> <p>Clean the face</p> <p>Toner, essence, face cream, sunscreen</p> <p>End</p>	
--	--	--	---

Treatment recommends

Ten times for a course of treatment, after one time, the skin is clean and transparent, supplementing the needed nutrients and moisture, and a treatment will stimulate the collagen regeneration at the bottom of the skin, so that the skin can better absorb the product and firm and elastic, and the skin contours of the three treatments Clear, firm and full.

Calming Soothing: 60 minutes, once a week

<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser + Cleaning Shovel + Spray Gun + Cold Hammer + Essence + Mask</p>	<p>Cleansing makeup, 5 minutes</p> <p>Wipe the face with a cotton pad and use the skin shovel for 5-10 minutes (the nose part with blackheads can be focused)</p> <p>Clean the face</p> <p>Use the spray gun to hydrate the water, 5-8 minutes</p> <p>Evenly apply the essence to the face and use the cold hammer to operate (from the chin, the line is raised upwards) 5-10 minutes</p> <p>Apply mask for 15 minutes + led light seal operation (led light can choose blue light,yellow light, color light)</p> <p>Clean the face</p> <p>Rubbing toner, essence</p> <p>End</p>	 
---	--	---	--

Treatment recommends

Ten times for a course of treatment, after one time, the skin is refreshing and cool, and the redness and swelling of the skin is eliminated. A course of treatment improves the fragility of sensitive skin and reduces the sensitivity rate. The three courses stabilize the skin condition, and the skin is kept clean and moist.

Acne Care: 60 minutes, once a week

<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser + Cleaning Shovel + Hot and Cold hammer + Spray Gun + BIO and Photon + High Frequency + Essence + Mask</p>	<p>Cleansing makeup, 5 minutes Wipe the face with a cotton pad and use the skin shovel for 5-10 minutes (the nose part with blackheads can be focused) The spray gun is filled with pure water for secondary cleaning and hydration for 3-5 minutes (one line, one line, upward circular injection) Use a hot hammer to open the pores of the skin for 3-5 minutes (a line of lines squatting upwards) Ultrasonic introduction of acne essence 5-10 minutes (a line of squatting in a line) High-frequency acne operation 5-10 minutes (select the appropriate operating head for full face circular operation, acne area is strengthened) Apply mask for 15 minutes. Combine with led light. Sealed operation (optional blue, violet) Clean the face Cold hammer operation 5-10 minutes (one line laps up the line) Rubbing toner, acne essence End</p>	
<p>Treatment recommends Ten times for a course of treatment, once done, the acne muscle can anti-inflammatory and sterilize, stabilize the activity of acne, smooth down the acne muscle after a course of treatment, eliminate redness and swelling, fundamentally cure acne problem, improve acne skin after three courses of treatment To reduce the rate of acne outbreaks, when the skin returns to normal.</p>			
<p>Overall Skin Conditioning: 90 minutes, once a week</p>			
<p>Adjust energy: 1-5 levels Mode: A, B</p>	<p>Cleansing Oil + Cleanser + Black export liquid + Cleaning Shovel +Microdermab rasion + Electric Ion + Ultrasound + EMS</p>	<p>Cleansing makeup, 5 minutes Apply black export liquid + thermal spray for 5-8 minutes (sensitive muscle can be about 2 minutes, many blackheads can last up to 10 minutes) Wipe the face with a cotton pad and use the skin shovel for 5-10 minutes (the nose part with blackheads can be focused), Clean the face Microdermabrasion operation 5-10 minutes (select the appropriate operating</p>	

	<ul style="list-style-type: none"> + Magic gloves + High Frequency + BIO and Photon + Essence + Mask 	<p>head and suction, the face and forehead are pulled up in a line, the nose can be pulled and pulled).</p> <p>Clean the face</p> <p>Apply facial cleansing cream or cleansing lotion or skin lotion, and transfer with electric ions for 5-10 minutes.</p> <p>Clean the face</p> <p>Ultrasonic introduction of facial essence 5-10 minutes (From the chin, a line of squats in a line of upwards, can be introduced into the whitening, anti-wrinkle, acne, repair and other essence)</p> <p>Ultrasonic introduction of eye essence for 3-5 minutes (a small circle around the eye, the whole eye is introduced)</p> <p>Ion the first line of the electric ion to lead the circle into the vc essence or collagen essence 5-10 minutes (the cotton piece is wrapped around the ion-implanted rod and fixed after the vc essence is introduced)</p> <p>EMS operation (both hands start from the chin at the same time, lining up the line and lifting up the circle to select the appropriate operation of the head) 5-10 minutes</p> <p>Overall lifting operation of the glove (from neck to face) 5-8 minutes (energy from small to large)</p> <p>Acne instrument operation (choose the appropriate operating head, face type operation) 5-8 minutes</p> <p>Apply mask + LED operation for 15 minutes</p> <p>Clean the face</p> <p>Cold hammer operation 5-8 minutes (face type operation)</p> <p>Toner + essence + cream + isolation</p> <p>End</p>	 <hr/>  <hr/>  <hr/> 
<p>Treatment recommends</p> <p>Ten times for a course of treatment, after the completion of the skin, the skin is deeply cleaned in two layers, completely removing the skin waste and dirt that has remained for a long time, and at the same time replenishing nutrition and moisture. After a course of treatment, the skin is clear and refreshed, hydrating and whitening, and slowly tightening, three The treatment will create a natural, beautiful skin that is healthy, firm, supple and fair.</p>			

