

# User Manual

SR-NF106



## Preface

To our valued users,

Thanks for choosing our 3-in-1 beauty equipment for deep cleansing, hydrating, moisturizing, and skin rejuvenation and brightening. This is multifunctional equipment, which can effectively fix problems like clogged pores on the facial skin, hydroperic skin, and uneven skin tone. It applies high-tech and achieves looks improving and skin care safely and effectively. SR-NF106 is professional beauty equipment, which needs to be operated by people with professional training. Any improper use of this equipment may bring adverse consequences to the body. Thus, we advise all people to read this manual thoroughly and strictly follow the instructions specified before operation.

We believe our quality product will give you a good return and our perfect after-sales service will make you worry-free.

Thank you.

# Table of Contents

Preface
Part I
Introduction
Advantages
Skin Cleansing & Purifying
Skin Rejuvenation with Active Oxygen
Part II
Detailed Operations
Technical Parameter
Contraindications
Precautions on Use
Troubleshooting & Solutions
FAQs
Packing List
Operational Diagrams

# Part I

## 1. Introduction

At present, diamond microdermabrasion, transparent tube, and oxygen injector is the most popular instrument for deep pore cleansing. It can effectively replace the traditional facial cleansing and care programs in a beauty salon, operate easily, conveniently, and swiftly, and solve skin problems for people pursuing beauty. Moreover, this equipment works safely and effectively and requires no injection, no medication, or surgery. It has no side effects and is operated externally throughout, and has an instant effect. Diamond microdermabrasion uses the vacuum suction technique. By decreasing internal air pressure, it effectively removes useless stratum corneum. It sucks wastes as blackheads and redundant grease out from pores, thus making the skin breathe and circulate freely, dredging pores, relieving large pores, and brightening skin color. Meanwhile, the matched oxygen injector can replenish moisture to the skin and make skin cool and fresh.

## 2. Advantages

- (1) The 3-in-1 multifunctional beauty equipment fixes facial care problems.
- (2) It has multiple handpieces, which can be changed per different requirements.
- (3) The diamond tips can effectively relieve facial skin problems and recover and clean the skin
- (4) It suits all kinds of skin, can improve skin quality, and flatten fine lines.
- (5) It's comfortable, painless, non-invasive, and injury-free during treatment. And it asks for no convalescence, requires no stay, and will not affect regular work and life after treatment.
- (6) It has no consumption, with low cost, but with a quick return.
- (7) It has a broader treatment range and an evident effect.
- (8) Unevenness, bleeding, and swelling will not occur.
- (9) It can deeply clean skin and supplement moisture to and refine the skin.

## 3. Skin Cleansing & Purifying

### (1) Principle


#### Diamond Microdermabrasion

Embedded with tiny diamond particles of different sizes, the specialized diamond-toned wand, with the help of vacuum fiber, grinds off a shallow layer of the epidermis by rubbing it back and forth. It's non-invasive and physical dermabrasion. By removing the dead skin cells of the epidermis, it regenerates them, which improves skin quality. After the treatment, the skin's fine lines lessen, the skin turns smooth, flat,

and more tender. Meanwhile, it, coupled with hands, can achieve lifting.

**Physical Effect:** Diamond microdermabrasion uses the vacuum suction technique. By decreasing internal air pressure, it effectively removes useless stratum corneum. It sucks wastes as blackheads and redundant grease out from pores, thus making the skin breathe and circulate freely, dredging pores, relieving large pores, and brightening skin color.

Dermabrasion adopts diamond crystals that have been processed specially to ill skin, and use vacuum suction to remove necrotic cells from skin as well as remove the deep waste and dirty of pore, help skin double cleansing and promote growth of new skin.

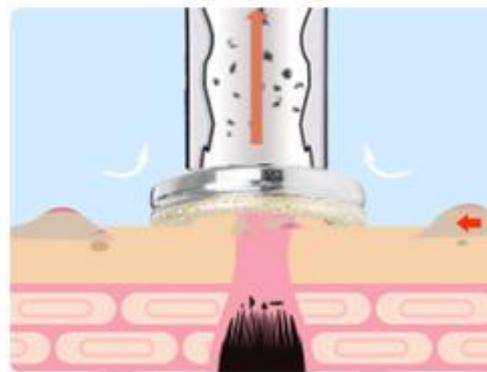

## Transparent Tube

Utilizing vacuum suction of over 50 cmHg, it sucks out obstructed substances from pores and profoundly cleanses them. Glass tubes of three different shapes deeply remove whiteheads, blackheads, and mites from the skin and completely open the obstructed pores up, doubling skin cleanliness.

MS-34P1 adopts vacuum glass set which use more than 50cmHg suction to get the specially demodex folliculorum, mites blackhead, whitehead located in the deep layer of skin, at the same time, the blocked pores will be opened totally.




## Oxygen Injector

By utilizing oxygen, water, and intense pressure, it acts on skin with molecular

particles smaller than 80 micrometers and spray, making active water molecules and pure oxygen enter into deep skin rapidly and accelerating skin nutrition absorption. Penetrating the pores and skin crack with strong pressure infiltrating from the epidermis to the dermis to stimulate cell regeneration, replenish adequate nutrition to the skin directly and rapidly, and remove wastes from the deeper epidermis. High-pressure oxygen and nutrient solution can stimulate the regeneration of the dermis's fibrous, promote metabolism, and achieve moisturizing, skin rejuvenating, and yellowish complexion elimination.

**Physical Effect:** The oxygen injector atomizes hydro-oxygen molecules. It rapidly injects hydro-oxygen molecules into each layer of the skin through pulsed impulse, supplements the needed moisture and oxygen to the cells, thus recovering cell activity and achieving skin plumpness, moisturizing, and shininess.

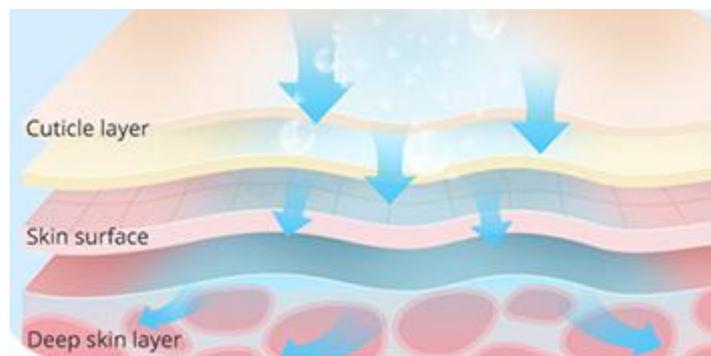

## (2)Efficacy

- 1) Cleanse deep skin and improve pores.
- 2) Remove the excessive stratum corneum.
- 3) Replenish moisture and nutrition.
- 4) Improve skin quality and increase skin elasticity.
- 5) Improve uneven and unsmooth skin.

## (3)Indications

- 1) Those with dry or hydropenic skin.
- 2) Those with large pores or coarse or lackluster skin.
- 3) Those with poor skin absorption or hydropenia of the underlying skin.
- 4) Those with dark skin or uneven skin color.
- 5) Those with oversecretion of grease or oily or dry skin.
- 6) Those who don't clean the skin properly in daily life or with unsmooth skin.

## (4)Contraindications

- 1) Those with severely sensitive skin or in the allergic period.
- 2) Those with extensive trauma on the face.

- 3) Women in pregnancy, or those with severe heart disease, serious hypertension, hyperglycemia, hyperlipidemia, other serious diseases, etc.
- 4) Those with skin disease or infectious disease.
- 5) Those who just had water light, microneedle, or with skin wound.

## (5)Matters Needing Attention After Treatment

- 1) Wash the face with warm water within three days.
- 2) Keep hydrating and protect yourself from the sun.
- 3) Don't use irritant skincare products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
- 4) Avoid staying up late, smoking, or drinking alcohol. Instead, eat more vegetables, fruits, and light food.
- 5) Apply a facial mask at least three times a week.


## 4. Skin Rejuvenation with Active Oxygen

### (1)Principle

#### Oxygen Injector

By utilizing oxygen, water, and intense pressure, it acts on skin with molecular particles smaller than 80 micrometers and spray, making active water molecules and pure oxygen enter into deep skin rapidly and accelerating skin nutrition absorption. Penetrating the pores and skin crack with strong pressure infiltrating from the epidermis to the dermis to stimulate cell regeneration, replenish adequate nutrition to the skin directly and rapidly, and remove wastes from the deeper epidermis. High-pressure oxygen and nutrient solution can stimulate the regeneration of the dermis's fibrous, promote metabolism, and achieve moisturizing, skin rejuvenating, and yellowish complexion elimination.

The oxygen injector atomizes hydro-oxygen molecules. It rapidly injects hydro-oxygen molecules into each layer of the skin through pulsed impulse, supplements the needed moisture and oxygen to the cells, thus recovering cell activity and achieving skin plumpness, moisturizing, and shininess.

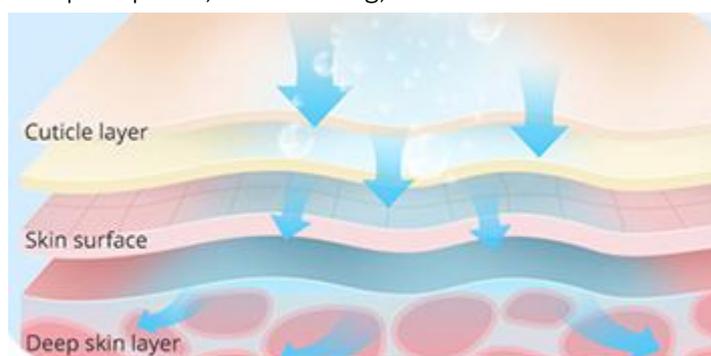

## (2)Efficacy

- 1) Replenish moisture to deep skin and replenish nutrition.
- 2) Shrink pores and whiten and rejuvenate the skin.
- 3) Do hydrating and moisturizing and lighten skin color.
- 4) Balance grease and improve oily skin.

## (3)Indications

- 1) Those with dry skin, fine lines, or lacking moisture.
- 2) Improve dark, uneven, or lusterless skin color.
- 3) Those with poor skin absorption or hydropenia of the underlying skin.
- 4) Those with oversecretion of grease or oily or dry skin.

## (4)Contraindications

- 1) Those with severely sensitive skin or in the allergic period.
- 2) Those with extensive trauma on the face.
- 3) Those with skin disease or infectious disease.
- 4) Those who just had water light, microneedle, or with skin wound.

## (5)Matters Needing Attention After Treatment

- 1) Wash the face with warm water within three days.
- 2) Keep hydrating and protect yourself from the sun.
- 3) Avoid staying up late, smoking, or drinking alcohol. Instead, eat more vegetables, fruits, and light food.
- 4) Apply a facial mask at least three times a week.

# Part II

## 1. Detailed Operations

### (1) Function Selection



On/Off



Rotating Knob for Suction Setting(Set the suction per individual situation.

Do not set it too high. Otherwise, it may trigger redness.)



SUCTION

Suction Hole. Insert one end of the rubber hose to the suction hole and the other end to the handpiece.



RELEASE

Vent Hole.

## (2)Detailed Operations for Diamond Microdermabrasion



Filter Cotton(A filter cotton must be installed when operating the machine. Otherwise, dirt may enter it, causing damages. Meanwhile, the cleaning effect can be seen.)



There are nine diamond tips, which can be chosen per individual skin condition.



This is the biggest, called the No.1 Diamond Tip(for large area or body)



No.2 Diamond Tip(for a large area or body parts)



No.3 Diamond Tip(for a large area on the face)



No.4 Diamond Tip(for a large area on the face, face lifting, extensive stratum corneum removal)



No.5 Diamond Tip(for removing stratum corneum on the cheeks, cleansing, and sucking pore dirt)



No.6 Diamond Tip(for sucking blackheads on the face and nose and pore obstruction)



No.7 Diamond Tip(for cleaning the face and nose and removing stratum corneum on them)



No.8 Diamond Tip(for sucking blackheads on the root of the nose and removing dirt from it)




No.9 Diamond Tip(for removing the deep-seated blackheads from the root of the nose and of the small area)



Rotating Knob for Suction Setting(Set the suction per individual situation.

Do not set it too high. Otherwise, it may trigger redness.)

### (3)Detailed Operations for Transparent Tube

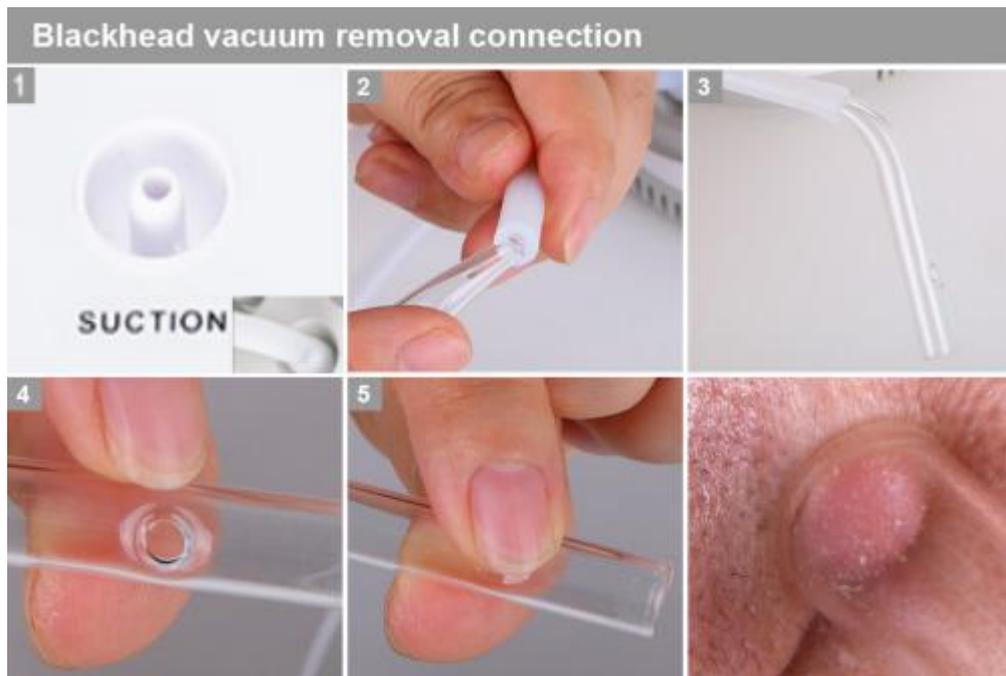

#### Glass tube



- 1: for massaging the small area of the face.
- 2: for fixing deep-seated clogged pores of a small area.
- 3: for cleaning blackheads of a small area.



Rotating Knob for Suction Setting(Set the suction per individual situation.

Do not set it too high. Otherwise, it may trigger redness.)

## (4)Detailed Operations for Oxygen Injector



Connect one end of the rubber hose to the vent hole and the other to the oxygen injector.



Pour water into the bottle(it can be filled with pure water, water-based serum, toner, normal saline, etc. The dilution rate for essence and distilled water is 2 to 1. Select per the skin condition.).



Rotating Knob for Spray Setting



Press the lever to spray the water mist. As shown in the following.



## 2. Technical Parameter

Vacuum Therapy Parameters

Power: 60W

Pressure: >250kPa

Air flow: >10L/minute

Noise level: <70dB (30cm away)

### 3. Contraindications

Those with the following situations should use the equipment with caution. Please consult a doctor or professional before using it. The details are as follow:

- (1) Women in pregnancy or lactation.
- (2) Those with heart disease or heart pacemakers.
- (3) Those with unhealed surgical wounds or in surgical recovery.
- (4) Those with epilepsy, severe diabetes, or hyperthyroidism.
- (5) Those with malignant tumors, hemophilia, or severe bleeding.
- (6) Those with skin disease or infectious disease should use it with caution.
- (7) Those with metal implants inside the body(such as a stent, pacemaker, etc.)or who are allergic to metals.

### 4. Precautions on Use

- (1) A plug with a ground pin must be used, and a power socket that is already grounded must be ensured before using the equipment.
- (2) Make sure the voltage of the equipment is adaptive. If the local power supply voltage is not stable, we suggest users add a voltage regulator with matching power between the mains supply and the equipment.
- (3) To guarantee the equipment's curative effect and normal service life, please uniformly use specified parts provided or suggested by the original manufacturer.
- (4) The equipment can't be placed in damp places or near water, and it also can not be exposed to direct sunlight.
- (5) Please don't place the equipment near an intense heat source since this may affect its service life and everyday use.
- (6) Please remove all the metal objects from the body before use to avoid unexpected situations which may affect the curative effect.
- (7) Please refrain from using the equipment to aim at eyes, thyroid, parathyroid, testicles, pregnant woman's abdomen, pacemaker, etc.
- (8) Those suffering from an illness should use it with caution unless they get permission from a doctor.
- (9) Please turn the equipment's power switch off if no one uses it, and ensure the main power is off after everything was settled. Thus the safety of the equipment can be guaranteed.
- (10) Start from the lowest energy intensity when start, and increase the intensity gradually after the client got used to it.
- (11) When using this equipment, parts to be treated must be kept moist, and dry skin treatment should be avoided.
- (12) The diamond microdermabrasion and glass tube are prohibited from using on the rim of the eyes, ears, the periphery of parts contracting skin diseases, and parts

had plastic surgery.

- (13) After the operation, clean the equipment with normal saline to ensure its cleanliness and hygiene and extend its service life.
- (14) Keep in mind that the air outlet and air inlet must not be mixed. And don't insert diamond microdermabrasion into the air inlet. Otherwise, the equipment may not work.
- (15) When in use, a rubber ring must be mounted. Otherwise, there is no suction, or the suction will be weakened.
- (16) When in use, filter cotton must be installed. Otherwise, dirt may enter the air duct causing damages to the main machine.

## 5. Troubleshooting & Solutions

- (1) The equipment can't be started, and the buttons in its back are not lighting up?
  - A. Make sure the power cord is plugged into a suitable power socket.
  - B. Check whether the fuse in its back is loose or burnt out.
  
- (2) When operating the diamond microdermabrasion, there is no suction?
  - A. Please check whether the connecting tube is properly connected to the main machine.
  - B. Please check whether the connecting tube is correctly or properly connected to the diamond microdermabrasion.
  - C. Please check whether the rotating knob for the suction setting is set to zero.
  
- (3) The oxygen injector doesn't have spray?
  - A. Please check whether the injector head is blocked.
  - B. Please check whether the setting part of the injector tail is tightly twisted.
  - C. Please check whether the injector lever is loose.

## 6. FAQs

- (1) Q: Will the skin become thin and sensitive after having diamond microdermabrasion treatment?

A: No. Diamond microdermabrasion only gets rid of the aging stratum corneum during treatment. Our skin refreshes every 28 days. Thus this treatment only makes the skin more smooth, tender, and transparent and causes no sensitive symptoms.

- (2) Q. Why does keratin must be removed?

A: The stratum corneum is the top layer of the epidermis, in which cells are entirely dead. Depending on body parts, the stratum corneum is consists of 15 to 20 layers of dead cells. The skin has a metabolic cycle of 28 days, precisely the cycle for people in adolescence. For adults, the cycle is their age plus 10. However, due to various reasons, the aging and dead keratinocyte may not shed timely, which causes acanthokeratoderma, makes the skin lose its transparency, affecting skin color and

metabolic turnover. Therefore, if the metabolism cannot be carried out under normal conditions, then an artificial method must be used to achieve the purpose.

(3) Q: Can I wear makeup after using this equipment?

A: Absolutely yes. After treatment, the skin turns moisturized and plump. It's more appropriate to wear makeup at this time. This treatment requires no skin breaking and is injury-free, safe, healthy, rapid, and effective.

(4) Q: How long does it take to see the effect?

A: The skin becomes clean and bright obviously after one-time treatment, which will become more evident if you stick to it. This treatment can relieve hydropenia of the deep skin, dark skin, fine lines, and aging.

(5) Q: Is there any dependency on this treatment?

A: No. Targeted skin problems, it improves and deeply cleanses the skin. And it maintains the skin and prevents it from having problems caused by other factors, helps the skin absorb nutrition, relieves skin that is dull yellow, hydropenic, coarse, and wrinkled. If you quit later, the skin will only back to its previous secretion and senility. In a word, it will not cause dependency.

## 7. Packing List

Main Machine x1  
 Diamond Tip x9  
 Diamond Toned Wand x3  
 Transparent Tube x3  
 Oxygen Injector x1  
 Rubber Hose x1  
 Power Cord x1  
 Filter Cotton x1 pack  
 Rubber Ring x1 pack

## 8. Operational Diagrams

Parameter Setting	Product	Technique	Diagram
Deep Cleansing: 60 Minutes/ Once a Week			
<b>Diamond Microdermabrasion</b> Operation and	Makeup Remover + Facial Cleanser +	1. Remove makeup and clean the face, 5 minutes. 2. Apply toner, 1 minute. 3. Apply export liquid and	Technique 4

<p>operating time can be adjusted per clients' skin types(Set the energy level, and then have a trial on the lower arms before operating it on the face. Ask the client whether he feels comfortable.)</p> <p><b>Transparent Tube</b> Operation and operating time can be adjusted per clients' skin types(Set the energy level, and then have a trial on the lower arms before operating it on the face. Ask the client whether he feels comfortable.)</p> <p><b>Oxygen Injector</b> Use it to operate when it sprays water mist.</p>	<p>Export Liquid + Cold &amp; Hot Steam + Essence + Facial Mask + Equipment</p>	<p>use hot steam(Apply the liquid to parts with large pores or severely clogged pores. Cheek can be applied to toner, and the face with sensitive skin can be applied to toner and used with cold steam. But it's not advised to use hot steam and apply the export liquid to it.), 5 to 8 minutes.</p> <p>4. Treat the whole forehead and nose with diamond microdermabrasion. Appress the device onto the forehead and leave it there for 1 second, then gently revolve the device with fingers and bring it up(forehead→nose→chin), 5 minutes.</p> <p>5. Treat nasal fossa with diamond microdermabrasion, then slide the device back and forth on nasal fossa, 1 minute.</p> <p>6. Treat cheeks. Start with the chin, align three points, treat it again after having every two to three points be treated, and move one line after another until Temple, 3 minutes.</p> <p>7. Do the same on the other side.</p> <p>8. Have more cleansing with the transparent glass tube.</p> <p>9. Forehead(select the flat operating head): Move slowly from the brow to the hairline direction and treat the whole forehead.</p> <p>10. Nose(select the prolate operating head): Start from the nasal fossa, moving</p>	<div data-bbox="1134 219 1262 421" data-label="Image"> </div> <p data-bbox="1038 488 1214 521">Technique 6</p> <div data-bbox="1134 551 1262 752" data-label="Image"> </div> <p data-bbox="1038 819 1214 853">Technique 9</p> <div data-bbox="1134 882 1262 1084" data-label="Image"> </div> <p data-bbox="1038 1151 1230 1184">Technique 11</p> <div data-bbox="1134 1214 1262 1415" data-label="Image"> </div>
--	---	---	--

	<p>slowly, 1 to 2 minutes.</p> <p>11. Chin &amp; Cheeks(select the curve operating head): Move slowly from the chin to the part below the ear, from the corner of the mouth to ear center, and from the wing of the nose to Temple, 1 to 2 minutes.</p> <p>12. Do the same on the other side.</p> <p>13. Clean the face, 2 minutes.</p> <p>14. Oxygen Injector: Pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into an oxygen injecting bottle(dilution rate for essence and distilled water is 2 to 1), 3 minutes.</p> <p>15. Move one line after another and circlewise on the whole face, from chin to the back of the ear, and from the corner of the mouth to ear center. Rub the treated parts gently with the other hand. Repeat two times.</p> <p>16. Treat from the forehead to hairline, and rub the treated parts gently with the other hand. Repeat two times.</p> <p>17. Apply a facial mask and wait for 15 minutes.</p> <p>18. Clean the face, 2 minutes.</p> <p>19. Apply toner, essence, facial cream, and sunscreen.</p> <p>20. Treatment is done.</p>	
--	---	--

**The Recommended Course of Treatment:**

Once a week. A course of treatment includes ten times. After one-time treatment, the skin turns clean and clear. After a course, the skin becomes moisturized and plump. Grease secreta decreases and water and oil returns to normal. After two courses, the

skin gets delicate, shiny, moisturized, and elastic.

## Skin Rejuvenation with Active Oxygen: 60 Minutes/ Once a Week

<p><b>Oxygen Injector</b> Use it to operate when it sprays water mist.</p>	<p>Makeup Remover + Facial Cleanser + Cold &amp; Hot Steam + Massage Cream + Moisturizing Mask + Essence + Facial Mask</p>	<ol style="list-style-type: none"> <li>1. Remove makeup and clean the face, 5 minutes.</li> <li>2. Apply a moisturizing mask and use hot steam, 10 minutes.</li> <li>3. Clean the face, 2 minutes.</li> <li>4. Apply toner, 1 minute.</li> <li>5. Apply massage cream evenly to the face, and caress it three times.</li> <li>6. Press Chengjiang(RN-24), Renzhong(DU-26), Dicang(ST-4), Jiache(ST-6), Yingxiang(LI-20), Jingming(BL-1), Cuanzhu(BL-2), Yuyao(EX-HN4), Sizhukong(SJ-23), Temple, Tongziliao(GB-1), Chengqi(ST-1), and Sibai(ST-2). Repeat three times.</li> <li>7. Caress the whole face three times.</li> <li>8. Lift in three lines with hands doing it alternately, from the chin to earlobe, from the corner of the mouth to Ermen(SJ21), from the wing of the nose to Temple, and from the lower eyelid to Temple, and lift the corner of the eye. Repeat three times.</li> <li>9. Do the same on the other side.</li> <li>10. Lift the forehead towards the hairline direction. Repeat three times.</li> <li>11. Flip and lift the face with rotating fingers. Repeat 3 to 5</li> </ol>	<p>Technique 5, 7</p> 
		<p>Technique 6</p> 	
		<p>Technique 8</p> 	
		<p>Technique 10, 12</p> 	
		<p>Technique 15</p> 	

	<p>times.</p> <p>12. Flip the forehead towards the hairline with one hand. Repeat 3 to 5 times.</p> <p>13. One side: Flip and lift the face with hands doing it alternately. Do it for 2 to 3 minutes.</p> <p>14. Do the same on the other side.</p> <p>15. Move zigzag on the whole forehead with middle and ring fingers, then slide to the front of the ear and lift to and fro for three times, and slide out from the back of the ear.</p> <p>16. Clean the face, 2 minutes.</p> <p>17. Oxygen Injector: Pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into an oxygen injecting bottle(dilution essence and distilled water is 2 to 1), 3 minutes.</p> <p>18. Move one line after another and circlewise on the whole face, from chin to the back of the ear, and from the corner of the mouth to ear center. Rub the treated parts gently with the other hand. Repeat two times.</p> <p>19. Treat from the forehead to hairline, and rub the treated parts gently with the other hand. Repeat two times.</p> <p>20. Apply a facial mask and wait for 15 minutes.</p> <p>21. Clean the face, 2 minutes.</p> <p>22. Apply toner, essence, facial cream, and sunscreen.</p> <p>23. Treatment is done.</p>	
<p><b>The Recommended Course of Treatment:</b></p>		

A course of treatment includes ten times. After one-time treatment, the skin becomes tightened and rejuvenated and is supplemented with the needed nutrition. After a course, the skin gets smooth, delicate, and shiny. After two courses, the skin turns firm, rejuvenated, bright, and elastic. After three courses, the elastic, shiny, and rosy skin maintain.