



Five in one

(Negative Pressure Roller Three handles
+ Inner Ball Roller Machine Two handles)

Please read the instruction manual carefully and follow the instructions before use.

A. Product Description

Welcome to choose the latest product of our company, the five-in-one multifunctional combination instrument, which is a combination of **Negative pressure Roller Machine Three handles** and **inner ball roller Machine Two handles** to realize multi-function use of one machine.

Negative pressure Roller is an innovative beauty instrument based on pure physical and mechanical massage. It uses "Stereoscopic Negative Pressure Mechanical Stimulation" technology. It is composed of a nursing head device with two-way electric roller combined with vacuum negative pressure, and another nursing head device is composed of vacuum negative pressure combined with electric flaps. Head in each part of the human body through the high-tech care for scroll machinery, suction, kneading, clamp, bounce and other mechanical massage action to activate and repair of connective tissue, activate the diastole and contraction, muscle tissue and soften the circulatory system and the lymphatic system, and by the activation of autologous skin renewal from the inside out, make its reactivation to complete different treatment goals, Such as metabolism fat, eliminate edema, smooth fine lines, firm skin, elastic and tight and anti-aging, both healthy and safe to let you perfect metamorphosis. It is used for facial beautification and anti-aging, physical therapy and slimming.

Negative pressure Roller is focused on beauty and wellness. Its biggest benefit is that it is non-invasive, safe and 100% natural. All forms of connective tissue transformation, muscle and circulatory disease can be treated. Use different body and face probes to send powerful, efficient, comfortable and painless activation signals to the deep layer of the skin to stimulate skin

activation, repair and renewal in an all-natural way, restoring natural physiological processes. Using negative pressure suction, two-way roller force, double d take force of sunny and realize mechanical massage naturally activate cells and stimulate collagen and elastin regeneration, recover skin luster and elasticity, but also can promote lymphatic drainage and improve blood circulation of skin detox, break down fat deposits at the same time, to reduce fluid retention, smooth the orange peel, firming skin, fade wrinkles, shape, Relieve fatigue, reduce muscle tightness pain, eliminate sub-health state. 100% natural and safe intelligent beauty, organic fitness and slim anti-aging concept, known as the "fashion figure designer", deeply favored by women, Hollywood stars, models and athletes.



inner ball roller machine uses innovative compression micro-vibration + infrared technology. This technology is realized by a 360° intelligent rotating cylindrical roller handle. A roller device composed of 50 silicone balls is used for body care; another smaller handle is a roller device composed of 60 silicone balls for face or neck care. Silicone balls are placed in a honeycomb shape with a specific density and diameter. The direction of rotation and pressure ensure that the micro-compression is transmitted to the tissue. The unique handle produces mechanical compression and micro-vibration on the skin, which cannot be replicated by hands or other technologies. "Compression micro-vibration therapy" replaces the traditional method of "suction traction massage", using micro-compression instead of suction, and micro-vibration instead of traction. When it used for the body, which can help to improve

lymphatic drainage and circulation, deep detoxification, relieve muscle stiffness and soreness, break down subcutaneous fat deposits, eliminate cellulite, reduce cellulite and shape contours. After a few procedures, your body will immediately reduce its volume by 1-2 sizes. When it is used for the face, which can help to improve the formation of blood vessels, thereby supporting the production of collagen and elastin, increasing oxygen delivery, helping to nourish and brighten tissues from the inside, in addition, it can relax and regulate muscles and help reduce expression wrinkles. Appears, reduces swelling, fights tissue sagging, and rejuvenates the face significantly. Compression micro-vibration therapy is specially designed for non-invasive skin rejuvenation, body shaping and physical therapy.


It is the latest generation of body care for the face, neck, shoulders, arms, chest, back, waist, abdomen, buttocks, legs and feet, etc. Compression micro-vibration therapy is a treatment that can reactivate blood circulation, a method that can care for complex diseases such as lymphatic stasis, lymphedema and cellulite, so it can improve skin nutrition and cell oxygenation, compression micro-vibration therapy It is an innovative combination between beauty and health.

B. Working Principle

Negative pressure Roller adopts "Three-dimensional Negative Pressure Mechanical Stimulation" technology, which is a non-invasive vacuum negative pressure massage therapy. The principle is that through the bidirectional electric roller combined with vacuum negative pressure of the nursing head device, and the electric flap combined with vacuum negative pressure of the nursing head device, the skin tissue to carry out rolling, suction, kneading, clamping, bouncing and other mechanical aerobic exercise, skin tissue repeated rhythmic deep massage and mechanical stimulation. Continuity can be deeper into the connective tissue and adipose tissue, subcutaneous tissue, lymph circulation system and blood circulation system, and thus to be skin organization exerts special physical stimulation and the actuating effect, make skin tissue under the oppression and wrinkled, and under the effect of positive and negative pressure produces delicate mechanical stimulation to deep skin tissue, the treatment areas produce biochemical reaction, Thus, it can effectively activate the decomposition of subcutaneous adipocytes and reduce the accumulation of fat in honeycomb tissue, increase the circulation of connective tissue blood and lymph, take away metabolic waste, eliminate excessive water and toxin in the body, eliminate honeycomb tissue, relieve muscle pain, and achieve the effect of thin body. It also stimulates the activity of fibroblasts, promotes the proliferation of collagen, elastin and hyaluronic acid, makes the whole body skin firm, full and elastic, reduces the orange peel lines, wrinkles and improves the distribution of fat layer, restores the health and vitality of skin tissue, and shapes a slimmer, more contorted body and young skin. He is the expert of "healthy body sculpture management". The treatment needs to wear special tights, so that the roller can give the body a smooth deep massage.

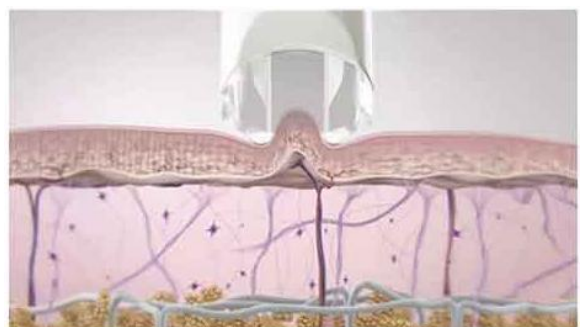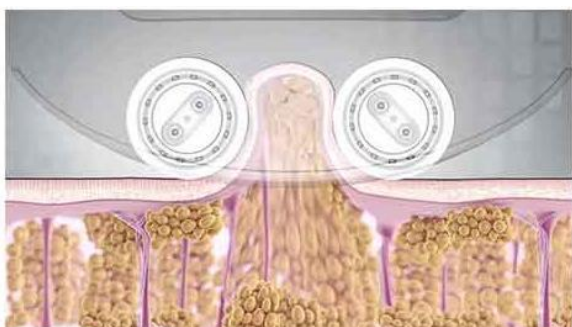


The **inner ball roller** machine is a non-invasive mechanical compression micro-vibration + infrared treatment. The principle is to generate compression micro-vibration by rolling the silicone ball along the 360° rotation of the roller. As the ball rotates and exerts pressure on the skin, it produces a "pulsation compression" effect, which realizes the reciprocating motion of continuous pushing, pulling and kneading, and the tissue will experience some pressure. And lifting action, will not squeeze or damage the skin, its principle is to release infrared rays while pressure is applied to the tissues to stretch the cells to naturally and deeply stimulate cell activity, blood flow and oxygenation, and the fat deposits are pressurized and thus loosened for final decomposition , Reduces cellulite and removes cellulite; also exerts pressure on deep muscle groups to fully soften and stretch, thereby reducing muscle stiffness and soreness, accelerating metabolism, eliminating stagnation and fluid accumulation, conditioning tissues and re-tightening skin tissues, reshaping your body . It can also stimulate fibroblasts, increase the production of collagen and elastin, increase blood flow and increase oxygen. As a result, wrinkles are smoothed, puffiness and bags under the eyes are reduced, and the skin is rejuvenated and tightened. This technology has been clinically proven to help tone muscles and sculpt the body and face to rejuvenate. It can also help redesign and tighten the chest.

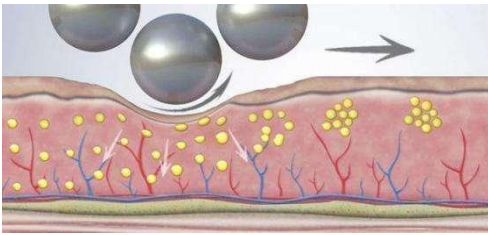


Reshaping action

Reshapes contours and smooths out wrinkles.

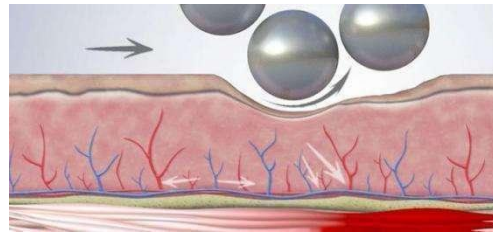


Muscle action

Tones muscles and tightens tissues.

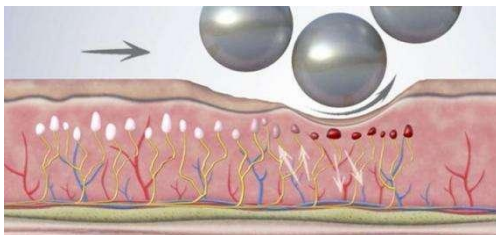


Sedation

Improve oxygen and eliminate pain.

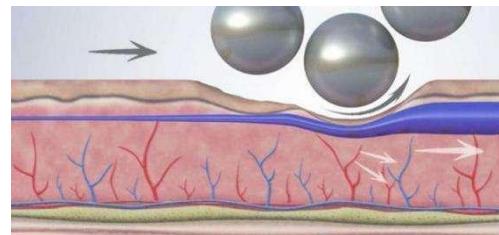

Drain action


Eliminate toxins and help lymphatic drainage.

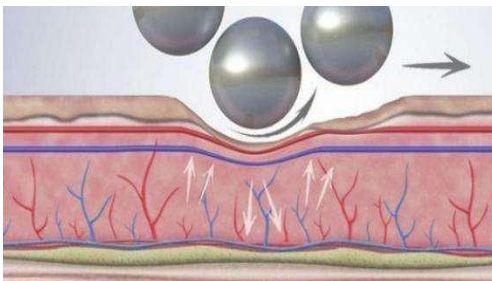

Vascular effect

Improve blood circulation and nourish tissues.

C. Inner ball roller Machine-Five Collaboration effects

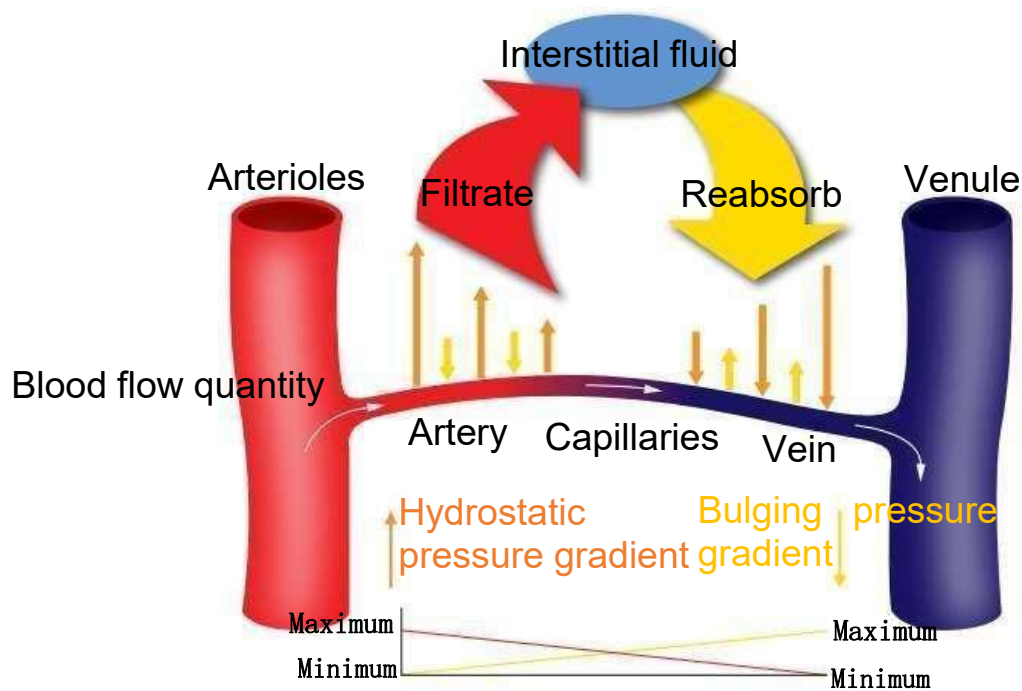


Physiology of capillary circulation

① Vascular effect

The balance between hydrostatic pressure and bulging pressure usually allows fluid and nutrients to flow from the arterial side, and fluid and catabolites to re-enter the venous side. The increase in hydrostatic pressure is due to the slowing down of venous outflow, which results in water stagnation in the extracellular fluid, forming edema inside the tissue matrix.

Due to the special arrangement of the spheres, they are in a honeycomb shape, through pressure and lifting movements to obtain gradual tissue compression, achieving the effect of vascular exercise.

The "compression microvibration" therapy allows us to balance the hydrostatic pressure of the veins by reversing the internal metabolism and hemodynamic exchange mechanism.

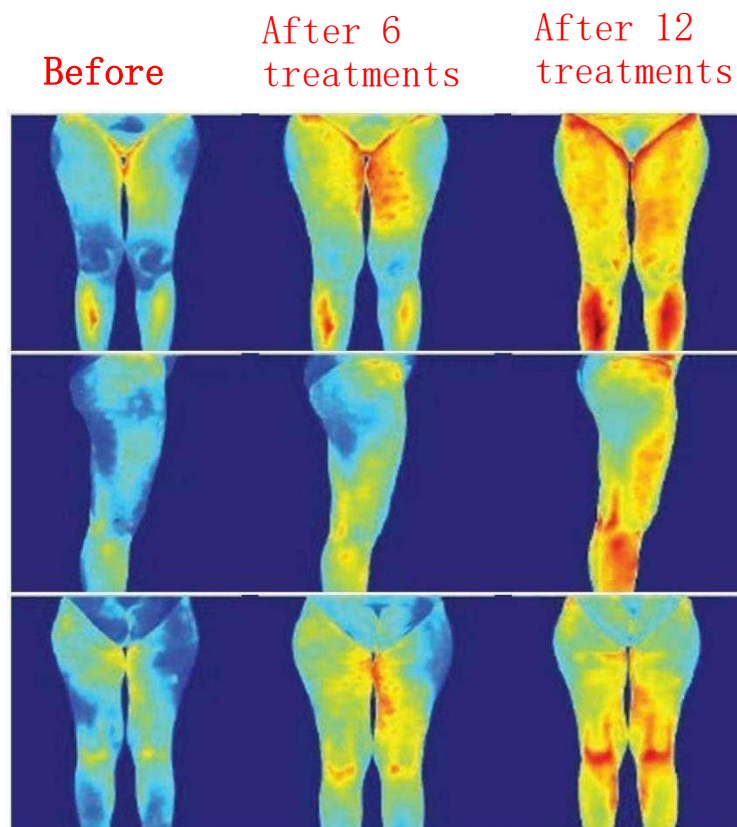

This activity has been scientifically confirmed by the thermal imaging evaluation on the right. This heat increase effect will lead to "increased skin perfusion and oxygenation, increased tissue metabolism, decomposition of fat aggregates and activation of the anti-inflammatory process of tissue changes. "To reduce and eliminate the pain caused by inflammation.

② Drainage effect

Edema is the result of an imbalance between fluid supply and drainage, so water accumulates in the gaps of the organism. The increase in the volume of interstitial fluid leads to the number of stagnant tissues or serous cavities in the latter, which is relevant from a clinical point of view. The most obvious symptom of this condition is swelling, which is manifested as palpable and compressible swelling (pitting). Over time, the retention of toxic substances in the tissues will change the state of the interstitium .

The direction of rotation of the sphere is opposite to the direction of application, which strengthens the activity of the lymphatic system and produces a pumping effect with the gradual movement of the liquid, thereby restoring and maintaining the stability of the blood vessels and the extracellular matrix.

"Compression micro-vibration" therapy is a rhythmic pulsating compression effect, which can stimulate lymphedema, lipoedema and other typical interstitial stasis components, improve deep lymphatic drainage, and eliminate tissue edema and fluid stagnation. It does not cause any skin traction or suction. The use of "compression micro-vibration" therapy in the state of interstitial stasis is more effective than manual lymphatic drainage.

③&④ Toning to relax muscles and relieve pain

This mechanical rotation exerts rhythmic pulsating compression on the tissues, which in turn generates vibration stimulation, so that the stiff and sore deep muscles are fully soft and stretched, thereby eliminating pain and contractures. The non-invasive "compression micro-vibration" patented system is more special and in-depth than manual treatment.

In addition, the compressed micro-vibration also relieves the simulated contraction action of facial muscles. Vascularization increases. Vascularization allows fibroblasts to naturally stimulate the production of collagen, elastin and hyaluronic acid, making tired, dull or sagging skin tone. Rejuvenate and re-awaken your natural beauty. This treatment works deep in the skin, conditioning facial muscles, relaxing contracted muscles (expression lines), fighting tissue sagging and reducing the appearance of wrinkles.

⑤ Remodeling effect

Due to the synergy between mechanical compression micro-vibration and infrared rays, it improves blood circulation and lymphatic flow in the tissues, breaks down fat aggregates and fibrous membranes, reduces cellulite, improves cellulite, makes them less hardened and makes the skin more Firm and smooth. Therefore, it can reduce blemishes and produce remodeling effects from the first few treatments.

This activity is amplified by the response of the underlying muscle layer, which provides active resistance and strengthens the response action. In this way, it can also cover the 24 most ingrained cellulite types and the most serious conditions, such as changes in the adipose tissue of the chest or hip. These activities determine the local skin remodeling, which is conducive to the physiological reorganization of the tissues. Thanks to blood vessels, metabolism and purification activities, as well as remodeling the connected system.



Before

Ultrasound examination showed tissue remodeling after 6

Clinical observations have shown that compression microvibration therapy is an effective treatment method for stimulating blood volume, increasing oxygenation, improving lymphatic drainage and improving microcirculation. This therapy is also an ideal method for treating venous stasis and eliminating toxins.

D、 Negative pressure Roller-Three Management



Facial Beauty and Anti-aging: Through facial operation head, it can revitalize human fibroblasts to generate collagen, elastic fibers, hyaluronic acid and other substances, and restore skin vitality;

Can solve the facial skin yellowish dull, contour is not clear, lack of elasticity, skin slack, wrinkles, double chin, and facial depression.

Slim body Shaping: For the decomposition of adipocytes, but also through the mobilization of α_2 receptors and β receptors to suppress, decompose lipids, into muscle energy, reduce fat volume, to tighten the skin, shape effect. It can solve cellulite accumulation, body obesity, orange skin tissue, moisture edema and etc.

Physical therapy: Revitalize endothelial cells, expel excess water and harmful substances from the body, and rejuvenate the body, away from injuries and muscle aches caused by some sports or other reasons;

It can solve myofascial pain, muscle strain swelling, traumatic swelling and inflammation, muscle strain, muscle tissue fibrosis, lactic acid accumulation muscle acid swelling and other sub-health state.

E、Features

1. The body handle is equipped with a 2.8-inch touch screen, which is easier to control and adjust.
2. Frequency conversion negative pressure attraction: pure physical property has no damage to the skin, the best effect is the most in-depth
3. Two-way power roller: the contact surface is wider, the function area is larger, the function depth is deeper.
4. Personal exclusive collet: three-dimensional effect deep, intelligent infinitely variable speed stimulation depth is different.
5. Special shaping clothes: double the effect of sculpting parts, enhance comfort, and take into account personal privacy and health.
6. Unique 360° intelligent rotating drum handle, continuous long-term operation mode, safe and stable.
7. There is an LED display on the handle to display the time and speed, and an LED display light pole, which makes it easier to control and adjust the rotation direction and speed on the body handle.
8. One-key switch between forward and reverse directions.
9. The silicone ball is flexible and smooth, effortless, the rolling process is gentle and does not sting, the movement is soft and evenly pushed, massaged and lifted to achieve the best effect.
10. No need for beautician laborious massage, simple and safe operation.

F、 Application part



G、Accessories

body Negative pressure Roller Handle

2.8 Inch Touch Screen



- The body negative pressure roller handle combines negative pressure suction + mechanical electric roller technology. Rolling, Use the massage head to perform powerful and gentle rolling, suction, and kneading movements on the skin, Generating multi-dimensional

physical stimuli while promoting various physiological mechanisms, activating connective tissue from the inside out, wake up sleeping cells, Restart the skin's self-renewal mechanism, dredge lymphatic drainage, reduce swelling, smooth cellulite, improve body pain, Reduce cellulite and achieve body line management.



Standard /1 pc: Roller nursing head-80mm

- suitable for parts with large body area, such as thigh, hip, abdomen, back and so on.



Standard /1 pc: Roller nursing head-70mm

- suitable for moderate area, such as thigh, hip, abdomen, back.



Standard /1 pc: Roller nursing head-50mm

- suitable for small and difficult to reach areas or narrow areas, such as shoulders, arms, calves and so on.

M30 roller handle




- The M30 roller handle combines roller (rolling) + negative pressure suction (suction) technology. The treatment roller can rhythmically lift and release the body's cortex, perform deep massage through manual rolling friction, activate connective tissue, promote blood and lymphatic

circulation, replenish nutrients and oxygen for cells, and take away metabolic waste toxins. Targeting the hard-to-reach blind areas of the body.

- Suitable for small area, and complementary body handle can not operate the area, such as shoulders, small hands. etc.

Flap clamping handle

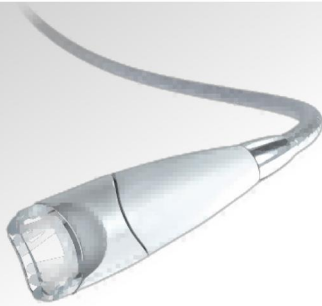

- The flap clamping handle combines negative pressure + clamping + elastic technology, through the variable frequency Flap clamping nursing head, it produces multi-dimensional physical stimulation on the skin, Triggers a biological response ,

Produce beneficial substances such as collagen and elastin that are beneficial to the skin, Wake up fibroblasts, make the skin plump, firm, and even, achieving improved facial anti-aging.



Standard /1 pc: Flap clamping nursing head-large

- suitable for face, neck, etc.




Standard /1 pc: Flap clamping nursing head-small

- suitable for narrow areas, such as around the eyes and lip contour, fingers, etc.

2. Inner Ball Roller Appendix

Body Roller Handle

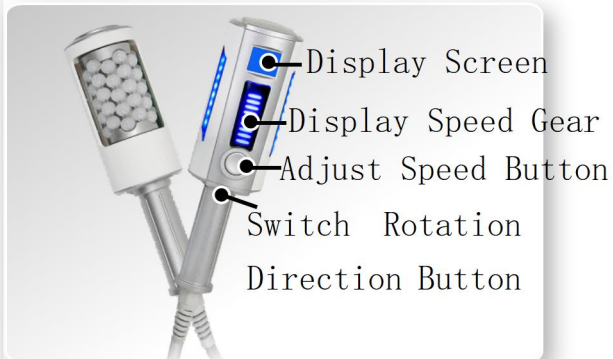

- The ball body handle combines mechanical compression micro-vibration and infrared ray technology in one, the reciprocating motion of continuous push-pull kneading on the skin, the tissue will experience some pressure and lifting actions, which helps to tone and firm the skin tissue, reduce cellulite and shape contour.



Standard /1 pc:
Silicone Large Head




Optional: Metal -
Gold Plated
Large Head




Optional: Metal -
Silver Plated
Large Head

Facial Roller Handle

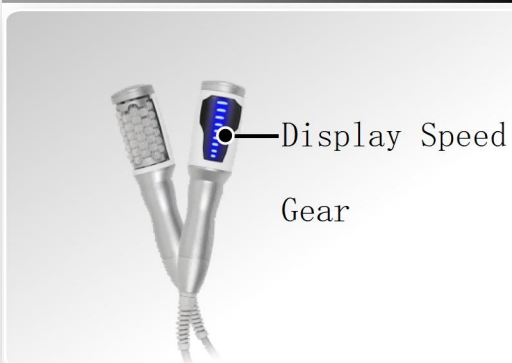

- The facial roller handle combines mechanical compression micro-vibration and infrared technology in one, and the reciprocating motion of continuous pushing, pulling and kneading of the skin will cause the tissue to undergo some pressure and lifting actions, effectively reducing swelling, resisting tissue sagging, and rejuvenating the face significantly.

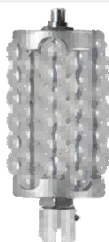

Standard/1 pc :
Silicone Small Head



Optional: Metal -
Gold Plated
Small Head



Optional: Metal -
Silver Plated
Small Head

- Silicone is soft and comfortable good for lifting.

- Metal is good for lymphatic.

H、 Product Instructions

1. Installation: Align the handle with the handle socket on the back of the device, insert and place it;
2. Insert the power cord into the rear socket of the device, turn on the power and turn on the power switch, and then press the power button to start the device.




Installation note: align the hole position, insert the handle and rotate the nut to lock the handle to ensure that the handle is locked to prevent the plug wire of the handle from loosening.



Note: If you need to use the M30 roller handle, remove the Flap clamping handle and replace the M30 roller handle.




3. Filter Indication and Maintenance:

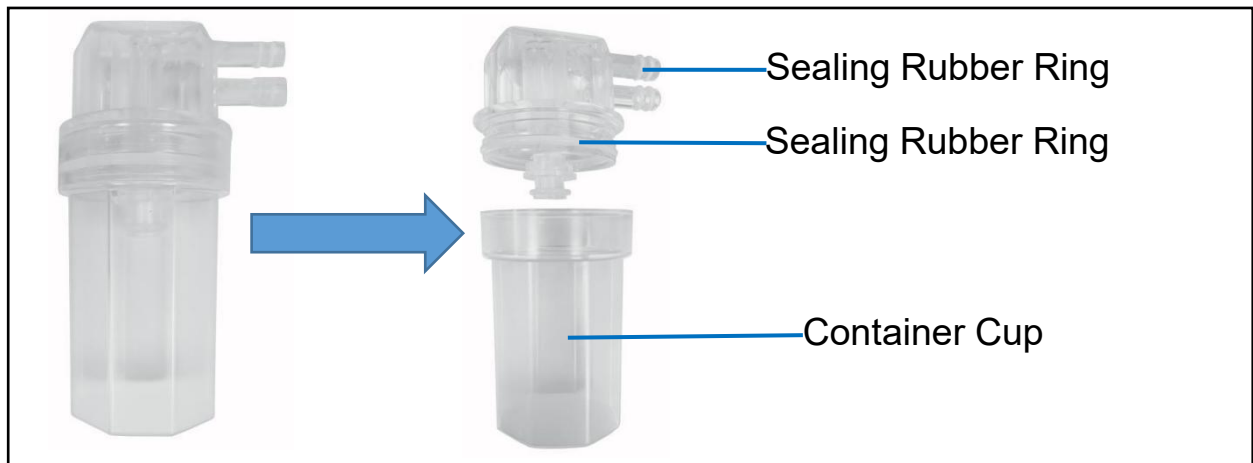

- How to disassemble: Pull down while rotating to twist out the container cup.
- Installation method: Push up while rotating to install the container cup.
(If it is difficult to install and insert, it is recommended to wipe a little oil or water before installing and inserting)
- Notice! Before each use, please check the filter cup for foreign matter or water, please clean it before use.
- Notice! When twisting out the container cup to clean, pay attention to the sealing rubber ring between the cup body and the filter body, and check whether the rubber ring is damaged or displaced. If the rubber ring is damaged, please replace it with a new one. It snaps into the groove and installs flat to avoid air leakage!


4. The installation of the skin flap massage head is shown in the following figure:


- ①As shown in the figure, align the massage head with the handle and push it in a straight line;
- ②As shown in the figure, install the massage head.

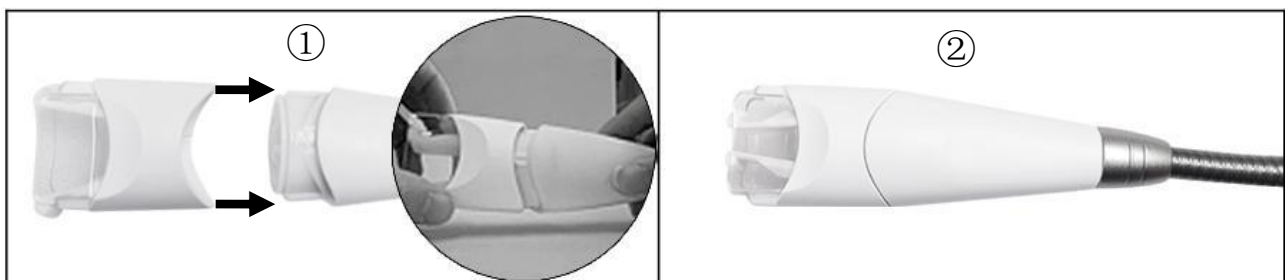

5. Negative pressure Roller Handle Indication

- 2.8-inch Touch Screen Button Introduction:

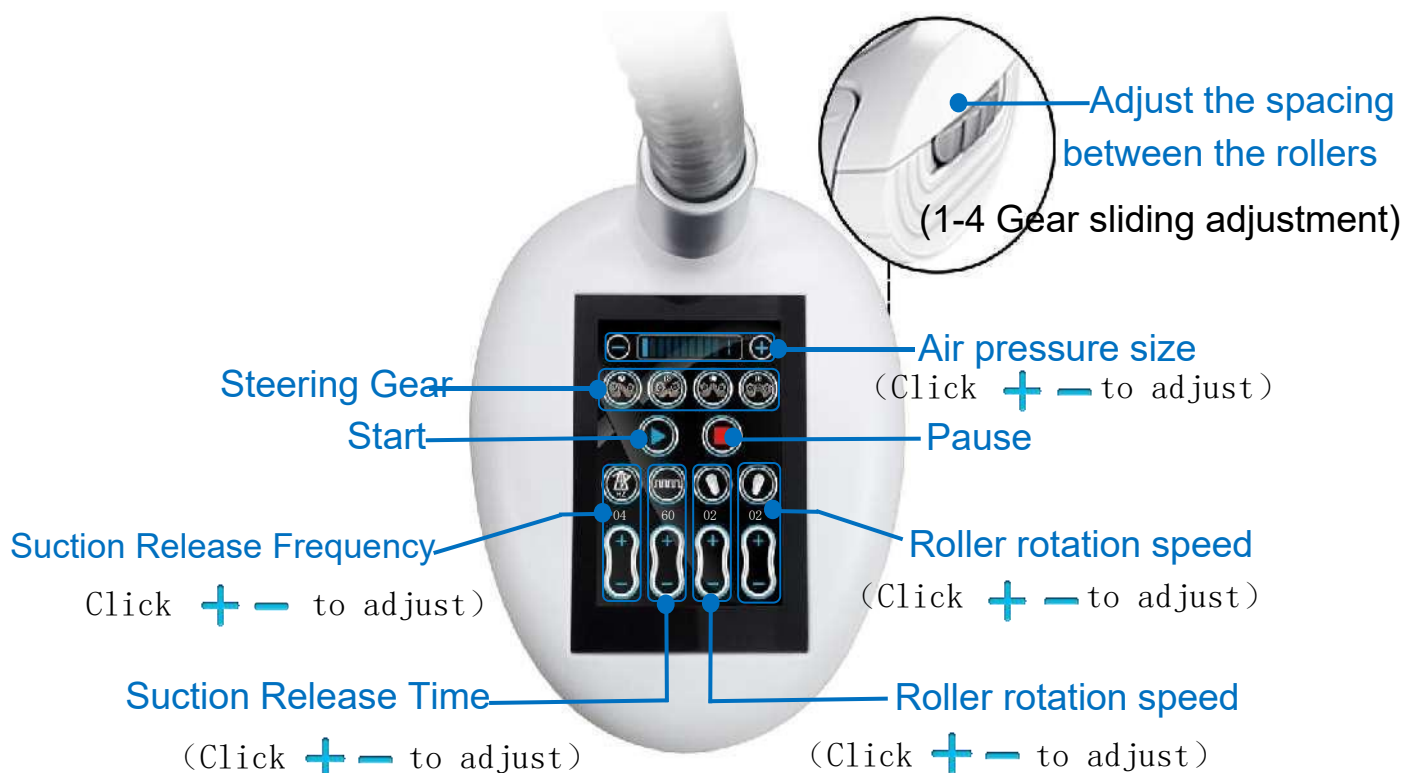

-  : Pressure size adjustment, the range is 1-10, click **【+】** **【-】** to adjust, the proposed parameter is 3-5.
- Select wheel steering, Click the icon to switch.



Scroll up and the two rollers rotate upwards (clockwise).



Roll down, the two rollers turn down (counterclockwise);



Roll inward, and both wheels turn inward at the same time;




Roll outward, and both rollers spin outward at the same time.



6. The roller massage head is installed as shown in the figure below:

- ①As shown in the figure, the thimble of the handle corresponds to the chip of the massage head, and corresponds to the holes at the four corners;
- ②Push and pull the cover buttons on both sides of the massage head upward;
- ③Press down the cover buttons on both sides of the massage head to install the massage head;
- ④As shown in the figure, install the massage head.

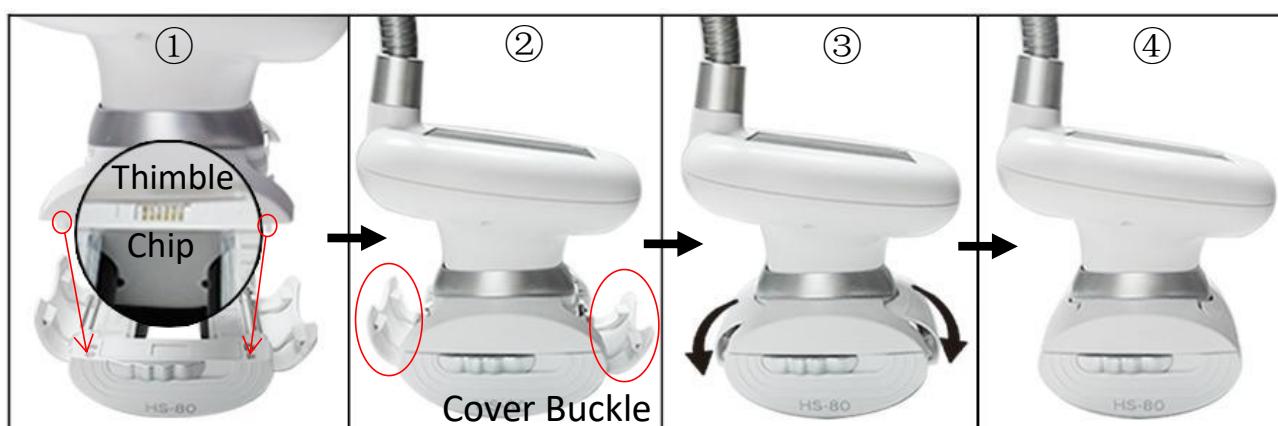

7.Bodysuit for Negative Pressure Roller:

- During the negative pressure roller treatment, it is necessary to wear a "tight jumpsuit", which fits the skin and is easier to operate, so that it can grasp the deep tissue more accurately and optimize the quality of treatment.
- Strictly for personal use and should be washed after each use for hygienic and aesthetic reasons.
- Clothes can only be hand-washed or dry-cleaned. When washing, it should be gentle. Avoid scrubbing vigorously or scrubbing with hard brushes.

- Three sizes:

L size for 40-75kg;


XL size for 75-95kg;

XXL size for 95-125kg.




8. Inner Ball Roller Handle Indication

- Inner Ball Roller Handle display screen & Button Introduction:

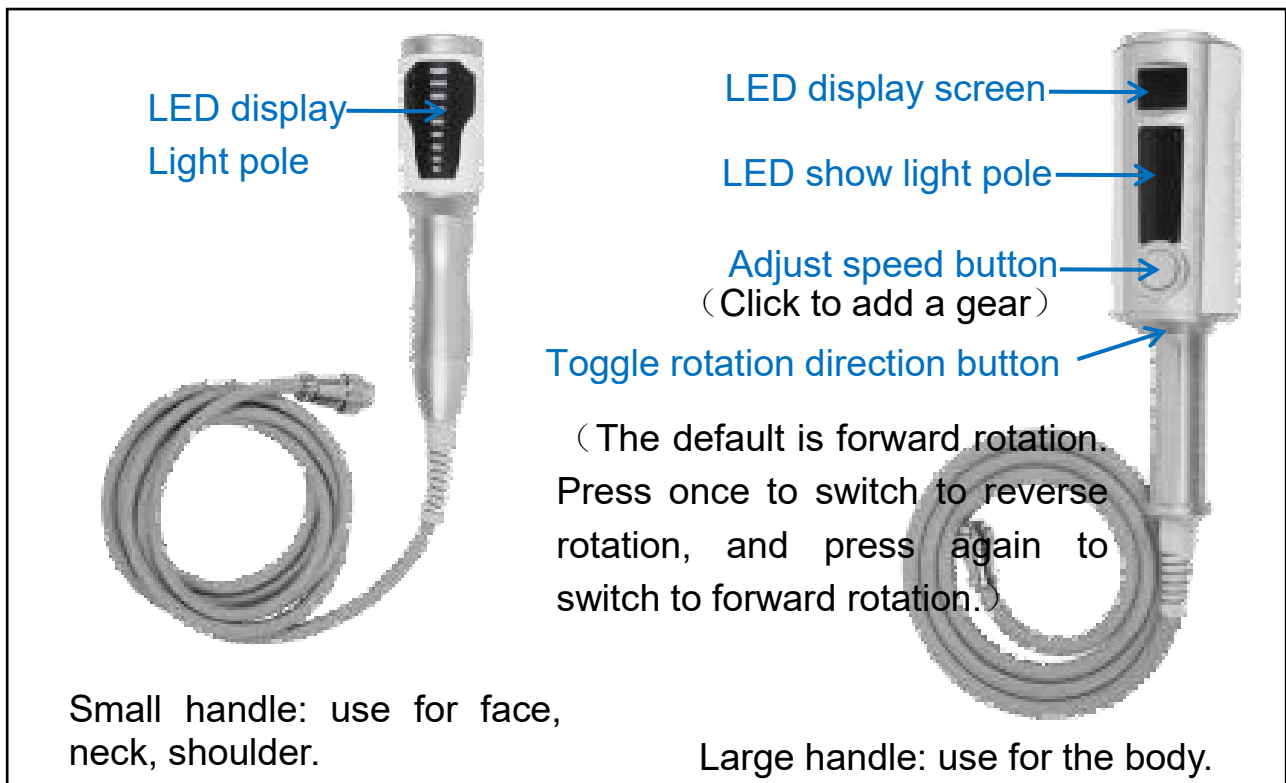

- The replacement and installation of the roller built into the handle is shown in the figure below:

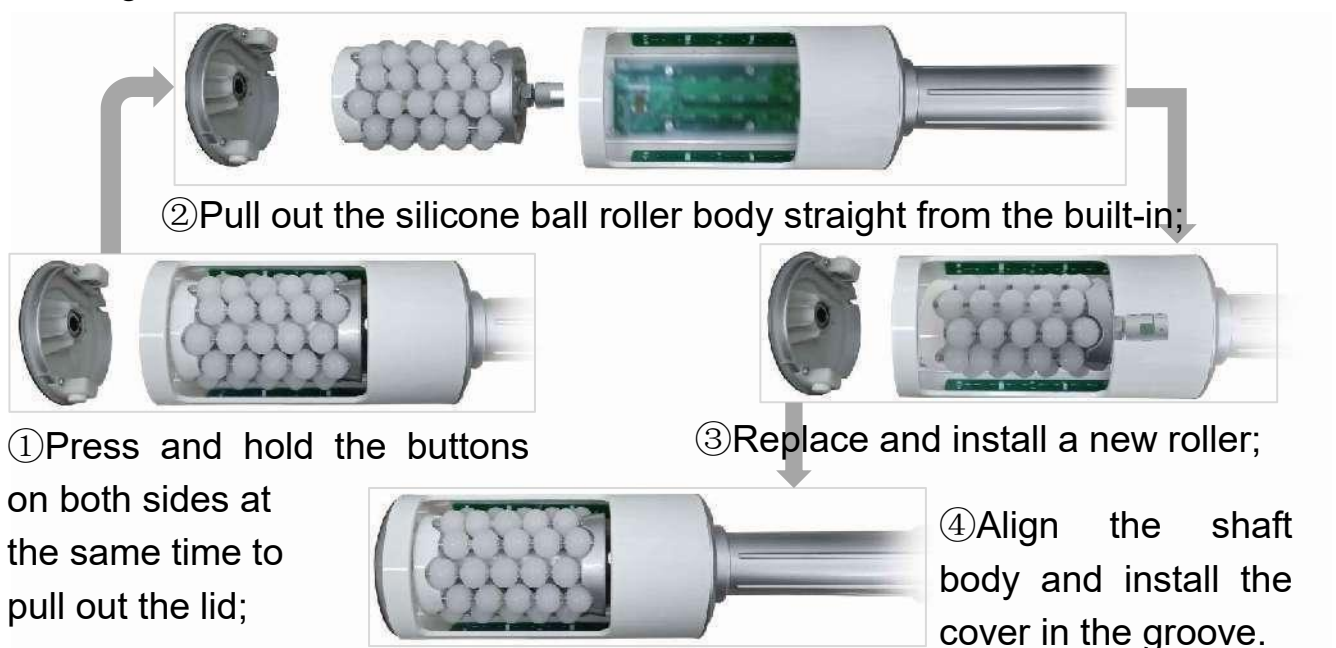

Note: During the working process of the handle, do not put your fingers into the groove of the handle to avoid pinching your hands.

I. Instrument Interface Button Introduction

1. After starting up, the function selection interface (Figure 1) will appear, Select the corresponding function interface according to the icon.



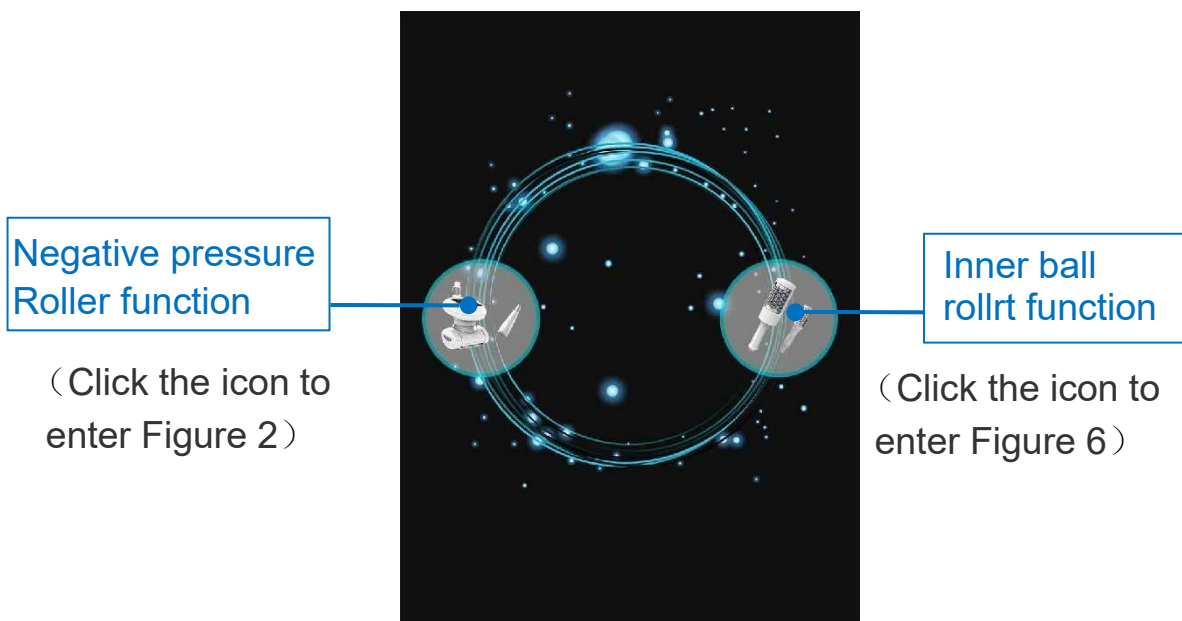
- click  to enter (Figure 2) the Negative pressure Roller interface .
- click  to enter (Figure 6) the Inner ball rollrt Dual-control & Single- control interface .

Figure 1: Function selection Interface



2. Select to set (male or female, age) user information in Figure 2;


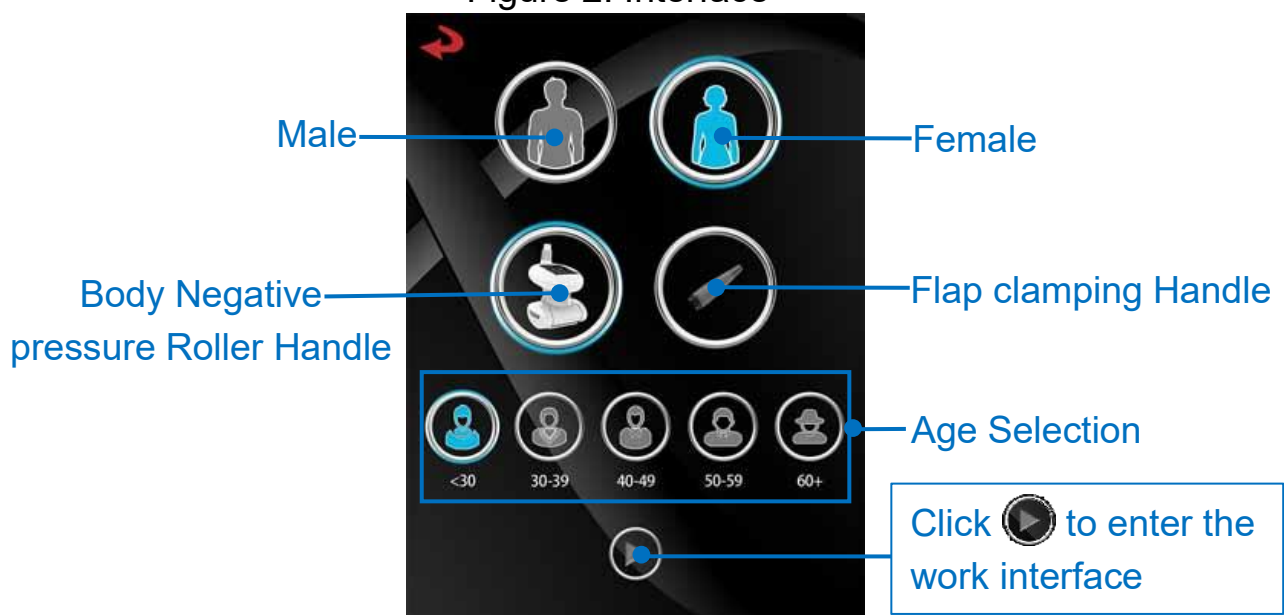
In Figure 2, select the handle icon that needs to be worked, and click  to enter the work interface.

Figure 2: Interface











3、In Figure 2, select the  handle icon, and then click  the machine to enter the working interface (Figure 3-1).



Figure 3-1: Working Interface

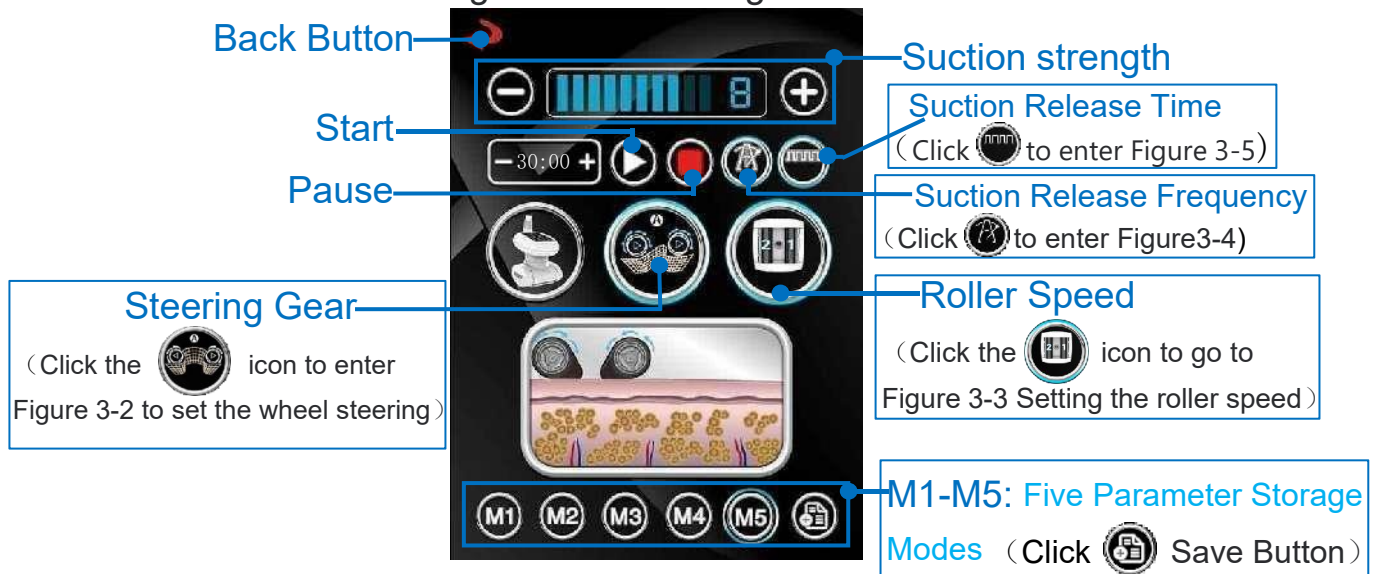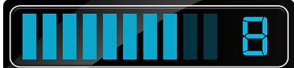


- Figure 3-1,  : Suction strength, the range is 1-10 gears, click **【+】** **【-】** to adjust add or subtract, the proposed parameter is 5-8 gears.

Note: The size of negative pressure (suction strength) must be adjusted based on the experience and skin quality. Be careful not to add too much on thin and sensitive areas, such as the inner arms and thighs.

- Click **【▶】** turn into **【▶】** Indicates the device starts to work.
- Click **【◻】** turn into **【◻】** Indicates pause, the device is in standby state.
- **【🕒】**:Steering Gear, Click “” icon to enter Figure 3-2 to set the steering direction.
- **【📊】**:Roller Speed, Click “” icon to enter Figure 3-3 to set the Roller Speed.
- **【⌘】**:Suction Release Frequency, Click “” icon to enter Figure 3-4 to set the Frequency.
- **【🕒】**:Suction Release Time, Click “” icon to enter Figure 3-5 to set the Suction Release Time.

- **【M1】 — 【M5】** Indicates the selection of 5 parameter storage modes or recall parameters. You can customize the settings to store ideal parameters for later recall. Click  to save the parameter storage.
- Click  Indicates returning to the previous interface.

- In Figure 3-1, click “” this icon to enter Figure 3-2 to set the steering direction.



- In Figure 3-1, click “” this icon to enter Figure 3-3 to set the Roller Speed.

Figure 3-2



Roller Speed

(Click  to enter Figure 3-3)

Displays the currently selected roller steering
(Click  to enter Figure 3-2)

Roller speed adjustment

Roller speed adjustment
(Click  adjust)

Figure 3-3



- Figure 3-2, select roller steering. Built in 4 scrolling modes.



:Up roll, Smoothing fat masses (Reduce fat accumulation) ,
increase blood and lymphatic circulation, and reshape body shape.



:Down roll, Smoothing fat masses (Reduce fat accumulation) ,
increase blood and lymphatic circulation, and reshape body shape.




:outward Roll, Stimulate the production of collagen and
Elastin, regulate the skin tissue and tighten the skin.



: Inward Roll, Targeting the deposition of thick fat,
mobilizing adipose tissue, stimulating adipocytes, and activating
lipid breakdown.

- Figure 3-3, Set the roller rotation speed within the range of 00-10 gears, click **【+】** **【-】** to adjust add or subtract. refer to the recommended 03-05 gears.

- In Figure 3-1, click “” this icon to enter Figure 3-4 to set the Suction Release Frequency.


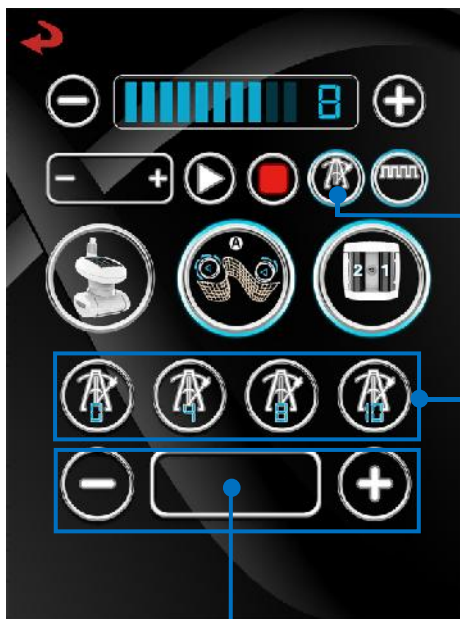
- In Figure 3-1, click “” this icon to enter Figure 3-5 to set the Suction Release Time.

Figure 3-4



Suction Release Frequency Adjustment

(Click to adjust)

Suction Release Time
(Click to enter
Figure 3-5)

Suction Release Frequency
(Click to enter
Figure 3-4)

Shortcuts: Suction
Release Frequency

Shortcuts:
Suction Release Time

Figure 3-5



Suction Release Time Adjustment

(Click to adjust)

- Figure 3-4, shortcut button: Suction Release Frequency. When the frequency is set to 0, it means that the suction will not release for a long time.



: Suction Release Frequency Adjustment, the range is 00-10Hz, click **【+】** **【-】** to adjust add or subtract. The slower the frequency, the deeper the suction.

- Figure 3-5. Shortcuts: Suction and Release Time.



:30% Suction,70% Release;



:40% Suction,60% Release;



:60% Suction,40% Release;



:80% Suction,20% Release;



: Suction Release Frequency Adjustment, the range is 30-80, click **【+】** **【-】** to adjust add or subtract.














4、Select the handle  icon in Figure 2 and click  to enter the working interface (Figure 4-1).

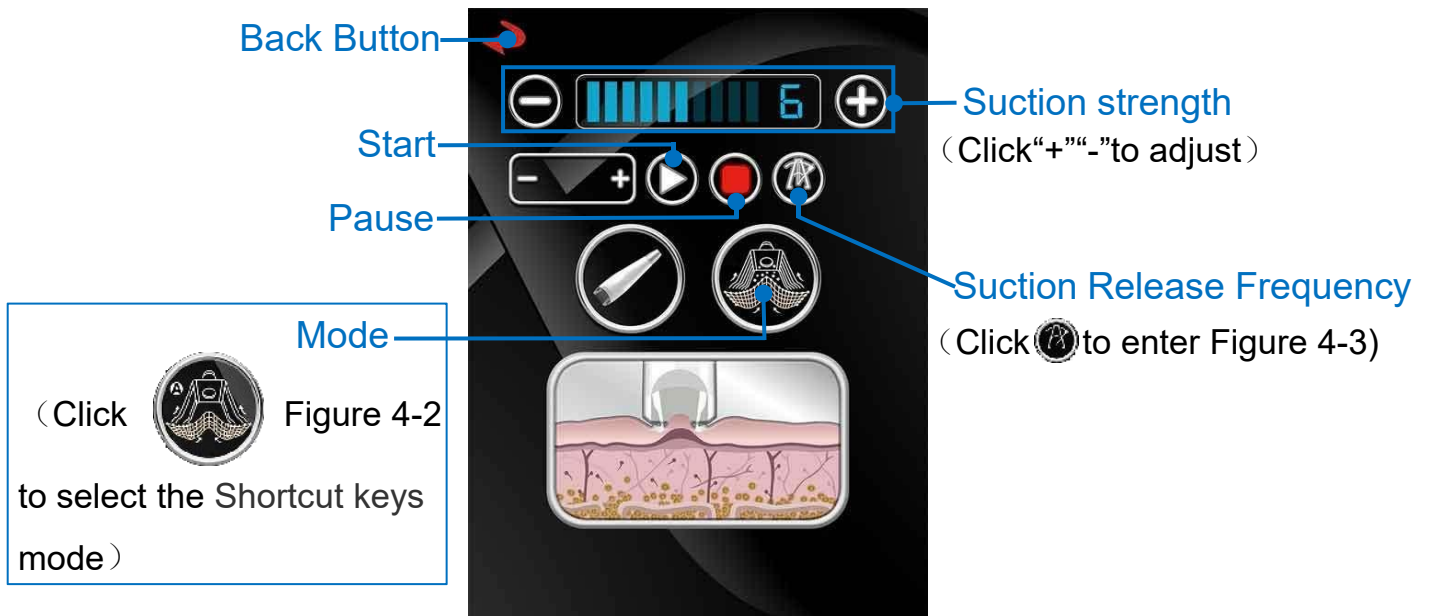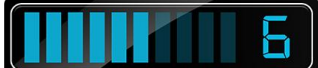Figure 4-1: Working Interface



- Figure 4-1, : Suction strength, the range is 1-10 gears, click **【+】** **【-】** to adjust add or subtract, the proposed parameter is 2-6 gears.

Note: The size of negative pressure (suction strength) must be adjusted based on the experience and skin quality. Be careful not to add too much on thin and mature skin.

- Click **【】** turn into **【】** Indicates the device starts to work.
- Click **【】** turn into **【】** Indicates pause, the device is in standby state.
- **【】**: Mode, Click “” icon to enter Figure 4-2 and select the Shortcut keys mode.
- **【】**: Suction Release Frequency, Click “” icon to enter Figure 3-4 to set the Frequency.
- Click **【】** Indicates returning to the previous interface.


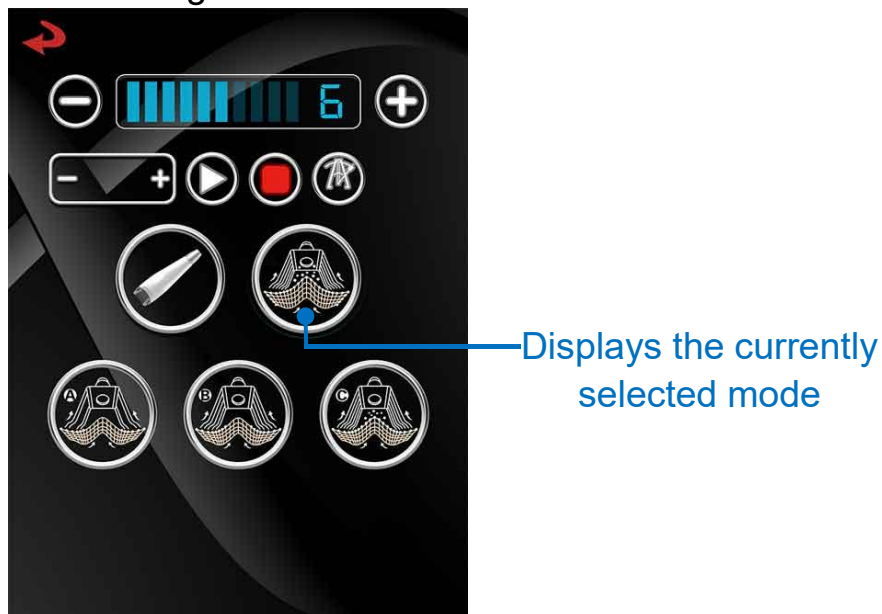
- In Figure 4-1, click “” this icon to enter Figure 4-2 to select the frequency Shortcuts keys mode.


Figure 4-2



- Figure 4-2, frequency Shortcuts keys A/B/C mode.



: A mode/2Hz

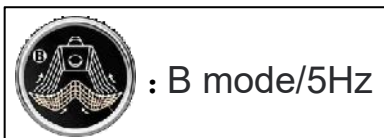

: B mode/5Hz



: C mode/10Hz


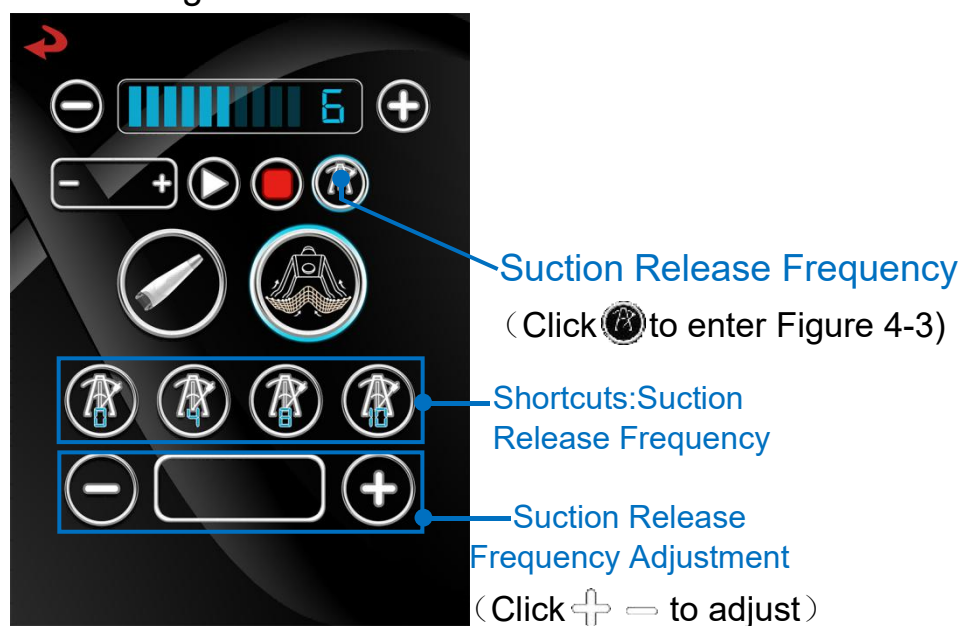
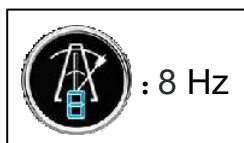
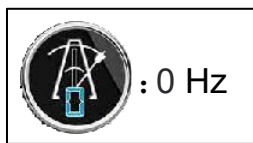

- In Figure 4-1, click “” this icon to enter Figure 4-3 to set the Suction Release Frequency.

Figure 4-3



- Figure 4-3, shortcut button: Suction Release Frequency. When the frequency is set to 0, it means that the suction will not release for a long time.



 : Suction Release Frequency Adjustment, the range is 00-10Hz, click **【+】** **【-】** to adjust add or subtract. The slower the frequency, the deeper the suction.



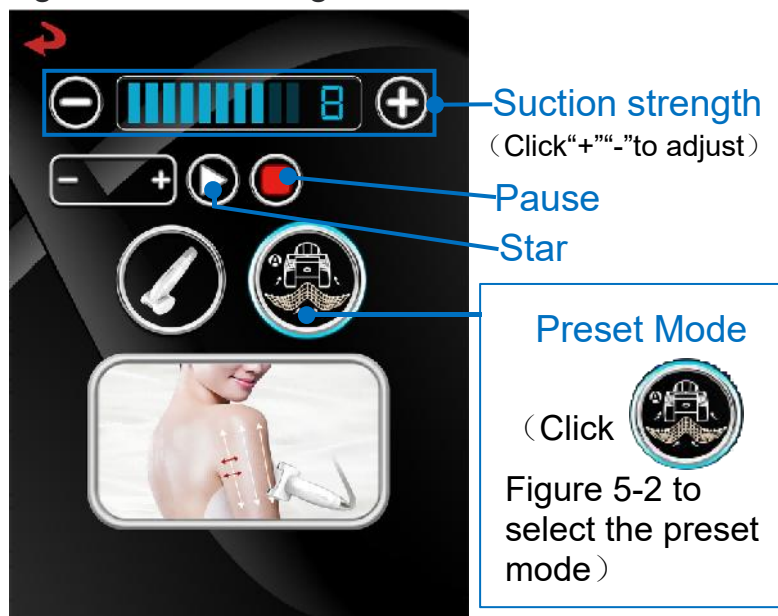

5、 Select the handle  icon in Figure 2-1 and click  to enter the working interface (Figure 5-1)。

Figure 2-1: interface



Figure 5-1: Working Interface



- Figure 5-1,  : Suction strength, range 1-10 gears, click **【+】** **【-】** to adjust add and subtract. the proposed parameter is 2-8 gears.

Note: The size of negative pressure (suction strength) must be adjusted based on the experience and skin quality. Be careful not to add too much on thin and sensitive areas, such as the inner arms and thighs.









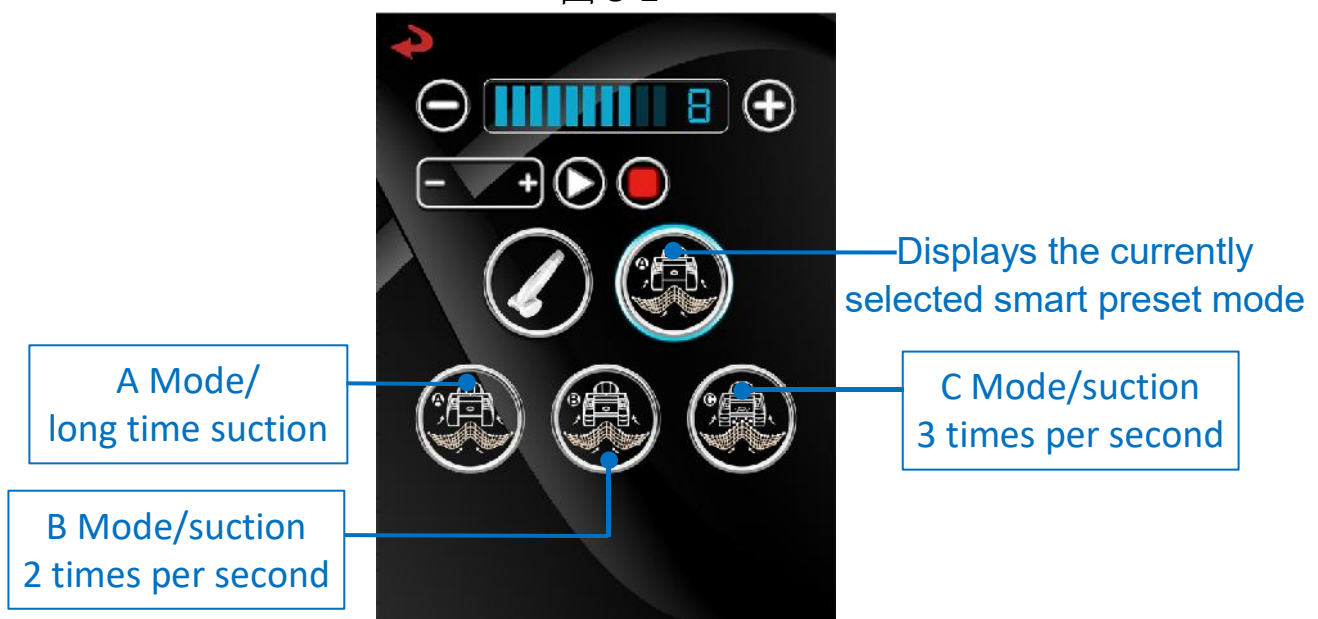
- Click  turn into  Indicates the device starts to work.
- Click  turn into  Indicates pause,the device is in standby state.
-  :Mode, Click “” icon to enter Figure 5-2 and select the select the Preset Mode.
- Click  Indicates returning to the previous interface.
- In Figure 5-1, click “” this icon to enter Figure 5-2 to select the Preset Mode.

图 5-2



- Figure 5-2, preset mode.



:A Mode /long time suction.



:B Mode /suction 2 times per second.



:C Mode /suction 3 times per second.


6.In Figure 1, click  the handle icon to enter the interface of selecting Dual-control & single-control of the ball rollrt (Figure 6).

Figure 1: Function selection Interface


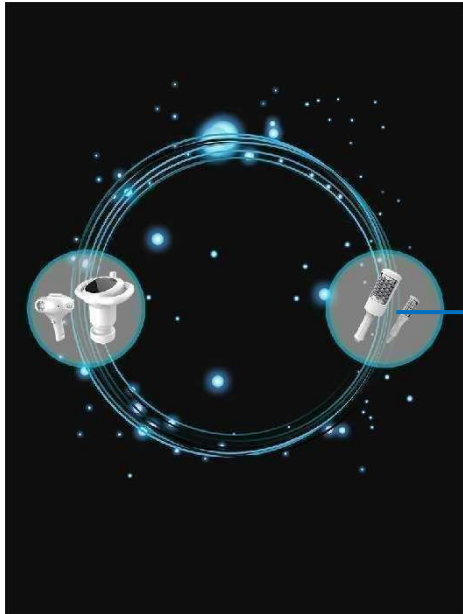




Figure 6: Dual-Control & Single-Control Interface

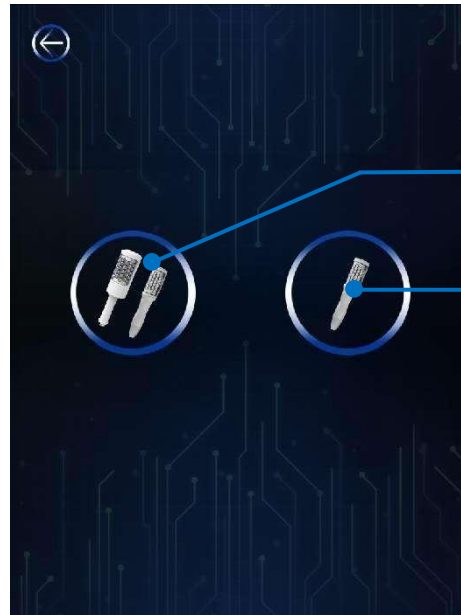

①Click  enter (Figure 7) Dual-channel dual-control work interface, The two handles can work separately and independently, or the two handles can work at the same time, and the two handles can independently adjust different parameters.

②Click  enter (Figure 8) Dual-channel single-control working interface: Only one handle works.


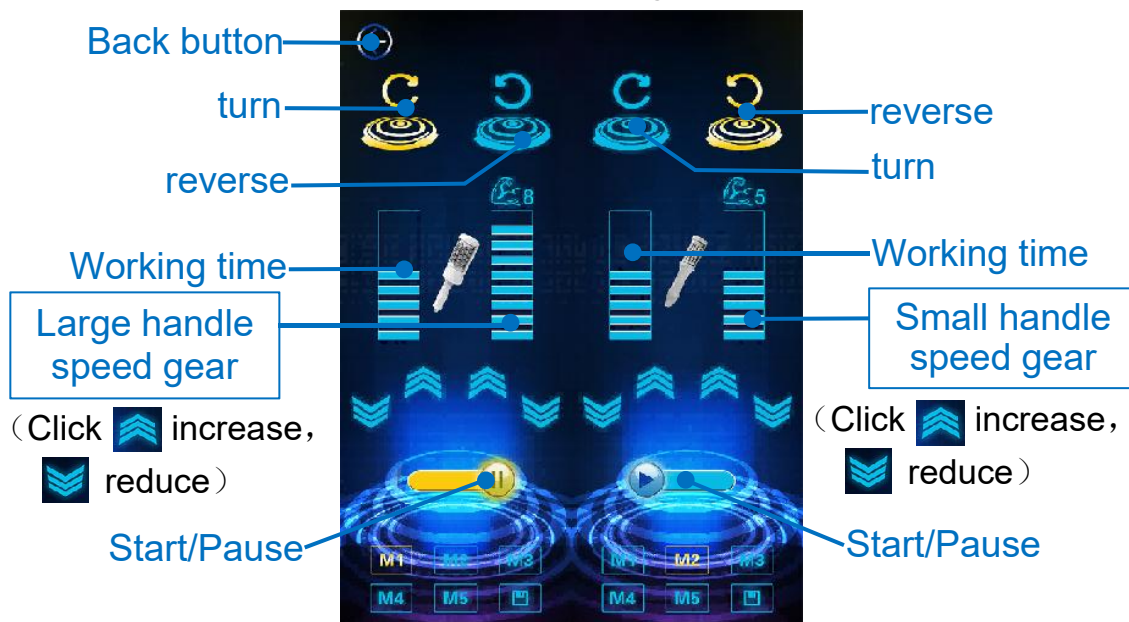
7.In Figure 6, click  the handle icon to enter the interface of selecting Dual-control of the ball rollrt (Figure 7).

Figure 7: Dual-channel dual-control working interface




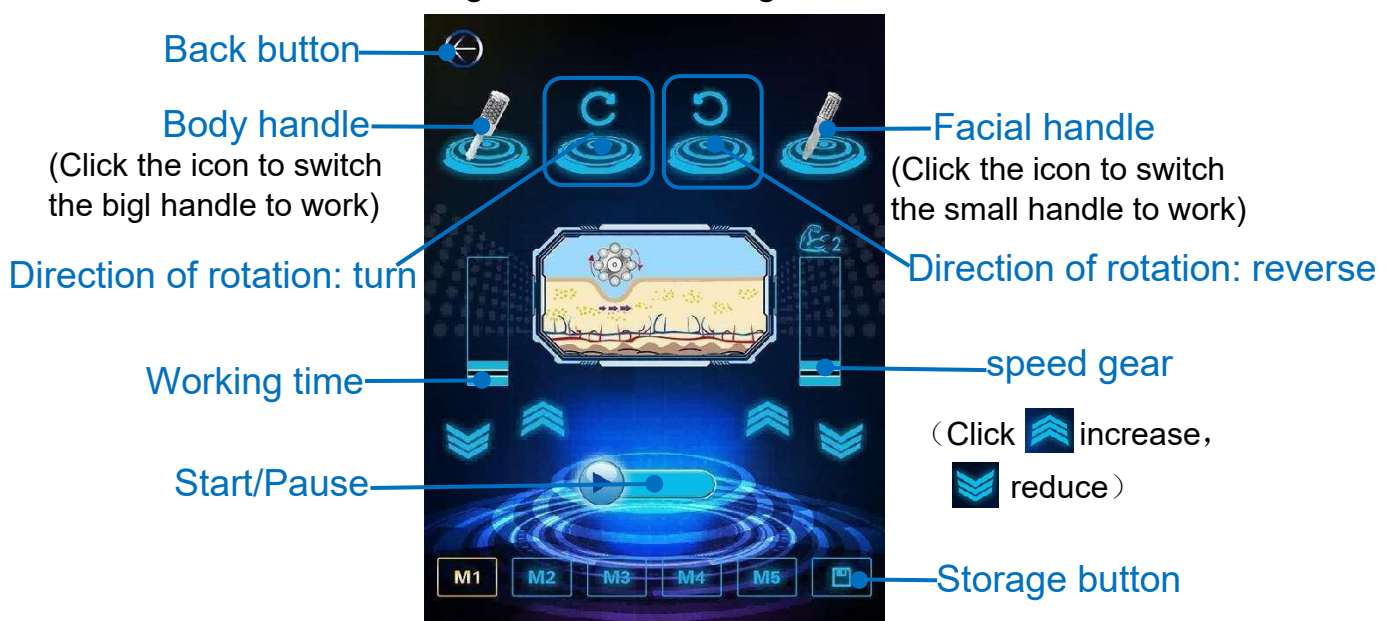
8. In Figure 6, click  the handle icon to enter the interface of selecting single-control of the ball roller (Figure 8).


Figure 8: Dual-channel single-control working interface




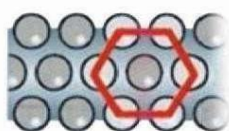
• The keys of the work interface are introduced as follows:

Click on  Switch the big handle to work;


Click  Switch the small handle to work;

Click on  Switch the direction of rotation: Clockwise rotation (Abbreviation: clockwise rotation).


Click on  Switch the direction of rotation: Rotate counterclockwise (Abbreviation: reverse).



Honeycomb sphere arrangement

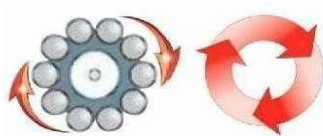

Clockwise rotation
(Abbreviation: clockwise rotation)

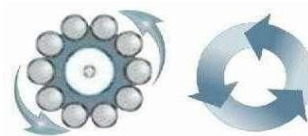

Rotate counterclockwise
(Abbreviation: reverse)



【90:00】 operating time: 90 mins, 1-9 ↑ gear, 10 minutes for each additional gear;



【】 Rotating speed: 0-9 gears, adjust the speed, click on  increase,



【】 reduce;

The rotating speed of small handle: First gear 225rpm, Second gear 345rpm, Third gear 375rpm, Fourth gear 420rpm, Fifth gear 480rpm. Sixth gear 525rpm, seventh gear 600rpm. eighth gear 645rpm, ninth gear 675rpm.

The rotating speed of large handle: First gear 172rpm, Second gear 240rpm, Third gear 307rpm, Fourth gear 375rpm, Fifth gear 450rpm. Sixth gear 510rpm, seventh gear 585rpm. eighth gear 625rpm, ninth gear 675rpm.

Click on  the button becomes  to indicate that the instrument starts to work.

Click on  the button becomes  to indicate that the instrument pause to work.

【M1】 — 【M5】 indicate 5 parameter storage modes or options for recalling parameters, click on  , save parameter storage. Click on  show to return.

J. Effect Comparison Diagram



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before Treatment After Treatment Before Treatment After Treatment Before Treatment After Treatment



Before

After

Before

After



Before

After

Before

After

Before

After



Before

After

Before

After 6
times

After 12
times



Before

After

Before

After



Before

After

Before

After



Before

After



Before

After




Before

After



Before

After

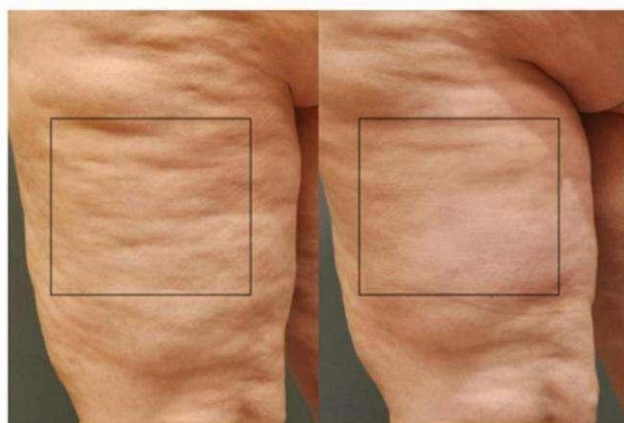

Before

After



Before

After



Before

After



Before

After

K. Technical Parameter

Product Name	Five-in-one (Negative Pressure Roller Three handles+ Inner Ball Roller Machine Two handles)
Touch screen	15 inch
Roller Speed	60-300RPM
Vacuum Negative Pressure	10-70KPA
Inner ball rollrt handle speed	675rpm
operating voltage	AC110V~220V±10%， 50Hz-60Hz
power	10-800W
Airbox size	63×64×118cm
Gross weight	About 96.3kg