


# User Manual

VT-7201K

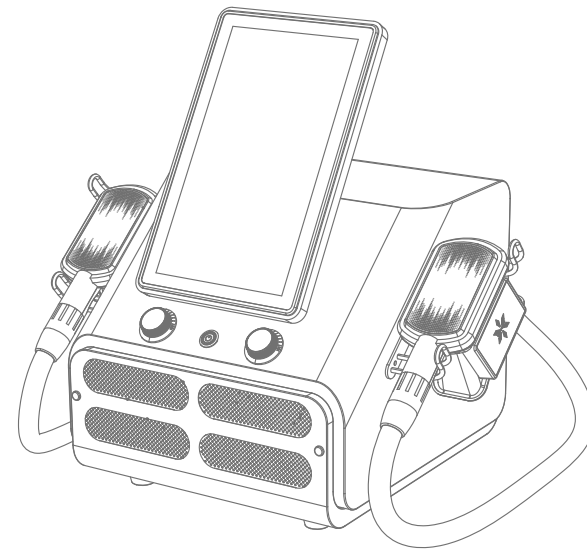

## Preface

Dear User:

Welcome to the BeautyFreeze Sculptor, our latest beauty device. This professional body sculpting device should only be operated under the guidance of a trained professional. Improper operation may result in adverse effects for the individual undergoing treatment. Please read this manual carefully before operation, and follow the instructions in the manual during operation.

We believe that this high-quality product will deliver excellent results, and our exceptional after-sales service will alleviate any concerns you may have.

Thank you!

Contents

1.Introduction..... 1

2.Precautions..... 1

3.Contraindications..... 2

4.Technical parameters..... 2

5.Packing list..... 3

6.Introduction for use..... 4

    How to install..... 4

    Interface Introduction..... 8

7.Troubleshooting..... 10

8.FAQ..... 11

9.Treatments & Application..... 12

    Abdomen Sculpting..... 12

    Legs Sculpting..... 14

    Back Sculpting..... 16

    Hips Sculpting..... 18

10.Treatment schedule..... 20

11.During treatment..... 20

12.After treatment..... 21

Introduction

BeautyFreeze Sculptor, the highly coveted beauty device, represents an innovative alternative to many salon beauty devices. This device offers user-friendly operation and is designed to deliver highly effective body-shaping solutions. Moreover, it focuses on specific body areas and offers a safe, non-invasive, medication-free, and surgery-free treatment with no adverse effects, resulting in a more refined and attractive body contour.

Precautions

- 1.This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near flammables and explosives to prevent damage.
- 2.Always use the designated accessories provided or recommended by the original manufacturer to ensure the performance of the instrument and sustain its lifetime.
- 3.Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
- 4.Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
- 5.Patients should refrain from using this instrument unless they have obtained permission from their doctor.
- 6.The application of an anti-freezing membrane on the operating area is necessary, do not contact the skin with the applicator directly.
- 7.Power off the instrument when not in use to ensure its safety.

- 8.If you are currently taking anti-adiposity drugs, please discontinue their use for 1 - 2 months before using this instrument. Failure to do so may result in an extended course of treatment.
- 9.To maintain hygiene and extend the instrument's service life, sterilize it with saline.

## Contraindications

- 1.Pregnant or lactating woman.
- 2.Individuals with hypertension, heart disease, or a pacemaker installed.
- 3.Individuals with unhealed wounds or those in the recovery period from surgery.
- 4.Individuals with epilepsy, severe diabetes, and hyperthyroidism.
- 5.Individuals suffering from malignant tumors, hemophilia, and hemorrhage.
- 6.Individuals with skin disease or infectious disease.
- 7.Individuals who have had a cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months should refrain from using this instrument.
- 8.For your safety, refrain from using this device if you've had plastic surgery or have implants/metal objects near the treatment area within 3 months.

## Technical parameters

Product Name: BeautyFreeze Sculptor

Product Model: VT-7201K

Shell material: Metal plate

Applicator material: ABS

Screen size: 15.6 inches

Product dimensions: 65.4cm\*59.0cm\*59.7 cm

Packing dimensions: 58cm\*62cm\*65.5 cm

Product weight: 25kg


Packing weight: 43kg

Input voltage: 110V/220V 50~60Hz


Rated power: 220W

## Packing list


Product details include:

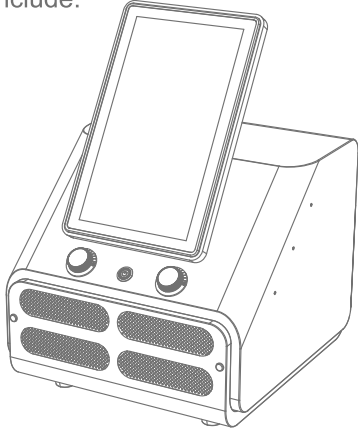


Main unit\*1

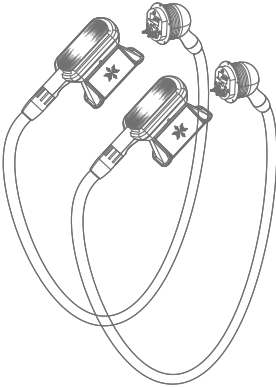


Applicator\*2

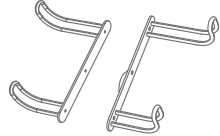


Holder for applicator\*2

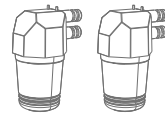


Filter\*2

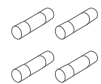

5A  
Fuse\*4

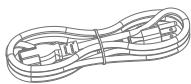

Power cord \* 1

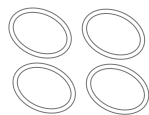

Large silicone  
ring for filter\*4



Small silicone  
ring for filter\*8




Silicone ring for  
aviation connector \* 8

Note: make sure the packaging is complete before use, then remove all packaging materials and confirm that all accessories of the appliance are complete.


## Introduction for Use

### How to install


Step 1: Secure the two holders onto the main unit using screws.

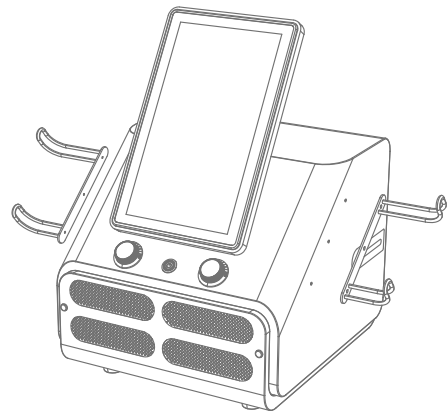


Step 2: Place the applicators in the holder and connect them to the main unit.

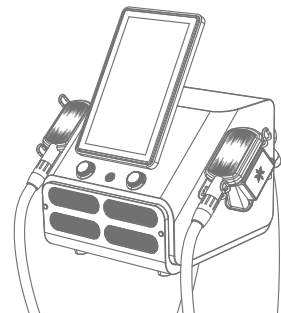


Step 3: Fasten the metal latch located at the back of the aviation plug.

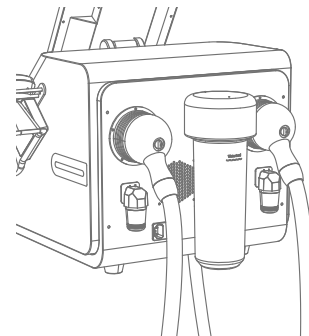


Step 4: Take off the bottle on the back of the main unit and fill in water, the water level inside the bottle should remain above the red alert mark

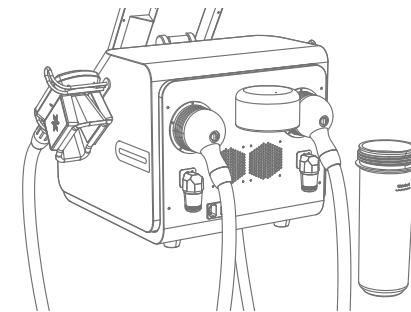








Step 5: Put the bottle back and secure it.

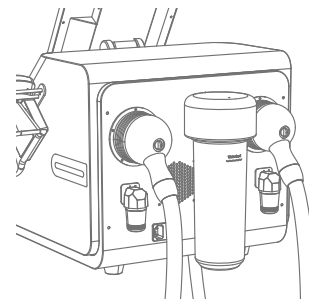

Step 6: Connect the power cord, and finish.

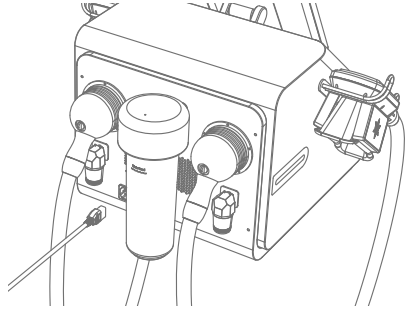

How to replace the fuse

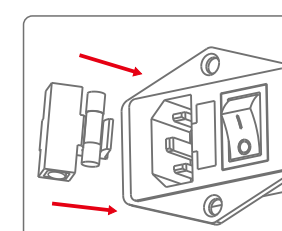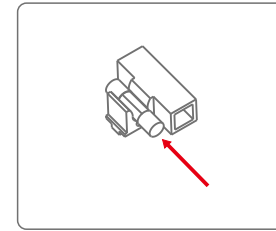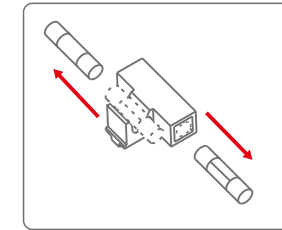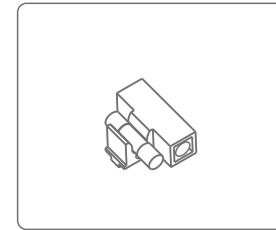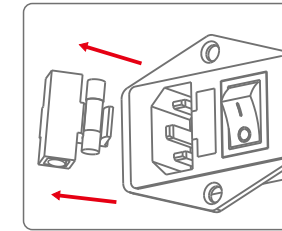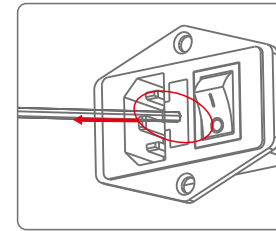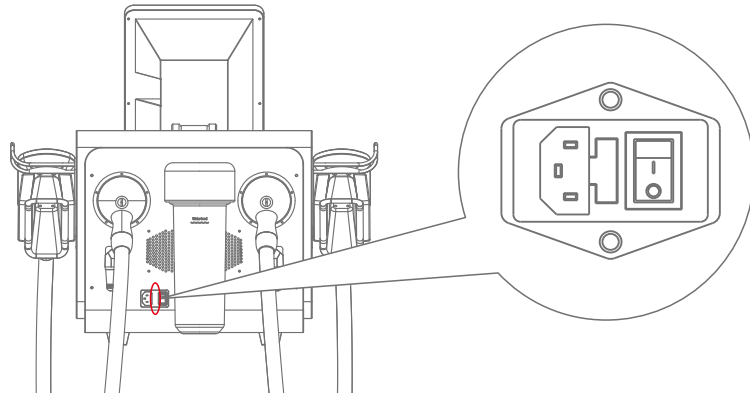


## Interface function


### Welcome interface



### Interface Introduction

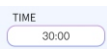

Time display

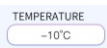

Temperature display (recommended temperature range: Between 0℃ with 4℃, it is recommended to start at 5℃ for the first time, and decrease the temperature after the user becomes accustomed to it.)



Time and temperature set buttons



On/off button of the applicator



Reset button



Main on/off button (To start the treatment, select the desired applicator first, then tap this button.)




Applicator's temperature alert, a yellow exclamation mark appears when the applicator's temperature is abnormal.



Tank alert, a yellow exclamation mark appears when the tank is out of water or abnormal.

Power button on the front side of the main unit.

Vacuum knob, set the intensity according to the individual's tolerance. Start the treatment at the lowest level of vacuum intensity and gradually increase it as the user becomes accustomed to the treatment.

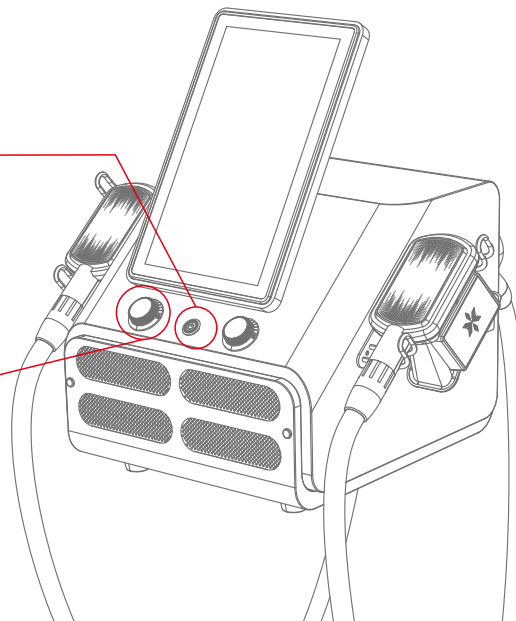

## Troubleshooting

### 1.Fail to turn on the instrument.

- A.Make sure the power cable is connected to a valid wall socket.
- B.Check if the fuse is loose or burned.
- C.If the above methods don't work, please contact the dealer for assistance.

### 2.The instrument doesn't work after pressing the Start button.

- A.Check if the bottle is short of water, the instrument may fail to detect the water flow due to insufficient water level.
- B.Check for any water leakage from both the applicator and the bottle.
- C.If the above methods don't work, please contact the dealer for assistance.

### 3.The water drop icon in the top left corner of the screen changes to a yellow exclamation mark (the first water drop symbolizes the left applicator, and the second one signifies the right applicator).

- A.Check for any water leakage from the instrument.
- B.If the above method doesn't work, please contact the dealer for assistance.

### 4.The thermometer icon in the top left corner of the screen changes to a yellow exclamation mark (the first thermometer symbolizes the left applicator, and the second one signifies the right applicator).

- A.The applicator's temperature is abnormal, please contact the dealer for assistance.

### 5.The instrument succeeded in starting up, but an error message appeared on the screen.

- A.Please unplug the power cord, wait for 1 minute before plugging in the power cord again, and then restart the instrument.
- B.If the above method doesn't work, please contact the dealer for assistance.

## FAQ

### 1.How does the device work?

A: When the applicator is applied to the skin, it cools the adipose tissue to 0°C to 4°C. The adipose tissue is then decomposed and metabolized by the body's circulation, resulting in body-sculpting effects.

### 2.Do I need to diet and exercise after treatment?

A: Yes, diet and exercise are recommended to keep the results. This treatment only targets the adipose tissue decomposition, while the metabolism enhancement is not included. There are chances that the resolved issues could reappear without a healthy diet and regular exercise.

### 3.How long does it take to notice the result of the treatment?

A: Approximately 1 – 2 months, although it can vary based on individual factors and metabolism. Due to the low tolerance of adipose tissue to cold temperatures, maintaining a constant low temperature between 0 °C to 4°C can expedite the breakdown of adipose tissue. Subsequently, the body's circulation facilitates the metabolism of these decomposed tissue. This process unfolds over a period, and visible results may become apparent during this time.

#### 4. Why the application of the anti-freezing membrane is necessary?

A: The anti-freezing membrane serves a dual purpose: preventing direct contact between the applicator and the skin to avoid frostbite, and aiding in maintaining the temperature at the treatment area. This contributes to an augmented body-sculpting effect.

## Treatment & Application

### Abdomen Sculpting

#### 1. Efficacy


(1) Enhance skin tightness and diminish accumulations of adipose tissue on the abdomen.

#### 2. Indications


- (1) People with excess adipose tissue on the abdomen, or those with sagging skin post-birth.
- (2) People with love handles due to a sedentary lifestyle.

#### 3. Steps (30 minutes)


Preparation: Towel, anti-freezing membrane

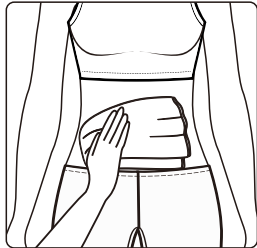

1. Clean the abdomen

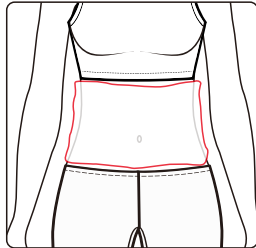

2. Apply the anti-freezing membrane.

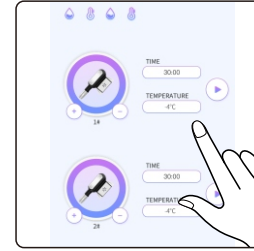


#### 3. Set parameters


Recommended duration: 30 minutes

Recommended temperature range: 0 °C ~ 4 °C

Adjust the suction intensity according to individual endurance.

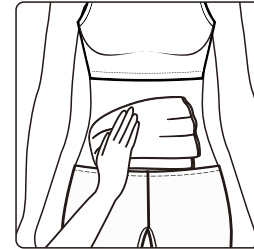

5. Remove the applicator and the membrane, and then cleanse the treatment area with a towel.

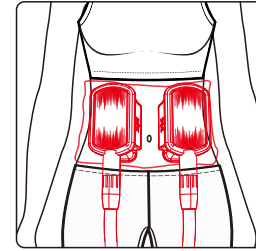

#### 4. Applicator operation

Apply the applicator to the abdomen with the anti-freezing membrane in place, and wait for 30 minutes.



## Legs Sculpting

### 1.Efficacy


Improve the appearance of legs.

### 2.Indications


- (1)People with poor circulation, edema, and excess adipose tissue in the lower body.
- (2)People with concerns about their leg appearance.

### 3.Steps (30 minutes)


Preparation: Towel, anti-freezing membrane

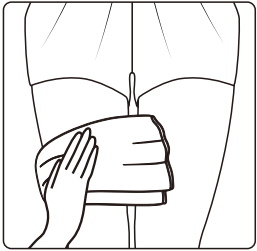

1.Clean the legs.

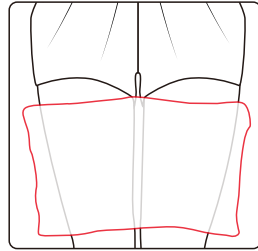

2.Apply the anti-freezing membrane.

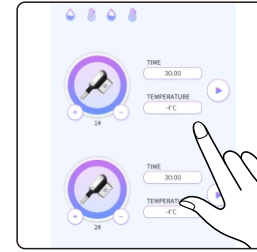


### 3.Set parameters


Recommended duration: 30 minutes

Recommended temperature range: 0 °C ~ 4 °C

Adjust the suction intensity according to individual endurance.

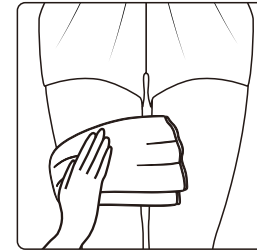

5.Remove the applicator and the membrane, and then cleanse the treatment area with a towel.

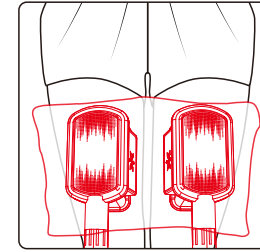

### 4.Applicator operation

Apply the applicator to the legs with the anti-freezing membrane in place, and wait for 30 minutes.

## Back Sculpting

### 1.Effect


Improve the appearance of the back.

### 2.Indications


Individuals with excess adipose tissue on their back, giving an unattractive appearance in clothing.

### 3.Steps (30 minutes)


Preparation: Towel, anti-freezing membrane

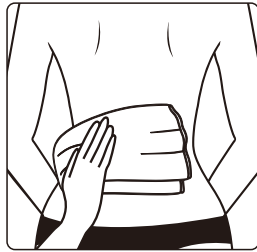

1.Cleanse the back.

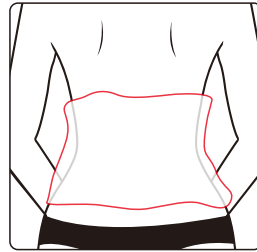

2.Apply the anti-freezing membrane.

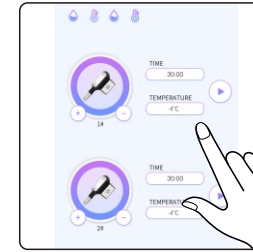


### 3.Set parameters


Recommended duration: 30 minutes

Recommended temperature range: 0 °C ~ 4 °C

Adjust the suction intensity according to individual endurance.

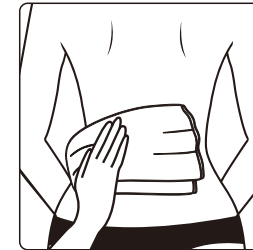

5.Remove the applicator and the membrane, and then cleanse the treatment area with a towel.

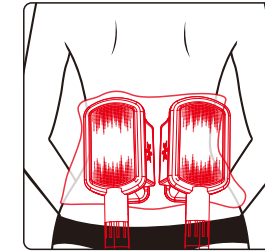

### 4.Applicator operation

Apply the applicator to the lumbar with the anti-freezing membrane in place, and wait for 30 minutes.

## Hips Sculpting

Preparation: Towel, anti-freezing membrane

### 1.Effect

Enhance the contour of the hips and alleviate sagging.


### 2.Indications

- (1)People with sagging hips and excess adipose tissue.
- (2)Individuals with an unattractive buttocks shape or those with flat buttocks.


### 3.Steps (30 minutes)



1.Clean the hips.

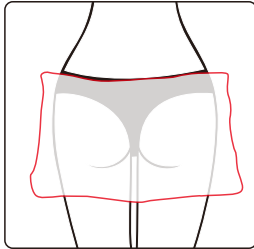

2.Apply the anti-freezing membrane.

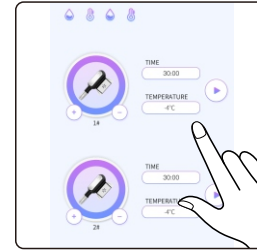


### 3.Set parameters


Recommended duration: 30 minutes

Recommended temperature range: 0 °C ~ 4 °C

Adjust the suction intensity according to individual endurance.

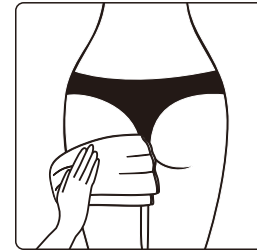

5.Remove the applicator and the membrane, and then cleanse the treatment area with a towel.

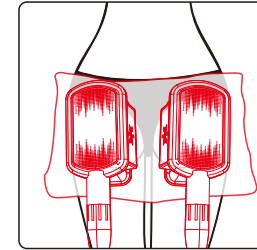

### 4.Applicator operation

Apply the applicator to the hips with the anti-freezing membrane in place, and wait for 30 minutes.

## Treatment schedule

Typically, visible results require multiple sessions. We recommend 3 – 4 sessions with a 6–8-week interval between each session, although this may vary based on individual circumstances. A single session lasts 30 minutes, and the user may experience mild redness, bruise, or regional numbness. These are normal and should subside within 1-2 weeks. After 3-4 sessions, the thickness of adipose tissue should be reduced by 20%-25%. However, the exact amount of adiposity decrease is influenced by various factors, and it does not necessarily lead to achieving your desired weight. A single session is insufficient to achieve the desired results. Please undergo multiple sessions for optimal outcomes.

## During treatment

Caution:

- 1.During the treatment, adipose tissue accumulations are gradually drawn into the applicator. The stronger the suction, the more tightly they are pulled in.
- 2.When the applicator is applied to the treatment area, there may be air trapped between your skin and the membrane, resulting in the formation of bubbles due to the suction from the applicator.
- 3.It's normal to experience mild swelling, redness, or bruising after treatment, so there's no cause for concern.

## After treatment

- 1.Stay warm, avoid cold food, or catch a cold after treatment.
- 2.Take a shower 4 - 6 hours later after treatment.
- 3.Drink plenty of warm water to stay hydrated and promote metabolism.
- 4.Do not overeat or stay up late.
- 5.Avoid intake of cold, raw, spicy food, and have enough sleep.
- 6.Avoid crop tops, backless dresses, miniskirts, and hotpants.
- 7.Do not scratch the treatment area.
- 8.Do not press the treatment area.

## Warranty & Support

A one-year warranty is provided from the date of purchase.

Product Name:

BeautyFreeze Sculptor

Product Model:

VT-7201K

Quality Inspection:

PASS