

User Manual

WL-7008C



Content

1.Safety Precautions	1
2.Contraindications	2
3.Overview	2
4.Control Panel Introduction	3
5.Technical Specification	5
6.Use Precautions	5
7.Maintenance & Cleaning	6
8.Ask & Answer	6
9.FAQ	8
10.Operation Guide	9
11.During treatment	10
13.After treatment	10
14.Warranty & Support	11

1.Safety Precautions

- 1.Please read this manual carefully before use and familiarize yourself with the operation of this device and related technology.
- 2.As a high-precision electronic device, this device should be stored properly. Avoid humid conditions, direct sunlight, and proximity to flammable or explosive materials to prevent damage.
- 3.Always use the designated accessory provided or recommended by the manufacturer to ensure performance throughout its service life.
- 4.Do not place the device in high-temperature conditions, as this may negatively impact its service life and performance.
- 5.Do not apply the device on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pace-makers, etc.
- 6.Patients should consult their doctor before using this device.
- 7.The application of an anti-freezing membrane in the operating area is necessary. Do not allow the handle to come into direct contact with the skin, and avoid operating on dry skin.
- 8.Power off the device when not in use to ensure its safety.
- 9.If you are currently taking anti-adiposity drugs, please discontinue their use for 1 - 2 months before using this device. Failure to do so may result in an extended course of treatment.
- 10.To maintain hygiene and extend the device's service life, sterilize it with saline.

2. Contraindications


Before using this device, it is essential to consult your doctor or seek professional advice. Details below:

1. Pregnancy or lactation.
2. Hypertension, heart disease, or with a pacemaker installed.
3. Unhealed wounds or ongoing recovery from recent surgery.
4. Epilepsy, severe diabetes, and hyperthyroidism.
5. Malignant tumors, hemophilia, and hemorrhage.
6. Skin disease or infectious disease.
7. Cesarean section within the last six months, a miscarriage within the last three months, or a natural birth within the last two months.
8. Recent plastic surgery (within three months), or presence of implants or metal objects in the treatment area.

3. Overview

BeautyFreeze Sculptor brings you the latest and safest fat-freezing technology, ensuring a radiant transformation. It provides the perfect solution for people seeking improvements in body figures without injections, drugs, or side effects. This non-invasive device delivers visible enhancements with each use.

4. Interface function

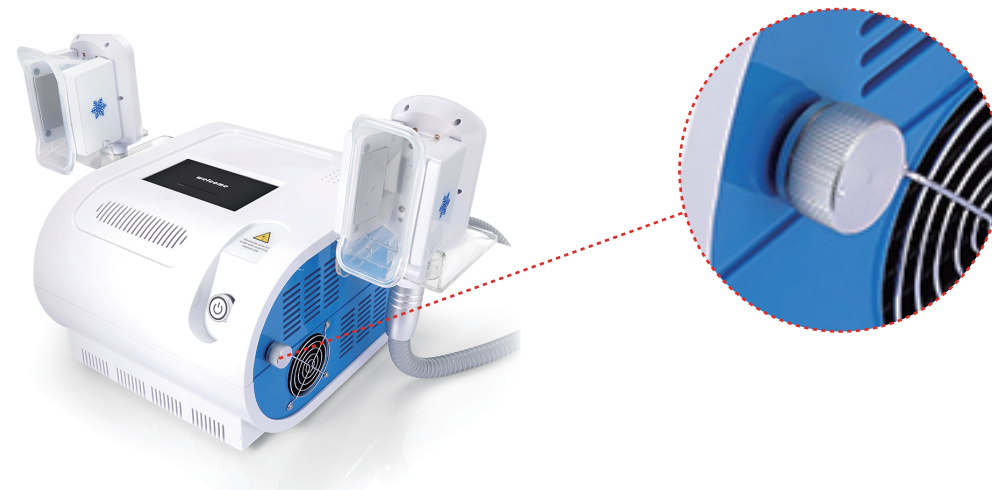


Water Filling

1. Assemble all the components on the main unit and put the handles into the corresponding rack.
2. Before adding distilled or pure water, you need to attach the funnel to the inlet and remove the overflow lid.
3. Fill the machine with water through the funnel until the water flows out from the overflow, then remove the funnel and attach the overflow lid.



The white plug should be removed after water filling. Ensure it remains removed during device operation.

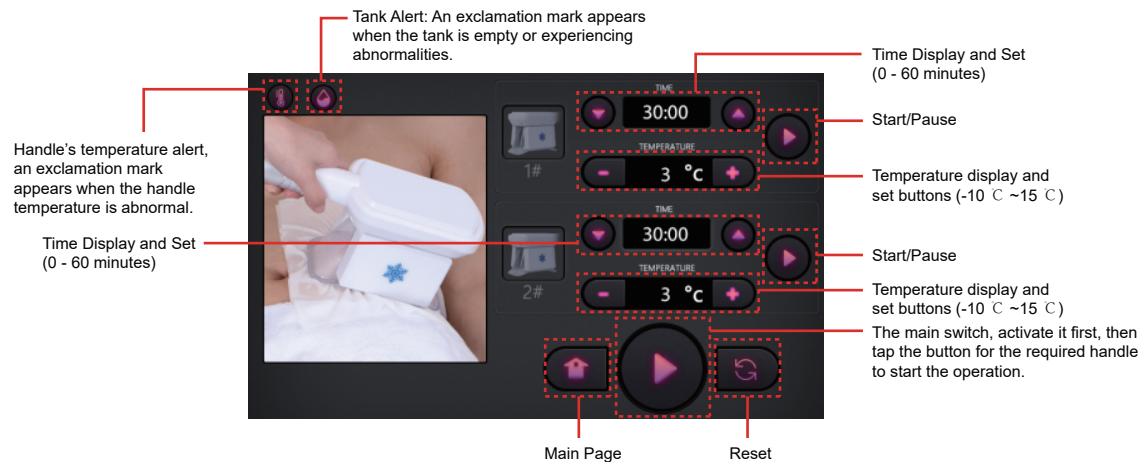

5. Technical Specification

Power Supply Input: AC110V-220V 50HZ-60HZ	
Power: 220W	Input Power: 220W
Dimensions: 14.5×17.3×9.1inches	Net Weight: 11.6KG
Handle Temperature: COLD: 23-59°F	Gross Weight: 13.2KG
Suction intensity: -73KPa	Vacuum flow rate L/min: 10L/min

6. Use Precautions

1. Strictly follow the instructions in the manual to use the device and train operator.
2. Before using the device, use a plug with a grounding pin and ensure the power socket is grounded.
3. Make sure the voltage of the device is adapted. If the local power supply voltage is unstable, we recommend adding a power-matched regulated power supply between the mains and the device.
4. Remove all the metal objects inside the body to avoid causing accidents or affecting treatment effects.
5. Stop the intake of anti-adiposity drugs 1 - 2 months before the treatment, or extend the treatment course if you want to take the treatment immediately.
6. Refrain from using the device on an empty stomach, and initiate the treatment one hour after a meal.

7. Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the device in a dry, cool, and well-ventilated place.
2. Clean and sanitize the handle with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.
3. After cleaning the device, cover it with a dustcloth or plastic wrap. Before use, ensure that the handle is sanitized again for hygienic reasons.
4. Disassembling the device and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

8. Ask & Answer

Ask		Cause		Answer
1. Fail to turn on the instrument.	→	The fuse may have burned out due to unstable voltage or prolonged use.	→	Please replace the fuse and restart the instrument 5 minutes later.
	→	The fuse is loose or burned.	→	Replace the fuse on the back of the instrument and restart it.

Ask		Cause		Answer
2. The device is not working and can't be restarted.	→	The power cord is disconnected from the outlet.	→	Check if the power cord is securely connected to the outlet and ensure that the outlet is functional.
3. The screen shows an error message.	→	Data error.	→	1. Turn off the device, disconnect the power cord, and wait for 1 minute to restart the instrument. 2. If the above method doesn't work, please contact the dealer.
4. The drop icon in the upper left corner of the interface transforms into a yellow exclamation mark.	→	Water supply failure.	→	1. Fill the device with water through the funnel. 2. If the above method doesn't work, please contact the dealer.
5. The thermometer icon in the upper left corner of the interface transforms into a yellow exclamation mark.	→	Temperature malfunction	→	1. If it appears upon turning on the device, probably the handle is not securely connected to the main unit. 2. If it appears upon starting the operation, power off the device, then wait for 1 minute before restarting it. 3. If the above methods don't work, please contact the dealer.

9. FAQ

1. How does the cryolipolysis treatment contribute to the body-shaping effect?

A: During cryolipolysis treatment, the cooling device is placed on the skin, cooling the subcutaneous tissue to approximately 4°C. This process accelerates the aging and subsequent death of adipose tissue, which is then naturally eliminated from the body through metabolic processes, resulting in a body-shaping effect.

2. Do I need to follow a diet and exercise plan after treatment?

A: After the treatment, it's important to maintain a healthy diet and exercise regularly since the treatment only reduces body fat and doesn't increase basal metabolic rate. Without incorporating exercise and a balanced diet into your lifestyle, there's a risk of regaining weight over time.

3. How long does it take to notice the result of cryolipolysis treatment?

A: Results can typically be seen in approximately 1-2 months, although this can vary based on individual factors and metabolism. The treatment utilizes a constant temperature of 4°C to cool the adipose tissue, which is then naturally eliminated through metabolism. It may take some time for the body to metabolize the treated tissue, so visible results may be noticed gradually over time.

4. Why is the anti-freezing membrane necessary?


A: The anti-freezing membrane helps regulate the temperature of the skin, ensuring even distribution of cold and heat in the treated area. Through clinical testing, it has been shown to effectively prevent frostbite on the skin during cryolipolysis treatment.

10. Operation Guide

Abdomen:




(30 minutes)

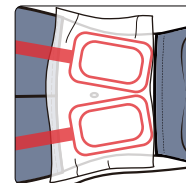

Hips:




(30 minutes)

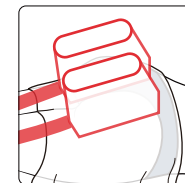

Waist:




(30 minutes)

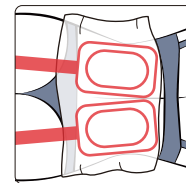

Leg:



(30 minutes)

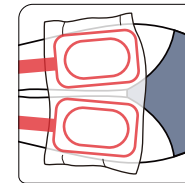

11. During Treatment

1. During the treatment, the treatment area is gradually drawn into the handle. The stronger the suction, the tighter the treatment area is pulled in.
2. You may use two handles simultaneously.
3. When the handle is applied to the treatment area, there may be air trapped between your skin and the membrane, resulting in the formation of bubbles due to the suction from the handle.
4. It's normal to experience mild swelling, redness, or bruising after treatment, so there's no cause for concern.

12. After Treatment

1. Stay warm and avoid cold food or catching a cold.
2. Take a shower 4 - 6 hours later after treatment.
3. Drink plenty of warm water to stay hydrated and promote metabolism.
4. Avoid overeating or staying up late.
5. Avoid consuming cold, raw, or spicy food, and ensure you get enough sleep.
6. Avoid crop tops, backless dresses, miniskirts, and hotpants.
7. Do not scratch the treatment area.
8. Do not press the treatment area.

13. Warranty & Support

We offer a one-year warranty on the main unit and a three-month warranty on components.

Product Name: BeautyFreeze Sculptor
 Product Model: WL-7008C
 Production Date: _____
 Quality Inspection: _____

