

User Manual

WL-V5



The company reserves the right of final interpretation for the content hereinafter.

Table of Contents

Part I
Product Overview
Theories
Product Display
Interface Display
Detailed Operation: Handle (A)
Detailed Operation: Handle (B)
Part II
Treatment Methods
Safety Precautions
Dos and Dont's
Troubleshooting
Machine Maintenance
Packing List
Warranty

Part I

Product Overview

V5 is a new modal integrating HIFU and RF technology, which helps achieve body shaping and body contouring. Please read this manual carefully before use and utilize this product only for its intended use as described in this manual.

Theories

HIFU

HIFU transforms condensed thermal points through source energy and reach deep into subcutaneous fat layers to destroy and reduce the excess fat cells, stimulate collagen contraction and thickening, at the same time stimulate the elastic fiber so that redundant adipose cells are disintegrated into triglyceride that later can be excreted through body metabolism and eventually achieves the body shaping effect.

RF

RF technology is adopted to travel thorough the epidermis and reach the subcutaneous tissues directly, so as to vibrate the skin and the water molecules in the subcutaneous tissues, so as to accelerate the generation of heat by tissue friction. The increase of temperature promotes the catabolic metabolism of subcutaneous fat, stimulates the proliferation and reorganization of collagen and elastic fibers, and achieves the effects of skin firming, shaping and lifting.

Product Display









Product Modal	V5	Power	160W
Modal		Voltage	220V
Working Voltage	36V	Probe	2 pcs
Package Size	52*46*28cm	Weight (package included)	19kg


Interface Display (Handle A)



Detailed Operation: Handle (A)

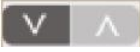1. Clean the treatment area before use; 
2. Connect the power cord and press the round switch button at the top of the screen  to turn on the device;
3. Click 'enter' to go to the interface above  ;
4. Click 'handle (A)' to go to the interface  of the larger probe; HIFU energy is 0-10, RF energy is 0-10, click on  to increase or decrease the energy level; It is advised to start with level 1 for first-time users, and increase accordingly based on one's comfort level; heat or stinging sensation during the process is within the normal range;





5. Click 'time' to set the working time  . For example, the recommended operation time for waist and abdomen is 30-45 minutes; Adjust working time according to the size of the treatment area;

6. After setting the working time and energy, click 'PAUSE'  . There is a round button on the handle to start and stop. Press the button on the handle and you will hear a beep sound to start and press the button on the handle to stop the treatment.


Interface Display (Handle B)



Detailed Operation: Handle (B)

1. Click 'enter' to go to the interface above  ;
2. Click 'handle (B)' to go to the interface  of the larger probe; HIFU energy is 0-10, RF energy is 0-10, click on  to increase or decrease the energy level; It is advised to start with level 1 for first-time users, and increase accordingly based on one's comfort level; heat or stinging sensation during the process is within the normal range;
3. Click 'time' to set the working time  . For example, the recommended operation time for waist and abdomen is 30-45 minutes; Adjust working time

according to the size of the treatment area;


4. After setting the working time and energy, click 'PAUSE' . There is a round button on the handle to start and stop. Press the button on the handle and you will hear a beep sound to start and press the button on the handle to stop the treatment.
5. Clean the surface of the probe after treatment and apply maintenance product.

Part II

Treatment Methods

1. Mark the treatment area and measure the marked site before the operation;
2. Apply the couplant on the treatment area and the probe.

Abdomen Treatment

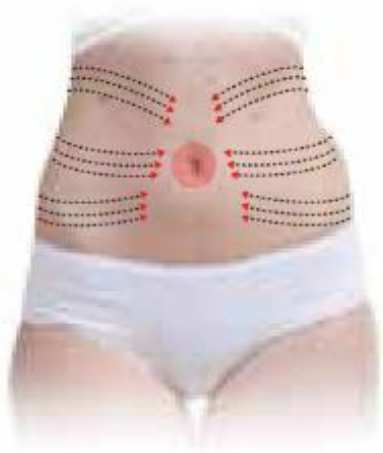

Move the probe in circular motions firstly from upper side of the waist to the navel, and then from the middle side of the waist to the navel, then the lower side of the waist to the navel, and finally from navel to groin; It is recommended to move the probe at a constant speed and even motion; Set the working time according to the user's degree of skin relaxation.

Leg Treatment




Firstly move the probe from the bottom to the top in circular motions, then popliteal fossa to the end of thigh, then from knee to groin; Set the working time according to the user's degree of skin relaxation.

Arm Treatment



Move the probe upwards and then lift towards the underarm from the outer side of the arm; Set the working time according to the user's degree of skin relaxation.

Back Treatment

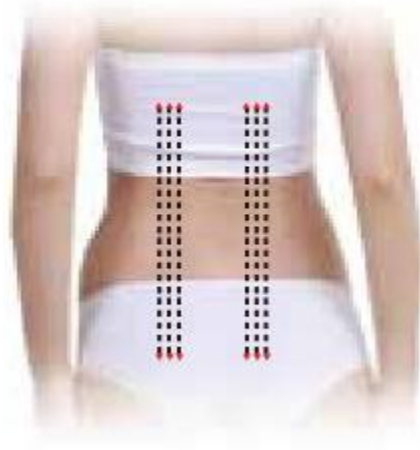

Move the probe from the lower side of the cervical spine to the lower back; Move the probe back and forth to cover the entire back.

Buttocks Treatment



Move the probe from the end of the thigh and pushes upwards.

Waist Treatment



Place the probe above the hip bone, and space one fist above the vertebrae; Follow the arrows to conduct the treatment.

Safety Precautions

1. It is not recommended to operate during menstruation;
2. It is not recommended to operate during pregnancy or one month after child delivery;
3. It is not recommended to operate around metal, plastic, silicon infills;
4. It is not applicable to those embedded with hearing aids, pacemakers and other medical electronic devices;
5. It is not applicable to those with severe '3H' (hypertension, hyperglycemia, hyperlipidemia), metal illness, etc;
6. It is not applicable to those with abnormal fat metabolism, including severe hyperlipidemia or severe fatty liver;
7. It is not applicable to those with skin allergy or inflammation or those with infectious skin diseases;
8. It is not applicable to those with long-term continuous use of anticoagulant polymerization drugs; For example: non-steroidal anti-inflammatory drugs or oral contraceptives, or those with long-term use of hormone drugs;
9. It is not applicable to children under the age of 12 and those with irrational expectations of cosmetic effects;

Dos & Don'ts

1. Hot bath in high temperature is not recommended within 4 hours after the treatment;
2. After operation, it is recommended to keep drinking water every day (2000cc);
3. Keep doing at least 10-15 minutes of exercise a day, preferably jogging or aerobic exercise and stay in a good mood;
4. Diet should be light; Do not overeat and try to keep a nutritional and balanced diet;

5. Keep a balance between work and rest; Do not drink excessively, eat late snacks, etc.

Troubleshooting

Problem	Solution
No response after plugged in	First, check whether the line plug is powered on and replace the outlet or socket
Can't get to the interface after turning on the device	Turn off and restart, or loosen the screws on the screen

Note: The device is a precision electronic product. It is prohibited to use in a humid environment, such as bath room; Once flooded with water, please stop using immediately and return it to the factory for repair!

Packing List

1× Main Machine	1× Power Cord
1× Handle A	1× Handle B

Machine Maintenance

1. Use a three-prong plug with grounding pin, and ensure that the power socket is well grounded;
2. Ensure that the voltage of the device is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase to the matching power voltage;
3. The instrument shall not be placed in a humid place or near the water source, nor shall it be directly exposed to the sunlight;
4. The instrument should not be too close to the strong pyrogen or strong radiation source (disinfection cabinet, induction cooker, microwave oven, etc.), which will affect the normal use of the device;
5. Always use the original qualified parts provided by the manufacturer;
6. After each operation, the operating handle shall be cleaned, disinfected and properly stored;
7. If the instrument is not in use, please turn off the power supply, pull out the plug and properly store all kinds of accessories.

Warranty

Dear users:

Thank you for your support and trust!

V5 warrants the device for a period of ONE YEAR after the original date of purchase, against defects due to faulty workmanship or materials. It is free of charge to repair during the warranty period. (Note: Repair costs shall be borne by the user for malfunction caused by the user).

Terms & Conditions

1. One year warranty starts from the date of purchase;
2. For any warranty claim, warranty card and the record of purchase must be provided;
3. Repair and replacement of the device components shall be charged with the amount listed as the replacement price if the warranty expires;
4. The customer shall NOT be entitled to any claim under the warranty to repair or replace the product under the following circumstances:
 - 4.1. Damage or malfunction caused by opening or disassembling the device;
 - 4.2. Failure to provide provide the warranty card and the record of purchase.

Modal
Date of Purchase
Name
Contact Info.
Malfunction Description