

User Manual

WL-VS17



Preface

To our valued users,

Thanks for choosing our latest beauty equipment: a far infrared sauna blanket. This newest instrument for detox and dispelling dampness applies far infrared technology. WL-VS17 is professional beauty equipment, which needs to be operated by people with professional training. Any improper use of this equipment may bring adverse consequences to the body. Thus, we advise all people to read this manual thoroughly and strictly follow the instructions specified before operation.

We believe our quality product will give you a good return and our perfect after-sales service will make you worry-free.

Thank you.

Table of Contents

Preface
Part I
Introduction
Advantages
Working Principle
Main Efficacy
Indications
Contraindications
Matters Needing Attention After Treatment
Part II
Detailed Operations
Technical Parameters
Precautions on Use
Common Problems & Solutions
FAQs
Packing List

Part I

1. Introduction


At present, instruments that adopt far infrared technology are the most popular machines for detox and dispelling dampness. It can effectively replace the traditional steaming programs in a beauty salon, operate easily, conveniently, and swiftly, and solve body problems for people pursuing beauty. Moreover, this equipment works safely and effectively and requires no skin breaking, injection, medication, or surgery. It has no side effects. Far infrared delivers its curative effect by utilizing the tepidity effect, accelerating blood circulation, promoting body metabolism, and enhancing self-immunity.

2. Advantages

- (1) It's painless and non-invasive during treatment; it requires no convalescence or stay and will not affect regular work and life after treatment.
- (2) It has zero consumption and is low-cost, but it gives a quick return.
- (3) Unevenness, bleeding, and swelling will not occur.
- (4) It refines the skin, detoxifies the body, enhances blood circulation, opens pores, changes skin quality, increases skin elasticity, and delays aging.
- (5) It reduces weight, contours the body, and relieves nervousness, pressure, insomnia, and headache.
- (6) It also reduces skin inflammation and disinfects the skin, presenting an evident effect and better comfort level.

3. Working Principle

It uses carbon fiber to produce infrared rays that are used for heating. Among all sunrays, far infrared ray is a light wave that can penetrate the skin and subcutaneous tissue and is hailed as "the light wave of life." The vibrational frequency of far infrared is close to that of cellular molecules inside the body. It can generate resonance with the body cells, enhance physiological redox, accelerate blood circulation, boost metabolism, and dissolve excess fat. Therefore, it removes toxins, slims the body, discharges acid, expels cold and dampness (can effectively dispel wind-damp and arthritis), and makes the skin smooth, delicate, and elastic.

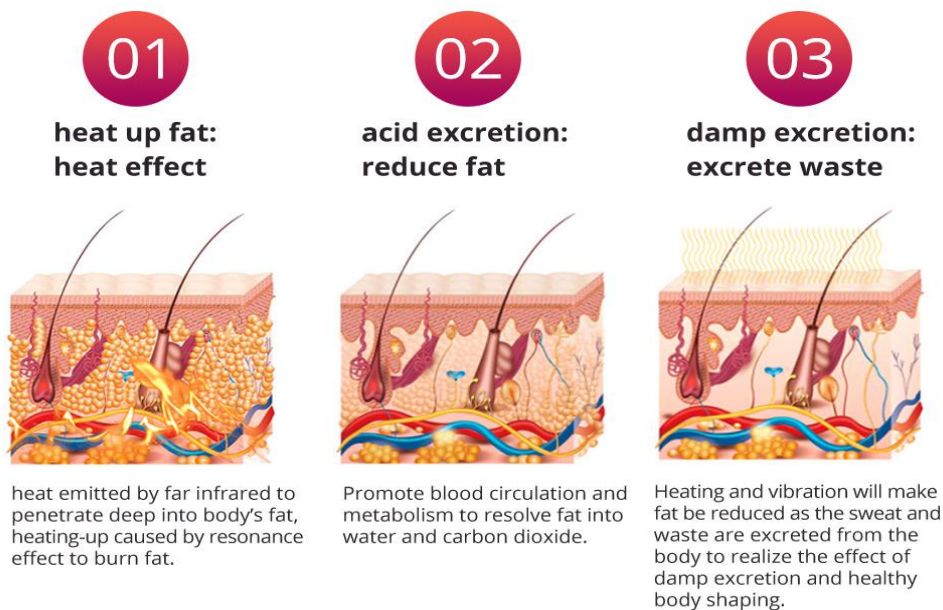

4. Main Efficacy

- (1) Activate blood, remove stasis, and dredge channels and collaterals.
- (2) Relieve fatigue and improve sleep.
- (3) Improve aged skin and increase skin elasticity.
- (4) Accelerate blood circulation and metabolism.
- (5) Reduce weight, slim body, enhance immunity.
- (6) Stimulate collagen hyperplasia and delay aging.

5. Indications

- (1) People with puffiness.
- (2) People who always feel tired.
- (3) People who engage in brainwork.
- (4) People with a cold-damp constitution.

6. Contraindications

- (1) People who recently injected hyaluronic acid, did skin booster, or had an injection for wrinkle removal or plastic surgery.
- (2) People with hypertension, heart disease, diabetes, severe thyroid, malignant

tumors, etc.

- (3) People in the allergic period or with severely sensitive skin.
- (4) People with skin trauma or wounds.
- (5) Overage people.
- (6) People in menstruation, pregnancy, lactation, or surgical recovery.
- (7) People with skin disease or infectious disease.
- (8) People who have an unrealistic illusion about the effect.

7. Matters Needing Attention After Treatment

- (1) Take a shower after 4 to 6 hours.
- (2) Do not catch a cold, and avoid having the fan or AC blowing at you directly.
- (3) Keep warm, and drink more hot water.
- (4) Avoid wearing shoulder-baring and backless clothes.

Part II


1. Detailed Operations



(1) Introduction of the Main Machine



Time Display

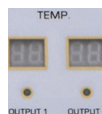

Output Temperature Display



Time Display for Output 1



Time Display for Output 2



Time & Temperature Setting & Display



Time & Temperature Setting (increase)



Time & Temperature Setting (decrease)



Time & Temperature Function Switch

(2) Detailed Operations

Properly connect the connecting wire between the main machine and sauna blanket.

Install the power cord to the main machine and plug it into a good power socket.

Press the power switch and turn on the device.

Let the client lie inside the blanket.

Adjust the operating time and temperature per the client's tolerance.

Just wait until the time runs out and the device stops working.

2. Technical Parameters

Color: Silver

Material: PVC

Power: 650W

Frequency: 50Hz-60Hz

Voltage: 110V or 220-240V

3. Precautions on Use

- (1) You must use a plug with a ground pin and ensure the power socket is already grounded before using the equipment.
- (2) Make sure the voltage of the equipment is adaptive. If the local power supply voltage is not stable, we suggest users add a voltage regulator with matching power between the mains supply and the equipment.
- (3) Please uniformly use specified parts provided or suggested by the original manufacturer to guarantee the equipment's curative effect and normal service life.
- (4) Do not put the equipment in a damp place, near a water source, or sites that expose it to direct sunlight.
- (5) Please don't place the equipment near an intense heat source since this may affect its service life and everyday use.
- (6) Please turn off the equipment's power switch if no one uses it, and ensure the main power is off after everything is settled. Thus, the equipment's safety can be kept.

- (7) Use the equipment and train the operators in strict accordance with the instructions specified in the manual.
- (8) Take a shower after 4 to 6 hours following the treatment.
- (9) Do not use it with a full or empty stomach.
- (10) When the treatment ends, cut off the power, spread the blanket until it cools down, and then put it away.
- (11) Do not use it if you wear jewelry; do not insert foreign matter or pierce it with a sharp object.
- (12) Those with the following situations can not use it: can not take care of themselves; insensitive to heat; with a heart problem, hypertension, skin disease, brain paralysis, or brain tumor; pregnant woman or baby.

4. Common Problems & Solutions

- (1) Does the interface not respond after clicking Start?
 - A. Make sure the power cord is plugged into a good power socket.
 - B. Check whether the fuse in the back is loose or burnt out.
- (2) Is there an error message on the monitor after the equipment is on?
 - A. Take out the plug from the back, wait about one minute, reinsert the plug, and restart the equipment.
 - B. If the solutions mentioned above are not working, please contact the equipment dealer for assistance.

5. FAQs

- (1) Q: Is far infrared ray harmful to the body?
A: Far infrared ray is invisible light. It delivers its curative effect by utilizing the tepidity effect to accelerate blood circulation and promotes metabolism. Thus, it causes no harm to the body.
- (2) Q: How long does a treatment take?
A: The time for one treatment is 20 to 30 minutes. We will use skincare products and the instrument together to deliver a noticeable effect.
- (3) Q: What functions does this blanket have?
A: It can dissolve the excess fat, promote blood circulation, enhance metabolism, remove toxins from the body, slim the body, discharge acid, expel cold and dampness, contour the body, refine the skin, nourish internal organs of the body, and boost immunity.

6. Packing List

1 set Mainframe

1 pc Power Wire
1 set Blanket
2 pcs Fuse