

User Manual

WL-WD678A



Foreword

Dear Users:

We're pleased to present you WL-WD678A, the multifunctional beauty device that's featuring in fat-blasting, firming, body-sculpting, lifting and anti-aging. It works effectively on wrinkle removal on face and eyes and fat-dissolving so as to shape up your body. By utilizing high-end technology, it delivers a safe and effective skin care experience and effect. WL-WD677 is professional beauty device is for professional use only, which needs to be handled by professionals with proper training. Any improper use will result in adverse consequences. Therefore we advise any personnel read this manual thoroughly and apply with the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

Foreword
Part I
Brief Introduction
Advantages
Working Principle & Treatments
Facial Firming
Eye Anti-aging
Shaping Waist & Abdomen
Warm Womb & Detoxification
Shaping Breasts
Body Shaping
Arms
Back
Buttocks
Legs
Part II
1. Detailed Operation
2. Technical Parameters
3. Safety Precautions
4. Dos and Dont's
5. Troubleshooting & Solutions
6. FAQs
7. Packing List
8. Operational Diagrams

Part I

Brief Introduction

RF and ultrasonic beauty machines are currently the most popular beauty devices in beauty parlors that featuring in firming, body-sculpting and anti-aging. Combined with effective fat dissolving results of laser pads, it's a good replacement for various of facial and body care programs in beauty salon, including weight loss and body-sculpting, etc. It delivers a safe, comfortable and and effective skincare experience. At the same time, it requires no injection, no medication and no surgery. It is used externally throughout the entire process and has an immediate effect. RF beauty device heat up deep skin layers, stimulate collagen regeneration, accelerate blood circulation on hypodermis, thus achieving effects like firming, plumping, anti-aging and so on.

Advantages

1. Multifunctional beauty machine that works on both of your facial and body care.
2. Ultrasonic fat burning replaces and transcends liposuction, bringing good news for treating obesity
3. Various work heads can be replaced in different parts according to different requirements.
4. The perfect combination of ultrasound and radio frequency solves the dilemma of losing weight. It helps with weight loss while at the same time maintains firm skin, increases skin elasticity, metabolism, and ultimately better health.
5. Adopting the cutting-edge technology of ultrasonic fat burning with a wider ranger and faster and more visible results.
6. Painless, non-invasive during the process. Skipping any recovery periods without affecting any work and life plans.
7. No consumption, low cost and quick returns.
8. Unevenness, bleed, swelling and stasis will not appear after treatment.
9. Laser fat dissolving: even fat dissolving presents safe and painless experience, leaving skin smooth and scar-free.
10. Ultrasound, negative pressure and radio frequency all have the energy release of red light respectively. They can dissolve fat and massage while at the same time accelerate metabolism, and sterilize skin. The effect is remarkable with a higher comfort level.
11. Red light RF is with a more effective result.
12. HIFU is with a higher energy level and evenness.

Facial Firming

Principle

BIO

BIO Cell Activation Instrument, which imitates human biological current, enters muscle cells through skin, stimulates ATP contained in cells, and restores normal operation and function of cells. It collects the latest technology to shape facial contour, thin face, reduce double chin and wrinkles and shrink pores for customers in the shortest time possible. BIO makes facial muscles tight and delicate, and rosy cheeks will appear naturally. It is more effective on loose skin, so as to achieve the goal of shaping a beautiful face.

RF

Radio frequency can alternate electrical polarities of biological tissues on the treated area millions of times per second and change directions in the same frequency concurrently. In the rapid change of electrodes, the dermal tissue naturally resists the radio frequency current and generates thermal energy, which stimulates the dermis to produce more collagen and fill up the gap of the lost collagens, therefore plumping up the skin and revive elasticity.

During multipolar RF treatment, polarities alternate consecutively, which produces a higher intensity energy release, heats up the bottom layers of skin. Through fast and consecutive heating of tissues, collagen regeneration is boosted, thus having a more visible result in a shorter period of time. It also comes with a wider and more intensifies treatment scope.

Biological Effect

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to $45^{\circ}\text{C}\sim 60^{\circ}\text{C}$, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

Effects

1. Firm and elevate, increase skin elasticity.
2. Improve sagging and loose skin; wrinkle removal.
3. Help absorb and lock nutrients; smooth skin.
4. Accelerate metabolism and blood circulation in hypodermis for healthier skin.
5. More plump and elastic skin.

Applicable Range

1. Those with facial dullness.
2. Those with skin relaxation and sagging.
3. Those with dark circles, eye bags, fine lines, nasolabial folds, crow's feet.
4. Those with lax skin after birth.
5. Those who are lacking in elasticity.

Inapplicable Range

1. Those who have just undergone plastic surgery which have implanted prostheses, metal materials and so on.
2. Those who recently experienced certain injections, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
3. Those with hypertension, heart disease, diabetes, severe thyroid and malignant tumors.
4. Those who are having an allergic reaction, or people with severe sensitive skin or allergic to metals.
5. Those with skin trauma or wound.
6. Those who are overaging.
7. Those in menstruation, pregnancy, lactation, surgical recovery.
8. Those with skin diseases and infectious diseases.
9. Those who are unrealistic about the effects.

Notes after Operation

1. Do not wash your face with overheated water within three days (lukewarm or cold water can be used).
2. Strengthen moisturizing and sunscreen.
3. Do not soak in hot springs, steam saunas or exercise strenuously, etc. within three days.
4. Use facial masks at least three times a week.
5. Spicy, greasy food should be avoided. Staying up late, smoking and drinking are prohibited. More vegetables, fruits, and less greasy food.
6. Avoid food that will cause 'three Hs'(high blood pressure, high cholesterol and high

blood sugar), less greasy food is advised.

Eye Anti-aging

Principle

BIO

The combination of micro-current and inductive current has a strong cosmetic effect. The function of microcurrent is to increase the synthesis of ATP (adenosine triphosphate), which is five times higher than usual. It can provide muscle and body energy, make you look energetic; Improve the permeability of cells, 30% - 40% higher than usual, help the human body better absorb nutrients and water, oxygen-containing blood, etc., and remove waste and toxins as soon as possible. Promote the synthesis of muscle fibers, 68% higher than usual, keep the skin moist, soft, compact and lustrous; Increase the level of collagen synthesis to 73%, reduce wrinkles, make the skin more soft, elastic and lustrous. The function of induced current is to balance cell's electric potential, increase cell ATP, accelerate blood circulation, improve collagen synthesis and increase skin elasticity.

BIO Cell Activation Instrument, which imitates human biological current, enters muscle cells through skin, stimulates ATP contained in cells, and restores normal operation and function of cells. It collects the latest technology to shape facial contour, thin face, reduce double chin and wrinkles and shrink pores for customers in the shortest time possible. BIO makes facial muscles tight and delicate, and rosy cheeks will appear naturally. It is more effective on loose skin, so as to achieve the goal of shaping a beautiful face.

Effects

1. Relieve eye fatigue, dark circles, bags and edema.
2. Reduce lines around eyes and crow's feet.
3. Reduce dark spots and accelerate blood circulation.
4. Accelerate metabolism and prevent hyperpigmentation.
5. Supplement nutrition to the eyes and moisturize skin.
6. Tighten and soften skin, lift the corners of the eyes.
7. Accelerate blood circulation around eyes and help skin absorb nutrients efficiently.

Applicable Range

1. Those with wrinkles, fine lines, bags and dark circles around the eyes.
2. Those with dry skin, dry lines and dynamic wrinkles.
3. Those with eyes prone to fatigue and dryness.
4. Those frequently face computer screens and mobile phones.

5. Those who often stay up late and have dark circles.
6. Those often stay in a dry or hot environment.

Inapplicable Range

1. Those who had just undergone plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe thyroid and malignant tumors.
3. Those who are having an allergic reaction, or people with severe sensitive skin or allergic to metals.
4. Those with skin trauma or wound.
5. Those who are overaging.
6. Those who are pregnant or convalescents.
7. Those with skin disease and contagious diseases.

Notes after Operation

1. Avoid excessive exposure to the sun. Sun protection is encouraged.
2. After operation, the treated area is relatively dry, moisturizing and sun protection is encouraged.
3. Within 1-3 days after operation, it is better not to use alcohol, AHA or other exfoliating products.
4. Avoid facial washing with overheated water, sauna steaming, hot springs or other strenuous exercise within 7 days after operation.
5. Moisturizers and eye masks are encouraged to use, preferably 3 times or more on eye masks a week.

Shaping Waist & Abdomen

Principle

40K&RF

Using the sound wave with a frequency of 4 MHz emitted by the strong sound wave head gathering strong sound waves, human adipocytes can produce a strong impact and friction movement between adipocytes after entering the human body, which can effectively consume heat, water and shrink adipocytes. In addition, when sound waves vibrate, it can make adipocytes smaller. Strong crack will come along, cells will burst instantly, adipocytes reduce, so as to achieve the effect of fat removal.

Cavitation Principle of Ultrasound: Tens of thousands of tiny bubbles, namely cavitation bubbles, are produced by vibration of liquid. These bubbles grow in the negative pressure region formed by the longitudinal propagation of ultrasound, and

close rapidly in the positive pressure region, thus being compressed and stretched under alternating positive and negative pressures. The bubbles will be compressed until burst, which will generate huge instantaneous pressure, generally up to tens of MPa to hundreds of MPa, and produce strong vibration and noise.

Advantages: only aim for low-density adipose tissue in a specific frequency, protect high-density tissue such as vascular and nerve tissue.

Vacuum&RF

Massage skin and muscles with the special negative pressure suction head, which can effectively improve body fluid flow and increase cell activity to achieve effects like improving skin elasticity, accelerating blood circulation of capillary blood vessels, clearing out toxins through lymphatic drainage, and improving and lowering the occurrence of dark spots, hyperpigmentation and blood stasis. Coupled with all sorts of massage techniques, release and suction function of negative pressure can achieve firming and shaping of body.

1. Skin layer

The absorption and discharge of air pressure:

- (1) It can improve the fluidity of cells, thus increasing the movement of cells, and can treat diseases related to blood stasis and blood stagnation.
- (2) draining lymph glands and veins.

Effects: redundant water is discharged from tissue fibers.

2. Vascular layer

The absorption and discharge of air pressure

- (1) Improve the blood circulation of microvessels, not only for the capillary system, but also the flow between the deeper veins and lymphatic networks.
- (2) Remove excess toxins in the body.

Effects: Strengthen blood vessels, eliminate toxins and relieve varicose veins.

3. Fibrous layer

The absorption and discharge of air pressure

- (1) Repair cell tissue, increase its activity and elasticity.

- (2) Stimulate the production of bone collagen and improve skin plumpness.
- (3) Improve the support capacity of skin and increase the consumption of carbon dioxide.

Effects: Break the hard fibers of cellulite, make it more elastic, so as to achieve the effect of body-shaping.

Lipo Laser Fat Removal

Laser fat-dissolving device uses red laser with a wavelength of 635nm-650nm, which is a visible spectrum. This wavelength of light has a strong penetrating power, which can effectively activate and repair fat cells, penetrate the fat layer, and dissolve subcutaneous fat by heat. Its treatment method is safe, without side effects or pain.

Low levels of laser energy chemical signals are sent to fat cells to break down triglycerides stored in the body's fat layer into free fatty acids and glycerol and release them through membrane channels. Fatty acids and glycerol are transported to tissues that produce metabolic energy. The release of fatty acids is a natural response when the body needs to use stored energy reserves, so there is no unnatural reaction in the body, nor does it affect or damage surrounding structures such as skin, blood vessels and peripheral nerves. After a period of exercise therapy, it will ensure that the free fatty acids of the body are eliminated through complete metabolism.

Biological Effect

LED uses 635nm - 650nm LED laser heating fat cells in the target zone to make it decompose. In a very short period of time, fat deposition is only absorbed by our body's natural metabolic processes and discharge. In the process of operation, the laser seal small blood vessels, significantly reduce the injuries, introduce the heat treatment area, stimulates the production of collagen, to make it smooth and tight and also achieve the effect of body-sculpting.

Laser mainly acts as a low-energy laser (biological stimulation) to reach fat in depth. It stimulates biological cells and induces or strengthens a series of physiological reactions by giving appropriate energy, including promoting local blood circulation, regulating cell function, enhancing immune function and promoting cell metabolism. It can at the same time tighten skin, wrinkles, reduce fat, and shape body. Laser penetrating power is very strong, and can activate or repair fat cells more effectively. It can also penetrate the fat layer, heat up and dissolve subcutaneous fat. The treatment method is safe, and has no side effects, no pain.

Using the latest non-operative, non-invasive laser fat decomposition technique, low levels of laser energy released. It produce a kind of chemical signals in the fat cells that will dissolve the trioxide triene stored in body into free fatty acid, glycerin and water molecules. And this is the natural reaction when using the energy reserves one need. The free fatty acids are transported into the body through the lymphatic system to provide energy for the body, just like the body would respond when lack of heat.

Effects

1. Improve loose and soft waist and abdomen skin.
2. Improve unwanted fat on waist and abdomen, such as swimming rings, and beer belly.
3. Tighten skin, improve stretch marks, fat marks and increase skin elasticity.
4. Accelerate metabolism, improve constipation and intestinal peristalsis.

Applicable Range

1. Those with cold hands and feet, cold womb and body.
2. Those with unwanted fat on waist and abdomen, protruding beer belly, postpartum loose skin.
3. Those who sit for long hours, and desire a curvier waistlines.
4. Those with fat marks, stretch marks
5. Those with constipation, and abdominal meridian blocking.

Inapplicable Range

1. Those with 'three Hs' (hypertension, hyperlipidemia, hyperglycemia) and heart disease.
2. Those who are during pregnancy, menstruation and lactation.
3. Those whose surgical wound is healing or convalescence.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor and hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Those with severe gynecological diseases.

Notes after Operation

1. Avoid wearing crop tops, and exposure to wind and cold.
2. Avoid overeating, staying up late, drinking, eating raw, cold, spicy and greasy food, and drinking more hot water.
3. Keep your abdomen warm and bathe 4-6 hours after the treatment.
4. Avoid sauna, hot spring or strenuous exercise within 7 days after operation.
5. Before going to bed at night, you can knead your abdomen clockwise with your hands to make the metabolic effect of weight loss better.

Warm womb & Detoxification

Ovary is an important organ of women, and can stimulate the hormone secretion of women again, increase the level of hormones in the body, also known as the source of life of women. Good ovarian maintenance can make facial skin fine, smooth and fresh,

always maintain toughness and elasticity. It can also promote reproduction and body health, regulate and secrete female hormones, and improve the quality of life of both sexes. Ovarian health can make a woman's bosom plump, compact, and mellow. Bad ovarian function will affect estrogen secretion, sexual function, quality and color of skin, and women BWH(bust,waist,hip) measurements, which results in yellowish face, bloated body, vaginal dryness, and advance entering to anile condition. So ovarian maintenance is very important for women. Women now have a lot of bad habits, such as dressing for grace and does not care about temperature, in addition to that they often eat some cold food. If continuous for a long time they are prone to have cold womb, which can cause symptoms such as dysmenorrhea, menstrual edema and even amenorrhoea. In a word cold womb does great harm to the health of women, so be sure to take means to warm the womb,which can also achieve the effect of body-shaping.

Working Principles

RF

RF energy works directly on the dermis through the skin epidermis and heats up the body, with its temperature reaching 40 °C-60 °C. Through biothermal effect it enhances the blood circulation and lymph circulation of the cervical region, fast decompose adipose tissue metabolism, thereby to regulate the uterus, increase the period flow, which is good for inhibiting cold womb.

Vacuum&RF

Through special negative pressure suction head on the skin, and muscle massage, can effectively improve the cells in the body fluids liquidity, increase cell movement can activate cells to improve the effect of skin elasticity, at the same time, accelerate the blood circulation of imperceptible blood-vessel, excess toxins through the lymphatic system the normal cycle of body outside, to reduce and improve the dark spots and pigment, as well as blood stasis.

Advantages: breaking through the previous single and simple working mode of negative air pressure, using different modes for different parts can be more efficient and more effective, slimming, shaping, and will never harm human body. Unique RF functional head design, RF and negative pressure are relatively independent but complementary to each other as one of the metabolic system, deep heating while helping massage and metabolism, so that the effect is twice the result with half the effort.

Effects

1. Improve women's gynecological problems.

2. Improve women's cold hands and feet, cold womb and body.
3. Regulate the function of female reproductive system and enhance the ovarian function.
4. Promote follicular development and regulate the secretion of sex hormones (estrogen, progesterone, androgen).
5. Maintain youthful appearance, delay the arrival of menopause and delay aging.
6. Tighten skin, improve stretch marks and increase skin elasticity.

Applicable Range

1. Those with cold hands and feet, cold womb and body.
2. Those with gynecological inflammatory menstruation and dysmenorrhea.
3. Those whose faces begin to sag.
4. Those who are prone to sore and soft waist.
5. Those with cold abdomen and protruding lower abdomen.

Inapplicable Range

1. Those with 'three Hs' (hypertension, hyperlipidemia, hyperglycemia) and heart disease.
2. Those who are during pregnancy, menstruation and lactation.
3. Those whose surgical wound is healing or convalescence.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor and hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Those with severe gynecological diseases.
8. Those whose gynecological diseases are being treated.

Notes after Operation

1. Avoid wearing crop tops, and exposure to wind and cold.
2. Avoid overeating, staying up late, drinking, eating raw, cold, spicy and greasy food, and drinking more hot water.
3. Keep your abdomen warm and bathe 4-6 hours after the treatment.
4. Avoid sauna, hot spring or strenuous exercise within 7 days after operation.

Shaping Breasts

Principle

RF

Through radio frequency heating to produce specific RF waves in the subcutaneous specific depth, through the skin epidermis direct effects on the dermis, allowing

organizations to heat, promote the catabolism of subcutaneous fat, at the same time stimulate collagen and elastic fibers hyperplasia restructuring, with the newborn dermis collagen and rearrange, lets loose skin after the treatment, immediately feel up, tight shaping effect. The form that massages at the same time promotes mammary blood to circulate, make blood accelerate circulatory dredge the clot of siltation and silt gas, will adipose below the circumstance of vibration, free to bosom. At the same time make more nutrition absorption to the breast, not only prevent lobular hyperplasia and the generation of breast cancer, but also let you have a natural, healthy, beautiful, strong breast, let the women who love the United States more confident.

Effects

1. Adjust breast shape and improve auxiliary breast.
2. Improve chest nodules, slight hyperplasia, and relieve menstrual breast swelling.
3. Improve chest enlargement.
4. Improve irregular menstruation and lack of elasticity of facial long spots.
5. Improve postpartum breast atrophy, breast laxity, and breast duct blockage.

Applicable Range

1. Those whose breasts are not good-looking and with auxiliary breasts.
2. Those with nodules in the chest, slight hyperplasia, and breast swelling pain during menstruation.
3. Those whose breast fat is free, saggy and outward expanding.
4. Those with low immunity.
5. Those with irregular menstruation, lack of elasticity of the face long spot crowd
6. Those who feel they have inadequate breasts.
7. Those with postpartum breast atrophy and breast relaxation.

Inapplicable Range

1. Those with cardiovascular and cerebrovascular diseases, diabetes, abnormal coagulation function, and impaired vital organs.
2. Those who are long-term or currently taking anticoagulant drugs, vasodilators, corticosteroids and other drugs.
3. Those whose chest skin has infection foci.
4. Those with severe fibroma and cyst of the chest.
5. Those who are during pregnancy and lactation.
6. Those who just had breast enlargement surgery.

Notes after Operation

1. Drink plenty of warm water and keep warm.

2. Wear undergarments that are designed to be comfortable, and don't squeeze your breasts.
3. Avoid exposure to cold air and shower 4-6 hours later.

Body Shaping

RF

Through radio frequency heating to produce specific RF waves in the subcutaneous specific depth, through the skin epidermis direct effects on the dermis, allowing organizations to heat, promote the catabolism of subcutaneous fat, at the same time stimulate collagen and elastic fibers hyperplasia restructuring, with the newborn dermis collagen and rearrange, after treatment, can make droopy skin feel up, tight shaping effect.

Biological Effect

Radio frequency can vibrate 1 million high-frequency radio waves per second, penetrate the epidermis layer, directly act on the dermis, deep heating collagen cells, stimulate the growth of collagen fiber, make it have a lot of new collagen support, make the skin become firmer and elastic. Acting on the deep dermis through the epidermis, water molecules are rapidly heated to produce safe biological heat. When the natural friction of collagen tissue is heated up to $45^{\circ}\text{C}\sim 60^{\circ}\text{C}$, it will immediately contract and stimulate the continuous growth of collagen. At the same time, biothermal energy can effectively accelerate the blood flow in fat cells and release free fatty acid (ffa), thereby increasing the dissolution of superficial fat and achieving the purpose of tightening the lifting and sagging parts of the body.

40K&RF

Using the sound wave with a frequency of 4 MHz emitted by the strong sound wave head gathering strong sound waves, human adipocytes can produce a strong impact and friction movement between adipocytes after entering the human body, which can effectively consume heat, water and shrink adipocytes. In addition, when sound waves vibrate, it can make adipocytes smaller. Strong crack will come along, cells will burst instantly, adipocytes reduce, so as to achieve the effect of fat removal.

Cavitation Principle of Ultrasound: Tens of thousands of tiny bubbles, namely cavitation bubbles, are produced by vibration of liquid. These bubbles grow in the negative pressure region formed by the longitudinal propagation of ultrasound, and close rapidly in the positive pressure region, thus being compressed and stretched under alternating positive and negative pressures. The bubbles will be compressed until burst, which will generate huge instantaneous pressure, generally up to tens of MPa to hundreds of MPa, and produce strong vibration and noise.

Advantages: only aim for low-density adipose tissue in a specific frequency, protect high-density tissue such as vascular and nerve tissue.

Vacuum&RF

Massage skin and muscle with special negative air pressure suction head can effectively improve fluid mobility of human cells and activate cells. Kneading effect generated by vacuum negative pressure liposuction can increase tissue activity of skin and muscle, thus helping to reduce rigid honeycomb tissue and increase skin tissue elasticity. In addition, the use of negative pressure adsorption and mobile role, can effectively shape shape, make thin and plastic at the same time to complete.

Advantages: breaking through the previous single and simple working mode of negative air pressure, using different modes for different parts can be more efficient and more effective, slimming, shaping, and will never harm human body. Unique RF function head design, RF and negative pressure are relatively independent but complementary metabolic system as a whole, RF has negative pressure, negative pressure has RF, each other as a whole. Compared to the ordinary single-stage RF can be more effective, faster and more uniform perfect remodeling body.

Lipo Laser Fat Removal

Laser fat-dissolving device uses red laser with a wavelength of 635nm-650nm, which is a visible spectrum. This wavelength of light has a strong penetrating power, which can effectively activate and repair fat cells, penetrate the fat layer, and dissolve subcutaneous fat by heat. Its treatment method is safe, without side effects or pain.

Low levels of laser energy chemical signals are sent to fat cells to break down triglycerides stored in the body's fat layer into free fatty acids and glycerol and release them through membrane channels. Fatty acids and glycerol are transported to tissues that produce metabolic energy. The release of fatty acids is a natural response when the body needs to use stored energy reserves, so there is no unnatural reaction in the body, nor does it affect or damage surrounding structures such as skin, blood vessels and peripheral nerves. After a period of exercise therapy, it will ensure that the free fatty acids of the body are eliminated through complete metabolism.

Biological Effect

LED uses 635nm - 650nm LED laser heating fat cells in the target zone to make it decompose. In a very short period of time, fat deposition is only absorbed by our body's natural metabolic processes and discharge. In the process of operation, the laser seal small blood vessels, significantly reduce the injuries, introduce the heat treatment area, stimulates the production of collagen, to make it smooth and tight and also achieve the effect of body-sculpting.

Laser mainly acts as a low-energy laser (biological stimulation) to reach fat in depth. It

stimulates biological cells and induces or strengthens a series of physiological reactions by giving appropriate energy, including promoting local blood circulation, regulating cell function, enhancing immune function and promoting cell metabolism. It can at the same time tighten skin, wrinkles, reduce fat, and shape body. Laser penetrating power is very strong, and can activate or repair fat cells more effectively. It can also penetrate the fat layer, heat up and dissolve subcutaneous fat. The treatment method is safe, and has no side effects, no pain.

Using the latest non-operative, non-invasive laser fat decomposition technique, low levels of laser energy released. It produce a kind of chemical signals in the fat cells that will dissolve the trioxide triene stored in body into free fatty acid, glycerin and water molecules. And this is the natural reaction when using the energy reserves one need. The free fatty acids are transported into the body through the lymphatic system to provide energy for the body, just like the body would respond when lack of heat.

Shaping Arms

Effects

1. Stimulate collagen hyperplasia in the bottom of the skin to shape and firm it.
2. Improve loose skin.
3. Improve worship sleeves and thick arms.
4. Soothe and firm loose skin.
5. Dissolve the fat, and say goodbye to flabby arms.
6. Accelerate blood circulation and dredge channels and collaterals.

Applicable Range

1. Those with thick arms and want to look better in clothes.
2. Those with jiggly and flabby arms.
3. Those with loose arm skin.
4. Those whose arms are prone to pain and numbness.

Inapplicable Range

1. Those who have just had plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe thyroid gland, malignant tumor, etc.
3. Those who are during allergic period and has severely sensitive skin.
4. Those with skin trauma or cut.
5. Those who are overly old.
6. Those who are pregnant and who are recovering from surgery.
7. Those with skin diseases and infectious diseases.

Notes after Operation

1. Keep warm after operation. Do not eat cold food.
2. 4-6 hours later in the shower.
3. Drink plenty of warm water to stay hydrated.
4. Refuse to overeat or stay up late.
5. Avoid sauna, hot spring or strenuous exercise within one week after operation.

Shaping Back

Effects

1. Relieve shoulder and back soreness, and improve the lump on back of the neck.
2. Dredge channels and collaterals, and improve channels and collaterals blocking.
3. Increase blood circulation and metabolism.
4. Improve blood supply to the head and sleep.
5. Regulate the functions of viscera and strengthen the body.
6. Firm skin and prevent sagging.
7. Improve excess fat on the back and shape the back.

Applicable Range

1. Those with sore shoulders and backs and stiff necks.
2. Those with insomnia, dreaminess and fading memory.
3. Those who are prone to fatigue, drowsiness and poor circulation of qi and blood.
4. Those with thick back and want to look better in clothes.
5. Those who has a lump on back of the neck.

Inapplicable Range

1. Those with metal implants in the body, such as stents and pacemakers, are allergic to metal.
2. Those are during pregnancy, menstruation and lactation.
3. Those whose surgical wound is healing or convalescence
4. Those who has “three Hs” , heart disease, epilepsy and severe diabetes and hyperthyroidism.
5. Those with malignant tumor and hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Those who are too weak.
8. Those who are drunk, full, empty, thirsty and overworked.

Notes after Operation

1. Keep warm, avoid cold air and drink plenty of hot water.
2. Shower 4-6 hours after operation.
3. Avoid staying up late, drinking and overeating.
4. Avoid raw, cold and spicy foods and get enough sleep.
5. Avoid shoulder-baring and backless clothing.

Shaping Buttocks

Buttocks are located in the middle of human body, and are the key hub of meridian qi and blood operation. They are the main switch of six meridian and also the bridge connecting the upper jiao qi and blood operation of human body. They are the important factor of the body management whose S curve increases the feminine charm.

Effects

1. Improve blood circulation and speed up metabolism.
2. Improve menstrual pain, irregular menstruation, abnormal leucorrhea and other female diseases.
3. Improve sleep quality, improve female sexual function and tighten the vagina.
4. Activate the warm nest function, stimulate the secretion of glands, and increase the feelings between husband and wife.
5. Make complexion ruddy, fade color spots and return to young state.
6. Shape hips, improve sagging and outward expansion of hips, tighten skin and increase elasticity.

Applicable Range

1. Those with saggy hips and fat accumulation.
2. Those whose butt shape is not good-looking, flat and soft outward expansion.
3. Those with cold and cool hips with low hip temperature.
4. Those with dysmenorrhea, dysmenorrhea, irregular menstruation, gynecological inflammation and other problems.
5. Those with decreased estrogen levels and poor sex lives.

Inapplicable Range

1. Those who are in menstruation, pregnancy, lactation and operation recovery.
2. Those with hypertension, heart disease, diabetes, severe thyroid gland, malignant tumor, etc

3. Those with skin diseases, patients with infectious diseases and skin sensitive period.
4. Those with wounds during recovery from surgery.
5. Those with allergic and severely sensitive skin.
6. Those who have just had liposuction.
7. Those who are overly old.

Notes after Operation

1. Keep hips warm and avoid wearing miniskirts and shorts.
2. Shower 4-6 hours after operation.
3. Drink plenty of hot water to avoid cold air.
4. Avoid staying up late, drinking and overeating.
5. Avoid eating raw, cold and spicy foods and get enough sleep.
6. Avoid sauna, hot spring or strenuous exercise for 7 days after operation.

Shaping Legs

Effects

1. Tighten skin and prevent sagging.
2. Stimulate collagen regeneration and improve fat lines.
3. Increase leg circulation and detoxification metabolism.
4. Promote blood circulation and remove blood stasis, dredge channels and collaterals, and prevent varicose veins.
5. Tighten legs more than proud flesh and shake off big legs.

Applicable Range

1. Those with poor circulation of lower limbs, edema and obese people
2. Those who have low immunity and feel discomfort and pain all over the body are prone to colds.
3. Those with constipation and has coarse, flabby skin.
4. Those whose channels and collaterals of the legs are blocked, and the proportion of the legs is poor.

Inapplicable Range

1. Those who are in menstruation, pregnancy, lactation and operation recovery.
2. Those with hypertension, heart disease, diabetes, severe thyroid gland, malignant tumor, etc
3. Those with skin diseases, patients with infectious diseases and skin sensitive period.
4. Those with wounds during recovery from surgery.
5. Patients with severe varicose veins and tumors.

6. Those with allergic and severely sensitive skin.
7. People who have just had liposuction.
8. Those who are overly old.
9. Those who are pregnant or recovering from surgery.


Notes after Operation

1. Keep warm after operation. Do not eat cold food.
2. 4-6 hours later in the shower.
3. Drink more warm water to replenish water and speed up metabolism.
4. Refuse to overeat or stay up late.
5. Avoid sauna, hot spring or strenuous exercise for 7 days after operation.
6. Wear pants when you're done. Avoid miniskirts and shorts.

Part II

1. Detailed Operation

Detailed installation is as follows:

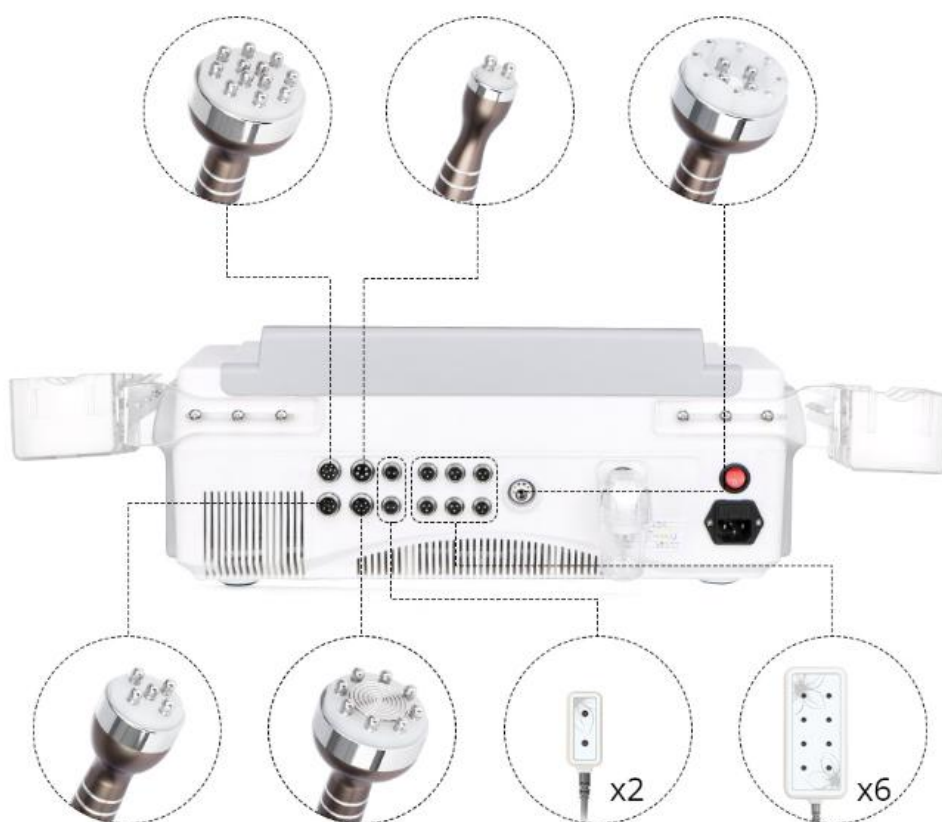


Note: Insert the filter the properly. Avoid going the wrong way or not inserting all way in.

After checking that the instrument is connected properly, the following starting interface will appear when the power supply is turned on.

Welcome

Then it goes to the interface below

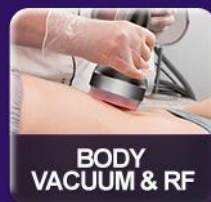

1.1 Function Selection



Working Time



Home



Start/Pause



Restore



Suction Tuning

Twist counterclockwise to lower suction level and clockwise to increase suction level.
Adjust suction level based on your own comfort level.

1.2 Detailed Operation: 40K & RF



Click to go to the interface below




Working Time

Reference Time: 10-20 min on each treatment area



Time +/-

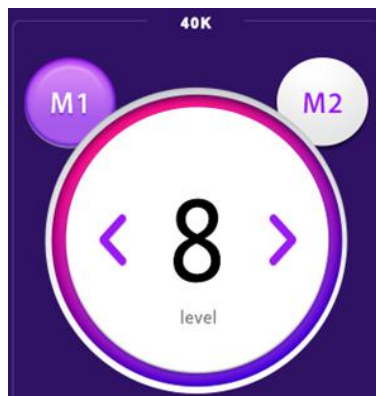

40K Parameters



Consecutive Mode

Under M1 mode, the probe starts to emit energy after clicking start. It is suitable for those who have fat accumulation and thick layers of fat



Intermittent Mode

Under M2 mode, the probe emits energy intermittently. It is suitable for first-timer users and those who takes time to adapt




40K Energy Level


Reference Energy: 2-8



40K Energy +/-

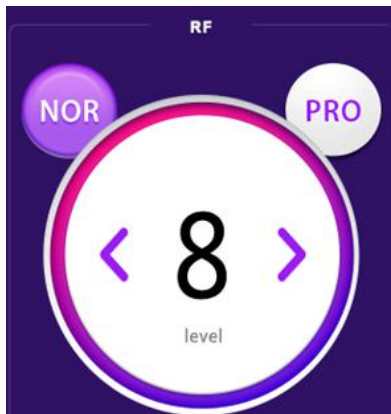

RF Parameters

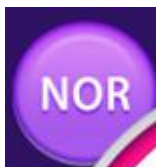

Set Mode

RF reaches the pre-set energy level and red LED turns on after clicking start



Smart Mode

RF probe emits energy slowly and red LED turns on after the probe comes into contact with the skin



RF Energy Level

Reference Energy: 3-8



RF Energy +/-



Home



Start/Pause



Restore

1.3 Detailed Operation: Body Vacuum & RF



Select to go to the interface below



Working Time


Reference Time: 20 min on each treatment area



Time +/-



RF Parameters

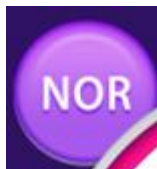

Set Mode

RF reaches the pre-set energy level and red LED turns on after clicking start



Smart Mode

RF probe emits energy slowly and red LED turns on after the probe comes into contact with the skin



RF Energy Level

Reference Energy: 3-8



RF Energy +/-




Suction Time

Reference Suction Time: 5-10 bars

Higher value means longer suction time.

It is advised to set suction time longer than release time.

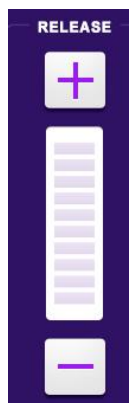

Release Time

Reference Release Time: 0-5 bars

Higher value means longer release time.

When release time is 0, the probe is under direct suction mode.

Direct suction mode can be applied to heavy internal dampness, lymphatic blockage for strong detoxification, weight loss, scraping and body shaping.

Under intermittent suction and release mode, it can be used for body detoxification, massage, muscle relaxation, sore muscle relief, channel dredging, skin tightening and body shaping.



Suction & Release Time +/-



Home



Start/Pause



Restore



Suction Tuning

Twist counterclockwise to lower suction level and clockwise to increase suction level.
Adjust suction level based on your own comfort level.

1.4 Detailed Operation: Body RF




Select to go to the interface below




Working Time


Reference Time: 20 min on each treatment area

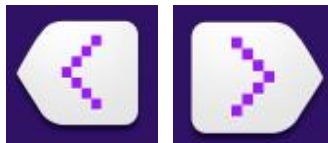

Time +/-

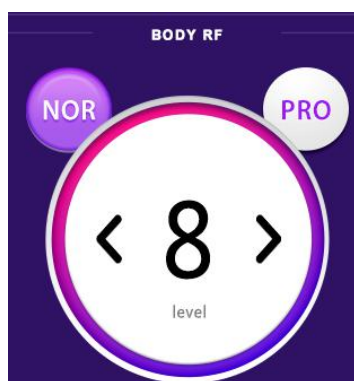

RF Parameters

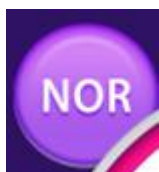

Set Mode

RF reaches the pre-set energy level and red LED turns on after clicking start



Smart Mode

RF probe emits energy slowly and red LED turns on after the probe comes into contact with the skin



RF Energy Level

Reference Energy: 3-8



RF Energy +/-



Home



Start/Pause




Restore

1.5 Detailed Operation: Face RF




Select **FACE RF** to go to the interface below

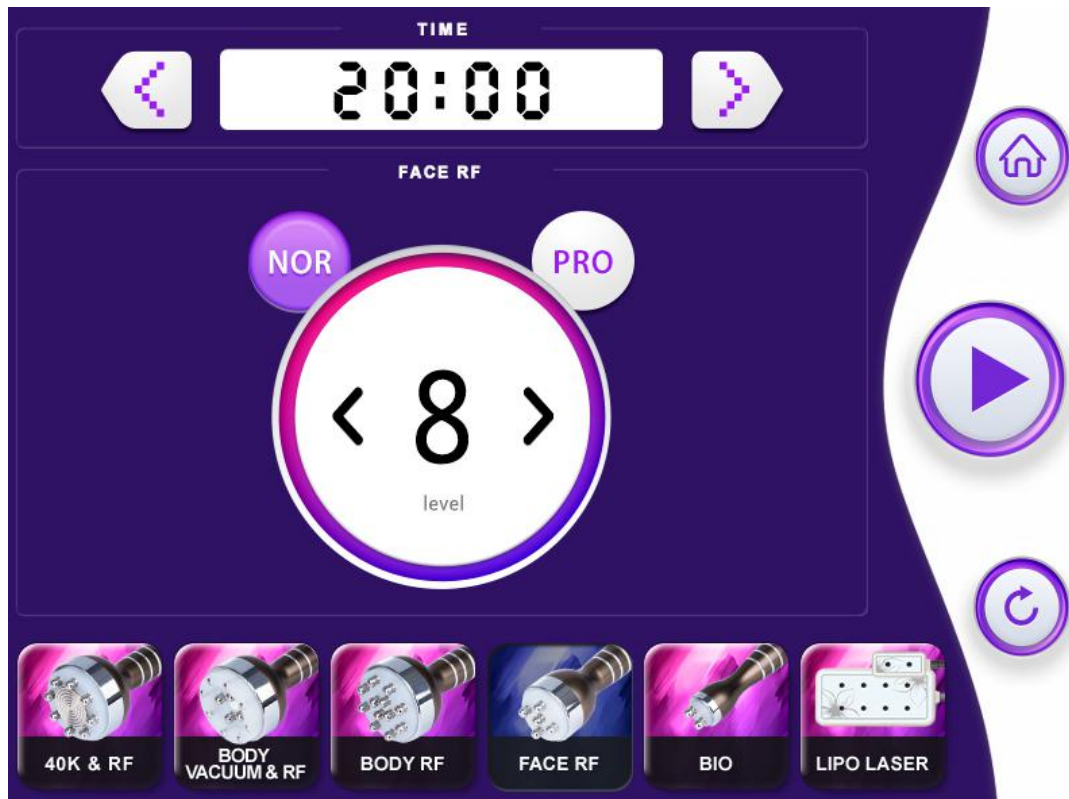


Working Time


Reference Time: 20 min on each treatment area

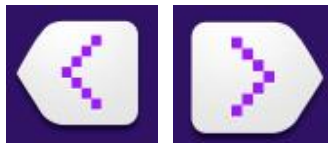

Time +/-

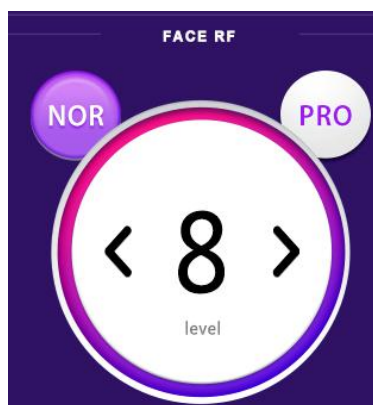

RF Parameters

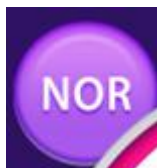

Set Mode

RF reaches the pre-set energy level and red LED turns on after clicking start



Smart Mode

RF probe emits energy slowly and red LED turns on after the probe comes into contact with the skin



RF Energy Level

Reference Energy: 3-8



RF Energy +/-



Home



Start/Pause




Restore

1.6 Detailed Operation: BIO



Select


to go to the interface below

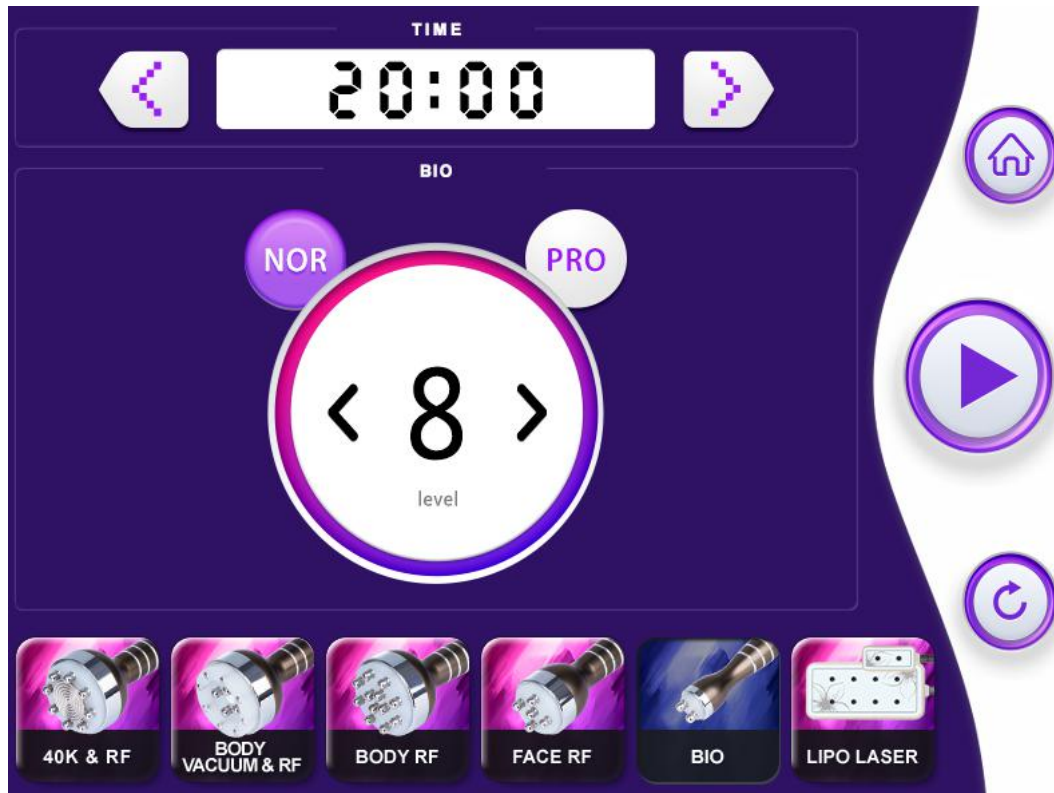


Working Time



Time +/-

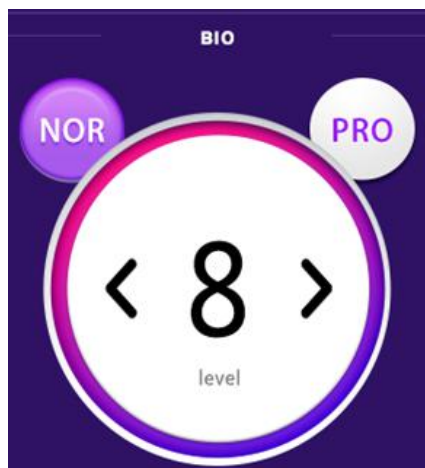

BIO Parameters

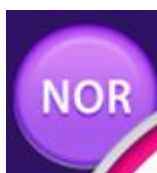

Consecutive Mode

Under consecutive mode, the probes works consecutively after clicking start. It is applicable to tightening, lifting and brightening skin.



Intermittent Mode

Under intermittent mode, the probe emits energy intermittently. It can be applied to products penetration, skin tightening and refining.



BIO Energy Level

It is advised to treat around 15 min on each spot



BIO Energy +/-



Home



Start/Pause




Restore

1.7 Detailed Operation: Lipo Laser



Select

to go to the interface below

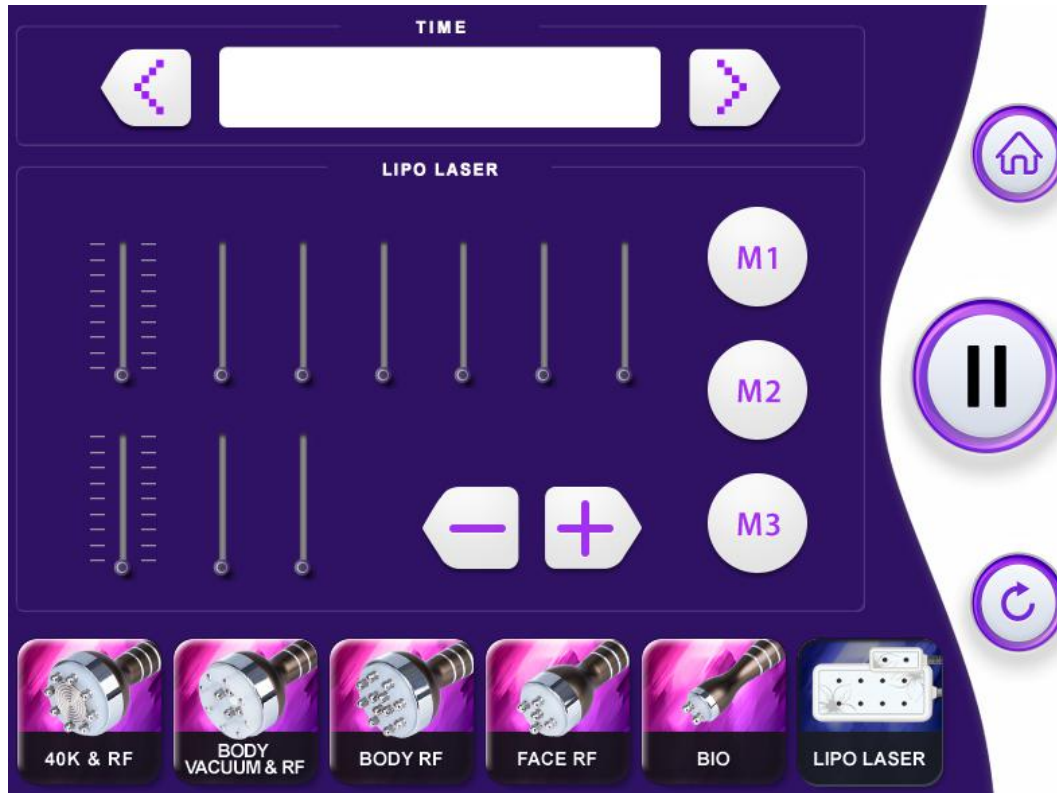

Small lipo laser pads can be applied on small areas such as arms, calves, etc.
Big lipo laser pads can be applied on any big treatment areas of the body.




Working Time

Reference Time: 20-30 min on each treatment site



Lipo Laser Mode

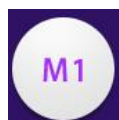

Consecutive Mode

Under consecutive mode, the light of lipo laser pads stays on consecutively after clicking start. It is applicable for those who have thick layers of fat and want to lose weight.



Slow Intermittent Mode

Under slow intermittent mode, the light flashes slowly after clicking start. It is applicable to those who opt for a comfortable slimming experience and stable results.




Fast Intermittent Mode


Under fast intermittent mode, the light flashes quickly after clicking start. It is applicable to those who have thick layers of fat and opt for evident slimming results.



Lipo Laser Energy +/-

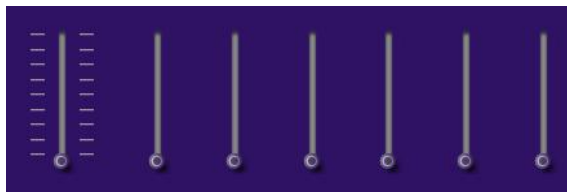


Big Lipo Laser Energy +/-

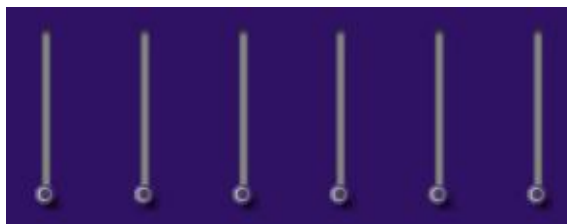

Current Individual Big Lipo Laser Energy



Small Lipo Laser Energy +/-

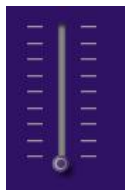

All Lipo Laser Energy +/-



Home



Start/Pause



Restore

2. Technical Parameters

Power supply Input: 110V/220V 60HZ/50HZ

Power Output: Max 110W

40K RF Cavitation Head

Frequency: 40KHz

Power: 45W

Vacuum+RF head:

Power supply output: 66V

Frequency: 3MHz

Power: 90W

12-polar RF head:

Power supply output: 66V

Frequency: 5MHz

Power: 50W

5 Polar RF for Body

Frequency: 1MHz

Power: 40W

Bipolar RF head For eye:

Power supply output: 66V

Frequency: 1MHz

Power: 30W

Lipo Laser

Wavelength – 635nm

Energy output – 52 x 5mW

Totalled output: 52 x 5mW = 260mW

Power output consumption – 100VA

Red wavelength: 650 - 730 nm

3. Safety Precautions

Patients with the following symptoms should be cautious about using this instrument.

Please consult a doctor or a professional before using the instrument. The details are as follows:

1. Those who are pregnant or lactating.
2. Those with heart disease or pacemaker equipped.
3. Those whose wounds have not healed and who are recovering from the operation.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor, hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Avoid wearing metal jewelry during operation.
8. It is not allowed to be use around metal, plastic, silicon and other parts embedded in the body.

4. Dos and Dont's

1. After each use of the instrument, please clean the treatment head with water and keep it properly.
2. The plug with grounding pin must be used before the instrument is used and the power socket of the instrument is properly grounded.
3. Ensure that the voltage of the instrument is suitable. If the voltage of the local power supply is not stable, we suggest that the user add a voltage regulator with matching power between the mains and the instrument.
4. In order to ensure the treatment effect and normal service life of the instrument, please use the specified parts provided or suggested by the original manufacturer.
5. The instrument should not be placed in damp places or near water sources, and should not be directly exposed to sunlight.
6. Do not place the instrument near a strong heat source, as this may affect the life and normal use of the instrument.
7. Please remove all metal objects from your body before treatment, so as not to affect the curative effect due to unexpected conditions.
8. Do not use in the eyes, thyroid gland, parathyroid gland, testicles, pregnant woman's abdomen and atrial pulse generator.
9. Patients who are currently ill should also use this instrument with caution unless approved by a physician.
10. Please turn off the power switch of the instrument when no guest is using it, and make sure to turn off the main power of the instrument before leaving after daily use to ensure the safety of electrical products.
11. Use the instrument or train the operator strictly according to the instruction of the manual.
12. If you are taking other weight-loss drugs, it is recommended to stop taking the drugs for 1 to 2 months before you lose weight. If you want to lose weight immediately, you should extend the course of treatment
13. Do not use the instrument on an empty stomach. Only after at least 1 hour after a

full meal can you carry out the weight-loss treatment.

14. During operation, the instrument should be in full contact with the skin to avoid uneven heating.

15. Start with the lowest energy at the beginning of operation, and gradually add energy after adaptation.

16. When using this instrument, the operating area must be kept moist and avoid dry skin operation.

17. After the operation, scrub the instrument with normal saline to ensure its cleanliness and extend its service life.

18. Fixate the lipo laser pads on the customer before turning on the device. Adjust energy from low to high level based on the customer's feedback.

19. Apply products that contain high volume of oil to keep it moisturized.

20. Make sure the entire surface of the lipo laser pad is in full contact with the skin.

21. Lipo laser pad can be used separately on intended treatment area, then adjust the energy level and working time. It is advised to treat for 20 min or so on each site. It brings better result when combined with massage, RF and ultrasonic devices.

5. Troubleshooting & Solutions

1. The instrument cannot be started, and the key light on the back of the instrument does not work.

A. Make sure the power cord is connected to a valid power socket.

B. Whether the fuse tube on the back of the instrument is loose or burnt out.

2. No RF output of the instrument?

A. Please check whether the plug connecting the instrument handle and the body is tightly connected.

B. Please check whether the treatment area has been cleaned. Grease or oil essential oil products may cause poor contact between the treatment handle of the instrument and human body, resulting in no rf output.

3. Weakened RF output?

A. Please check whether the plug connecting the instrument handle and the body is tightly connected.

B. Please check whether non-conductive grease and other substances are stained on the treatment handle, which will cause poor contact and weaken the output.

C. Please check whether the product used is the adaptive product specified by the instrument.

4. No suction or very little suction?

A. Please close the instrument and check the filter element specially used for the instrument, which may need to be replaced.

B. Please check the rubber ring of the handle plug, because the rubber ring may be

worn and cause air leakage.

C. Please check whether the oil filter cup outside the filter is tightened and whether the rubber ring is worn. It is possible that air leakage at this position may lead to poor air pressure.

D. If the above methods cannot be handled, please contact the instrument dealer for assistance.

5. I can start the instrument, but the screen show's error message?

A. Please unplug the power plug behind the instrument and wait for about 1 minute before restarting the instrument.

B. If the above methods cannot be handled, please contact the instrument dealer for assistance.

6.FAQs

1. Q: How long can I start seeing results of RF treatment?

A: Under normal circumstances within that day or a week. Skin collagen tissue is heated and produces contractions, and you can obviously feel skin tightening. Because RF works by stimulating subcutaneous tissue and bring about sustainable collagen regeneration, so the more you do it, the more visible effects you shall receive.

2. Q: Is RF harmful to skin?

A: RF treatment falls under the category of non-surgery operation. It stimulates collagen regeneration at the skin bottom and boosts metabolism, therefore has no damage to the skin. Partial redness and fever is the normal phenomenon of accelerated blood circulation, and will be resolved by oneself after a moment, so there's no need to worry.

3.Q: How long does the body firming treatment take?

A: One operation time is 60 minutes. We will combine professional techniques and instruments to achieve remarkable results.

4.Q: What are all the functions of this instrument?

A: Dissolve fat, body shaping, smooth skin, firming, lifting, and anti-aging. It can be operated all over the body. In terms of facial care, it can contour facial outline, alleviate wrinkles and saggy skin. In terms of body care, it can reduce regional fatness and build S curve, promote the metabolism and detoxification of the whole body. Meanwhile, it can also enhance the functions of viscera and regulate the sub-health of the body.

5.Q:Which one is better in terms of weight loss, liposuction or this machine?

A: The principle of liposuction is to suck out the excess fat in one part of the body through the method of negative pressure suction, so as to achieve the purpose of

rapid weight loss on parts of the body. It achieves fast results, requires anesthesia, recovery period. It falls under the category of surgery operation and comes with risks. However, our machine is risk-free and has no side effects. While losing weight, it can stimulate collagen regeneration to lift and tighten skin. By lifting, it can also shape perfect curves and accentuate women's natural charm.

6.Q: Will I experience rebound after operation?

A: RF works by dissolving fat to reduce weight, and are not prone to rebound. Because what's dissolved through RF treatment is fat, not water. The formation of fat needs a longer period of time to accumulate, so it is not prone to rebound.

7.Q: Do I need to be on a diet?

A: Pay attention to what you eat. Because radio frequency and ultrasonic treatment end with blasting adipose and accelerated metabolism. It's best to avoid eating spicy, oily, or fried food, to avoid affecting regular metabolism. Also proper workout and sweating help with the process, in a way weight loss will be more visible as well.

8.Q: Does it have any side effects on the body?

A: RF beauty treatment is non-invasive and so far the most safe and effective method to remove wrinkles and shape body. Generally speaking side effects won't appear. A small number of people may experience transient redness or swelling that will disappear after a few hours. Those with dry skin may experience atrophy after the initial treatment. The skin will lose moisture due to RF heat. However, the skin will become plump at the beginning of collagen regeneration and these symptoms will disappear after three days. There are no side effects on health.

9.Q: Can RF instrument be used for chest shaping?

A: As the growth of age and female physiology period, sub-health state, gravity, stimulation during lactation, a large number of nutrition loss... All the above will cause collagen loss, decreased blood flow, damaged elastic tissue, flabby ligament gland, and prolapsed breasts. Through RF energy conduction, collagen protease can be promoted to repair elastic tissue and ligament glands, thus achieving the effect of chest lifting and shaping.

10. Q: Can ultrasound be operated all over the body?

A: Ultrasonic cavitation ruptures fat cells and causes mechanical disruption of fat cell membrane. Liquefied fat cells then cleared away by metabolic pathways. Our heart is a hollow organ, and sensitive to high frequency sound waves. Back and forth reflex occur due to the fact that muscle tissue and blood in the heart do not conduct sound waves in the same way. The reflex force can detach cardiac valves from cardiac muscles. If directed at the eye, it can cause retinal detachment. In a word, avoid eye and heart area when using ultrasonic beauty device. (It is also advised not to use it on waist, back and chest.)

11.Q: Does ultrasound have side effects on human body?

A: Ultrasonic beauty machine requires no surgery, no opening up and no anesthesia, therefore is indeed non-invasive. It relies on ultrasound (mechanical sound wave) - blast fat through cavitation - ultrasound focusing. And the results are fat smash - fat dissolving - skin firming and body sculpting respectively. It only targets at low density fat cells while leaving high density tissues like blood vessels intact, therefore having no side effects on human body. Slight tinnitus might appear to happen during the treatment, which is within normal range so there's no need to worry.

12.Q: Why does tinnitus occur?

A: It's because ultrasound comes with a high frequency vibration, which could go up to 20kHz. It also goes deep into skin fat layers, which could go down to 20mm below epidermis. Highly intensifies ultrasound cause high speed friction among fat cells, resulting in them bursting and heating up, and eventually emulsified. And during the process, ultrasound can cause microvibration to the tissue that make us feel like as "tinnitus".

13.Q: What is collagen?

A: Collagen is a kind of biological macromolecule substance, a kind of white, opaque and non-branched fibrous protein. It can supplement the nutrition needed by all layers of the skin, enhance the collagen activity in the skin, lock moisture, nourish the skin, delay aging, beautify, relieve facial relaxation, provide nutrients for the hair and so on. Collagen is a nutrient that human body must replenish to delay aging. With the growth of age, collagen will gradually be lost. At the age of 20, women have begun to age, collagen gradually decrease. At the age of 25, collagen loss reaches its peak. At the age of 40, the content is less than half of which one has at age 18. It is the loss of collagen and water that causes the wrinkles on the face of the elderly. The breakage of collagen fibers and elastic mesh will cause skin tissue to be oxidized, atrophied and collapsed. The skin will be dry, wrinkled, flabby and inelastic etc,. Therefore, in order to delay aging, collagen must be supplemented.

14. Q: Why do you need breast maintenance?

A: The lymph nodes of bosom rank the top among other parts of the body, which also lead to toxin accumulation very easily, and urbanite life rhythm is very fast now, working pressure is very big, which can bring about the hyperplasia of different level. The saying goes that bosom is the cradle of children, the garden of the woman, the symbol of female identity! Breast maintenance can make your figure more beautiful, add scores for your whole temperament, thus turn head rate increases 10 times certainly! Good figure and temperament will also ensure husband's loyalty!

The evolution of breast cancer: breast milk residual, secretions, toxins -- nodules and blockages -- lobular hyperplasia, ductal hyperplasia, breast hyperplasia -- duct blockage and adhesion -- fibroma, breast cyst -- breast cancer.

15.Q: Why do you need hip maintenance?

A: Because it can help the body lymphatic system discharge poison, improve the gynaecological disease, and avoid harm brought by buttock blockage. Hip impassability surely bring about gynaecological issues. What's inside buttocks is pelvic cavity, bowel. It connects daimai, lumbar vertebra, sciatic nerve from above, uterus ovary, uterine adenexa in front, anus, vagina, inguinal lymphatic from below.

If suffering from squeeze of shangjiao and xiaojiao meridian channels, hip would be most vulnerable to cold, wet and blood stasis. When hip is affected by cold, it can cause meridian contraction of hips, dysmenorrhea, irregular menstruation, dark menstruation color, clot, deterred blood flow and meridian channels. It also affects fertility.

16.Q: How does negative pressure detoxify?

A: Negative pressure can congest capillary, stimulate cell to increase vitality. The suction and release of pressure during operation can make local pores continue to open and close, therefore promote skin breathing, increase the amount of skin oxygen, speed up the elimination of waste. So it achieves the effects of promoting qi and blood circulation, dispelling coldness and dampness, dredging channels and collaterals, clearing toxins, heat and so on.

17. Q: Will I experience rebound after lipo laser fat removal?

A: The number of fat cells in the body is fixed. With the growth of years to adulthood, the increase of body weight is caused by the increase in the volume of fat cells. Laser fat-dissolving removes the local fat structure and reduces the volume of fat. Normally, as long as the diet structure is reasonable, instead of binge eating or drinking all day, you will not experience rebound.

18. Q: Do I need to be on a diet and exercise while receiving lipo laser slimming treatments?

A: Lipo laser slimming results are quite remarkable, but you cannot blindly rely on laser to reduce weight only. If you have reached the ideal limits using lipo laser slimming approach, but does not pay attention to your diet, and starts to binge eat, then rebound is likely to happen. No weight reduction product in the world is guaranteed to last forever. So laser weight loss is an auxiliary method of weight loss. When your weight reached the ideal range, after stopping the laser weight loss, we should also reasonably plan our own diet, and exercise reasonably.

7. Packing List

1 x 40KHz Cavitation RF Head




1 x 12 Polar RF Head for Body







1 x Five polar RF for face







1 x Bipolar RF Head for face and eye around




6 x Big paddle with 8 Light//Each 5mw
 2 x Small paddle with 2 light//Each 5mw
 1 x Holders for light paddles
 1 x Power Supply Cord







8. Operational Diagrams





Parameter Adjustment	Product	Techniques	Diagrams
Facial Firming: 60 min, 1-2 times/week			
RF energy: 3-8 Mode: A BIO energy: 2-8 Mode: B	Makeup remover oil+ Cleanser+ Toner+ Massage cream+ Essence+ Sheet mask	<ol style="list-style-type: none"> 1. Remove makeup and cleanse face, 5 min. 2. Apply toner, 1 min. 3. Apply cleaning base mask, 10 min. 4. Remove the mask and wash face clean, 3 min. 5. Apply massage cream evenly on face and massage face, 3 times. 6. Press on the following acupoints: Chengjiang (Ren-24), Renzhong (Du-26), Dicang (St-4), Jiache (St-6), Yingxiang (LI-20), Jingming (BL-1), Cuanzhu (BL-2), Yuyao (EM3), Sizhukong (TE-23), Tongziliao (Gb-1), the Temple, Chengqi (St-1), Sibai (St-2), 3 times. 7. Massage the entire face, 3 times. 8. Stroke upwards in succession, 20 times. 	<p>Technique6</p>  <hr/> <p>Technique12</p>  <hr/> <p>Technique13</p>  <hr/> <p>Technique14</p>







		<p>9. Stroke upwards in succession on one side of the face, 20 times.</p> <p>10. Do the same on the other side, once.</p> <p>11. Massage the entire face, once.</p> <p>12. Circle around eyes with both hands, 3 times.</p> <p>13. Tracing lower eyelid, push from inner eye corner to the temple on one side, 3-5 times.</p> <p>14. Use scissor hand gesture to lift eye corner on one side.</p> <p>15. Do the same on the other side.</p> <p>16. Alternate hands in lifting in three lines, tracing from chin to Yifeng (耳尖), mouth corner to Tinggong (耳门), nose wing to temple in parallel lines, 3 times.</p> <p>17. Lift towards hairline on forehead, 3 times.</p> <p>18. Do the same on the other side.</p> <p>19. Massage the entire face, 3 times.</p> <p>20. Move in Z shape on forehead, then slide to the front of the ear. Repeat back and forth for 3 times, then slide to the back of the ear.</p> <p>21. RF Move from chin to earlobe, mouth corner to ear gate, nose wing to temple in upward circular motions, 3 times.</p> <p>22. Lift from chin to earlobe, mouth corner to</p>	 <p>Technique16</p>  <p>Technique17</p>  <p>Technique20</p>  <p>Technique21</p>  <p>Technique22,23,32</p>  <p>Technique25,26,34,36</p>
--	--	---	---

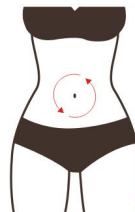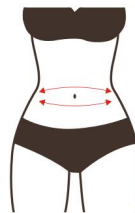



		<p>ear gate, nose wing to temple in upward motions, 3 times.</p> <p>23. Coupled with hand, lift from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>24. Do the same on the other side.</p> <p>25. Lift towards hairline on forehead, 3 times.</p> <p>26. Coupled with hand, lift towards hairline, 3 times.</p> <p>27. Wash face clean, 2 min.</p> <p>28. Apply (anti-aging) essence evenly all over the face, 1 min.</p> <p>29. BIO</p> <p>Start from chin, lifting along the jawline towards ear, 3 times.</p> <p>30. Lift towards cheek along the jawline, 3 times.</p> <p>31. Slide back and forth along the jawline, 3-5 times.</p> <p>32. Coupled with hand, lift from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>33. Do the same on the other side.</p> <p>34. Lift towards hairline on forehead, 3 times.</p> <p>35. Push slowly from the beginning of the eyebrow to the end of the eyebrow, 3 times.</p> <p>36. Coupled with hand, lift towards hairline, 3 times.</p> <p>37. Apply hydrating essence evenly, 1 min.</p> <p>38. Apply sheet mask, 15</p>	 <p>Technique29</p>  <p>Technique31</p>  <p>Technique35</p>  <p>Technique38</p>  <p>Technique40</p> 
--	--	---	---

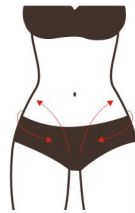		min. 39. Remove the mask and wash face clean, 2 min. 40. Apply toner, essence, eye cream, cream, and sunscreen.	
Suggested Treatment: 10 times for a period of treatment, and once after a finish skin tight feeling, after a period of treatment brightens the skin, repair the skin barrier, two treatment increase the elasticity of skin, increase skin metabolism and detoxification, help fade out pigment, and dark skin, sensitive skin can increase skin resistance, reduce the chances of sensitive and three courses down stable state of skin, prevent skin flabby aging, repair skin contour.			
Eye Anti-aging: 60min, 1-2 times/week			
Bio energy: 3-8 Mode: A	Makeup remover oil+ Cleanser+ Toner+ Eye essence+ Eye masks	1. Wash face clean, 2 minutes. 2. Apply toner, 2 minutes. 3. Apply eye essence evenly on eyes in circular motion, 1 minute. 4. Push the following acupoints Jingming (Bl-1), Cuanzhu (Bl-2), Yuyao (EM3), Sizhukong (TE-23), the Temple, Tongziliao (Gb-1), Chengqi (St-1) using middle and ring fingers, 3 times. 5. Move hands in circular motion on eyes and push temples, 3 times. 6. Move middle and ring fingers horizontally below lower eyelids and slide to back of the ear, 3 times. 7. Lift middle and ring fingers from inner eye corner to upper eyelids and slide to back of the ear, 3 times. 8. Lift eye corners with	Technique 3,5,10,11  Technique 4  Technique 6  Technique 7






		<p>"scissor" hand gesture, 3-5 times.</p> <p>9. Repeat the previous operation on another side, 3 times.</p> <p>10. Move hands in circular motion on eyes and push temples, 3-5 times.</p> <p>11. Move hands on eyes as if writing an "8" number, 3 times.</p> <p>12. BIO move the device in small circles from lower eyelids to temple, 3-5 times.</p> <p>13. Combined with hands: lift from lower eyelids to temple, 3-5 times.</p> <p>14. Lift the device from beginning till the end of eyebrow, 3-5 times.</p> <p>15. Move the device as if writing an "8" number on temple.</p> <p>16. Combined with hands: Lift from lower eyelids to temple, 3-5 times.</p> <p>17. Apply eye masks, 15 minutes.</p> <p>18. Take off masks and wash face clean, 2 minutes.</p> <p>19. Apply toner, essence, eye essence, facial cream, and sunscreen.</p>	 <p>Technique 8,9</p>  <p>Technique 12</p>  <p>Technique 13,16</p>  <p>Technique 14</p>  <p>Technique 15</p> 
--	--	--	--

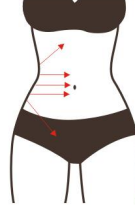


<p>Suggested Treatment:</p> <p>Ten sessions for one treatment. Once done, the eyes are lifted and tightened, increasing blood circulation to the eyes. After one session, it lightens fine lines, dark circles under the eyes and brightens the skin. After three treatments, the skin around the eyes is firm, tender and shiny, improving and preventing the aging of the eyes.</p>			
<p align="center">Shaping Waist&Abdomen: 60 min, once/week</p>			
<p>40K&RF Energy: 3-8</p> <p>Mode: A</p> <p>Vacuum&RF Energy: 3-8</p> <p>Mode: A</p>	<p>Massage cream+ Gel</p>	<ol style="list-style-type: none"> 1. Apply and rub oil on abdomen with hands, 3 times. 2. Rub stomach back and forth with both hands, 3-5 times. 3. Knead abdomen with both hands using chiropractic techniques, 3 times. 4. Lift Dai Channel (BV) on both sides of waist with both hands alternately, 16 times. 5. Move hands as if writing an "8" number on waist, 3 times. 6. Overlap hands and massage the intestinal canal clockwise, 3 times. 7. Move hands in circular motion and caress the treatment area, 3 times. 8. Push on the following acupoints: Shangwan (Ren-13), Zhongwan (Ren-12), Xiawan (Ren-10), Shenque (Ren-8), Qihai (Ren-6), Guanyuan (Ren-4), Zhongji (Ren-3), Tuanshu (St-25), Daheng (SP-15), 2 	<p>Technique 1,7</p>  <hr/> <p>Technique 2</p>  <hr/> <p>Technique 3</p>  <hr/> <p>Technique 4</p>  <hr/> <p>Technique 5</p>

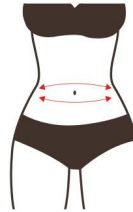		<p>times.</p> <p>9. Starting from zhongji point, push both thumbs to belly navel, slide them to waist and the lift upwards to groin, 3 times.</p> <p>10. Caress the treatment area with hands and slide to groin.</p> <p>11. 40K&RF Starting from one side of the waist, lift the device to abdomen and groin, 3 times.</p> <p>12. Repeat the previous operation on another side, 3 times.</p> <p>13. Move the device on abdomen in small circular motions, 3 times.</p> <p>14. Move the device on abdomen in big circular motions, 3 times.</p> <p>15. Starting from Dai Channel (BV) on one side of the waist, lift the device to another side of the waist back and forth, 3 times.</p> <p>16. Move the device on waist and abdomen vertically back and forth.</p> <p>17. Vacuum&RF Set it to direct suction mode, lift the device on one side of the waist to abdomen, 3 times.</p> <p>18. Repeat the previous operation on another side.</p> <p>19. Lift Dai Channel (BV) horizontally back and forth, 3 times.</p> <p>20. Lift the device on sides of the waist vertically back and forth to shape the</p>	 <p>Technique 6</p>  <p>Technique 8</p>  <p>Technique 9</p>  <p>Technique 10,21</p>  <p>Technique 11,12,17,18,22</p>  <p>Technique 13</p>
--	--	--	--

		<p>body.</p> <p>21. Move the device on abdomen in big circular motion, 3 times.</p> <p>22. Lift the device from one side of the waist to abdomen, 3 times.</p> <p>23. Wrap straps around the waist (loosely).</p> <p>24. Adjust mode and energy level. Strap the lipo laser pads around the waist. Wait for 20 min or so.</p> <p>25. Remove the pads.</p>	 <p>Technique 14</p>  <p>Technique 15,19</p>  <p>Technique 16,20</p> 
<p>Suggested Treatment:</p> <p>Ten treatments account for one full course of treatment. Once done, abdomen is heating up, metabolism is accelerated. After one course, fat slowly reduce through metabolism, body starts to slim down. Constipation and other issues are addressed at the same time. After 2 courses the effect gradually becomes obvious, skin is tightened and lifted, skin collagen increased. Stretch marks and flabby skin are also improved. 3 courses help reinforce the effect. Redundant adipose gradually disappear, lumbar abdomen curve start to take in shape, and slim waist starts to show.</p>			

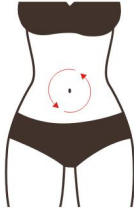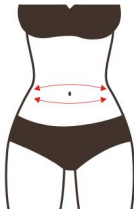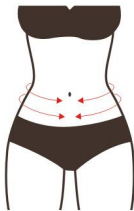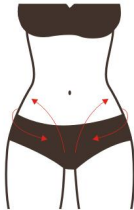Warm Womb&Detoxification: 60 min, once/week
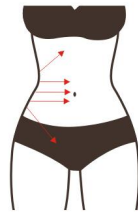
<p>RF Energy: 3-8 Mode:A</p> <p>Vacuum&RF Energy: 3-8</p> <p>Suction Time: 3-8 Release Time: 0-6</p>	<p>Massage cream+ Gel</p> <ol style="list-style-type: none"> 1. Apply and rub oil on treatment area with hands, 3 times. 2. Rub stomach back and forth with both hands, 3-5 times. 3. Lift Dai Channel (BV) on both sides of waist with both hands alternately, 16 times. 4. Move hands as if writing an "8" number on waist, 3 times. 5. Push channels and collaterals from top to bottom with hands alternately Ren channel (CV) - 2 cun away Kidney Channel (KI) - 4 cun away Stomach Channel (ST) - Spleen Channel (SP) - Liver Channel (LV) - Gallbladder Channel (GB) on two sides, 3 times. 6. Move hands in circular motion and caress the treatment area, 3 times. 7. Rub hands together until they're warm and put them on lower abdomen. 8. RF Starting from one side of the waist, lift the device to abdomen and groin, 3 times. 9. Move the device on abdomen in small circular motions, 3 times. 10. Move the device on abdomen in big circular 	<p>手法 1、6</p>  <hr/> <p>手法 2</p>  <hr/> <p>手法 3、12</p>  <hr/> <p>手法 4</p>  <hr/> <p>手法 5、11</p>  <hr/> <p>手法 8</p>
--	---	--

		<p>motions, 3 times.</p> <p>11. Push the device on abdomen through channels in bottom-up order to groin, 3 times.</p> <p>12. Combined with hands: lift Dai Channel (BV) on two sides of the waist, 16 times.</p> <p>13. Move the device on abdomen in small circles, 3-5 times.</p> <p>14. Vacuum&RF</p> <p>Set it at direct suction mode. Lift from side of the waist in parallel lines towards the abdomen, 3 times.</p> <p>15. Do the same on the other side.</p> <p>16. Push and pull along Dai Channel (BV) back and forth, 3 times.</p> <p>17. Slide up and down sides of the waist, 3 times.</p> <p>18. Move the probe in big circular motions on abdomen, 3 times.</p> <p>19. Lift from side of the waist in parallel lines towards the abdomen, 3 times.</p>	 <hr/> <p>手法 9、13</p>  <hr/> <p>手法 10</p>  <hr/>
--	--	--	--

Suggested treatment:

10 treatments account for a full course. Once done, abdominal temperature increases, meridian dredges. After a course, metabolism speeds up, constipation and dysmenorrhea improves. After 2 courses, menstrual flow returns to normal, cold feet improves. 3 courses help strengthens the function of warm womb, regulate the female reproductive system problems, reduce leucorrhea, increase vaginal secretion, delay menopause, delay aging, and skin giving out a healthy complexion.


Shaping Breasts: 45 min, 1-2 times/week

RF Energy:
3-8


Massage
cream+
Gel

1. Stand at the head of the bed, both hands from Shanzhong (Ren-17) point began to rub oil to armpit up at the same time lift the suspension ligament (this is to caress treatment area) 3 times.
2. Both hands thumb point: Shanzhong (Ren-17), Rugen (St-18), Dabao (SP-21), Yingchuang (St-16), Zhongfu (LU-1), Yunmen (LU-2), 3 times.
3. Caress the treatment area, 3 times.
4. Overlap both palms from Shanzhong (Ren-17) point to chest both sides back and forth as if writing an "8" number, 3 times.
5. Lift the accessory breast to the suspensory ligament alternately with both hands, first left, then right, 10 times.
6. Caress the treatment area, 3 times.
7. Sit beside the customer, first left, then right, with both hands to soothe and lift the chest, 3-5 times.
8. Alternately push the


Technique 1,3,6,10

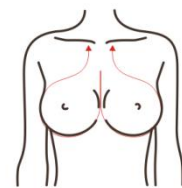


Technique 2

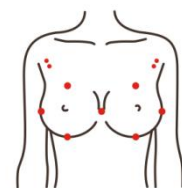







Technique 4

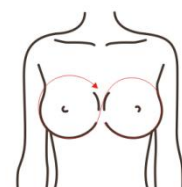


Technique 5

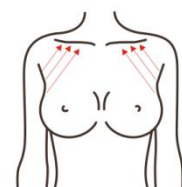

Technique 7,13

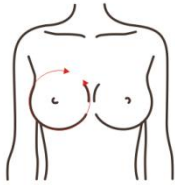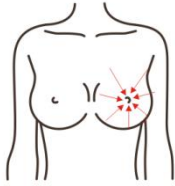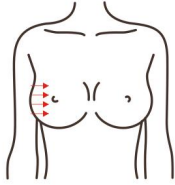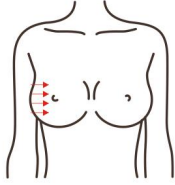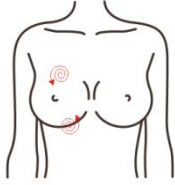
		<p>breast duct around the chest (one circle of the chest), 3-5 times.</p> <p>9. Use the thumb to dredge the nodules in the chest, 3 times.</p> <p>10. Lift the whole chest with hands, 3-5 times.</p> <p>11. The technique on the other side is the same as above.</p> <p>12. RF First left, then right, from the bottom to the top along the direction of the chest to the nipple, combined with hand pacification, 5-8 times.</p> <p>13. Draw half a circle along the chest to lift and pull from the bottom to the collarbone, combining with hand comfort, 5-8 times.</p> <p>14. Make small circles to clear the nodule site for 3-5 times.</p> <p>15. The customer lies on his side and stands up with her arms. The device burns fat in circles at the position of auxiliary breast, 5-8 times.</p> <p>16. The instrument is pushed from the position of underarm auxiliary breast to the chest (shaping and receiving auxiliary breast), 5-8 times</p> <p>17. The technique on the other side is the same as above.</p>	 <p>Technique 8</p>  <p>Technique 9</p>  <p>Technique 12</p>  <p>Technique 14</p>  <p>Technique 15</p>  <p>Technique 16</p>
--	--	---	---


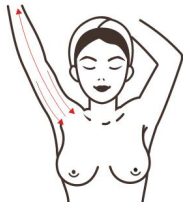
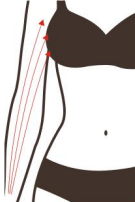
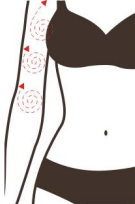


			
--	--	--	---

Suggested Treatment:

10 times for a course of treatment, after a complete chest fever to promote blood circulation, see a certain improvement effect, a course of treatment the effect is more obvious, improve the nodules. Three courses of skin firming and consolidation, increase the elasticity of the chest. Regulating endocrine makes women more attractive.

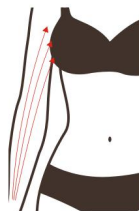Shaping Arms: 60 min, 1-2 times/week

RF Energy: 3-8 Mode: A 40K&RF Energy: 3-8 Mode: B	Massage cream+ Gel	1. Left-right order: lay the arm flatwise, apply and rub oil in from lower arm to the entire arm and slide to the fingers, 3 times. 2. Push the entire arm with both palms, 3 times. 3. Push hands upwards tracing three main collateral channels: Large Intestine Meridian(LI)	Technique 1,2,3,4,5,6  Technique 7,8
--	--------------------------	--	---

		<p>- Triple Energizer Meridian(TE) - Small Intestine Meridian(SI) to armpits with fingers spread and hukou pointing upwards, 3 times.</p> <p>4. Caress the treatment area, 3 times.</p> <p>5. Trace and rub three main collateral channels on arms with kneeling finger back and forth till them turn hot, 3 times.</p> <p>6. Caress the treatment area, 3 times.</p> <p>7. Lay inner arm upwards, and push hands tracing three yin channel on inner arm:</p> <p>Lung Meridian(LU) - Pericardium Meridian(PC) - Heart Meridian(HT) to armpit, with hukou pointing upwards, 3 times respectively.</p> <p>8. Rub three channels back and forth with kneeling fingers, 3 times.</p> <p>9. Caress the treatment area and slide to fingers.</p> <p>10. Do the same on the other side.</p> <p>11. Clean the treated area with hot towel and apply gel evenly.</p> <p>12. RF</p> <p>Lay arms flatwise and start from fat part of lower arm, tracing three channels to armpit, 3 times.</p> <p>13. Move the device in annular motion, tracing three channels to armpit, 3 times.</p>	 <hr/> <p>Technique 9</p>  <hr/> <p>Technique 12,15</p>  <hr/> <p>Technique 13,14</p>  <hr/> <p>Technique 16</p>  <hr/> <p>Technique 18</p>  <hr/>
--	--	---	--

		<p>14. Double the treatment on flabby arms if needed, 3 times.</p> <p>15. Trace three channels back and forth and then to the underarm, 3 times.</p> <p>16. Lay inner arm flatwise and push three channels on upper arm to armpit, 3 times.</p> <p>17. Trace three channels on upper arm and move the device in circles to armpit, 3 times.</p> <p>18. Trace three channels from upper arm to armpit.</p> <p>19. 40K&RF</p> <p>Lay arms flatwise and start from fat part of upper arm, tracing three channels to armpit, 3-5 times.</p> <p>20. Move in small circular motions evenly on arm fat, 3-5 times.</p> <p>21. Slide up and down arm fat, 3-5 times.</p> <p>22. Do the same on the other arm.</p> <p>23. Warp the straps around the arms (loosely).</p> <p>24. Adjust mode and energy level. Strap the lipo laser pads around the arms. Wait for 20 min or so.</p> <p>25. Remove the pads.</p> <p>26. Clean arms with hot towel.</p>	
--	--	---	--

Suggested Treatment:

Ten times for a course of treatment. Once done, it will feel warm and tight, and the arms will be relaxed. A course of treatment down arm excess fat began to reduce, the skin began to close to full, two courses of treatment began to model, strengthen the effect, dredging meridian, enhance the physique. Three courses consolidate stability and prevent rebound.

Shaping Back: 60 min, once/week

RF Energy:
3-8

Mode: A

Vacuum&RF
Energy:
3-8

Suction
Time: 3-8

Release
Time: 0-6


Essential
oil+
Gel

1. Apply oil on back and press Fengchi (Gb-20), and Fengfu (Du-16) points.
2. (Starting from hairline) stroke Da Ban Jin with thumb, 3-5 times.
3. Move thumb outwards from Du Channel (GV) to the 8 Crevice area and then to Fengchi (Gb-20), and Fengfu (Du-16) point.
4. Starting from neck to caudal vertebra, move hands in S-shape, 3 times.
5. Push Bladder Channel (BL) with thumb in left-right order, 3 times.
6. Push Bladder Channel (BL) to 8 Crevice area with both thumbs, at the same time, 3 times.
7. Push Bladder Channel (BL) in three kneeling fingers, 3 times.
8. Push scapula slot with both hands alternately in left-right order, 3-6 times.
9. Push scapula slot with both hands horizontally, 3-6 times.
10. Caress the back with both hands and push Fengchi (Gb-20), and Fengfu (Du-16) points, 3


Technique 1,10




Technique 2,15,25

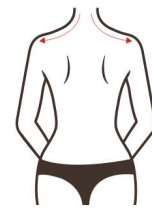




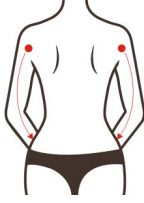
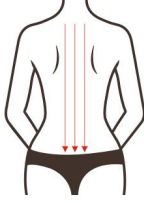
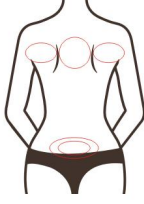
Technique 3



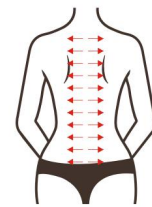


Technique 4

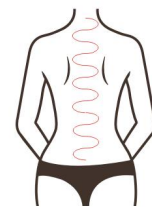


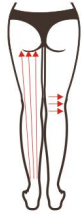


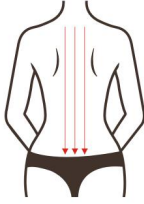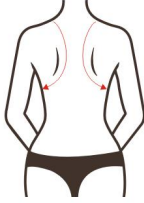
Technique 5,6,7

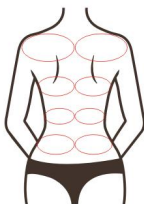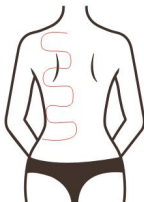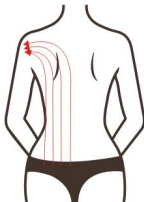

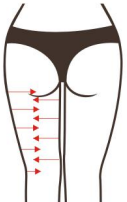
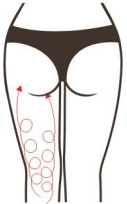
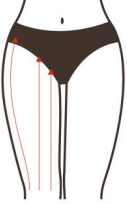

		<p>times.</p> <p>11. Overlap both thumbs and push Tianzong (SI-11) point and slide to the arm, 3 times.</p> <p>12. Repeat the previous action on another arm, 3 times.</p> <p>13. RF Starting from Du Channel (GV) to Bladder Channel (BL), move the device from neck to the 8 Crevice area, 3-5 times.</p> <p>14. Move the device in circles on Dazhui (Du-14) point, and the 8 Crevice area, 3-5 times respectively.</p> <p>15. From Da Ban Jin to neck, move the device back and forth, 3-5 times.</p> <p>16. Lift the device on scapula slot in left-right order, 3-5 times.</p> <p>17. Move the device horizontally as if writing an "8" number to the 8 Crevice area, 3 times.</p> <p>18. Move in annular motions from two sides till the underarm, 3 times.</p> <p>19. Starting from Du Channel (GV) to Bladder Channel (BL), move the device from neck to the 8 Crevice area, 3-5 times.</p> <p>20. Vacuum&RF Starting from Du Channel (GV) to Bladder Channel (BL), move the device from neck to the 8 Crevice area, 3-5 times.</p> <p>21. Move the device in circles on Dazhui (Du-14)</p>	 <hr/> <p>Technique 8,16,26</p>  <hr/> <p>Technique 9</p>  <hr/> <p>Technique11,12</p>  <hr/> <p>Technique 13,19,20</p>  <hr/> <p>Technique 14,21</p>  <hr/> <p>Technique 17</p>
--	--	--	---




		<p>point, and the 8 Crevice area, 3-5 times respectively.</p> <p>22. Coupled with hand, lift from two sides of the underarm towards Du Channel (GV) in parallel lines to the waist, 3 times.</p> <p>23. From left to right, move in annular motions from two sides till the underarm, 3 times.</p> <p>24. Lift upwards from sides of the waist to the underarm, 3-5 times.</p> <p>25. Lift Da Ban Jin horizontally back and forth, 3-5 times.</p> <p>26. Stroke back and forth along shoulder blades, 3-5 times.</p> <p>27. Warp the straps around the back (loosely).</p> <p>28. Adjust mode and energy level. Strap the lipo laser pads on the back. Wait for 20 min or so.</p> <p>29. Remove the pads.</p> <p>30. Clean treated area with hot towel.</p>	 <p>Technique 18,23</p>  <p>Technique 24</p> 
<p>Suggested Treatment:</p> <p>10 times account for a full course. Once done, back feels significantly relaxed, neck stiffness alleviated, and skin feels lifted. After a full course, shoulder feels relaxed, back fat lessens, the lump on the nape of the neck improves, and the back line starts to show. 2 courses help dredge meridians, regulate sub-health, and improve sleep. 3 courses help achieve thin back, curvy back line, better viscera function and digestion, detoxification and metabolism. (one course helps reduce size, two courses help strengthen the effects, three courses help with consolidation and avoid rebound.)</p>			
<p>Shaping Buttocks: 60 min, 1-2 times/week</p>			
<p>RF Energy: 3-8 Mode A</p>	<p>Massage cream+ Gel</p>	<p>1. Standing on the side, accumulate oil on hands and then slide to the waist</p>	<p>Technique 1,4,10</p>

<p>40K&RF Energy: 3-8 Mode: B</p>	<p>and then lift up along the hips from the waist, 3 times (this is to caress the treatment area.)</p> <p>2. Repeatedly push the 8 Crevice area with two thumbs.</p> <p>3. Caress the treatment area for 3 times and then point: Shenyu (Bl-23), 8 Crevice area, Changqiang (Du-1), Huantiao (Gb-30), Chengfu (Bl-36), 3 times.</p> <p>4. Caress the treatment area, 3 times.</p> <p>5. First left, then right, with both hands extending from the root of the thigh from bottom to top --Bladder Channel (BL) - Kidney Channel (KI) - Liver Channel (LV) - Gallbladder Channel (GB) - Dai Channel (BV) , 3 times each.</p> <p>6. Both hands follow from the root of thigh from bottom to top -- Bladder Channel (BL) - Kidney Channel (KI) - Liver Channel (LV) - Gallbladder Channel (GB) - Dai Channel (BV) , 3 times each.</p> <p>7. Overlap both palms and push up together from thigh root to pulse (ascension) ,3 to 5 times.</p> <p>8. Push your hands from bottom to top from both sides of your hips to the top of your hips, 3 times.</p> <p>9. Repeat the previous operation.</p> <p>10. Caress the treatment area.</p>	<div data-bbox="1114 210 1284 421"> </div> <p>Technique 2</p> <div data-bbox="1114 501 1284 712"> </div> <p>Technique 3</p> <div data-bbox="1114 792 1284 1003"> </div> <p>Technique 5,12,15</p> <div data-bbox="1114 1084 1284 1294"> </div> <p>Technique 6</p> <div data-bbox="1114 1375 1284 1585"> </div> <p>Technique 8,13,16</p> <div data-bbox="1114 1666 1284 1877"> </div> <p>Technique 14</p>
---	---	---

		<p>11. The technique on the other side is the same as above.</p> <p>12. RF Lift from the thigh root to the Dai Channel (BV) line by line, 3 times.</p> <p>13. Lift 3 times from bottom to top on both sides of the buttocks to the highest point of the buttocks.</p> <p>14. Circle your hips, 3 to 5 times.</p> <p>15. Lift 3 times from thigh root to Dai Channel (BV).</p> <p>16. Lift from bottom to top on both sides of the buttocks to the highest point of the buttocks, 3 times.</p> <p>17. The technique on the other side is the same as above.</p> <p>18. 40K&RF Lift from the thigh root to the Dai Channel (BV) line by line, 3 times.</p> <p>19. Lift 3-5 times from bottom to top on both sides of the buttocks to the highest point of the buttocks.</p> <p>20. Move in small circular motions around the buttocks, 3-5 times.</p> <p>21. Warp the straps around buttocks (loosely).</p> <p>22. Adjust mode and energy level. Strap the lipo laser pads on the buttocks. Wait for 20 min or so.</p> <p>23. Remove the pads.</p> <p>24. Clean treated area with</p>	
--	--	---	---

		hot towel.	
Suggested Treatment: 10 times account for a full course. After one treatment, buttocks feel lifted and heated. After a course, hip curve starts to show, excess fat slowly disappears, skin becomes tight. 3 courses help improve gynecological problems like cold body and cold uterus. Menstruation returns to normal. It helps consolidate the effect and increase the female sex appeal.			
<h2 style="text-align: center;">Shaping Legs: 70 min, once/week</h2>			
Vacuum&RF Energy: 3-8 Suction Time: 3-8 Release Time: 0-6 40K&RF Energy: 3-8 Mode: A	Essential oil+ Gel	1. Prone Position - Rear Leg Treatment In left-right order, apply and rub oil in from calf to thigh to heel, 3 times. 2. Move hands upwards to four main collateral channels:Bladder Channel (BL) - Kidney Channel (KI) - Liver Channel (LV) - Gallbladder Channel (GB) with fingers spread and hukou pointing upwards. 3. Push hands alternately to popliteal fossa, 3 times. 4. Twist both hands upwards alternately as if twisting a fired dough, 3 times. 5. Caress the treatment area, 3 times. 6. Push four main collateral channels upwards with kneeling fingers of both hands, 3 times. 7. Caress the treatment area, 3 times. 8. Do the same on the other side. 9. 40K&RF Move the device from popliteal fossa to Bladder Channel (BL) - Kidney	Technique 1,2,3,5,6,7,8,9  <hr/> Technique 4  <hr/> Technique 10,17  <hr/> Technique 11,16,18  <hr/> Technique 12,14,19

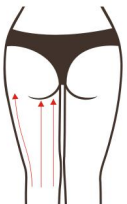		<p>Channel (KI) - Liver Channel (LV) - Gallbladder Channel (GB) , 3 times.</p> <p>10. Move the probe in small circular motion on calf, 3 times.</p> <p>11. Move the device upwards to popliteal fossa and caress the treatment area with hands, 3 times.</p> <p>12. Push from popliteal fossa tracing 4 main collateral channels to the end of thigh, 3 times.</p> <p>13. Move the device in circular motion from popliteal fossa to the end of thigh, 3 times.</p> <p>14. Push from popliteal fossa tracing main collateral channels to the end of thigh, 3 times.</p> <p>15. Clean with towel and apply gel evenly on the treatment area.</p> <p>16. Vacuum&RF</p> <p>Coupled with hand, lift from heel towards popliteal fossa in parallel lines, 3 times.</p> <p>17. Move the probe in small circular motions on calf, 3 times.</p> <p>18. Slide up and down the calf, 3-5 times.</p> <p>19. Stroke from popliteal fossa towards the end of thigh in parallel lines, 3 times.</p> <p>20. Move the probe in small circular motions on thigh, 3 times.</p> <p>21. Lift from sides of thigh towards the middle, 3-5 times.</p>	 <hr/> <p>Technique 13,15,20</p>  <hr/> <p>Technique 21</p>  <hr/> <p>Technique 22</p>  <hr/> <p>Technique 24,25,26,29,32,34</p>  <hr/> <p>Technique 30,35</p>  <hr/> <p>Technique 31</p>
--	--	--	---

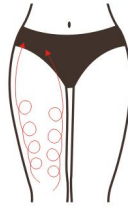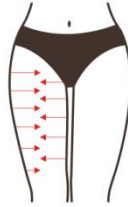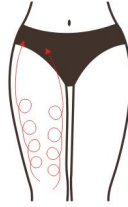		<p>22. Move in annular motions on two sides of the thigh to lift the treatment area, 3 times.</p> <p>23. Do the same on the other side.</p> <p>24. Fixate the lipo laser pads on leg fat. Wait for 20 min or so.</p> <p>25. Remove the pads.</p> <p>26. Supine Position - Foreleg Treatment</p> <p>Rub oil with both hands from heel to the thigh root (namely to caress the treatment area), 3 times.</p> <p>27. Move hands upwards to four main collateral channels: Spleen Channel (SP) - Stomach Channel (ST) - Liver Channel (LV) - Gallbladder Channel (GB) with fingers spread and hukou pointing upwards, 3 times.</p> <p>28. Push four main collateral channels upwards with kneeling fingers of both hands, 3 times.</p> <p>29. 40K&RF</p> <p>Lift the device towards knee while tracing 4 channels, 3 times(start from thigh if there's not enough unwanted fat on calf.)</p> <p>30. Move the device on two sides of calf in annular motion, 3 times(start from thigh if there's not enough unwanted fat on calf.)</p> <p>31. Stroke upwards from knee to the end of thigh in parallel line, 3 times.</p> <p>32. Move in small circular</p>	 <hr/> <p>Technique 36</p>  <hr/> <p>Technique 37</p>  <hr/>
--	--	--	--

		<p>motions on the entire thigh area, 3 times.</p> <p>33. Lift the device from knee to thigh root in annular motion, 3 times.</p> <p>34. Lift the probe from knee to thigh root in parallel lines, 3 times.</p> <p>35. Vacuum&RF</p> <p>Lift the device towards knee while tracing 4 channels, 3 times(start from thigh if there's not enough unwanted fat on calf.)</p> <p>36. Coupled with hand, lift from 2 sides of leg to the middle, 3 times.</p> <p>37. Move in small circular motions on thigh, 3 times.</p> <p>38. Move from two sides of the thigh towards the middle in parallel lines, 3-5 times.</p> <p>39. Move the probe on two sides of thigh in annular motions to lift the treatment area, 3 times.</p> <p>40. Do the same on the other side.</p> <p>41. Warp the straps around the legs (loosely).</p> <p>42. Adjust mode and energy level. Strap the lipo laser pads on leg fat. Wait for 20 min or so.</p> <p>43. Remove the pads.</p> <p>44. Clean with hot towel.</p>	
--	--	--	--

Suggested Treatment:

10 times for a course of treatment, after a course of treatment leg is relaxed, circulation is accelerated, a course of treatment leg becomes thin, skin collagen tissue because of heat and produce tightening effect, can obviously feel skin tight feeling, two course of treatment leg is fine tight effect obvious, three course of treatment consolidate effect. Super RF + super ultrasound is to stimulate the dermis of the skin, which can sustain collagen regeneration, so the treatment effect will be more and more obvious.